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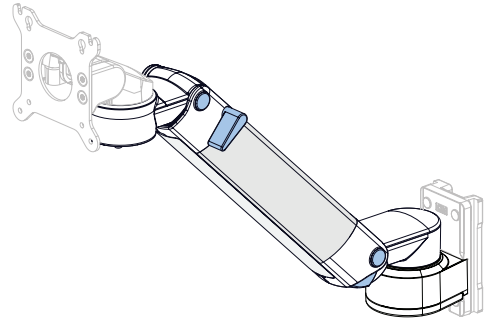
ASSEMBLY INSTRUCTION

HEIGHT ADJUSTABLE ARM

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Name: CIM height adjustable arm

Part no. code of the arm: 431

Part no. code of the monitor adaptation: A, AVS, C, E, F, FK, G, M, MD, PH, Q, Q1, SLX, SV, TN, WA

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TABLE OF CONTENT

Area of application/ Intended use	2
General notes	2
Warning installation, operation and maintenance	2
Tools required	3
Mounting the medical electrical equipment	3
Cleaning and disinfection	3
Maintenance: manner and frequency	4
Safekeeping	4
Assistance required	4
Name of manufacturer	5
CE mark	5
Authorized distributor	5
Scope of delivery	6
Monitor adaptation	7 - 8
Assembly steps	9 - 16
Trouble shooting	17
Maintenance plan	18

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GENERAL NOTES

PLEASE READ THIS INSTRUCTION CAREFULLY BEFORE INSTALLING!

Clarification of non-standardized Symbols:

Warning of a danger point

Important information

Danger of squeezing fingers

AREA OF APPLICATION/ INTENDED USE

Cable Integrated Mounting systems (CIM) are intended to mount and position monitors, mouse/keyboard, laptops and other medical devices or equipment (hereafter referred to as ME equipment). They are mounted to walls, carts, tables, other surfaces or machines using specific mounting adapters. CIM systems do not fulfil any specific medical purpose on their own.

GENERAL NOTES

Verify that the CIM wall channel has been installed and approved according to the CIM channel installation instructions. If you have a wall channel of another manufacturer, please read the corresponding assembly instruction of the manufacturer carefully.

If you want to attach the height adjustable arm (hereafter referred to as support arm) to an existing European rail or vertical pole, please verify that the rail can hold the weight. Please check the weight of the ME equipment to be mounted. The permissible total weight is mentioned on the corresponding data sheet.

WARNING Installation, Operation and Maintenance

This section contains warnings regarding the installation, operation and maintenance of the support arm. This section must be read in its entirety before installing, operating and maintaining the support arm. Failure to follow these warnings may result in damage to equipment or injury to people.

Installation, operation and maintenance may only be carried out by qualified personnel. Cable integration and assembly may only be done by qualified technical personnel. CIM med GmbH cannot be held responsible for any damage due to poor installation.

Do not position the support arm above a patient. Note that the support arm has a wide range of motion both up/down and side to side. Please consider carefully the ME equipment being mounted and the proximity of the mounting assembly to other equipment, hospital personnel, and the patient. CIM med GmbH recommends that the hospital's risk management personnel verify that the application is appropriate prior to installation and use of the support arm.

Your support arm contains a powerful spring, used to counterbalance the

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weight of the bearing load. Mounted ME equipments must be removed with the support arm in the highest vertical position, and with the height locking lever engaged. Failure to follow these guidelines could result in injury if this procedure is not followed.

The total load mounted may have changed due to adding or removing equipment (other than primary instrument) while the height locking lever was engaged. Changing the total weight may result in a sudden downward or upward movement of the arm when the height locking lever is disengaged.

Ensure that the weight of the instrument being mounted does not exceed the maximum load indication.

Product modifications, disassembling and reassembling and/or mounting any of the components in any other way than described below are permitted only after written approval by the CIM med GmbH construction and engineering department. CIM med GmbH cannot be held responsible for any damage or injury caused by product modifications and/or amendments without prior permission by CIM med GmbH.

⚠️ WARNING: MORTAL DANGER BY ELECTRIC SHOCK

The electric cables of ME equipment and ME systems may only be connected to the power source and to the medical device respectively after finishing installation. Prior to maintenance or disassembly all power cables must be disconnected from the power source. Only cables with double insulation may be used for running through the support arm.

⚠️ WARNING: EARTHING CONNECTION

The support arm is delivered with components' grounding. It is the health-care institution's responsibility to connect the support arm to the electrical grounding of the building. When mounting the support arm to wall channel or side channels of anesthesia machines the support arm is connected to the channel via the screws of the adapter.

TOOLS REQUIRED

hex key 2.5 for plastic cover caps
hex key 4.0 for alu cover cap
hex key 4.0 for tilt adjustment
slot head screwdriver size 1 for aluminum profile

MOUNTING THE MEDICAL ELECTRICAL EQUIPMENT

Ensure that the locking lever is engaged with the support arm in the highest vertical position. Do not disengage the locking lever until mounting is complete. Only when all screws are tightened and securely fixed can the ME equipment be mounted. For detailed instructions please see assembly instruction „monitor adaptation“.

CLEANING AND DISINFECTION

CIM support arms may be cleaned with mild, non-abrasive solutions commonly used in hospital environment. Do not use steel wool or other

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GENERAL NOTES

abrasive materials. Avoid all liquids entering the support arm. Your support arm has been designed to the highest specifications to make cleaning as easy as possible and to avoid liquids from penetrating. CIM med GmbH makes no claims concerning the efficacy of the disinfection processes. It is recommended to test any cleaning solution on a small area of the support arm not visible to verify compatibility. Please consult your hospital's infection control manager.

Exemplary disinfection tests have been carried out with the following active substance groups: alcoholic quick disinfectants (e.g. Bacillol AF/ Hartmann), aldehyde and quaternary ammonium compounds (e.g. B5/ Orochemie), alcohol-free quaternary ammonium compounds (e.g. Sani-Cloth AF3/PDI Healthcare). These compounds are VAH / DGHM listed and EPA registered and must be used as per the manufacturers' instructions. The surface finish will be permanently damaged by acetone and trichloroethylene.

MAINTENANCE: MANNER AND FREQUENCY

Very little maintenance is required during the life span of the support arm; however, it is recommended to check all pivot/ swivel points as well as all bearing points regularly. Please check all screws on a regular basis (at least once a year) and tighten them if necessary.

The support arm is designed for 26,000 movements or 13,000 cycles (one cycle = a full movement from one extreme to the other and back again). A daily number of 10 movements on 365 days (= operation on 7 days a week) means 3,650 movements per year which correspond to a durability of 7.1 years.

In case of damage done to the load-bearing parts of the system it has to be strictly refrained from further usage of the mounting system.

Inspection must include the following steps:

With the ME equipment mounted, release the height lock and move the support arm through its entire vertical range of motion. The load should maintain its position at every point in the travel of arm.

Grasp the support arm and pivot it from side to side at the attachment. The support arm should pivot with some tension or resistance.

Grasp the ME equipment and tilt it forward and back, through its entire range of motion. There should be enough tension or resistance in the tilt mechanism to prevent the ME equipment from tilting forward unexpectedly. If necessary, the tilt tension may be adjusted in accordance with section „Tilt adjustment“.

Inspect fasteners for looseness.

Failure to periodically inspect and adjust the support arm as instructed may result in damage to equipment or injury.

SAFEKEEPING

For any questions that may arise at a later time please safekeep our manual!

ASSISTANCE REQUIRED

If you are unable to adjust the arm or require service, contact CIM med technical support: Phone +49 89 978 94 080 or your local dealer.

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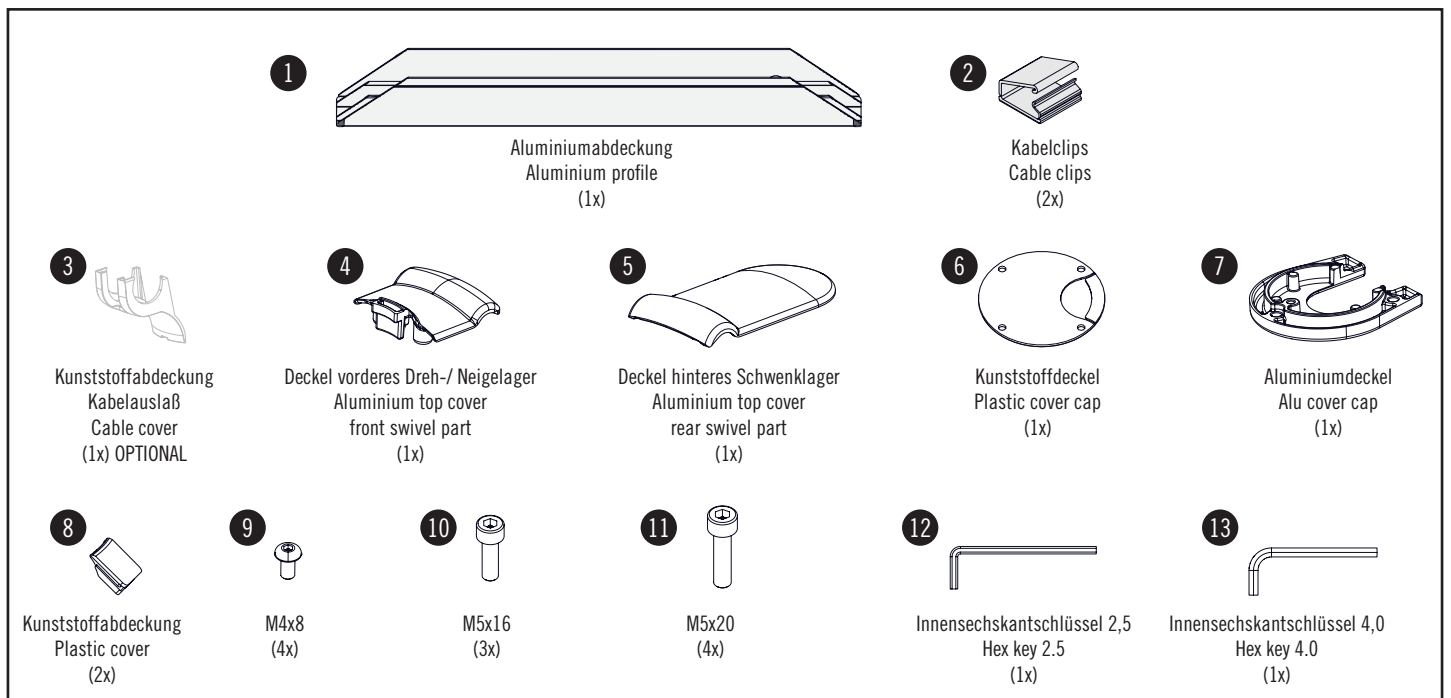
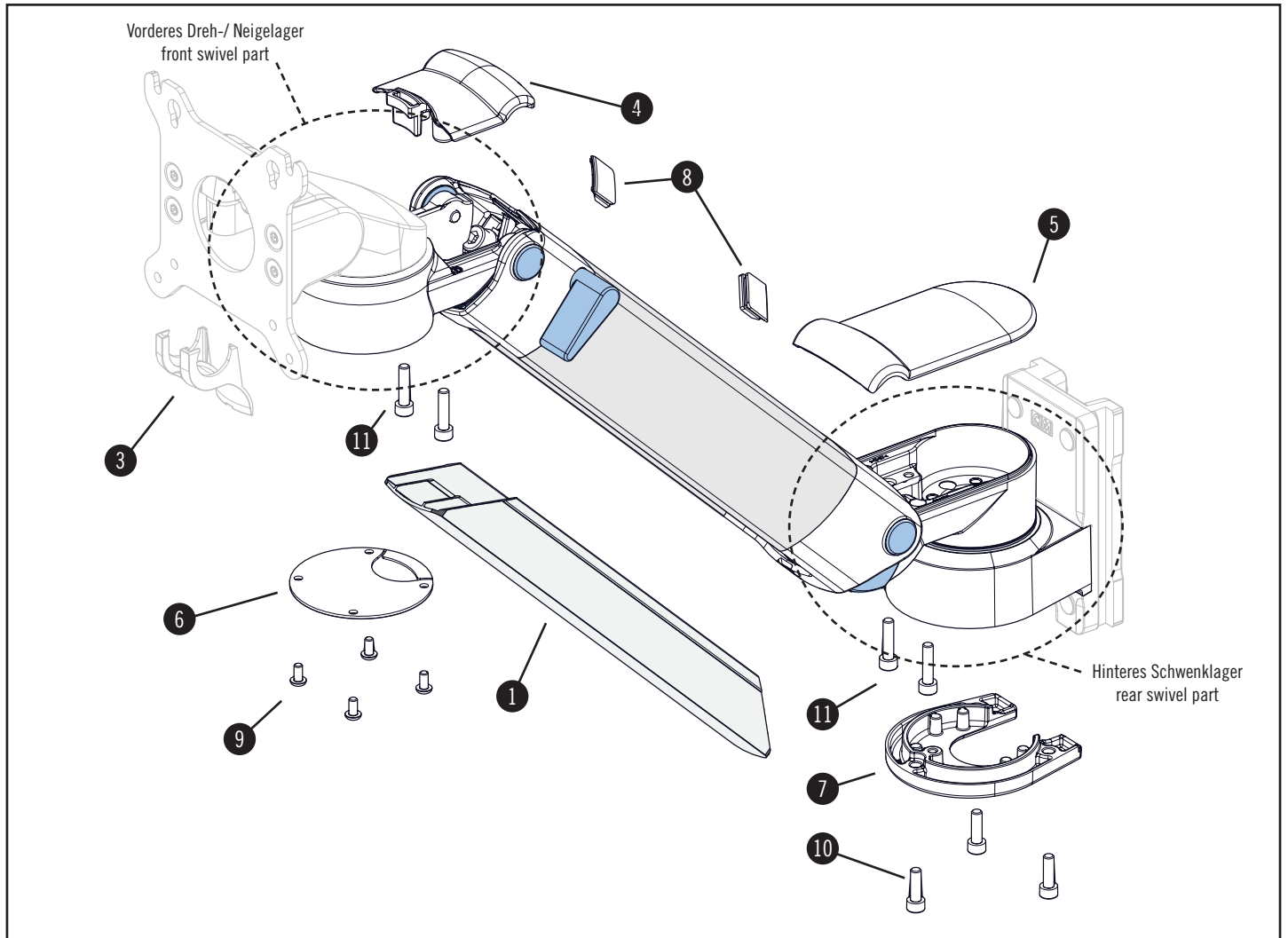
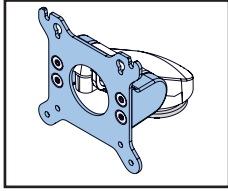
REAR-MOUNTED DEVICES

For rear-mounted devices, see separate assembly instructions:

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
VESA 75/100

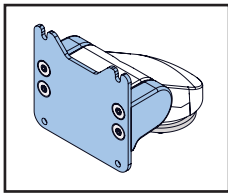- ➔ Assembly Instruction Monitor Adaptation VESA

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
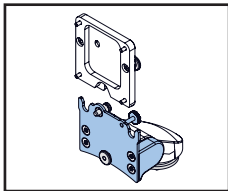
VESA 75 QUICK RELEASE

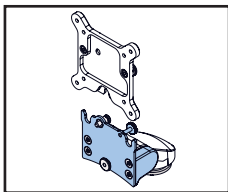- ➔ Assembly Instruction Monitor Adaptation Quick Release VESA 75, VESA 75/100, Mindray SMR

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
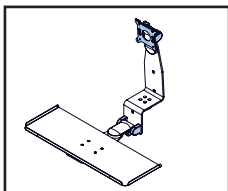
Z-BRACKET

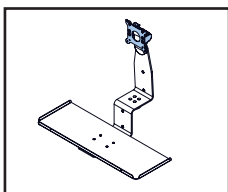- ➔ Assembly Instruction Monitor Adaptation Z-Bracket

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W-BRACKET

- ➔ Assembly Instruction Monitor Adaptation W-Bracket

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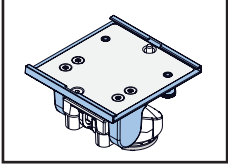
BOTTOM-MOUNTED DEVICES

For bottom-mounted devices, see separate assembly instructions:

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
5" PLUNGE PLATE

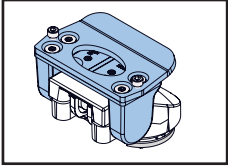- ➔ Assembly Instruction Monitor Adaptation 5" plunge plate

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
MINDRAY QUICK RELEASE

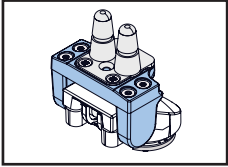- ➔ Assembly Instruction Monitor Adaptation Mindray Quick Release

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
PHILIPS TABLE TOP MOUNT

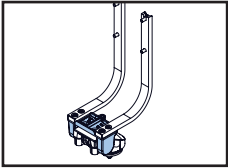- ➔ Assembly Instruction Monitor Adaptation Philips Table Top Mount

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
SPACELABS XPREZZON BACKPACK

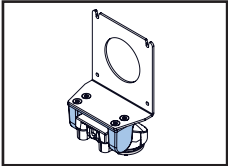- ➔ Assembly Instruction Monitor Adaptation Spacelabs XPREZZON Backpack

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
FUKUDA MOUNTING ADAPTER

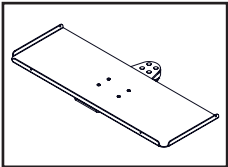- ➔ Assembly Instruction Monitor Adaptation Fukuda

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
KEYBOARD TRAY

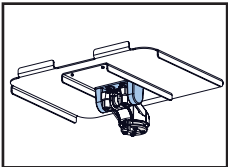- ➔ Assembly Instruction Keyboard Tray for height adjustable arm, support arm pivot, cart

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NOTEBOOK TRAY

- ➔ Assembly Instruction Notebook Tray for height adjustable arm, support arm pivot, cart

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
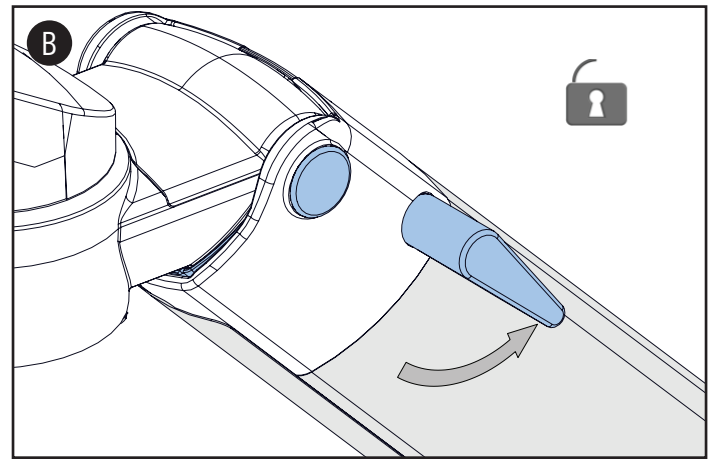
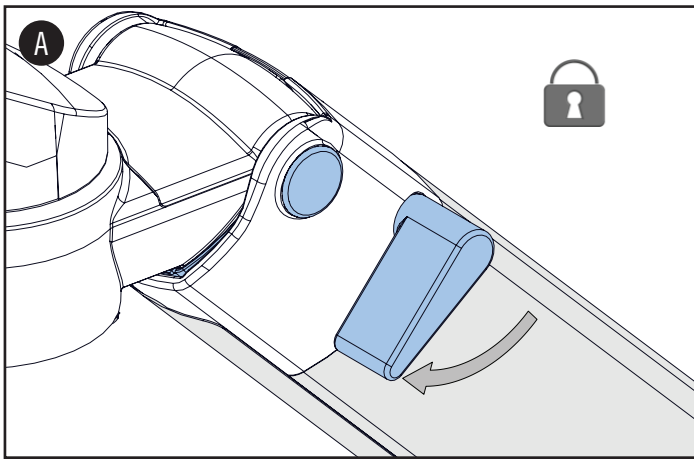
RELEASING THE PRETENTION

If your support arm is delivered with the locking lever engaged holding the support arm in a horizontal position you have to release the pretention before the installation. Hold the support arm at the middle of the aluminium part with one hand and disengage locking lever with the other hand. This ensures an upward movement in a slow and even tempo and brings the support arm into the highest mounting position.

Illustration A shows the support arm with the locking lever engaged.
Illustration B shows the support arm with the locking lever not engaged.

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
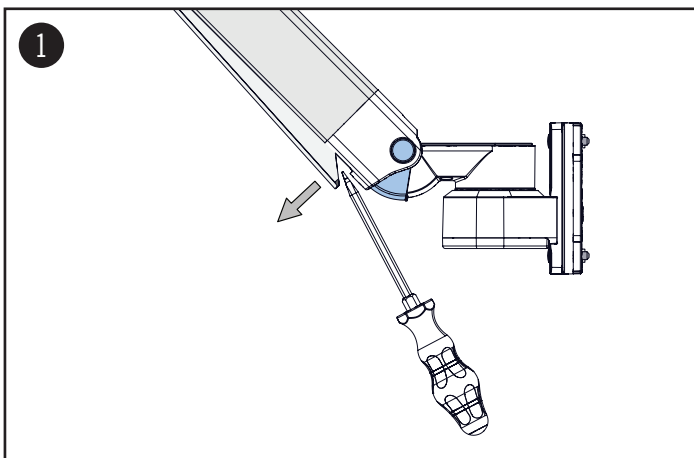
ASSEMBLY STEPS

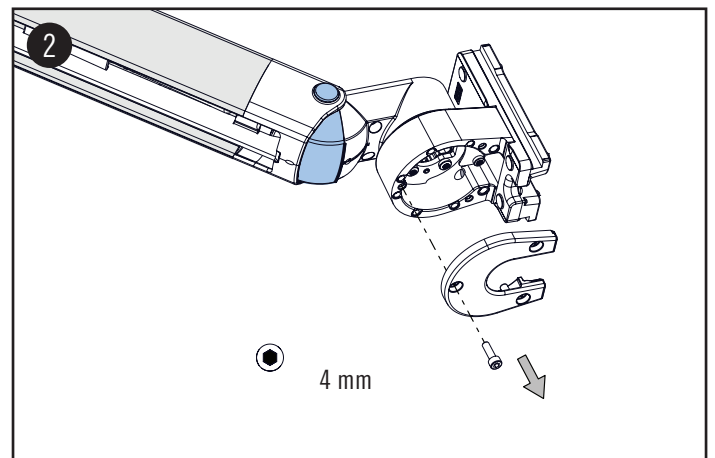1. Remove aluminium profile by using a slot head screwdriver.
2. Remove alu cover cap of the rear swivel part.

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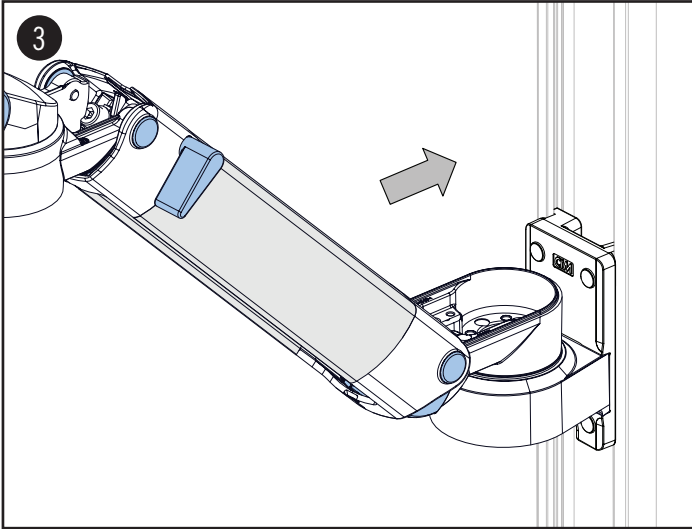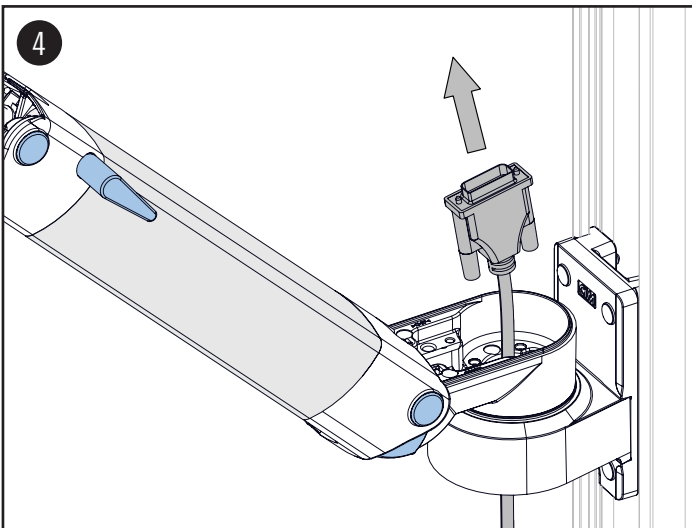
ASSEMBLY STEPS

3. Mount the support arm to its designated place (wall channel, tube, DIN rail etc.). The illustration shows a wall channel adapter. Please refer to the corresponding assembly instruction for all other adapters.

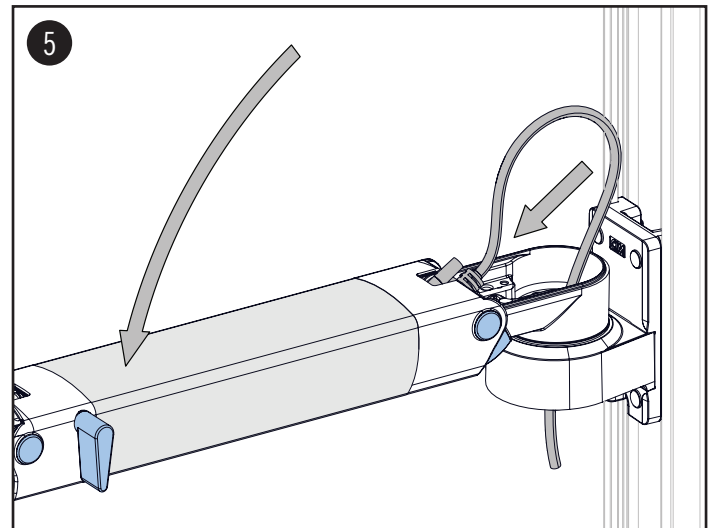
4. Feed the cables one after the other through the rear swivel part.

NOTE: Always start with the largest cable first if you have multiple cables.

5. Bring the support arm in a horizontal position and secure it by engaging the locking lever. Feed the cable through the support arm.

NOTE: Depending on the plug size, it may be necessary to adjust the position of the support arm slightly using the locking lever.


NOTE: Always start with the largest cable first if you have multiple cables to thread through the arm.

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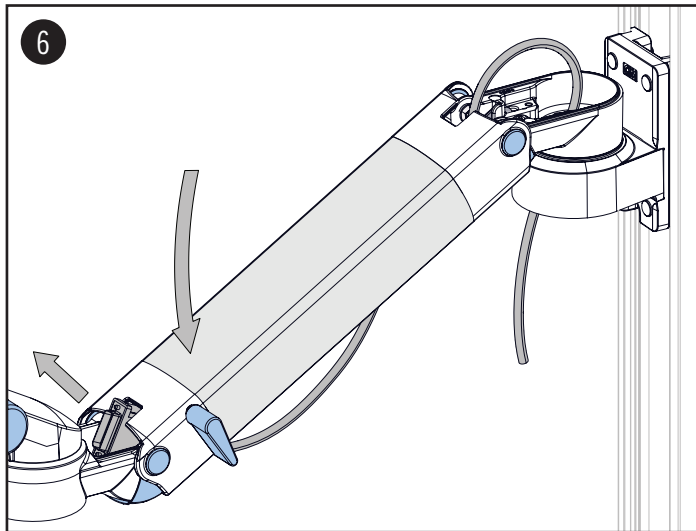
6. Bring the support arm to a lower position (approx. 30°) and feed the cable through the front swivel part.

NOTE: Always start with the largest cable first if you have multiple cables.

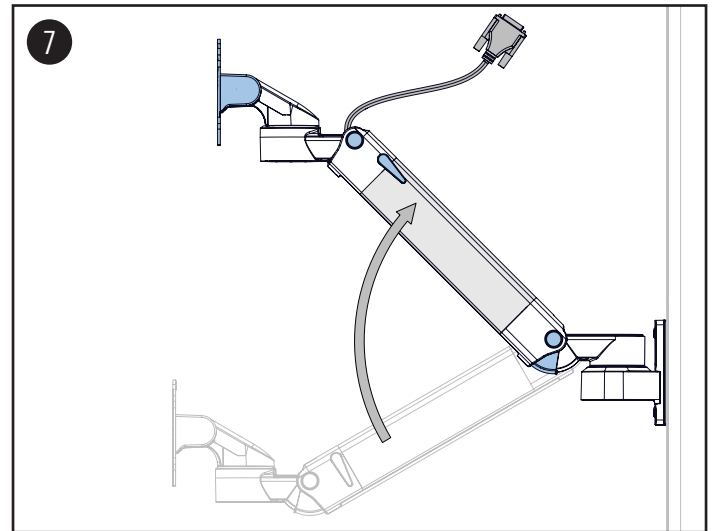7. Bring the support arm to its highest position.

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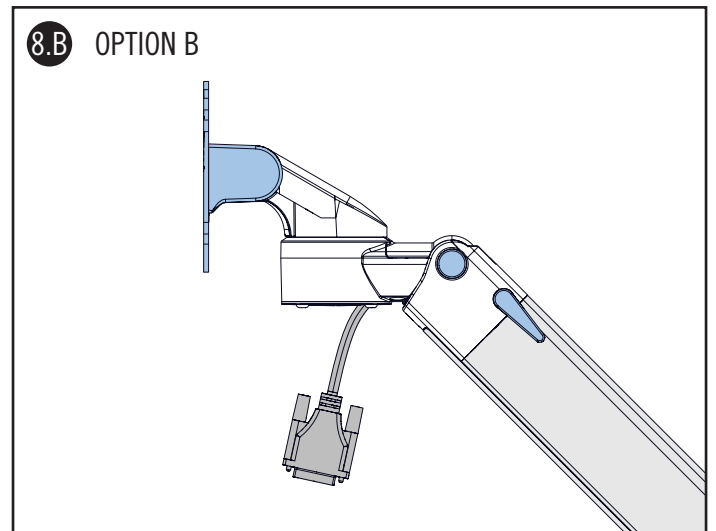
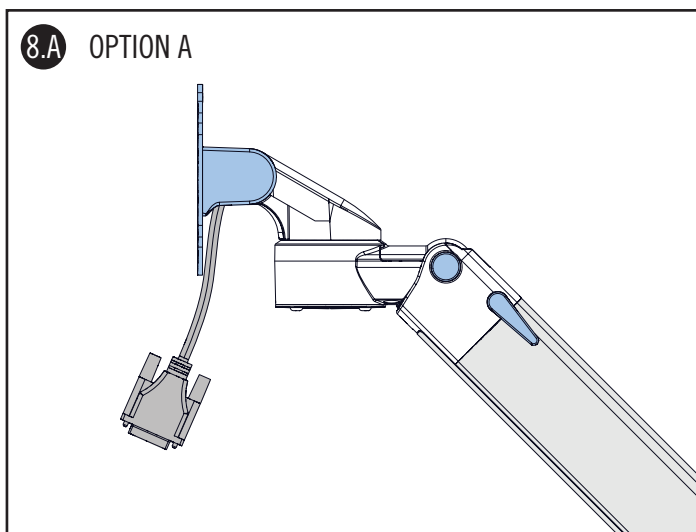
8. From this point there are two options for cable exits.
OPTION A: through the front swivel part (illustration 8.A) or
OPTION B: underneath the front swivel part (illustration 8.B).

Option A is suitable for 1 - 4 cables depending on the wire cross section.

Option B is recommended for easier cabling with 3 or multiple cables.

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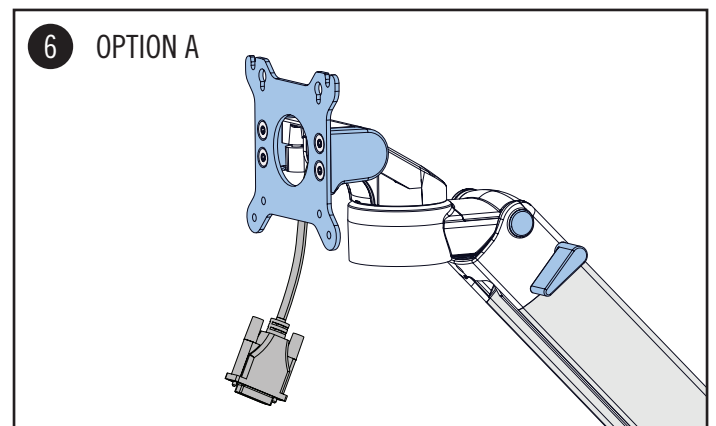
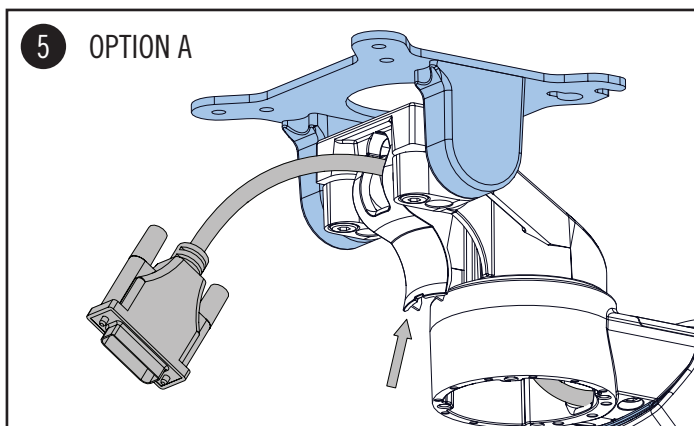
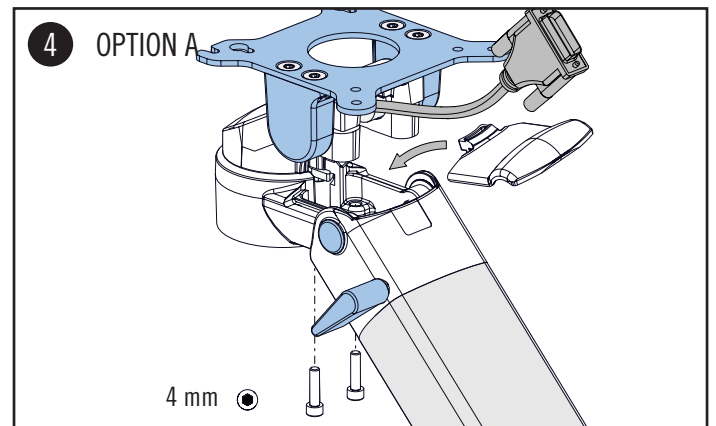
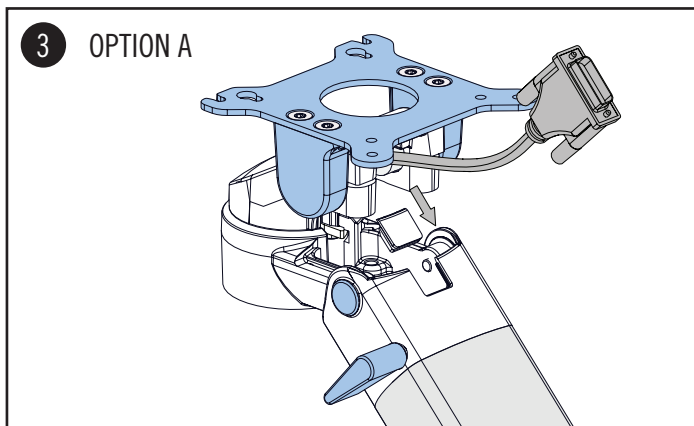
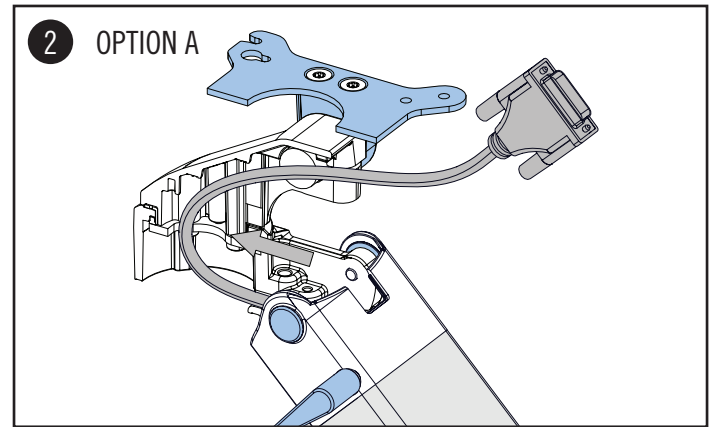
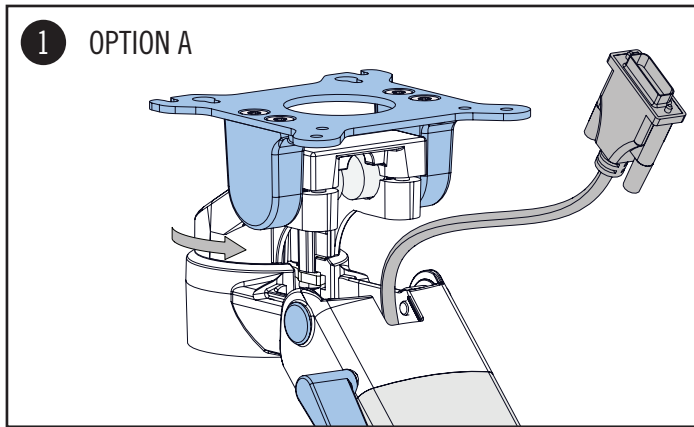
OPTION A: 1 - 4 CABLE

- Turn the front swivel part by 180° facing inwards towards the rear end of the support arm, so that the two cutaways line up. Vertical monitor adaptations have to be positioned horizontally (illu. 1).
- Push the cables all the way to back to the front swivel part forming a loop. Attach the little plastic cover before mounting the aluminium top cover to the front swivel part by using (2) M5x20 screws. Screw both securely (illu 2 - 4).







⚠ WARNING: Push the cables all the way to the back of the front swivel part in order to avoid cable pinching when turning the tilt to the head.

- Turn the front swivel part back to the starting position (except the Philips table top mount). Attache the cable cover (illu 5 - 6).

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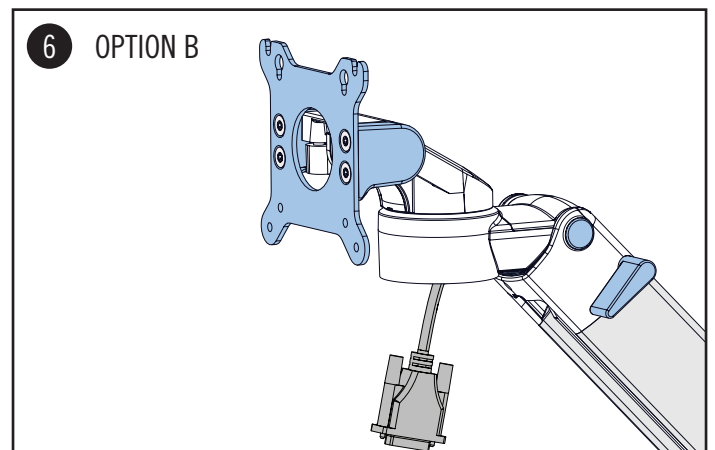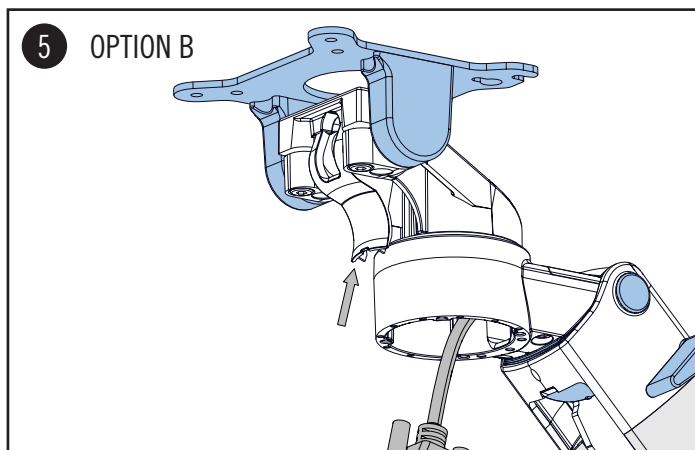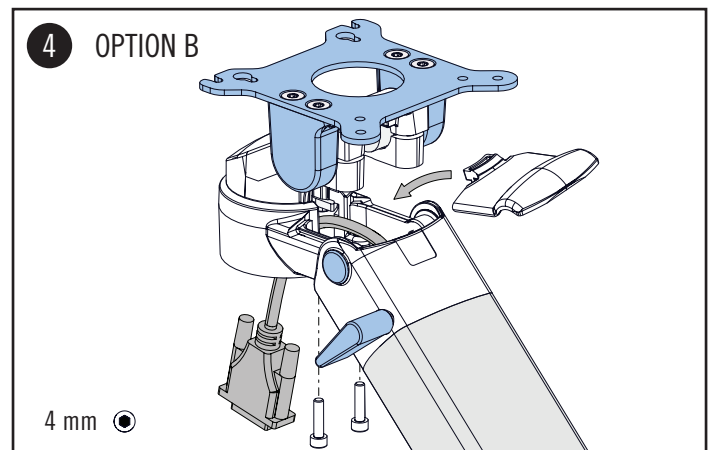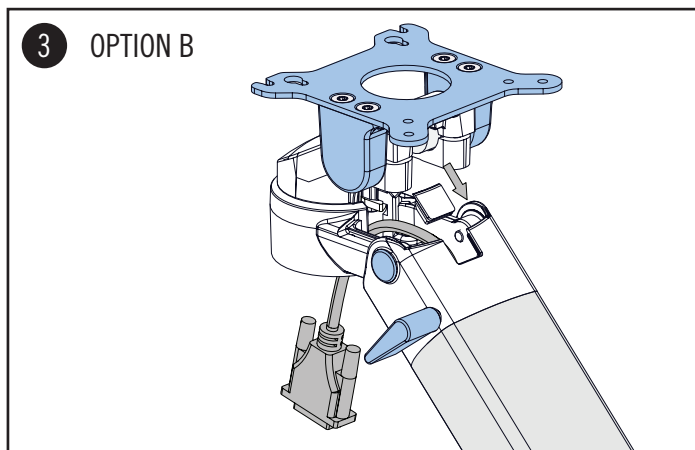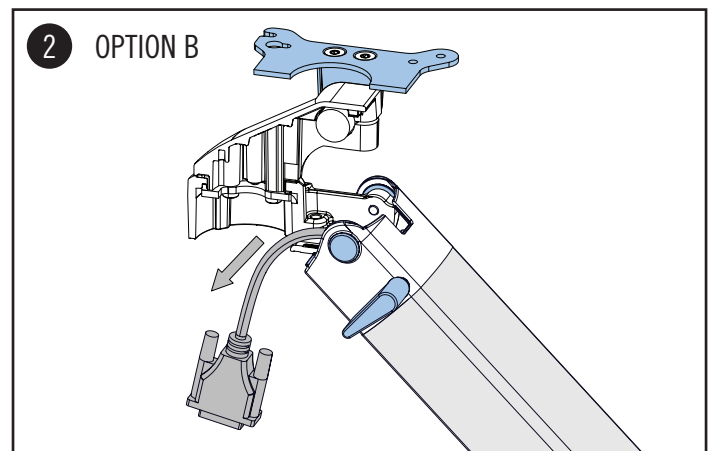
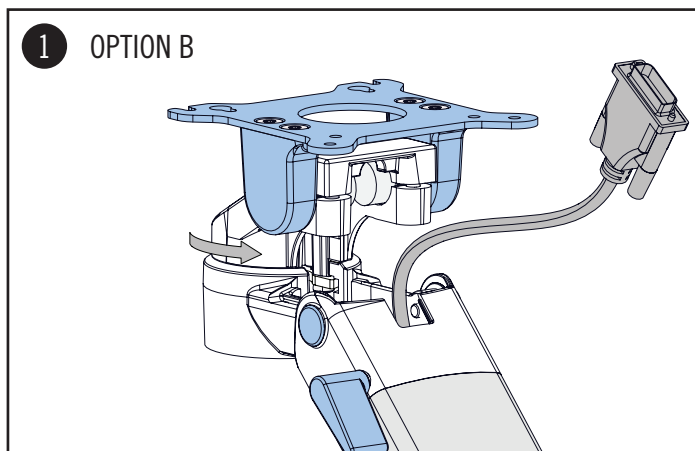
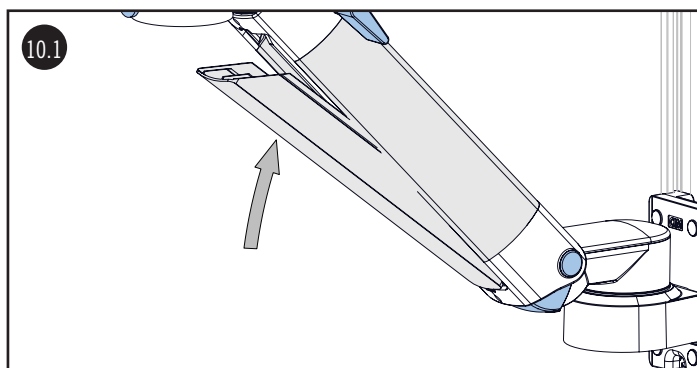
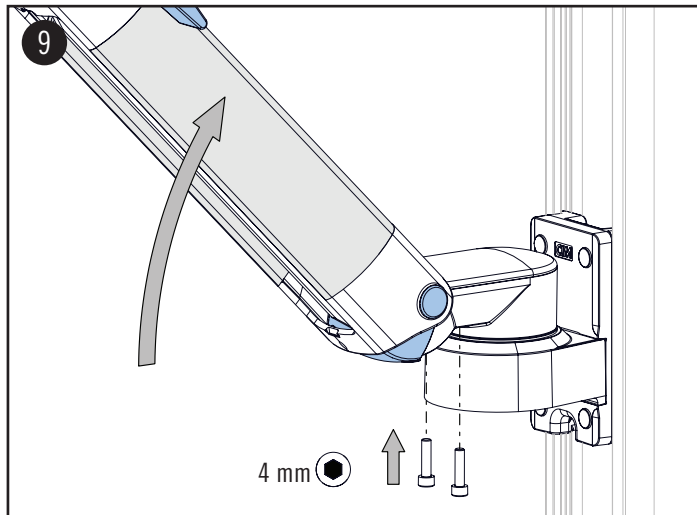
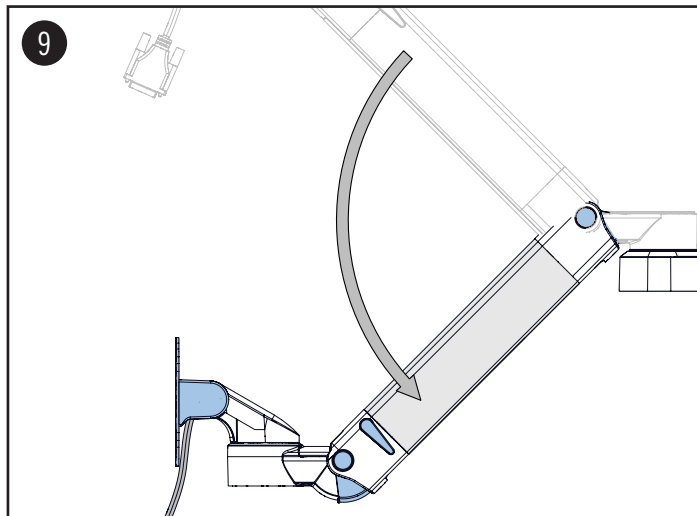
ASSEMBLY STEPS

9. Bring the support arm to the lowest position. Place the plastic cover into the slot and mount the aluminium top cover to the rear swivel part using the two M5x20 screws. Screw both securely.

⚠ WARNING: As soon as the support arm is in its lowest position the locking lever has to be engaged.

10. Clip the aluminium cover into the support arm.

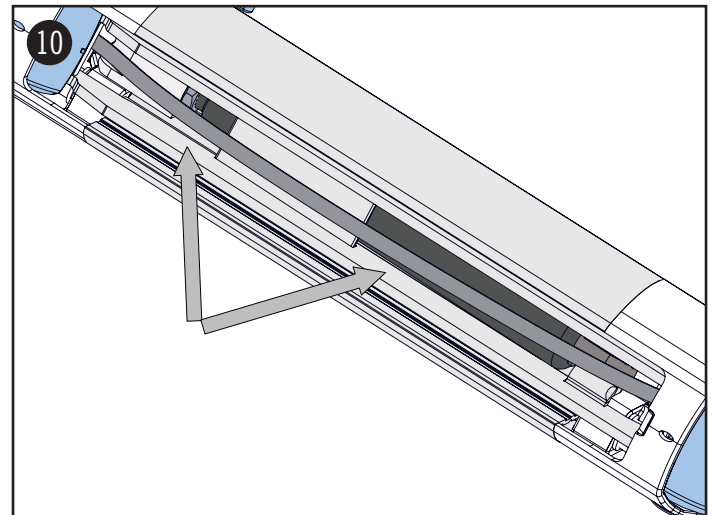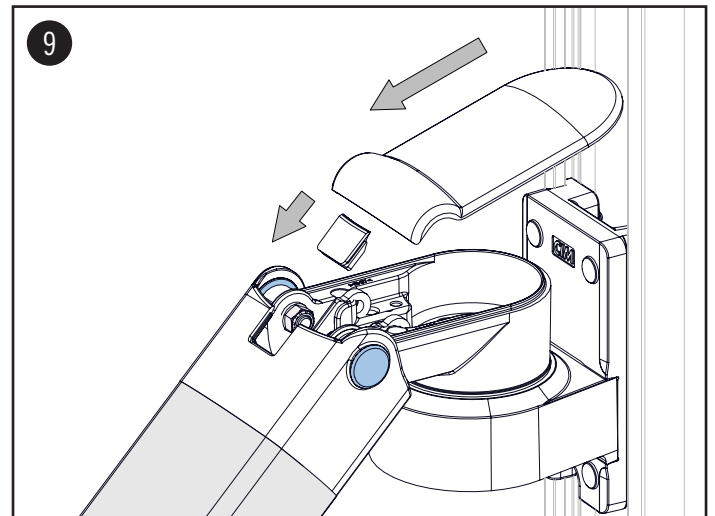⚠ WARNING: Make sure that the cables are placed along the intended cable channel (illustration 10).

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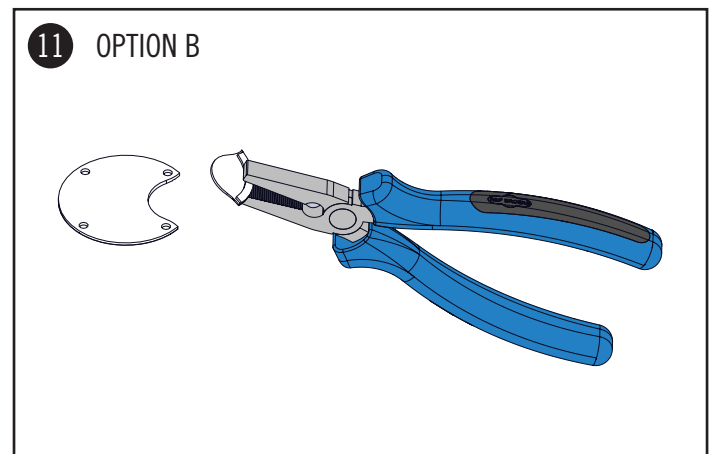
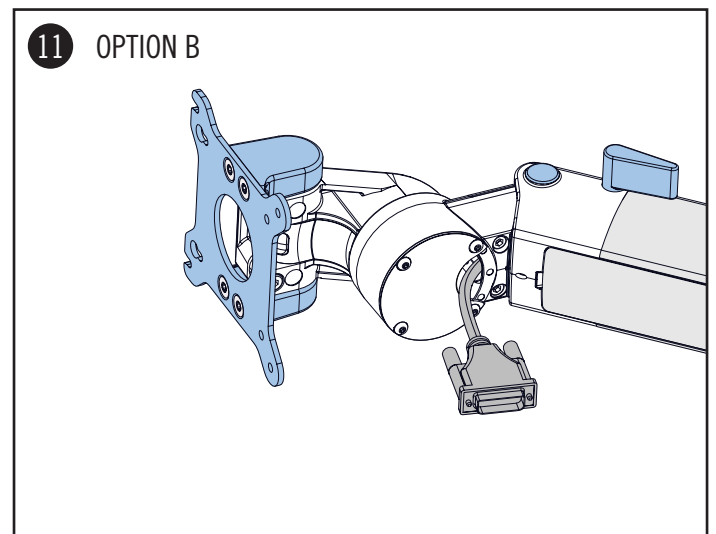
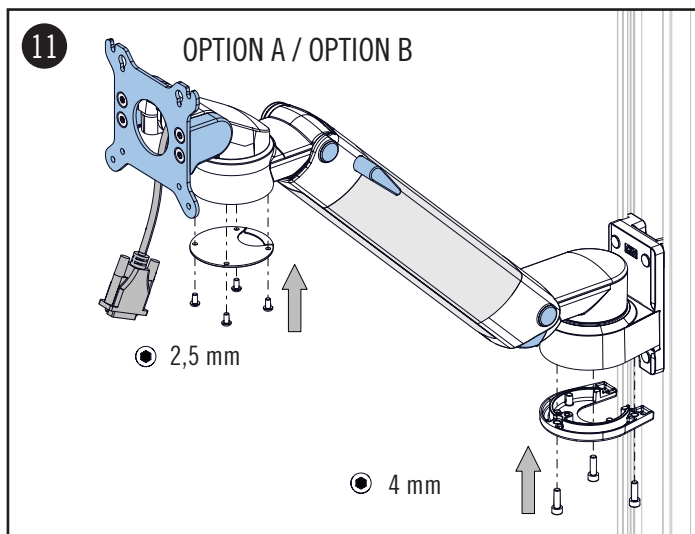
11. Mount the plastic cover cap to the front swivel part by using (4) M4x8 rounded head screws. Mount the aluminium cover cap to the rear swivel part by using (3) M5x16 pan head screws and screw all securely.

Only for option B: The plastic cover cap offer a snap-out section for cables which is only required for option B.

12. Mount the ME equipment according to the assembly instruction „monitor adaption“.

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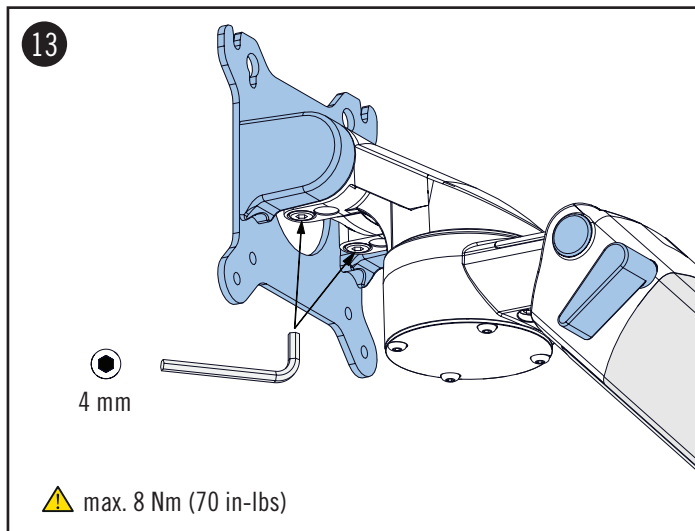
ASSEMBLY STEPS

TILT ADJUSTMENT

13. To adjust the overall tilt tension, tighten or loosen the both hex screws with the hex key 4 mm provided. By tightening both of the screws the tilt function can be locked completely. Maximum permissible tightening torque is 8 Nm (70 in-lbs).

⚠ WARNING: In case of overtightening the screw (> 10 Nm/ 88 in-lbs) the casting respectively the threads may break which leads to loss of function.

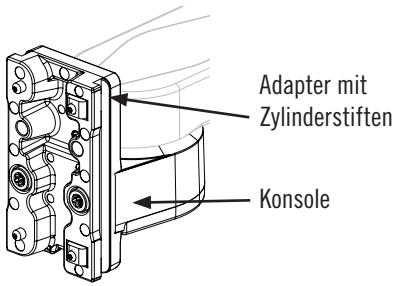
⚠ WARNING: When loosening both hex screws the ME equipment will tilt forward.

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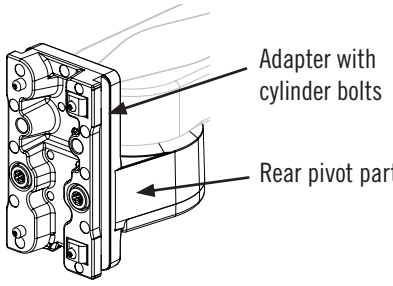
TROUBLE SHOOTING

Problem	Problem caused by	Remedy
ME equipment does not appear level or parallel to floor	Channel not even/ straight	Adjust wall channel or reinstall channel.
	Adapter and rear pivot part are not tightened completely. 	Please check that there is no crack or movement between these parts. Tighten the screws and make sure that adapter and console are pulled tight with a torque moment of 25 Nm.
	Adapter not parallel to wall channel: (4) cylinder bolts are pulled tight without having adjusted the adapter within the channel.	Remove cylinder bolts on both sides. Tighten first the cylinder pins on the left side. See assembly instruction adapter vertical/ horizontal wall channel.
Monitor adaption VESA / flat panel are not mounted parallel to floor.	Loosen the (4) M4 screws, adjust the ME equipment and pull the (4) screws tight again. Make sure that the ME equipment appears level or parallel to the floor.	

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CIM med GmbH recommends the following maintenance schedule for annual inspection.

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Die CIM med GmbH empfiehlt folgenden Wartungsplan zur jährlichen Kontrolle.