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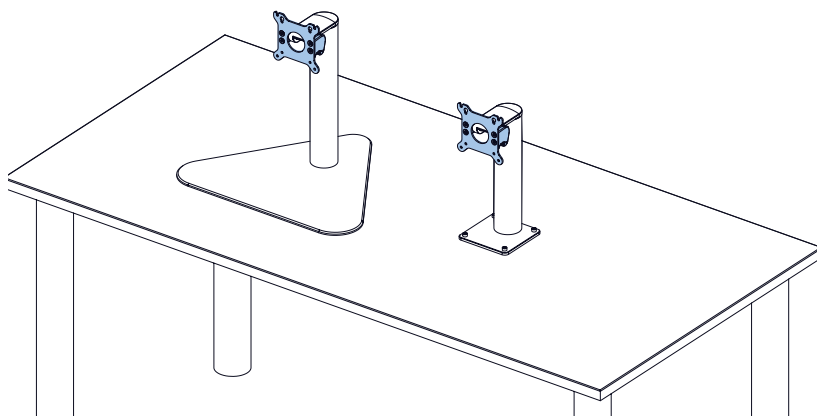
ASSEMBLY INSTRUCTION COUNTER TOP

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Name: CIM counter top
 Part no. code of the counter top: T-H
 Part no. code of the monitor adaptation: A, C, FK, G, K, MD, PH, Q, Q1, SLX

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PLEASE READ THIS INSTRUCTION CAREFULLY BEFORE INSTALLING!

Clarification of non-standardized Symbols:

-  Warning of a danger point
-  Important information
-  Danger of squeezing fingers

AREA OF APPLICATION/ INTENDED USE

CIM medical products are intended to mount and position medical electrical equipment (hereafter referred to as ME equipment) and accessories to those and are used in operating theatres as well as treatment rooms as auxiliary devices for doctors and medical staff.

GENERAL NOTES

It shall be the responsibility of the hospital, its consultants or contractors to determine that the table is suitable to safely and securely mount the counter top. This includes the selection of the suitable fasteners and the proper installation of the same. Please check the weight of the ME equipment to be mounted. The permissible total weight is mentioned on the corresponding data sheet.

WARNING Installation, Operation and Maintenance

This section contains warnings regarding the installation, operation and maintenance of the counter top. This section must be read in its entirety before installing, operating and maintaining the counter top. Failure to follow these warnings may result in damage to equipment or injury to people.

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GENERAL NOTES

Installation, operation and maintenance may only be carried out by qualified personnel. Assembly may only be done by qualified technical personnel. CIM med GmbH cannot be held responsible for any damage due to poor installation.

Please consider carefully the ME equipment being mounted and the proximity of the mounting assembly to other equipment, hospital personnel, and the patient. CIM med GmbH recommends that the hospital's risk management personnel verify that the application is appropriate prior to installation and use of the counter top.

Ensure that the weight of the instrument being mounted does not exceed the maximum load indication.

Product modifications, disassembling and reassembling and/or mounting any of the components in any other way than described below are permitted only after written approval by the CIM med GmbH construction and engineering department. CIM med GmbH cannot be held responsible for any damage or injury caused by product modifications and/or amendments without prior permission by CIM med GmbH.

WARNING: MORTAL DANGER BY ELECTRIC SHOCK

The electric cables of ME equipment and ME systems may only be connected to the power source and to the medical device respectively after finishing installation. Prior to maintenance or disassembly all power cables must be disconnected from the power source. Only cables with double insulation may be used for running through the counter top.

TOOLS REQUIRED

hex key 4.0 for tilt adjustment

CLEANING

CIM counter tops may be cleaned with mild, non-abrasive solutions commonly used in hospital environment. Do not use steel wool or other abrasive materials. Avoid all liquids entering the column. Your counter tops has been designed to the highest specifications to make cleaning as easy as possible and to avoid liquids from penetrating. CIM med GmbH makes no claims concerning the efficacy of the disinfection processes. It is recommended to test any cleaning solution on a small area of the column not visible to verify compatibility. Please consult your hospital's infection control manager.

Exemplary disinfection tests have been carried out with the following active substance groups: alcoholic quick disinfectants (e.g. Bacillol AF/Hartmann), aldehyde and quaternary ammonium compounds (e.g. B5/Orochemie), alcohol-free quaternary ammonium compounds (e.g. Sani-Cloth AF3/PDI Healthcare). These compounds are VAH / DGHM listed and EPA registered and must be used as per the manufacturers' instructions. The surface finish will be permanently damaged by acetone and trichloroethylene.

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GENERAL NOTES

MAINTENANCE: MANNER AND FREQUENCY

Very little maintenance is required during the life span of the counter top; however, it is recommended to check all pivot/ swivel points as well as all bearing points regularly. Please check all screws on a regular basis (at least once a year) and tighten them if necessary.

In case of damage done to the load-bearing parts of the system it has to be strictly refrained from further usage of the mounting system.

Inspection must include the following steps:

Grasp the counter top and pivot it from side to side at the attachment. The counter top should pivot with some tension or resistance.

Grasp the ME equipment and tilt it forward and back, through its entire range of motion. There should be enough tension or resistance in the tilt mechanism to prevent the ME equipment from tilting forward unexpectedly. If necessary, the tilt tension may be adjusted in accordance with section „Tilt adjustment“.

Failure to periodically inspect and adjust the counter top as instructed may result in damage to equipment or injury.

REPORTING OBLIGATION

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

SAFEKEEPING

For any questions that may arise at a later time please safekeep our manual!

ASSISTANCE REQUIRED

If you are unable to adjust the counter top or require service, contact CIM med technical support: Phone +49 89 978 94 080 or your local dealer.

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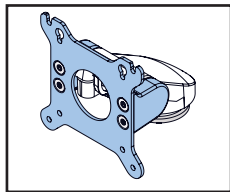
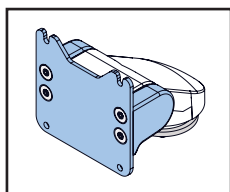
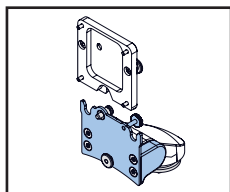
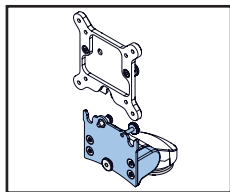
MONITOR ADAPTATIONS

REAR-MOUNTED DEVICES

For rear-mounted devices, see separate assembly instructions:

VESA 75/ 100

- ➔ Assembly Instruction Monitor Adaptation VESA 75, VESA 75/ 100

VESA 75

- ➔ Assembly Instruction Monitor Adaptation VESA 75, VESA 75/ 100

VESA 75 QUICK RELEASE

- ➔ Assembly Instruction Monitor Adaptation Quick Release VESA 75, VESA 75/ 100

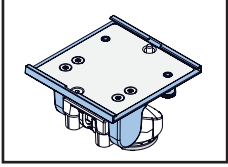
VESA 75/100 QUICK RELEASE

- ➔ Assembly Instruction Monitor Adaptation Quick Release VESA 75, VESA 75/ 100

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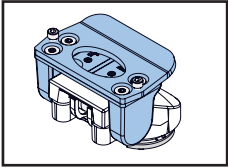
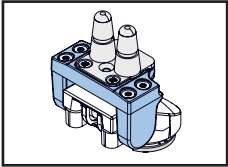
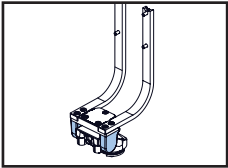
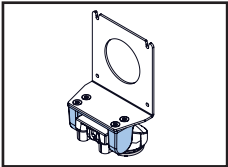
MONITOR ADAPTATIONS

BOTTOM-MOUNTED DEVICES

For bottom-mounted devices, see separate assembly instructions:

5" PLUNGE PLATE

- ➔ Assembly Instruction Monitor Adaptation 5" plunge plate

MINDRAY QUICK RELEASE

- ➔ Assembly Instruction Monitor Adaptation Mindray Quick Release

PHILIPS TABLE TOP MOUNT

- ➔ Assembly Instruction Monitor Adaptation Philips Table Top Mount

SPACELABS XPREZZON BACKPACK

- ➔ Assembly Instruction Monitor Adaptation Spacelabs XPREZZON Backpack

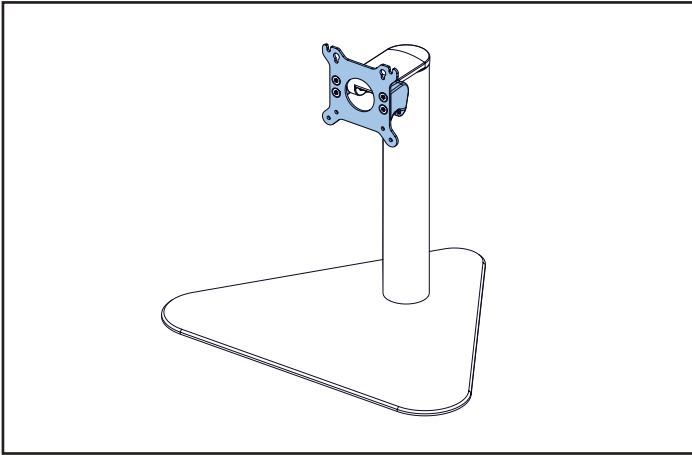
FUKUDA MOUNTING ADAPTER

- ➔ Assembly Instruction Monitor Adaptation Fukuda

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
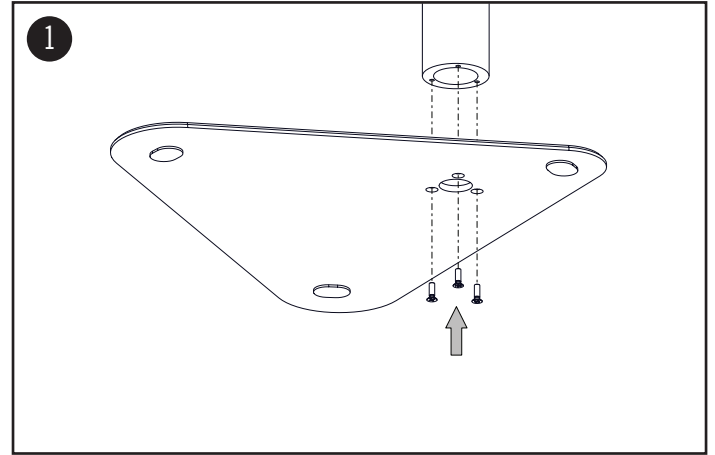
COUNTER TOP FREE-STANDING / WITHOUT BASE PLATE

1. Mount the counter top with (3) M5x16 counter sunk screws to the base plate or on the existing table area.

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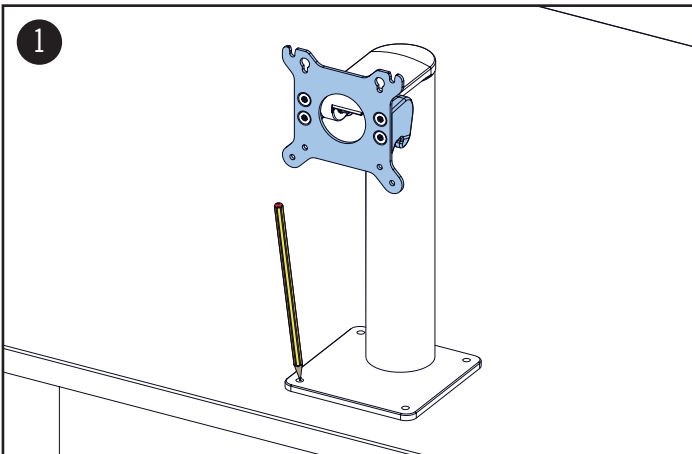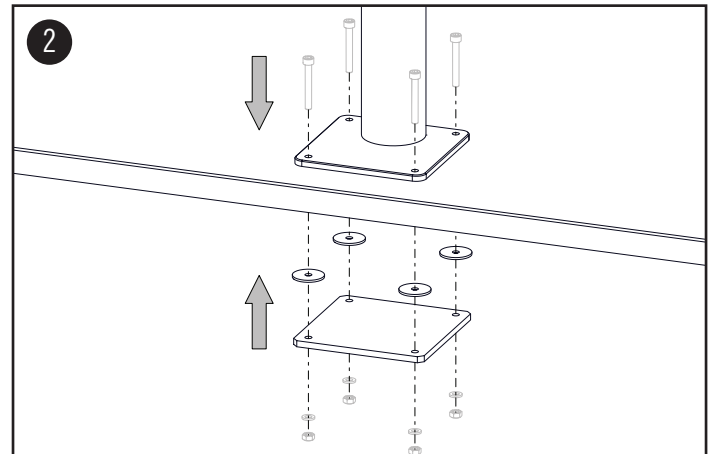
COUNTER TOP WITH BASE PLATE TO MOUNT

1. Position the counter top. Mark the requested bore position.
2. Mount the counter top using suitable mounting hardware and attached washers. Screw all securely.

ⓘ NOTE: Depending on the type of table, it is recommended to use the optional counter plate.

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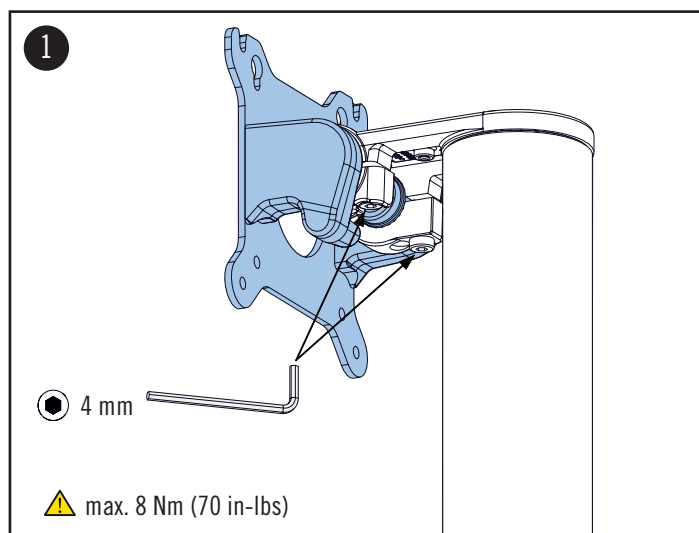
ASSEMBLY STEPS

TILT ADJUSTMENT

1. To adjust the overall tilt tension, tighten or loosen the both hex screws with the hex key 4 mm provided. By tightening both of the screws the tilt function can be locked completely. Maximum permissible tightening torque is 8 Nm (70 in-lbs).

⚠ WARNING: In case of overtightening the screw (> 10 Nm/ 88 in-lbs) the casting respectively the threads may break which leads to loss of function.

⚠ WARNING: When loosening both hex screws the ME equipment will tilt forward.

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CIM med GmbH recommends the following maintenance schedule for annual inspection.

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Die CIM med GmbH empfiehlt folgenden Wartungsplan zur jährlichen Kontrolle.