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VERTIKALE NORMSCHIENENKLEMME 10 x 25 MM

ASSEMBLY INSTRUCTION

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PLEASE READ THIS INSTRUCTION CAREFULLY BEFORE INSTALLING!

Clarification of non-standardized Symbols:

 Warning of a danger point

AREA OF APPLICATION/ INTENDED USE

CIM medical products are intended to mount and position medical electrical equipment (hereafter referred to as ME equipment) and accessories to those and are used in operating theatres as well as treatment rooms as auxiliary devices for doctors and medical staff.

The vertical rail clamps are exclusively to be used in conjunction with CIM support arms in the medical sector.

GENERAL NOTES

Before you want to attach the vertical rail clamp to an existing European rail, please verify that the rail can hold the weight. Please check the weight of the ME equipment to be mounted. The permissible total weight is mentioned on the corresponding data sheet.

WARNING Installation, Operation and Maintenance 

This section contains warnings regarding the installation, operation and maintenance of the vertical rail clamp. This section must be read in its entirety before installing, operating and maintaining the vertical rail clamp. Failure to follow these warnings may result in damage to equipment or injury to people.

Installation, operation and maintenance may only be carried out by qualified personnel. Assembly may only be done by qualified technical personnel. CIM med GmbH cannot be held responsible for any damage due to poor installation.

Product modifications, disassembling and reassembling and/or mounting any of the components in any other way than described below are permitted only after written approval by the CIM med GmbH construction and engineering department. CIM med GmbH cannot be held responsible for any damage or injury caused by product modifications and/or amendments without prior permission by CIM med GmbH.

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GENERAL NOTES

⚠ WARNING: EARTHING CONNECTION

The vertical rail clamp is delivered with components' grounding. It is the healthcare institution's responsibility to connect the rail to the electrical grounding of the building. The arm is connected to the channel via the grounding screw of the adapter.

TOOLS REQUIRED

Torx wrench T45
Hex key 3.0

CLEANING

The vertical rail clamp may be cleaned with mild, non-abrasive solutions commonly used in hospital environment. Do not use steel wool or other abrasive materials. The product has been designed to the highest specifications to make cleaning as easy as possible. CIM med makes no claims concerning the efficacy of the disinfection processes.

It is recommended to test any cleaning solution on a small area not visible to verify compatibility. Please consult your hospital's infection control manager.

MAINTENANCE: MANNER AND FREQUENCY

Very little maintenance is required during the life span. Please check all screws on a regular basis (at least once a year) and tighten them if necessary.

In case of damage done to the load-bearing parts of the system it has to be strictly refrained from further usage of the mounting system.

REPORTING OBLIGATION

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

SAFEKEEPING

For any questions that may arise at a later time please safekeep our manual!

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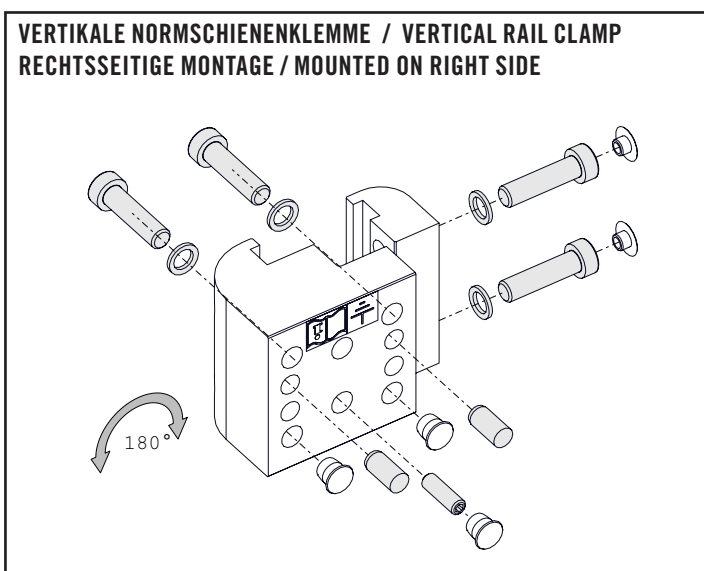
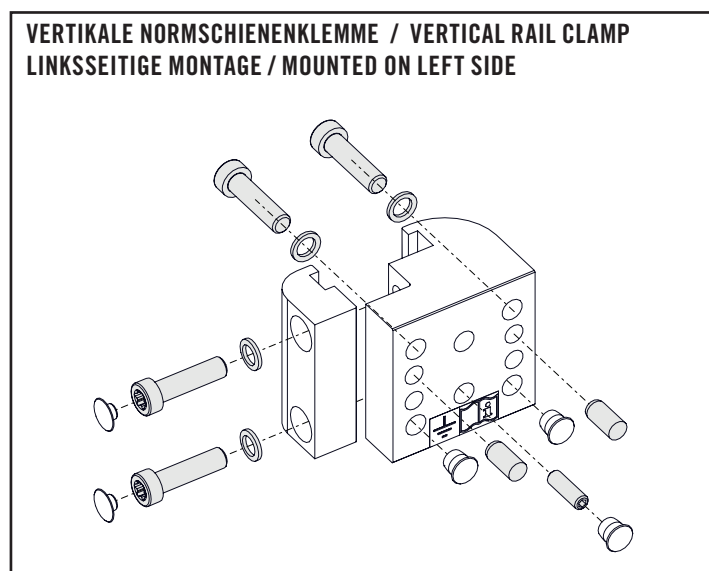
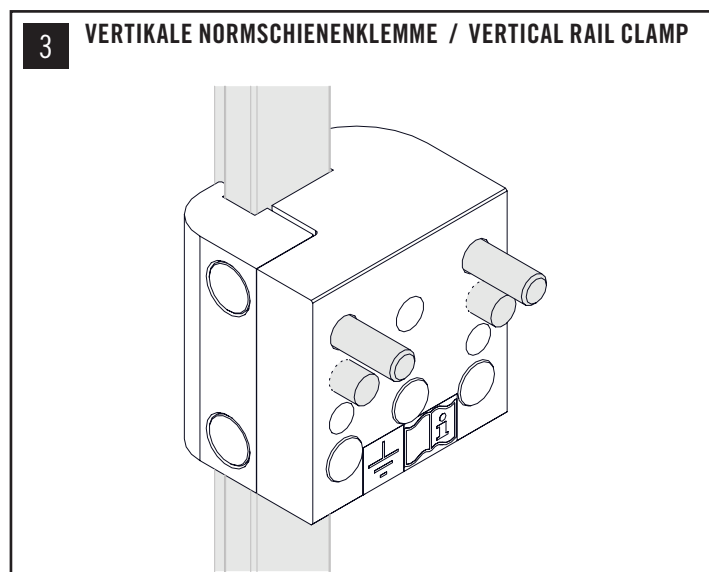
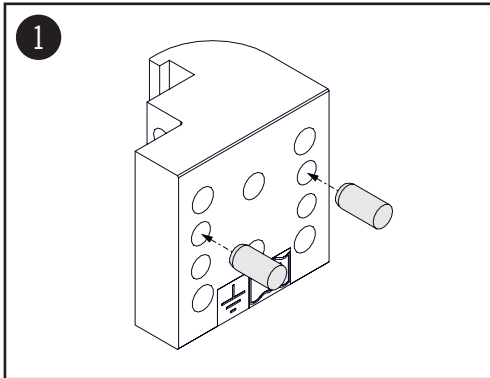
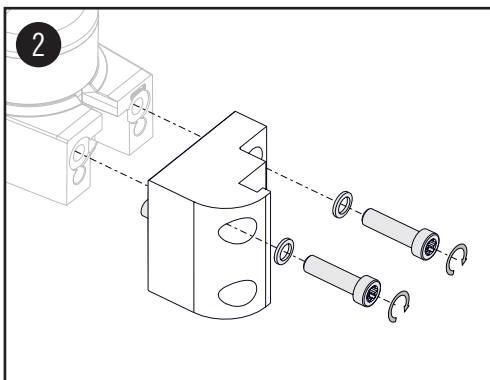
Place locating pins in the bore holes below the holes for the screws. It is the second row from top.

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
Mount the adapter to arm using the 2 x M8 screws and detent edged washers.
Torque moment: 25 Nm (221 in-lbs)

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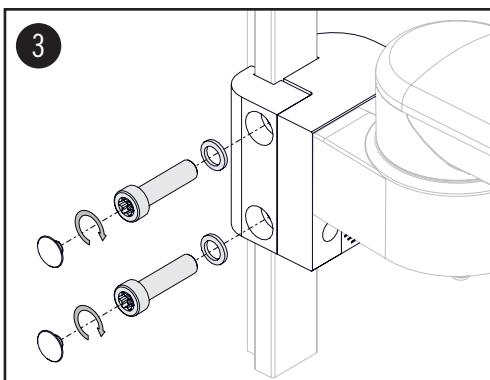
! To guarantee a secure components grounding it is essential to use the both detent edged washers.

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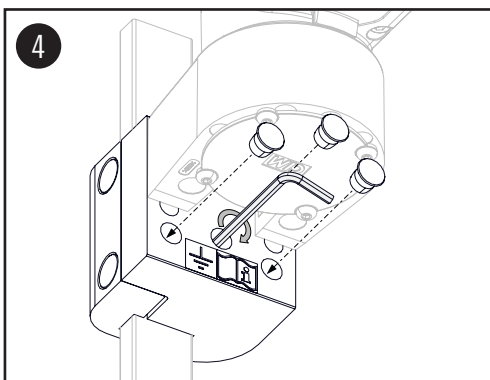Place the rail clamp to the vertical rail and fasten it with two TX45 screws.
Apply the cover caps on top of the screws.

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! **CAUTION**
The grounding screw must be tightened for a safe component grounding.