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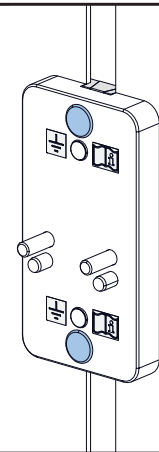
PLEASE READ THIS INSTRUCTION CAREFULLY BEFORE INSTALLING!

Clarification of non-standardized Symbols:

Warning of a danger point

AREA OF APPLICATION/ INTENDED USE

CIM medical products are intended to mount and position medical electrical equipment (hereafter referred to as ME equipment) and accessories to those and are used in operating theatres as well as treatment rooms as auxiliary devices for doctors and medical staff.

The adapter for Dameca anaesthesia machine is exclusively to be used in conjunction with CIM support arms in the medical sector.

GENERAL NOTES

Before you want to mount the adapter to an existing headwall, please verify that the rail can hold the weight. Please check the weight of the ME equipment to be mounted. The permissible total weight is mentioned on the corresponding data sheet.

WARNING Installation, Operation and Maintenance 

This section contains warnings regarding the installation, operation and maintenance of the adapter for Dameca anaesthesia machine. This section must be read in its entirety before installing, operating and maintaining the adapter. Failure to follow these warnings may result in damage to equipment or injury to people.

Installation, operation and maintenance may only be carried out by qualified personnel. Assembly may only be done by qualified technical personnel. CIM med GmbH cannot be held responsible for any damage due to poor installation.

Product modifications, disassembling and reassembling and/or mounting any of the components in any other way than described below are permitted only after written approval by the CIM med GmbH construction and engineering department. CIM med GmbH cannot be held responsible for any damage or injury caused by product modifications and/ or amendments without prior permission by CIM med GmbH.

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GENERAL NOTES

⚠️ WARNING: EARTHING CONNECTION

The adapter for Dameca anaesthesia machines is delivered with components' grounding. It is the healthcare institution's responsibility to control the grounding of the headwall. The adapter is connected to the channel via the grounding screw of the adapter.

TOOLS REQUIRED

Hex key 3.0
Hex key 5.0
Torx wrench T45

CLEANING

The adapter may be cleaned with mild, non-abrasive solutions commonly used in hospital environment. Do not use steel wool or other abrasive materials. The product has been designed to the highest specifications to make cleaning as easy as possible. CIM med makes no claims concerning the efficacy of the disinfection processes.

It is recommended to test any cleaning solution on a small area not visible to verify compatibility. Please consult your hospital's infection control manager.

MAINTENANCE: MANNER AND FREQUENCY

Very little maintenance is required during the life span. Please check all screws on a regular basis (at least once a year) and tighten them if necessary.

In case of damage done to the load-bearing parts of the system it has to be strictly refrained from further usage of the mounting system.

REPORTING OBLIGATION

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

SAFEKEEPING

For any questions that may arise at a later time please safekeep our manual!

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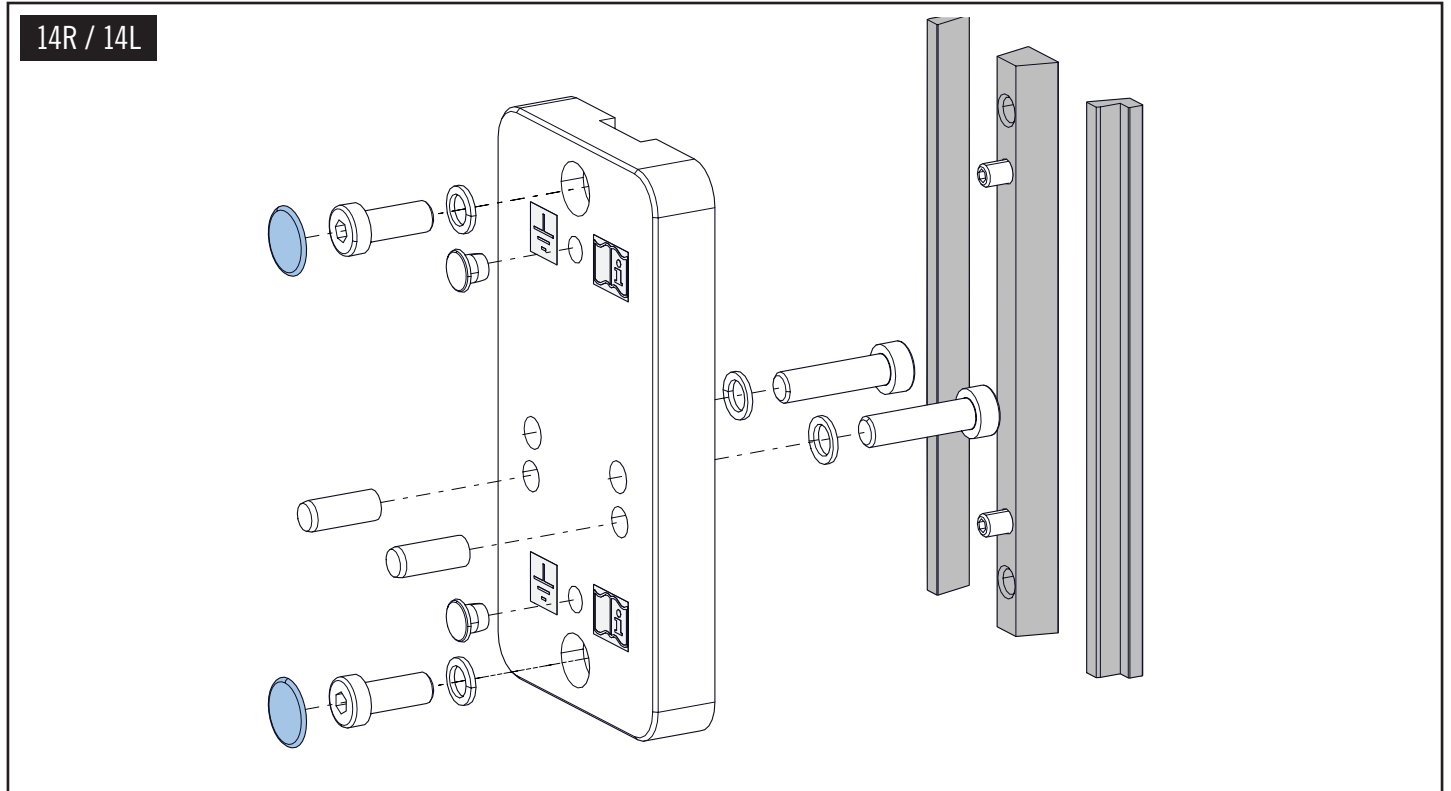
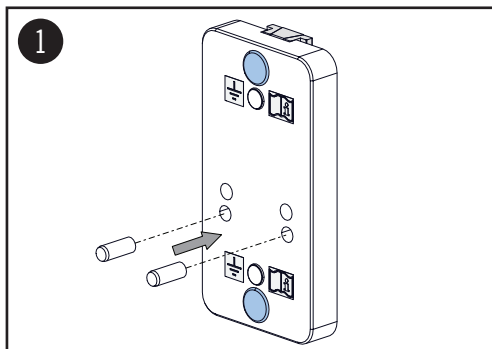
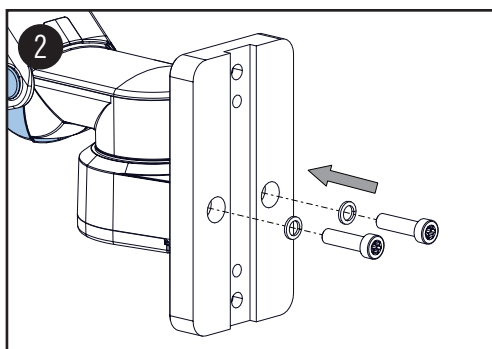
Insert (2) cylinder pins into the lower holes.

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Mount support arm with (2) M8x30 cylinder head screws (Torx) and (2) detent edged washers.

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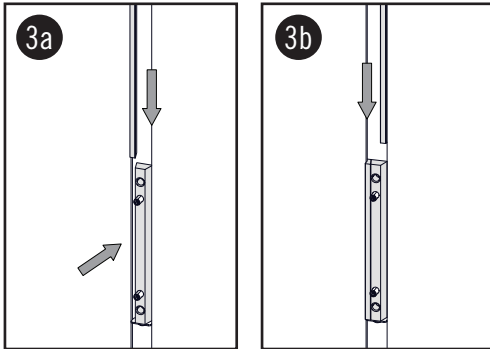
To guarantee a secure components grounding it is essential to use the both detent edged washers.

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
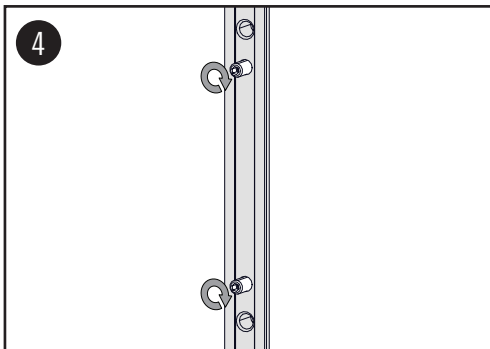
Place T-nuts in the channel of medical device as shown.

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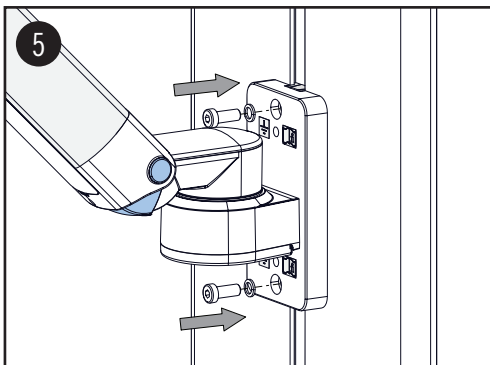Lightly tighten (2) grounding screws for pre-assembly.

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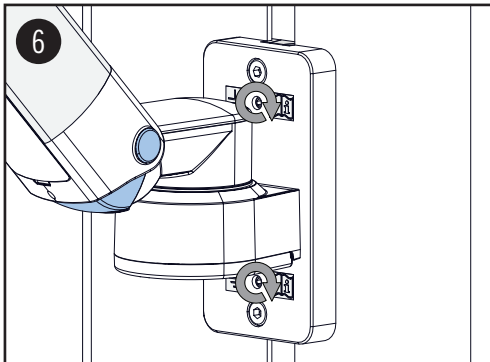
Place adapter on the T-nuts and mount it with (2) M8x20 cylinder head screws and (2) detent edged washers.

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⚠ WARNING:

The pre-assembled support arm may only be mounted without payload. To guarantee a secure components grounding it is essential to use the both detent edged washers.

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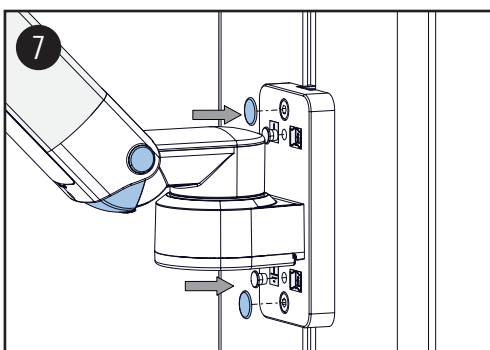
Tighten the (2) M6 x 20 grounding screws for final assembly.

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⚠ WARNING:

The grounding screw must be tightened for a safe component grounding.

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