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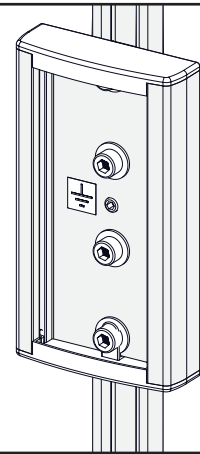
**PLEASE READ THIS INSTRUCTION CAREFULLY BEFORE INSTALLING!**

Clarification of non-standardized Symbols:

Warning of a danger point

**AREA OF APPLICATION/ INTENDED USE**

CIM medical products are intended to mount and position medical electrical equipment (hereafter referred to as ME equipment) and accessories to those and are used in operating theatres as well as treatment rooms as auxiliary devices for doctors and medical staff.

The wall channel adapter for Amico headwall is exclusively to be used in conjunction with CIM support arms in the medical sector.

**GENERAL NOTES**

Before you want to mount the wall channel adapter to an existing Amico headwall, please verify that the rail can hold the weight. Please check the weight of the ME equipment to be mounted. The permissible total weight is mentioned on the corresponding data sheet.

**WARNING Installation, Operation and Maintenance** 

This section contains warnings regarding the installation, operation and maintenance of the wall channel adapter. This section must be read in its entirety before installing, operating and maintaining the wall channel adapter. Failure to follow these warnings may result in damage to equipment or injury to people.

Installation, operation and maintenance may only be carried out by qualified personnel. Assembly may only be done by qualified technical personnel. CIM med GmbH cannot be held responsible for any damage due to poor installation.

Product modifications, disassembling and reassembling and/or mounting any of the components in any other way than described below are permitted only after written approval by the CIM med GmbH construction and engineering department. CIM med GmbH cannot be held responsible for any damage or injury caused by product modifications and/ or amendments without prior permission by CIM med GmbH.

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## GENERAL NOTES

### ⚠️ WARNING: EARTHING CONNECTION

The wall channel adapter for Amico headwall is delivered with components' grounding. It is the healthcare institution's responsibility to control the grounding of the Amico headwall. The wall channel adapter is connected to the channel via the grounding screw of the adapter.

### TOOLS REQUIRED

Hex key 3.0  
Hex key 4.0

### CLEANING

The wall channel adapter may be cleaned with mild, non-abrasive solutions commonly used in hospital environment. Do not use steel wool or other abrasive materials. The product has been designed to the highest specifications to make cleaning as easy as possible. CIM med makes no claims concerning the efficacy of the disinfection processes.

It is recommended to test any cleaning solution on a small area not visible to verify compatibility. Please consult your hospital's infection control manager.

### MAINTENANCE: MANNER AND FREQUENCY

Very little maintenance is required during the life span. Please check all screws on a regular basis (at least once a year) and tighten them if necessary.

In case of damage done to the load-bearing parts of the system it has to be strictly refrained from further usage of the mounting system.

### REPORTING OBLIGATION

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

### SAFEKEEPING

For any questions that may arise at a later time please safekeep our manual!

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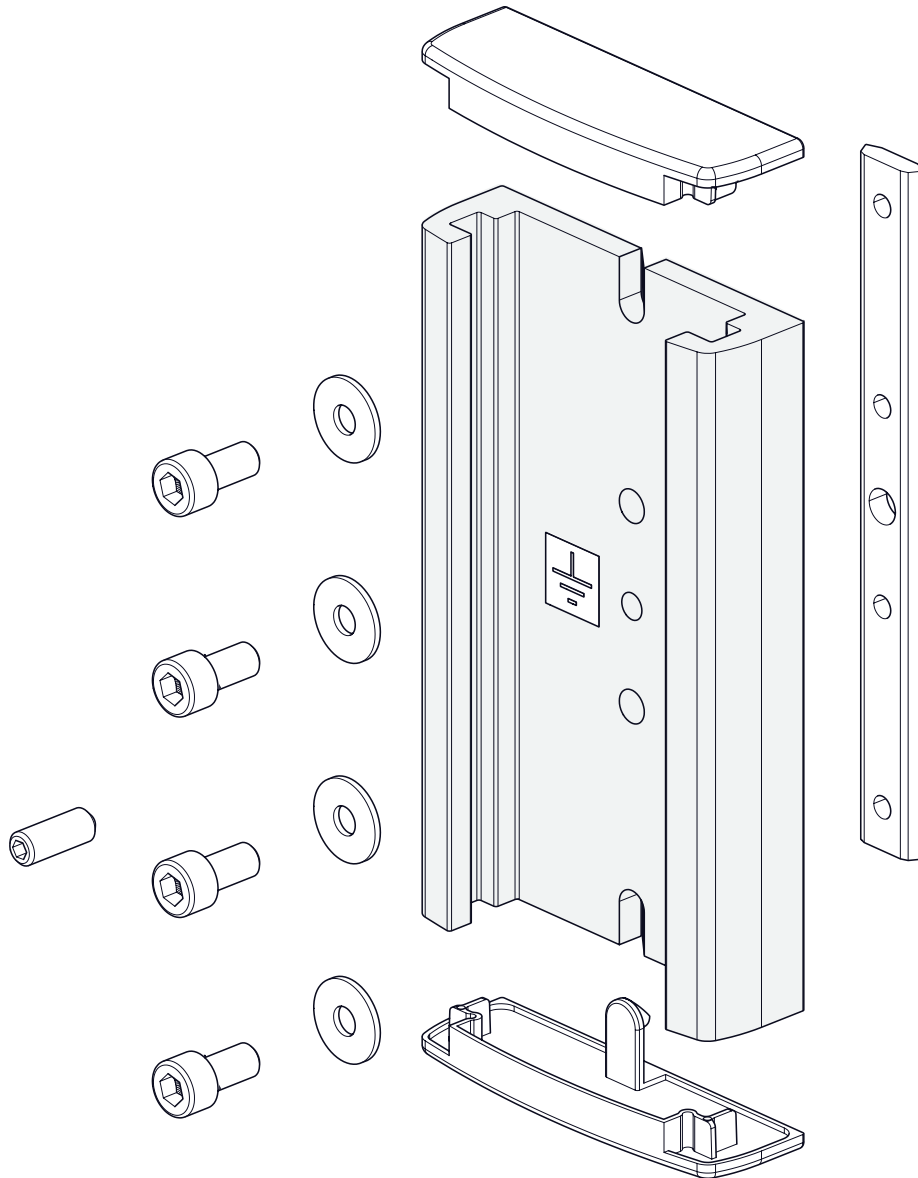
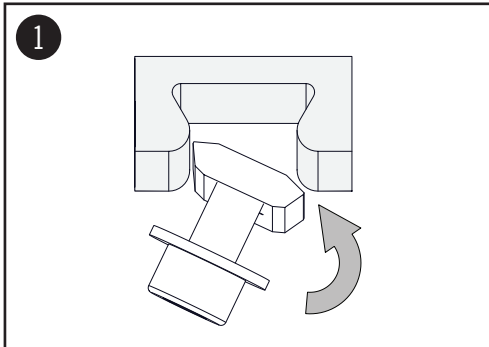
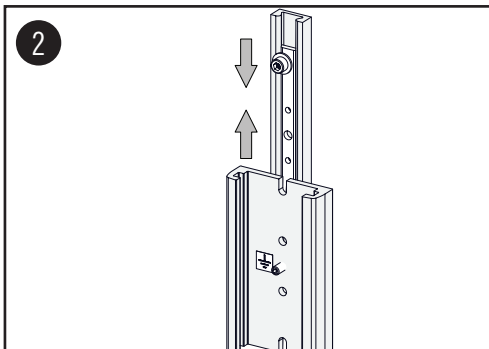
Put the trapezoidal strip with the pre-mounted screw in the Amico rail.


**⚠ CAUTION**

Pay attention to the positioning of the trapezoidal strip.

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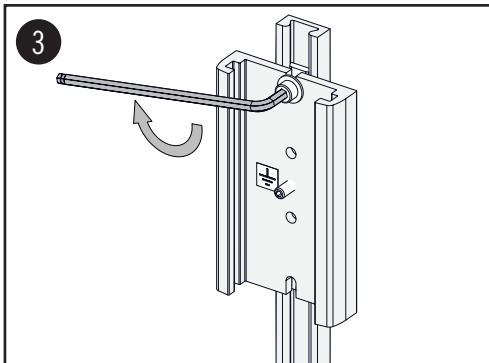
Position the wall channel on the rail and then connect it to the strip. Pay attention to the correct positioning of the washers.

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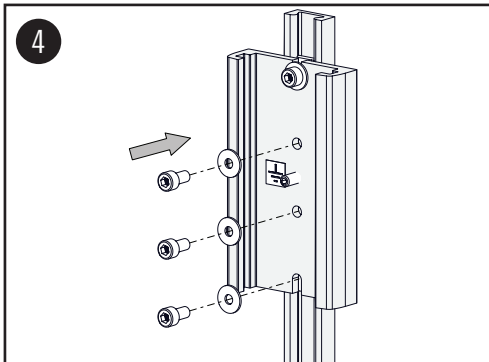
Tighten the pre-mounted M5x12 screw slightly.

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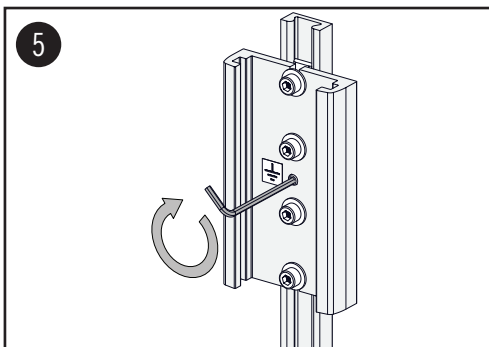Insert the other (3) M5x12 cylinder head bolts with the washers. Position the wall channel in the desired height and then tighten all (4) M5x12 screws.

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**⚠ CAUTION**

The grounding screw must be tightened for a safe component grounding.