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Innensechsrund Torx T30

**PLEASE READ THIS INSTRUCTION CAREFULLY BEFORE INSTALLING!**

Clarification of non-standardized Symbols:

Warning of a danger point

**AREA OF APPLICATION/ INTENDED USE**

CIM medical products are intended to mount and position medical electrical equipment (hereafter referred to as ME equipment) and accessories to those and are used in operating theatres as well as treatment rooms as auxiliary devices for doctors and medical staff.

**WARNING Installation, Operation and Maintenance** 

This section contains warnings regarding the installation, operation, and maintenance of handles. This section must be read in its entirety before installing, operating and maintaining. Failure to follow these warnings may result in damage to equipment or injury to people.

Installation, operation and maintenance may only be carried out by qualified personnel. Assembly may only be done by qualified technical personnel. CIM med GmbH cannot be held responsible for any damage due to poor installation.

Ensure that the weight of the instrument being mounted does not exceed the maximum load indication.

Product modifications, disassembling and reassembling and/or mounting any of the components in any other way than described below are permitted only after written approval by the CIM med GmbH construction and engineering department. CIM med GmbH cannot be held responsible for any damage or injury caused by product modifications and/ or amendments without prior permission by CIM med GmbH.

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## GENERAL NOTES

### CLEANING

The handle may be cleaned with mild, non-abrasive solutions commonly used in hospital environment. Do not use steel wool or other abrasive materials. The product has been designed to the highest specifications to make cleaning as easy as possible. CIM med makes no claims concerning the efficacy of the disinfection processes.

It is recommended to test any cleaning solution on a small area of the accessories not visible to verify compatibility. Please consult your hospital's infection control manager.

### MAINTENANCE: MANNER AND FREQUENCY

Very little maintenance is required during the life span. Please check all screws on a regular basis (at least once a year) and tighten them if necessary.

In case of damage done to the load-bearing parts of the system it has to be strictly refrained from further usage of the mounting system.

### REPORTING OBLIGATION

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

### SAFEKEEPING

For any questions that may arise at a later time please safekeep our manual!

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
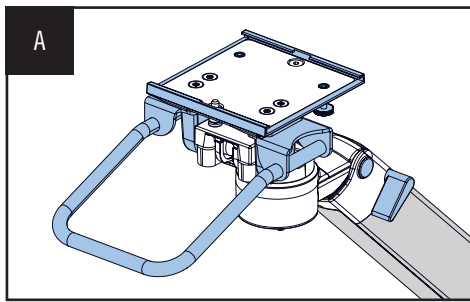
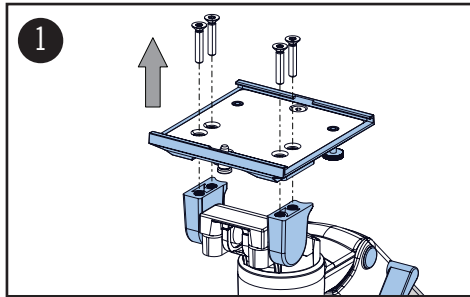
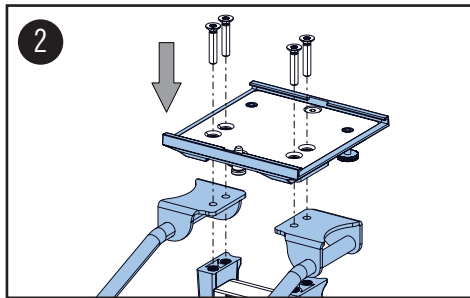
Unscrew the (4) 6x32 counter-sunk screws and remove the 5" plunge plate.

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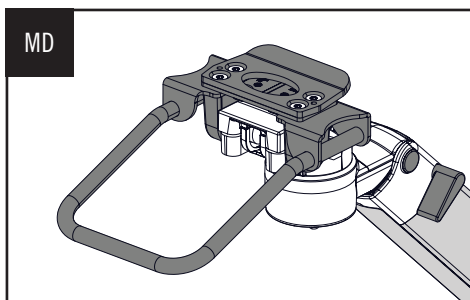
Mount handle and 5" plunge plate using the (4) 6x32 counter-sunk screws.

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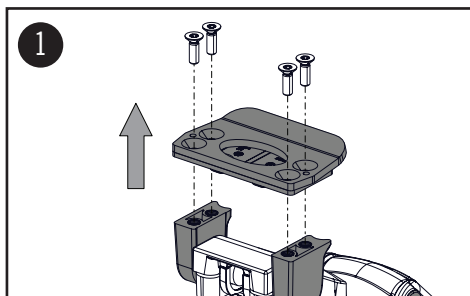
HANDLE FOR  
MINDRAY QUICK RELEASE MOUNTING ADAPTER

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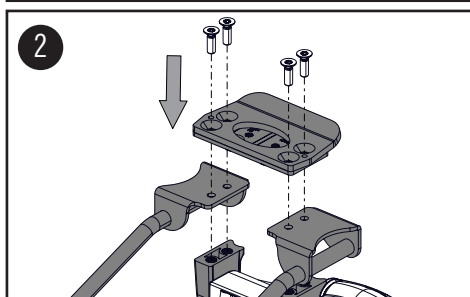Unscrew the (4) 6x19 counter-sunk screws and remove the Mindray quick release mounting adapter.

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
Mount handle and Mindray quick release mounting adapter using the (4) 6x19 counter-sunk screws.

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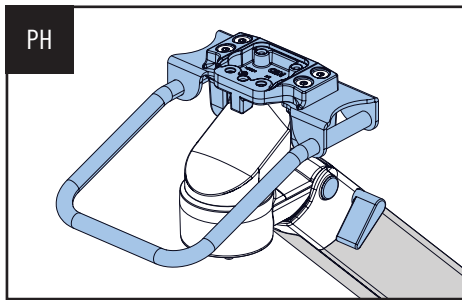
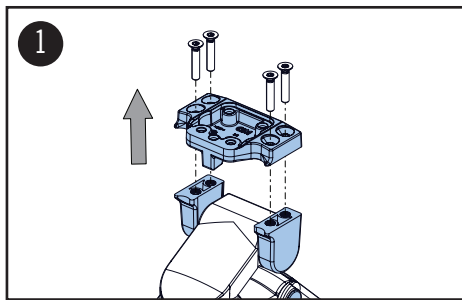
**HANDLE FOR  
PHILIPS MONITORING INTELLIVUE ADAPTER**

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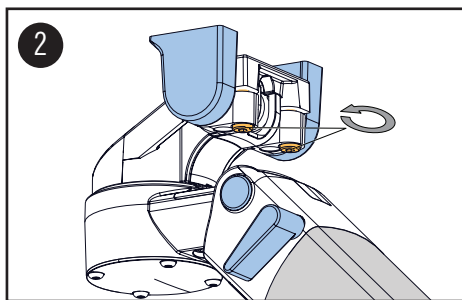
Unscrew the (4) 6x32 counter-sunk screws and remove the Philips Monitoring IntelliVue adapter.

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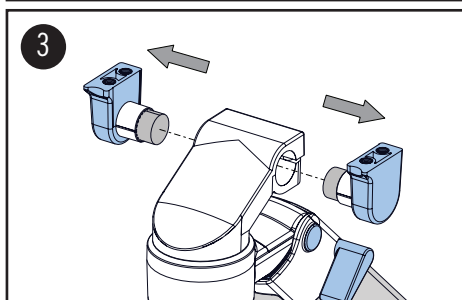
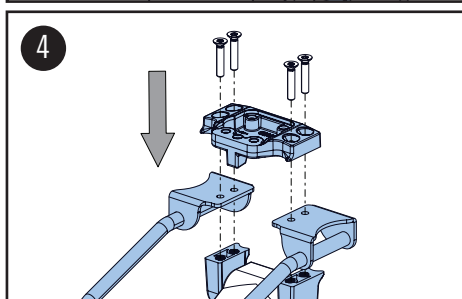
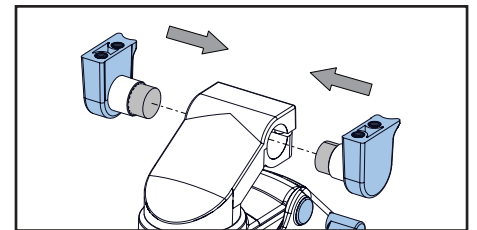
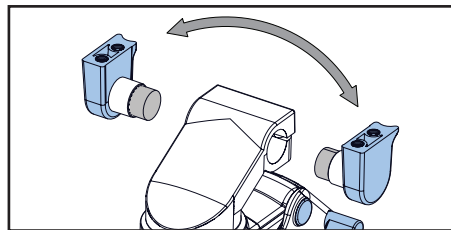
Change the right plate holder with the left one.

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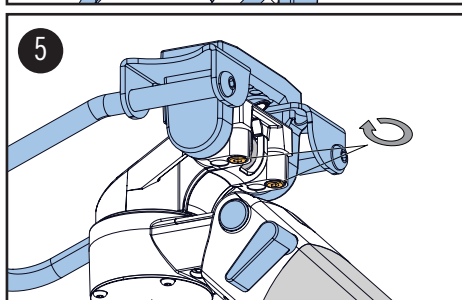Mount handle and Philips Monitoring IntelliVue adapter using the (4) 6x32 counter-sunk screws.

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Tighten the (2) M6x25 cylinder head screws until tilt function is adjusted as necessary.

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**⚠ CAUTION**

In case of overtightening the screw (> 10 Nm/ 88 in-lbs) the casting respectively the threads may break which leads to loss of function.