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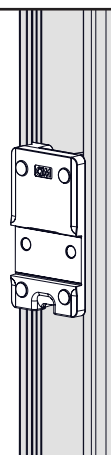
# ADAPTER VERTIKALER / HORIZONTALER WANDKANAL

ASSEMBLY INSTRUCTION

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**PLEASE READ THIS INSTRUCTION CAREFULLY BEFORE INSTALLING!**

Clarification of non-standardized Symbols:

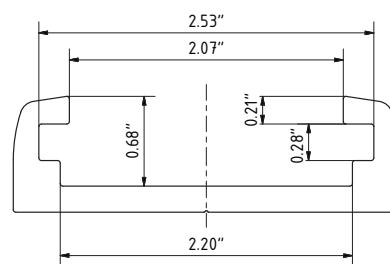
 Warning of a danger point

**AREA OF APPLICATION / INTENDED USE**

CIM medical products are intended to mount and position medical electrical equipment (hereafter referred to as ME equipment) and accessories to those and are used in operating theatres as well as treatment rooms as auxiliary devices for doctors and medical staff.

The adapters for vertical / horizontal wall channels are exclusively to be used in conjunction with CIM support arms in the medical sector. The max. weight load capacity is only in reference to tests being made to CIM arms being mounted to CIM wall channels. The adapters are also compatible to various wall channels used in the medical area (e.g. GCX)

Please note the compatibility (cross-section) and weight load of different wall channels provided by any other supplier.

**GENERAL NOTES**

Before you want to attach the adapter to an existing wall channel, please verify that the wall channel can hold the weight. Please check the weight of the ME equipment to be mounted. The permissible total weight is mentioned on the corresponding data sheet.

**WARNING Installation, Operation and Maintenance** 

This section contains warnings regarding the installation, operation, and maintenance of the adapter. This section must be read in its entirety before

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## GENERAL NOTES

installing, operating, and maintaining the adapter. Failure to follow these warnings may result in damage to equipment or injury to people.

Installation, operation and maintenance may only be carried out by qualified personnel. Assembly may only be done by qualified technical personnel. CIM med GmbH cannot be held responsible for any damage due to poor installation.

Product modifications, disassembling and reassembling and/or mounting any of the components in any other way than described below are permitted only after written approval by the CIM med GmbH construction and engineering department. CIM med GmbH cannot be held responsible for any damage or injury caused by product modifications and/ or amendments without prior permission by CIM med GmbH.

### **⚠️ WARNING: EARTHING CONNECTION**

The CIM support arms are delivered with components' grounding. The arm is connected to the wall channel via the screws of the adapter. It is the healthcare institution's responsibility to connect the wall channel to the electrical grounding of the building.

### **TOOLS REQUIRED**

Torx wrench T45  
Hex key 3.0

### **CLEANING**

The adapter may be cleaned with mild, non-abrasive solutions commonly used in hospital environment. Do not use steel wool or other abrasive materials. The product has been designed to the highest specifications to make cleaning as easy as possible. CIM med makes no claims concerning the efficacy of the disinfection processes.

It is recommended to test any cleaning solution on a small area of the adapter not visible to verify compatibility. Please consult your hospital's infection control manager.

### **MAINTENANCE: MANNER AND FREQUENCY**

Very little maintenance is required during the life span. Please check all screws on a regular basis (at least once a year) and tighten them if necessary.

In case of damage done to the load-bearing parts of the system it has to be strictly refrained from further usage of the mounting system.

### **REPORTING OBLIGATION**

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/ or patient is established.

### **SAFEKEEPING**

For any questions that may arise at a later time please safekeep our manual!

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
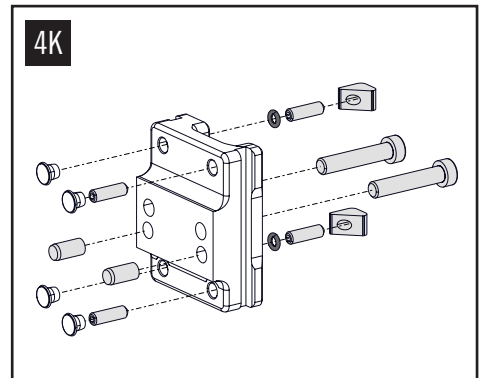
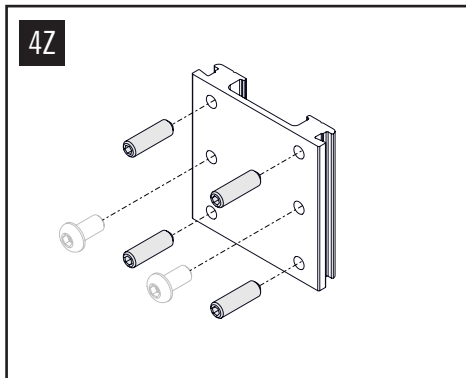
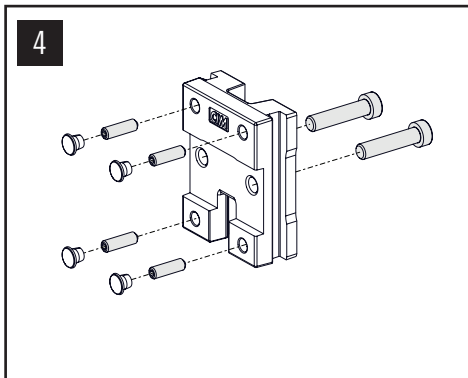
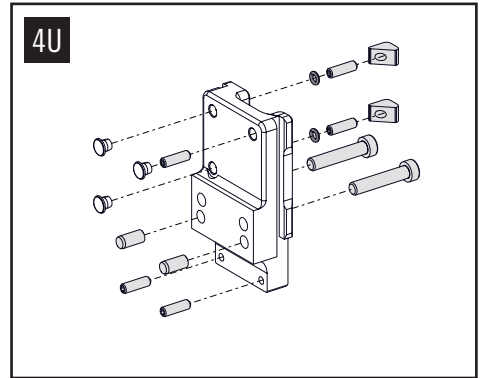
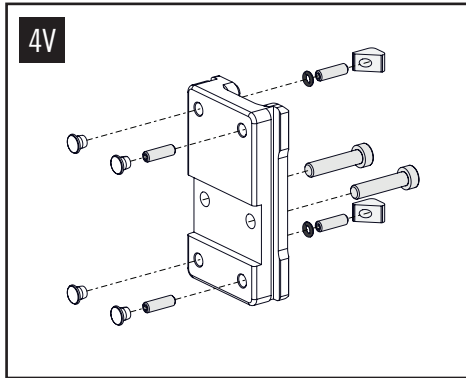
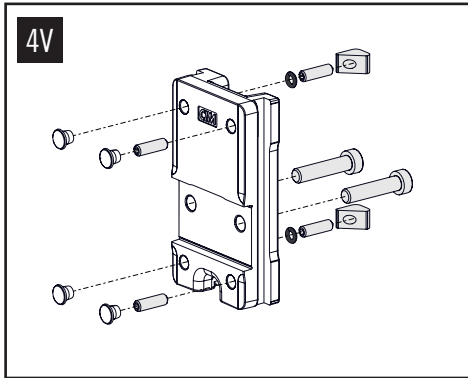
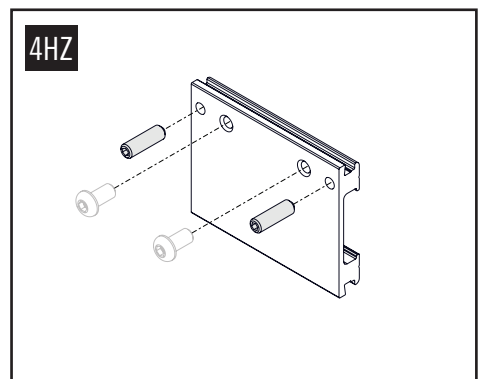
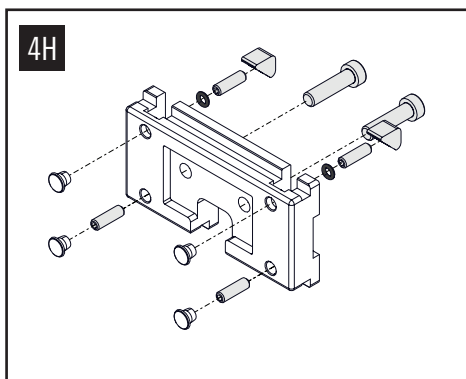
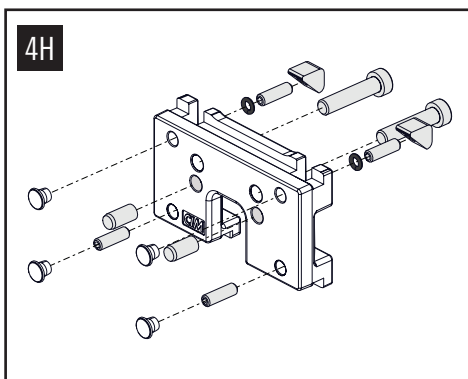
### ADAPTER FOR HORIZONTAL WALL CHANNEL

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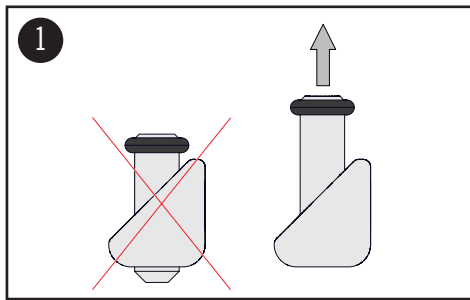
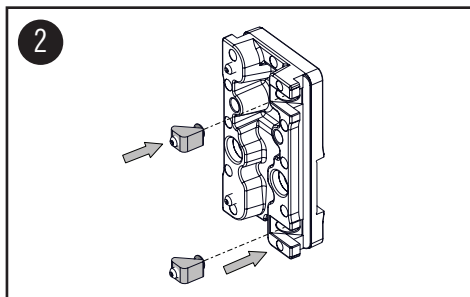
Clamping blocks with O-rings.  
Grub screw must be fully extended and the ring  
placed on the top few turns.

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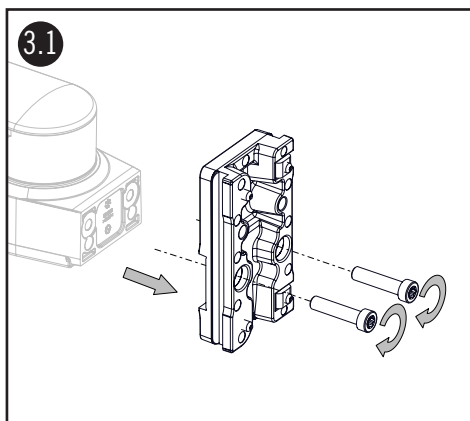
Press clamping block with grub screw and O-ring  
into the hole of the adapter.

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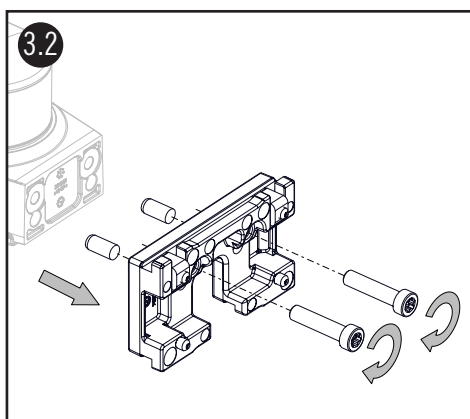**Adapter vertical/horizontal without pins**  
Mount the adapter to arm using the (2) M8 screws.  
Torque moment 23 Nm (204 in-lbs).

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


**Adapter vertical/horizontal with pins**  
Place locating pins into the bore holes. Tighten  
the (2) M8 screws.  
Torque moment 23 Nm (204 in-lbs).

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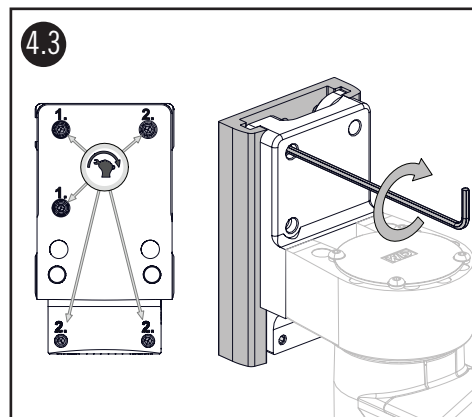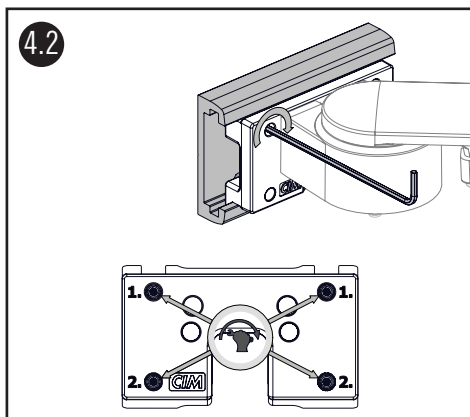
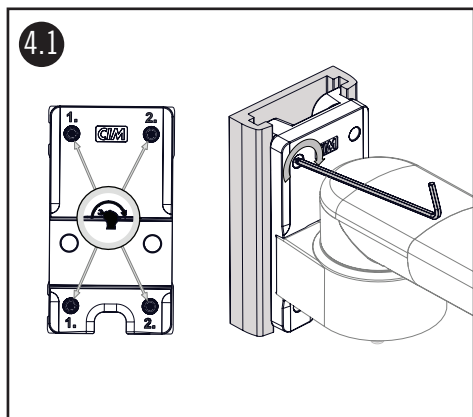
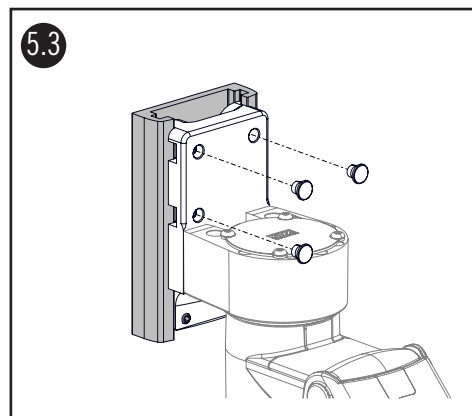
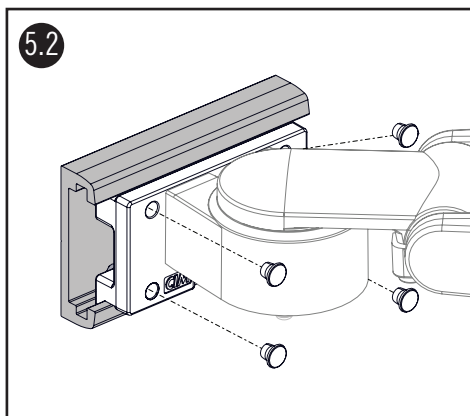
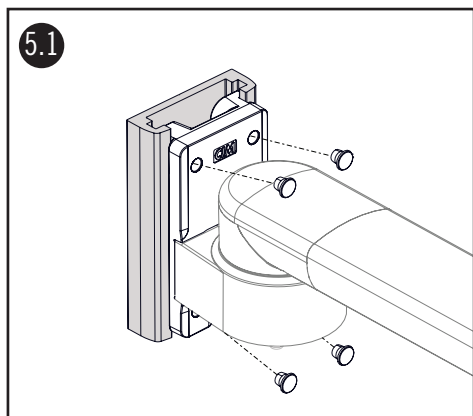
Slide vertical/ horizontal adapter into wall channel. Tighten the grub screws on the left side first with the adapter for vertical wall channel or the upper grub screws with the horizontal wall channel adapter. This creates a secure ground connection between the adapter and the wall channel.

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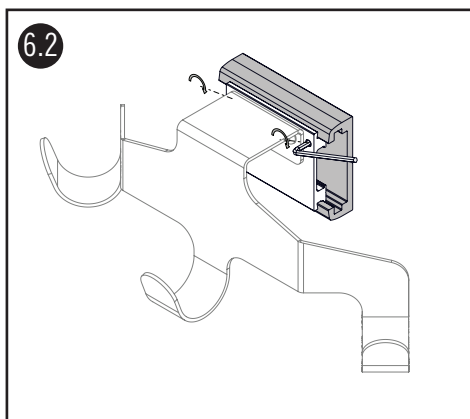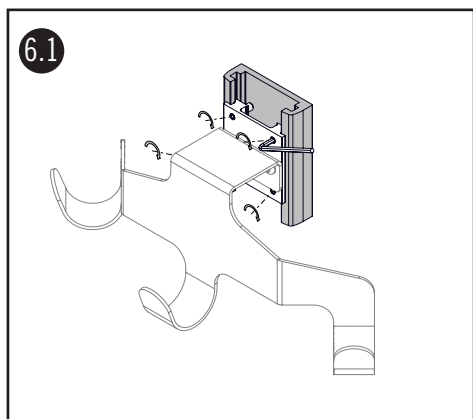The (4) cover caps press into the holes of the grub screws.

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
**Adapter vertical/horizontal for accessories**  
Mount the adapter to wall channel using the M6x20 screws.

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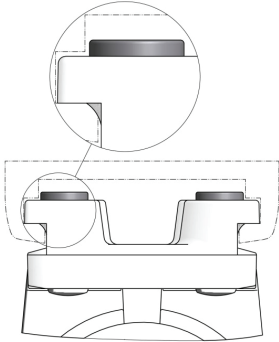Problem	Problem caused by	Trouble shooting
<p>Adapter cannot be slid into wall channel.</p>	<p>Adapter cannot be slid into wall channel. When installing to wall channel of other manufacturer (e.g. GCX) the M8 screw may collide with root face of the channel. The M8 screws have been tightened with a higher torque moment than 23 Nm (204 in-lbs). Torque moment higher than 25 Nm (220 in-lbs) increases the risk of bulging (side of the adapter). Especially with GCX wall channels the convexity prevents the adapter from being slid into the channel (see illustration).</p>	<p>Do not tighten M8 screws with more than 23 Nm (204 in-lbs).  <b>Remedy:</b> Change adapter</p> 