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**CIM WANDKANAL**

## ASSEMBLY INSTRUCTION

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**PLEASE READ THIS INSTRUCTION CAREFULLY BEFORE INSTALLING!**

Clarification of non-standardized Symbols:

Warning of a danger point

**AREA OF APPLICATION/ INTENDED USE**

CIM medical products are intended to mount and position medical electrical equipment (hereafter referred to as ME equipment) and accessories to those and are used in operating theatres as well as treatment rooms as auxiliary devices for doctors and medical staff.

**GENERAL NOTES**

It shall be the responsibility of the hospital, its consultants or contractors to determine that the wall channel is suitable to safely and securely mount the monitor/ device. This includes the selection of the suitable fasteners and the proper installation of the same.

**WARNING Installation, Operation and Maintenance** 

This section contains warnings regarding the installation, operation and maintenance of the CIM wall channel. This section must be read in its entirety before installing, operating and maintaining the wall channel. Failure to follow these warnings may result in damage to equipment or injury to people.

Installation, operation and maintenance may only be carried out by qualified personnel. Assembly may only be done by qualified technical personnel. CIM med GmbH cannot be held responsible for any damage due to poor installation.

Do not position the wall channel above a patient. When mounting the wall channel consider horizontal swivel and vertical height adjustment range. Please make sure to avoid other equipment, personnel and patient within swivel range. CIM med GmbH recommends that the hospital's risk management personnel verify that the application is appropriate prior to installation and use of the wall channel.

Product modifications, disassembling and reassembling and/or mounting any of the components in any other way than described below are permitted

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## GENERAL NOTES

only after written approval by the CIM med GmbH construction and engineering department. CIM med GmbH cannot be held responsible for any damage or injury caused by product modifications and/ or amendments without prior permission by CIM med GmbH.

### ⚠️ WARNING: EARTHING CONNECTION

The wall channel is equipped with earth connection. It is the healthcare institution's responsibility to connect to the building's grounding. The support arm's component grounding requires connection to the building's grounding.

### MOUNTING MATERIAL

Mounting material is not included in the scope of delivery. It is important to check the load capacity of the wall and to make sure that the adequate and recommended mounting material is used. Please consult your technical department.

### DRYWALL

The wall channel must be anchored into the studs. Make sure that the appropriate fasteners and mounting material is used and the manufacturers' instructions for proper installation are adhered to. Make sure that the studs are safe to mount the wall channel.

### CONCRETE WALL

Make sure that the appropriate fasteners and mounting material is used and the manufacturers' instructions for proper installation are adhered to. Refer to qualified personnel.

### METAL SHEET OR WOODEN WALL

Make sure that the appropriate fasteners and mounting material is used and the manufacturers' instructions for proper installation are adhered to. Make sure that the metal sheet or wooden wall is safe to mount the wall channel.

### CLEANING

The CIM wall channel may be cleaned with mild, non-abrasive solutions commonly used in hospital environment. Do not use steel wool or other abrasive materials. The product has been designed to the highest specifications to make cleaning as easy as possible. CIM med makes no claims concerning the efficacy of the disinfection processes.

It is recommended to test any cleaning solution on a small area not visible to verify compatibility. Please consult your hospital's infection control manager.

### MAINTENANCE: MANNER AND FREQUENCY

Very little maintenance is required during the life span. Please check all screws on a regular basis (at least once a year) and tighten them if necessary.

In case of damage done to the load-bearing parts of the system it has to be strictly refrained from further usage of the mounting system.

### REPORTING OBLIGATION

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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---

### SAFEKEEPING

For any questions that may arise at a later time please safekeep our manual!

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
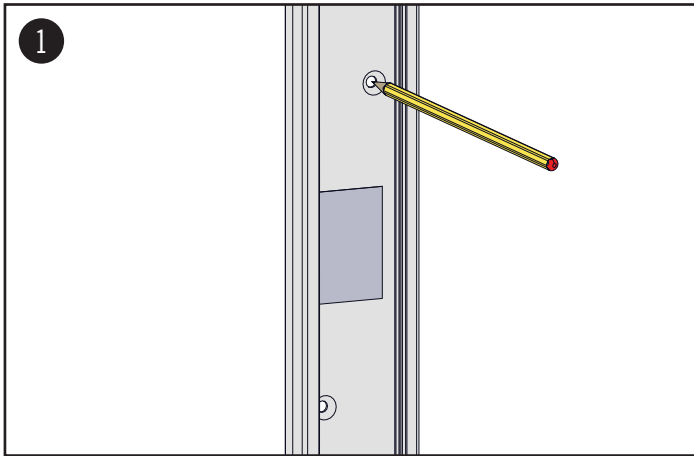
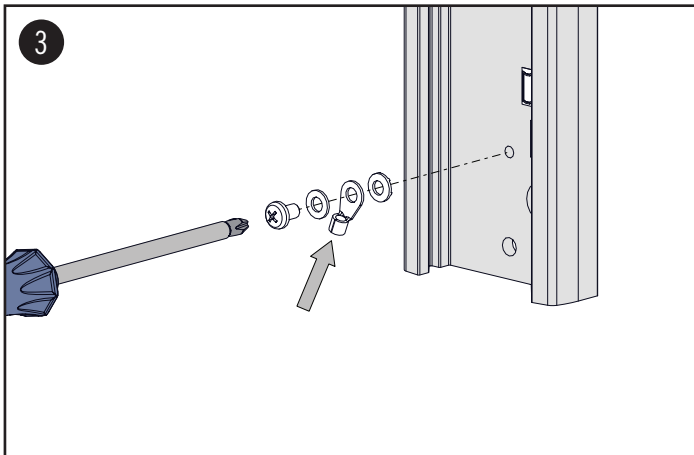
# ASSEMBLY STEPS

1. Position the wall channel. Mark the requested bore position by using wall channel's hole pattern.

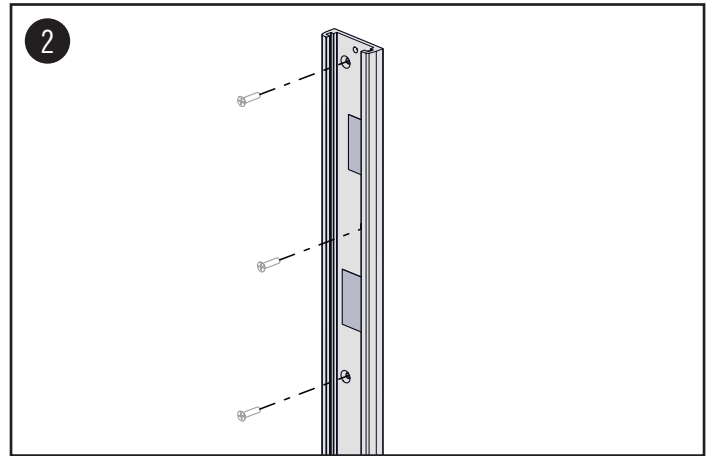**NOTE:** To level the wall channel use an air level.

2. Mount the wall channel using suitable mounting hardware. Make sure to firmly tighten all screws.
3. Connect the wall channel to the building's grounding by using the pre-assembled eyelet connector. Make sure that the contact disk are positioned as shown.