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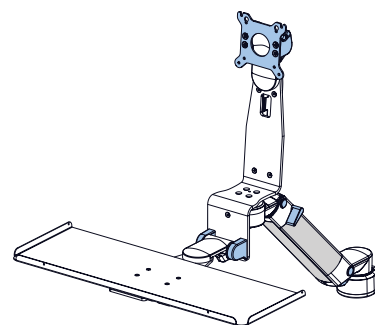
## ASSEMBLY INSTRUCTION

## MONITOR ADAPTATION Z-BRACKET

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**PLEASE READ THIS INSTRUCTION CAREFULLY BEFORE INSTALLING!**

Clarification of non-standardized Symbols:

Warning of a danger point

**AREA OF APPLICATION/ INTENDED USE**

CIM medical products are intended to mount and position medical electrical equipment (hereafter referred to as ME equipment) and accessories to those and are used in operating theatres as well as treatment rooms as auxiliary devices for doctors and medical staff.

**WARNING Installation, Operation and Maintenance** 

This section contains warnings regarding the installation, operation and maintenance of monitor adaptation z-bracket. This section must be read in its entirety before installing, operating and maintaining the monitor adaptation z-bracket. Failure to follow these warnings may result in damage to equipment or injury to people.

Installation, operation and maintenance may only be carried out by qualified personnel. Assembly may only be done by qualified technical personnel. CIM med GmbH cannot be held responsible for any damage due to poor installation.

Do not position the support arm with z-bracket above a patient. Note that the support arm with z-bracket has a wide range of motion. Please consider carefully the ME equipment being mounted and the proximity of the mounting assembly to other equipment, hospital personnel, and the patient. CIM med GmbH recommends that the hospital's risk management personnel verify that the application is appropriate prior to installation and use of the support arm.

The total load mounted may have changed due to adding or removing equipment (other than primary instrument) while the height locking lever was engaged. Ensure that the weight of the instrument being mounted does not exceed the maximum load indication.

Product modifications, disassembling and reassembling and/or mounting any of the components in any other way than described below are permitted only after written approval by the CIM med GmbH construction and engineering department. CIM med GmbH cannot be held responsible for any damage or injury caused by product modifications and/ or amendments without prior permission by CIM med GmbH.

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CIM med GmbH, Margot-Kalinke-Str. 9, 80939 München

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## GENERAL NOTES

### ⚠️ WARNING: MORTAL DANGER BY ELECTRIC SHOCK

The electric cables of ME equipment and ME systems may only be connected to the power source and to the medical device respectively after finishing installation. Prior to maintenance or disassembly all power cables must be disconnected from the power source. Only cables with double insulation may be used for running through the arm.

### ⚠️ WARNING: EARTHING CONNECTION

The support arm with z-bracket is delivered with components' grounding. It is the healthcare institution's responsibility to connect the support arm with z-bracket to the electrical grounding of the building. When mounting the arm to wall channel or side channels of anesthesia machines the arm is connected to the channel via the screws of the adapter.

### CLEANING

The monitor adaptations z-bracket may be cleaned with mild, non-abrasive solutions commonly used in hospital environment. Do not use steel wool or other abrasive materials. The product has been designed to the highest specifications to make cleaning as easy as possible. CIM med makes no claims concerning the efficacy of the disinfection processes.

It is recommended to test any cleaning solution on a small area of the monitor adaptation not visible to verify compatibility. Please consult your hospital's infection control manager.

### MAINTENANCE: MANNER AND FREQUENCY

Very little maintenance is required during the life span. Please check all screws on a regular basis (at least once a year) and tighten them if necessary. In case of damage done to the load-bearing parts of the system it has to be strictly refrained from further usage of the mounting system.

### REPORTING OBLIGATION

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

### SAFEKEEPING

For any questions that may arise at a later time please safekeep our manual!

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
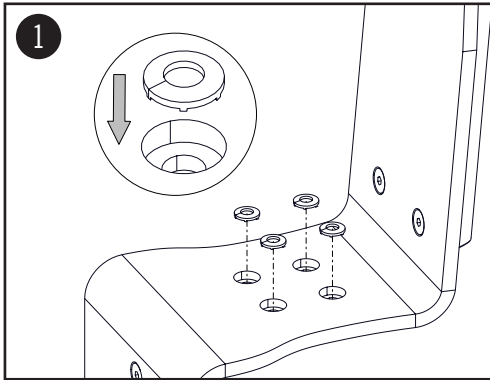
Place the (4) contact wheels in the holes as shown.

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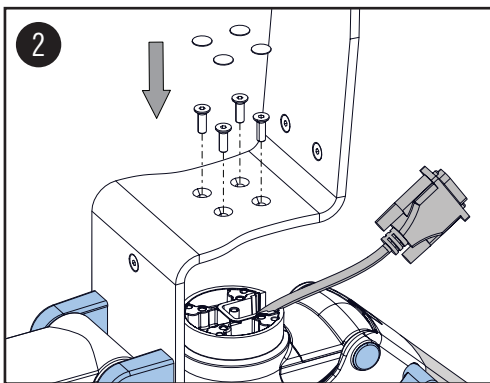**⚠ WARNING**

To guarantee a secure components grounding it is essential to use the (4) contact wheels.

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
Mount the z-bracket to the swivel head by using the (4) M5 x 16 counter-sunk screws. Attach (4) cover caps to the screws.

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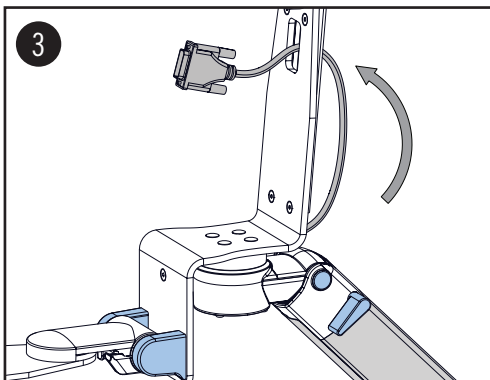
**⚠ WARNING**

Take care the screw head is completely counter-sunk.

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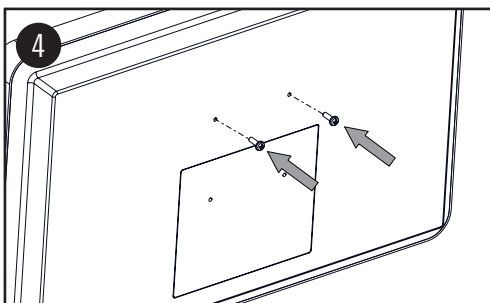
Feed the cables through the bracket from behind.

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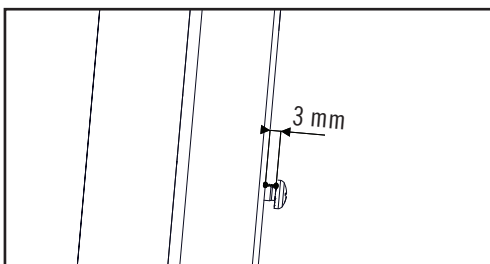
For easy mounting fix the two upper M4 pan head screws to the monitor rear side.

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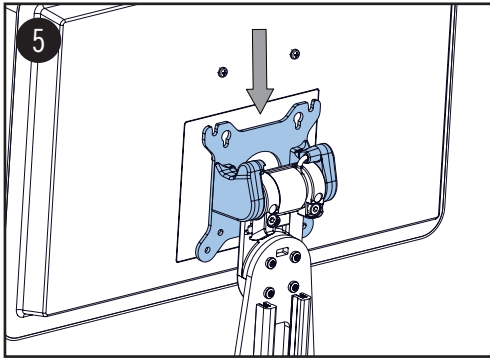
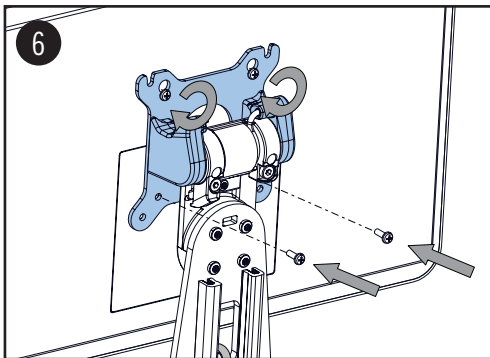
Fix the lower two screws to the monitor. Tighten the (4) screws.

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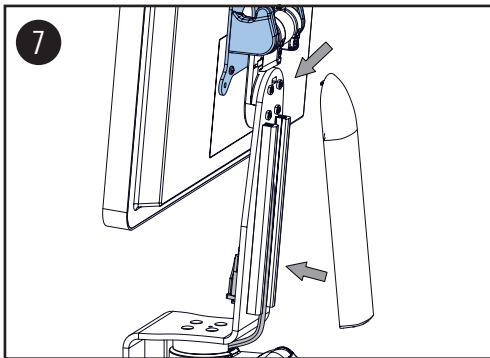For tensioning the friction joint, see "tilt adjustment" in accompanying arm instructions.

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
Clip the plastic cover into the bracket.

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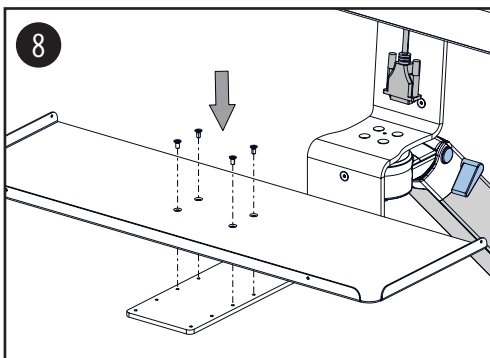
**⚠ WARNING**

Make sure that the cables are placed along the intended cable channel.

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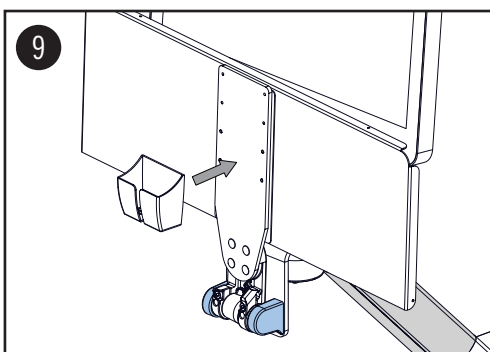Mount keyboard tray by using M4 x 8 counter-sunk screws.

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Mouse garage with velcro tape to be fixed at the lower side of the keyboard.