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KOMPONENTENERDUNG AN TRAGARMEN

ASSEMBLY INSTRUCTION

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PLEASE READ THIS INSTRUCTION CAREFULLY BEFORE INSTALLING!

Clarification of non-standardized Symbols:

 Warning of a danger point

 Important information

AREA OF APPLICATION/ INTENDED USE

Cable Integrated Mounting systems (CIM) are intended to mount and position monitors, mouse/keyboard, laptops and other medical devices or equipment (hereafter referred to as ME equipment). They are mounted to walls, carts, tables, other surfaces or machines using specific mounting adapters. CIM systems do not fulfil any specific medical purpose on their own.

GENERAL NOTES

Make sure that the housing provided is properly mounted and can withstand the weight load. Please check the weight of the ME equipment to be mounted. The permissible total weight is mentioned on the corresponding data sheet.

WARNING Installation, Operation and Maintenance 

This section contains warnings regarding the installation, operation and maintenance of the support arm. This section must be read in its entirety before installing, operating and maintaining the support arm. Failure to follow these warnings may result in damage to equipment or injury to people.

Installation, operation and maintenance may only be carried out by qualified personnel. Cable integration and assembly may only be done by qualified technical personnel. CIM med GmbH cannot be held responsible for any damage due to poor installation.

Product modifications, disassembling and reassembling and/or mounting any of the components in any other way than described below are permitted only after written approval by the CIM med GmbH construction and engineering department. CIM med GmbH cannot be held responsible for any damage or injury caused by product modifications and/or amendments without prior permission by CIM med GmbH.

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GENERAL NOTES

⚠️ WARNING: EARTHING CONNECTION

CIM arms are equipped with component grounding. It is the responsibility of the customer to connect the arm to the building's grounding in accordance with these installation instructions.

TOOLS REQUIRED

Torx T20 for grounding strap

MAINTENANCE: MANNER AND FREQUENCY

Very little maintenance is required during the life span of the support arm; however, it is recommended to check all screws on a regular basis (at least once a year) and tighten them if necessary.

In case of damage done to the load-bearing parts of the system it has to be strictly refrained from further usage of the mounting system.

REPORTING OBLIGATION

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

SAFEKEEPING

For any questions that may arise at a later time please safekeep our manual!

ASSISTANCE REQUIRED

If you are unable to adjust the support arm or require service, contact CIM med technical support: Phone +49 89 978 94 080 or your local dealer.

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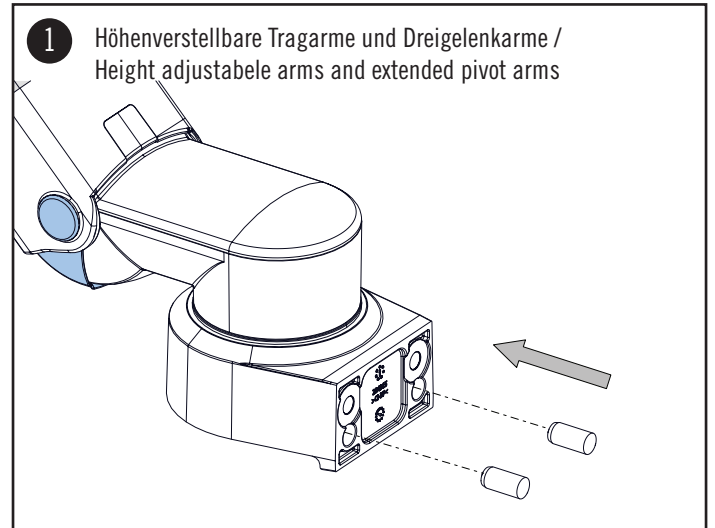
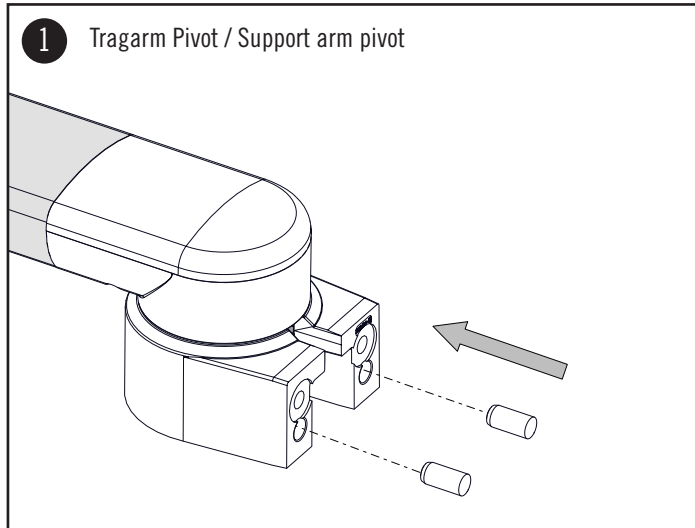
1. Push the Ø8x18 dowel pins into the bore holes.

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There are two ways to connect the support arm to the component grounding of your device:

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OPTION A:

Mounting of the support arm on a housing which is conductively connected to the grounding of the device.

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OPTION B:

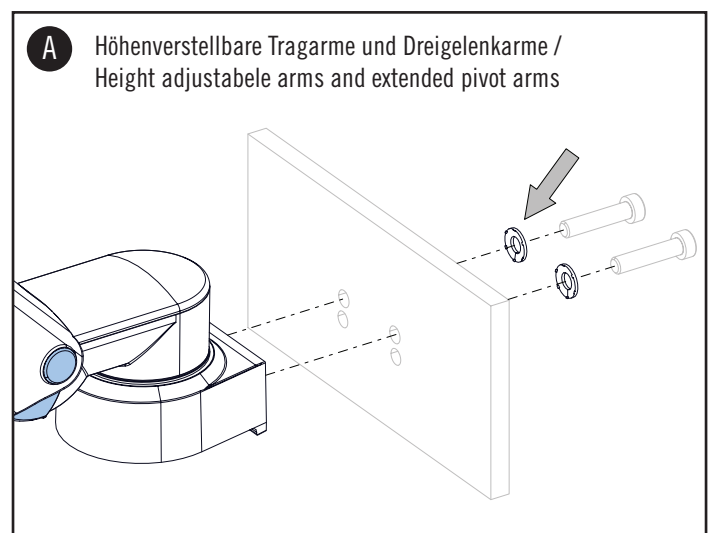
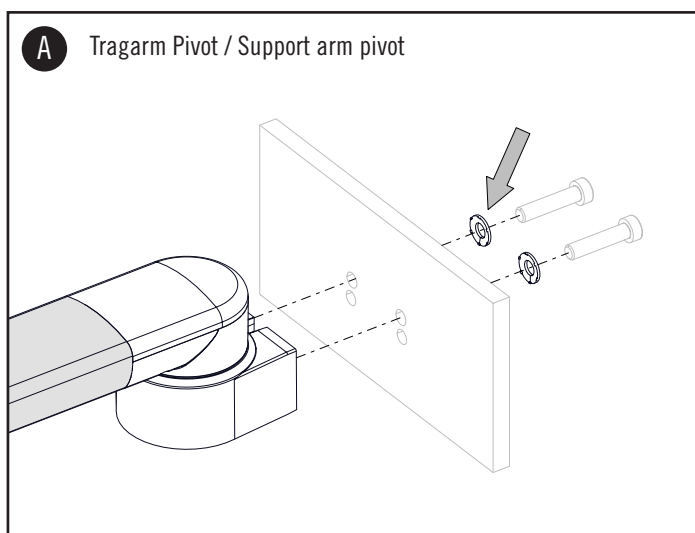
The support arm is mounted on a non-conductive housing.

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When mounting the support arm, use the enclosed contact disks. Torque moment 23 Nm

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
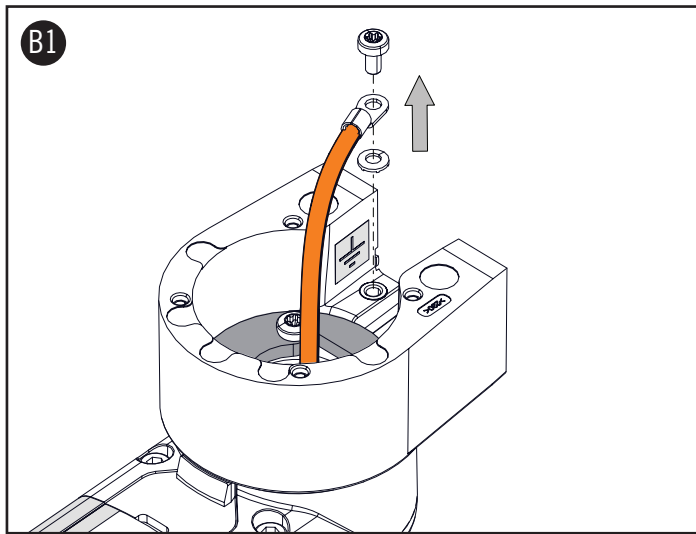
ASSEMBLY STEPS OPTION B - SUPPORT ARM PIVOT


1. Loosen the M4x8 pan-head screw with which the earthing strap is fastened.
2. Use the enclosed cable lug to ground the component.

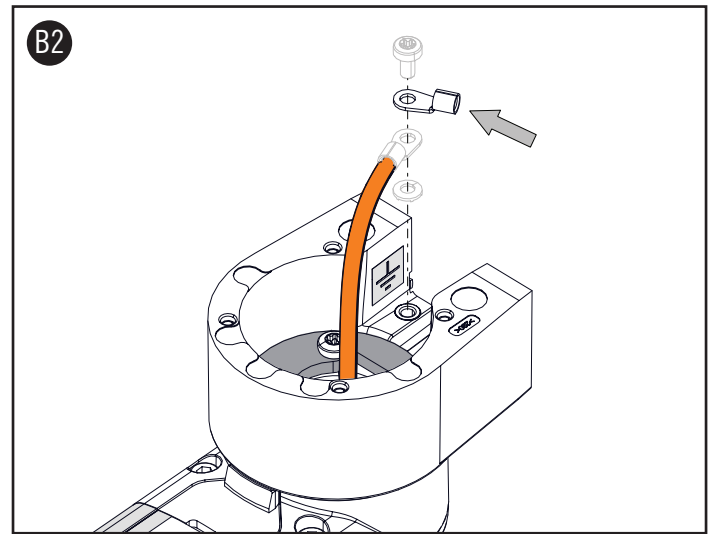ⓘ NOTE: Make sure to reconnect the component grounding of the support arm.

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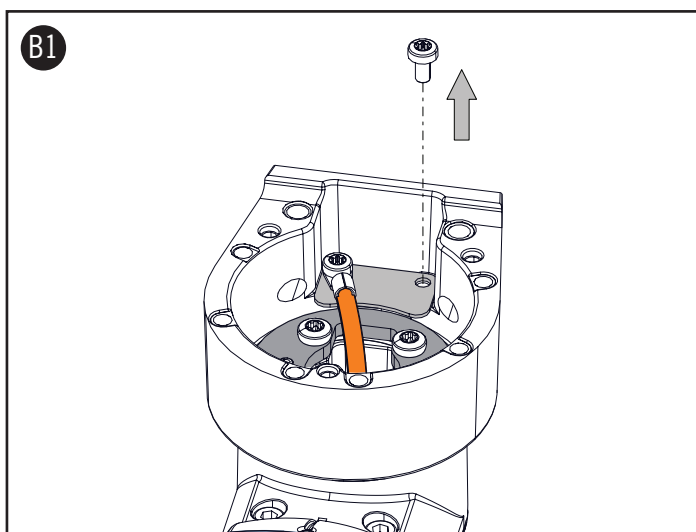ASSEMBLY STEPS OPTION B - HEIGHT ADJUSTABLE ARMS AND EXTENDED PIVOT ARMS:

1. Loosen the M4x8 pan-head screw to which no grounding strap is attached.
2. Use the enclosed cable lug to ground the component.