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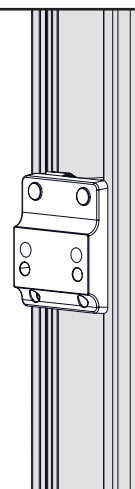
ADAPTER VERTIKALER / HORIZONTALER WANDKANAL

ASSEMBLY INSTRUCTION

ADAPTER VERTICAL / HORIZONTAL WALL CHANNEL

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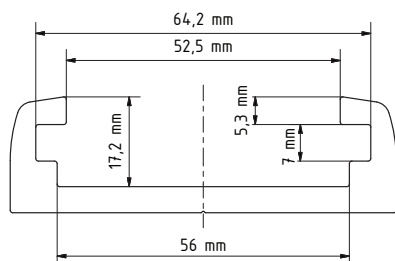
WARNING ⚠

Please check the weight of the ME equipment to be mounted. The permissible total weight is mentioned on the corresponding data sheet.

Failure to follow these warnings may result in damage to equipment or injury to people.

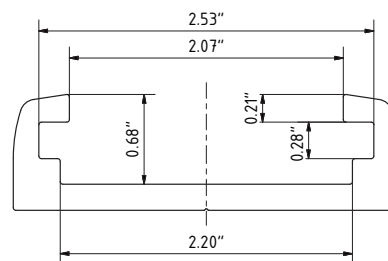
CIM med GmbH cannot be held responsible for any damage due to poor installation.

Product modifications, disassembling and reassembling and/or mounting any of the components in any other way than described below are permitted only after written approval by the CIM med GmbH construction and engineering department. CIM med GmbH cannot be held responsible for any damage or injury caused by product modifications and/ or amendments without prior permission from CIM med GmbH.

The additional instructions for use and the inspection plan are intended for end users.

COMPATIBILITY

The adapters are also compatible to various wall channels used in the medical area (e.g. GCX). Please note the compatibility (cross section) and weight load of different wall channels provided by any other supplier.

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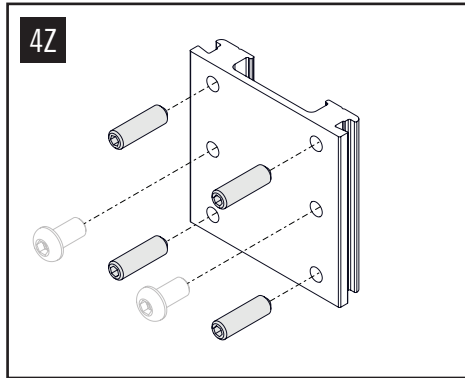
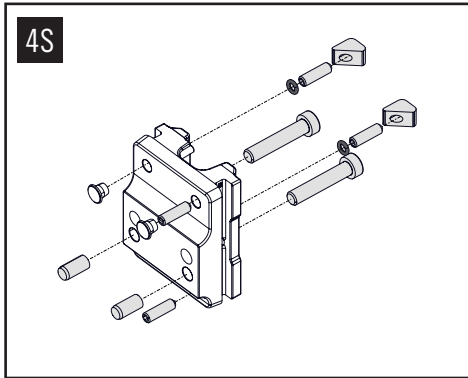
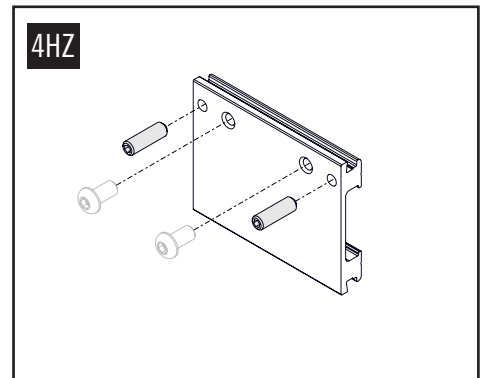
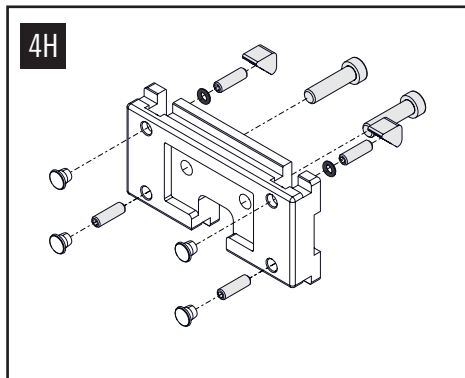
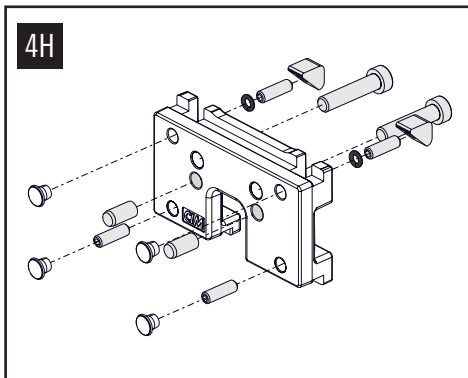
ADAPTER FOR HORIZONTAL WALL CHANNEL

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
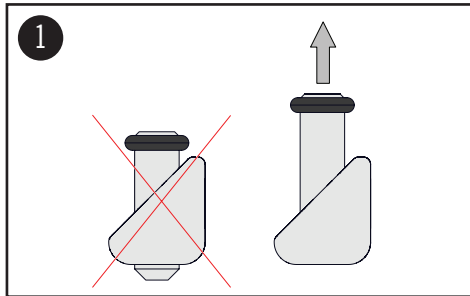
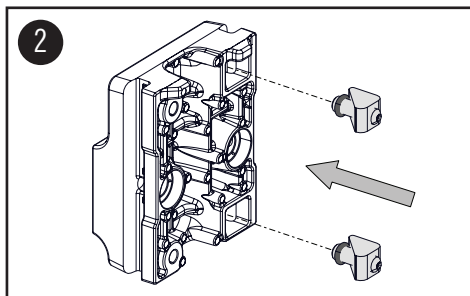
Clamping blocks with O-rings.
Grub screw must be fully extended and the ring placed on the top few turns.

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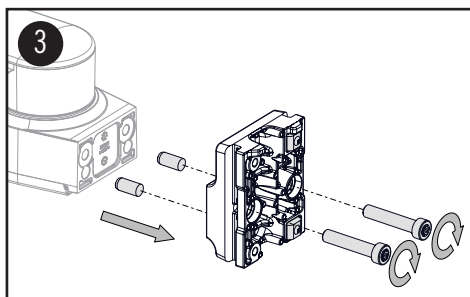
Press clamping block with grub screw and O-ring into the hole of the adapter.

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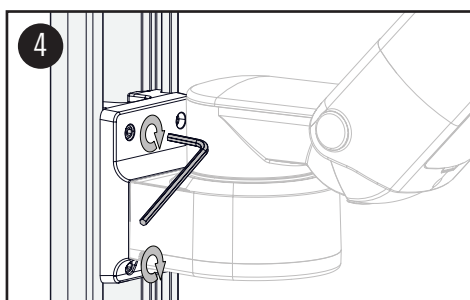To attach the support arm, first locate the pins into the bore hohles and then tighten the (2) M8 screws.
Torquere moment 23 Nm (204 in-lbs).

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
Slide the adapter vertically (or horizontally) into the wall channel. First tighten the threaded pins on the left side (or on the upper side). This ensures the correct adjustment of the adapter.

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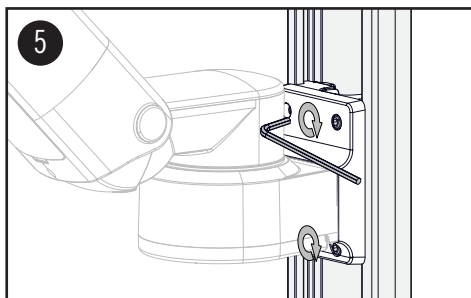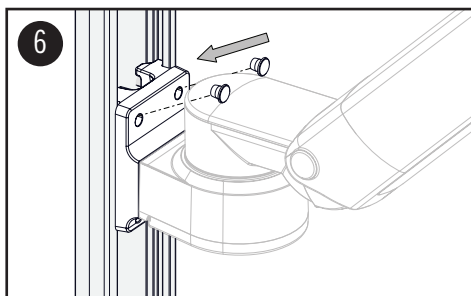
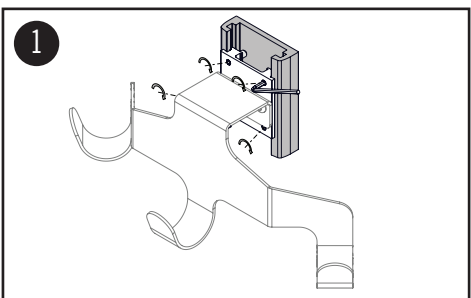
ADAPTER VERTICAL / HORIZONTAL

Tighten the threaded pins on the right side (or on the lower side).


⚠ WARNING:
The grounding screw must be tightened for a safe component grounding.

The (2) cover caps press into the holes of the grub screws.

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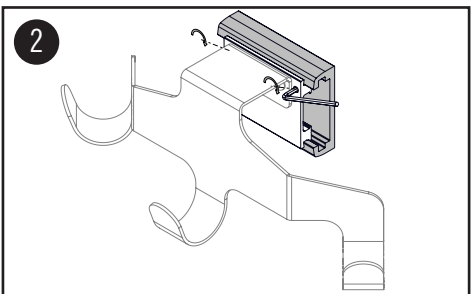ADAPTER VERTICAL / HORIZONTAL FOR ACCESSORIES

Adapter vertical for accessories
Mount the adapter to wall channel using the M6x20 screws.

Adapter horizontal for accessories
Mount the adapter to wall channel using the M6x20 screws.