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FIXE NORMSCHIENENKLEMMEN

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PLEASE READ THIS INSTRUCTION CAREFULLY BEFORE INSTALLING!

Clarification of non-standardized Symbols:

Warning of a danger point

AREA OF APPLICATION/ INTENDED USE

CIM medical products are intended to mount and position medical electrical equipment (hereafter referred to as ME equipment) and accessories to those and are used in operating theatres as well as treatment rooms as auxiliary devices for doctors and medical staff. The fixed rail clamp are exclusively to be used in conjunction with CIM support arms in the medical sector.

GENERAL NOTES

Before you want to attach the fixed rail clamp to an existing European rail, please verify that the rail can hold the weight. Please check the weight of the ME equipment to be mounted. The permissible total weight is mentioned on the corresponding data sheet.

WARNING Installation, Operation and Maintenance

This section contains warnings regarding the installation, operation and maintenance of the fixed rail clamp. This section must be read in its entirety before installing, operating and maintaining the fixed rail clamp. Failure to follow these warnings may result in damage to equipment or injury to people.

Installation, operation and maintenance may only be carried out by qualified personnel. Assembly may only be done by qualified technical personnel. CIM med GmbH cannot be held responsible for any damage due to poor installation.

Product modifications, disassembling and reassembling and/or mounting any of the components in any other way than described below are permitted only after written approval by the CIM med GmbH construction and engineering department. CIM med GmbH cannot be held responsible for any damage or injury caused by product modifications and/or amendments without prior permission by CIM med GmbH.

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GENERAL NOTES

⚠️ WARNING: EARTHING CONNECTION

The fixed rail clamp is delivered with components' grounding. It is the healthcare institution's responsibility to connect the rail to the electrical grounding of the building. The arm is connected to the channel via the grounding screw of the adapter.

CLEANING

The fixed rail clamp may be cleaned with mild, non-abrasive solutions commonly used in hospital environment. Do not use steel wool or other abrasive materials. The product has been designed to the highest specifications to make cleaning as easy as possible. CIM med makes no claims concerning the efficacy of the disinfection processes.

It is recommended to test any cleaning solution on a small area of the fixed rail clamp not visible to verify compatibility. Please consult your hospital's infection control manager.

MAINTENANCE: MANNER AND FREQUENCY

Very little maintenance is required during the life span. Please check all screws on a regular basis (at least once a year) and tighten them if necessary.

In case of damage done to the load-bearing parts of the system it has to be strictly refrained from further usage of the mounting system.

REPORTING OBLIGATION

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

SAFEKEEPING

For any questions that may arise at a later time please safekeep our manual!

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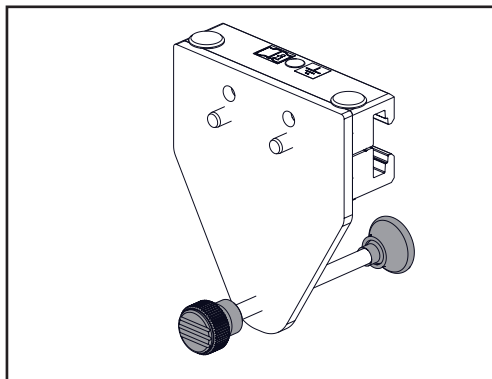
FIXED RAIL CLAMP WITH LEVELLING FOOT (2F)

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

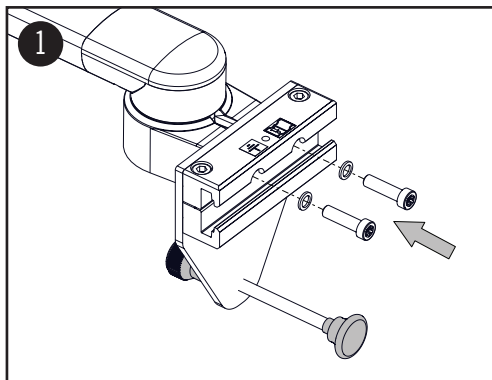
ASSEMBLY STEPS

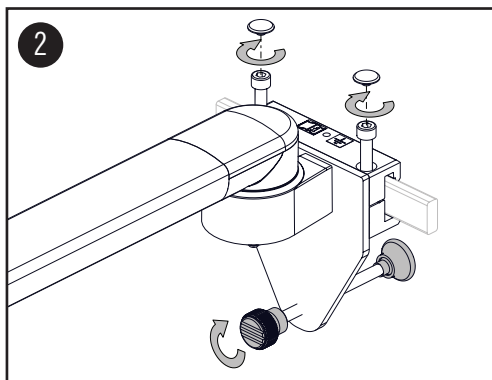Mount the arm to the rail clamp using the (2) M8x30 Torx 45 pan head screws and detent edged washers M8.
Torque moment 25 Nm (221 in-lbs)

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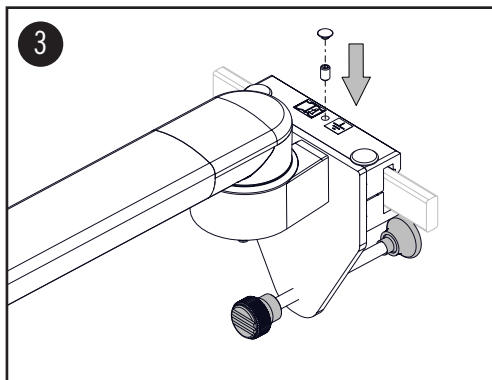Loosen the two pre-assembled hexagon socket head screws M8x45. Attach the rail clamp to the DIN rail and mount it by tightening the hex screws. Cover the screws with the end caps provided.
The levelling foot must be adjusted for support against the wall.

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
⚠ CAUTION

The grounding screw must be tightened for a safe component grounding. Cover the screw with the end cap provided.

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
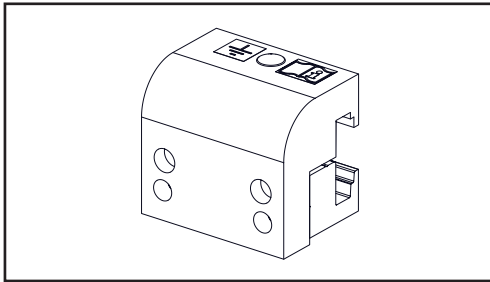
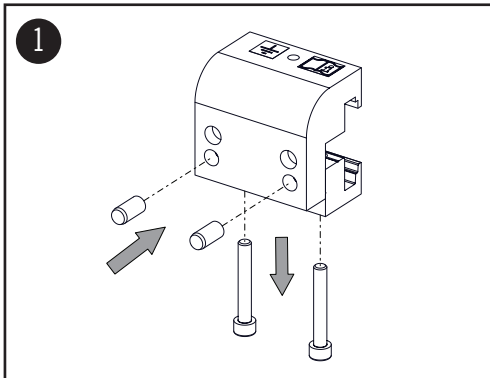
ASSEMBLY STEPS

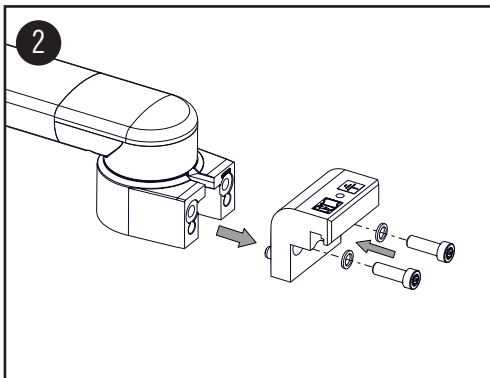Push dowel pins Ø8x18 into the bore holes. Loosen the two hexagon socket head screws M6x40 of the lower clamping jaw.

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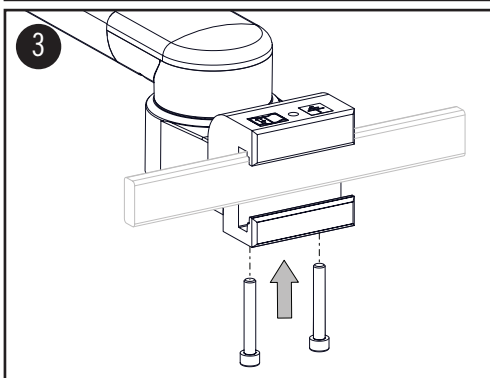
Mount the top clamp to the arm using the (2) M8x25 Torx 45 pan head screws and detent edged washers M8. Torque moment 25 Nm (221 in-lbs)

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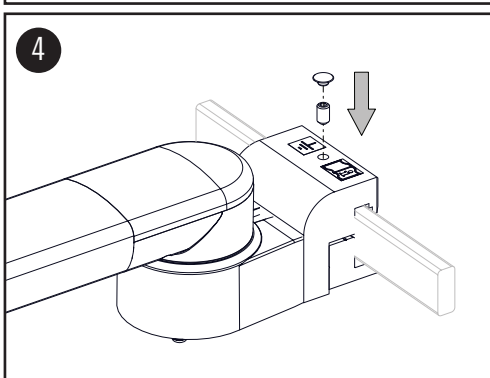Attach the rail clamp to the DIN rail and mount it by tightening the two hex screws M6x40 in the lower clamping jaw.

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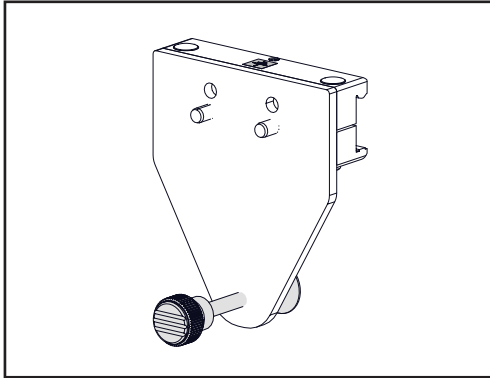
FIXED RAIL CLAMP WITH LEVELLING FOOT (2W)

FOR SKANDINAVIAN RAILS UNIFLEX, DUOFLEX, MODURA

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

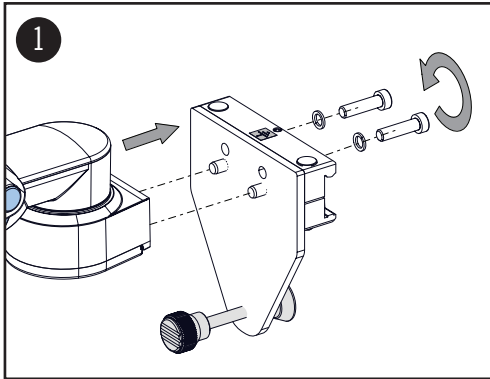
ASSEMBLY STEPS

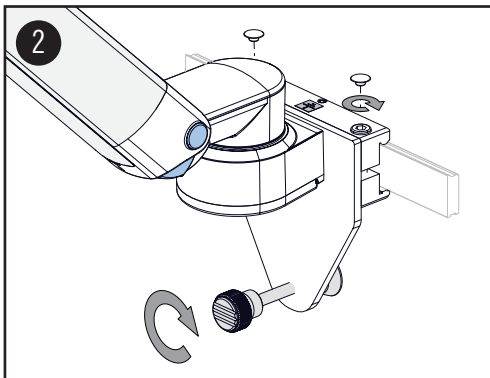Mount the arm to the rail clamp using the (2) M8x30 Torx 45 pan head screws and detent edged washers M8. Torque moment 25 Nm (221 in-lbs)

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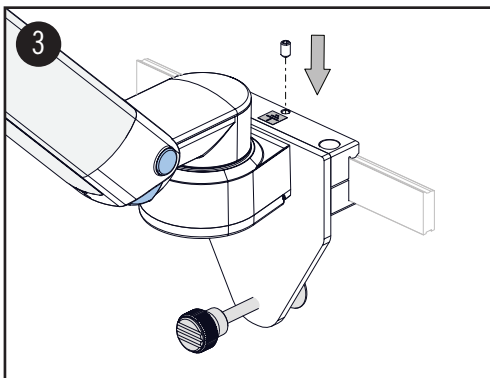Loosen the two pre-assembled hexagon socket head screws M8x45. Attach the rail clamp to the DIN rail and mount it by tightening the hex screws. Cover the screws with the end caps provided. The levelling foot must be adjusted for support against the wall.

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⚠ CAUTION

The grounding screw must be tightened for a safe component grounding.