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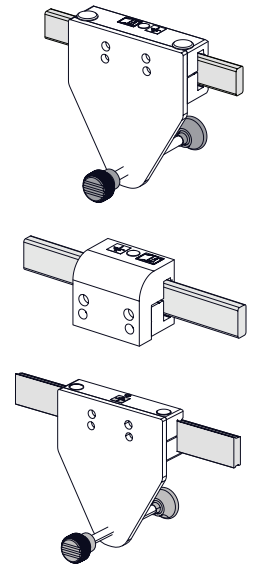
FIXE NORMSCHIENENKLEMMEN

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ASSEMBLY INSTRUCTION

FIXED RAIL CLAMPS

Fixed rail clamp with levelling foot (2F, 2K, 2L)

Fixed rail clamp (5)

Fixed rail clamp with levelling foot - for Scandinavian rails (2W, 2WK, 2WL)

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WARNING ⚠

Please check the weight of the ME equipment to be mounted. The permissible total weight is mentioned on the corresponding data sheet.

Failure to follow these warnings may result in damage to equipment or injury to people.

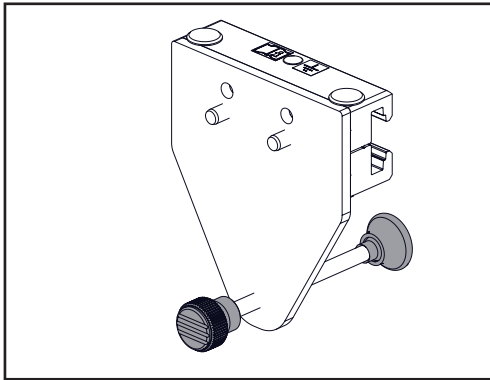
CIM med GmbH cannot be held responsible for any damage due to poor installation.

Product modifications, disassembling and reassembling and/or mounting any of the components in any other way than described below are permitted only after written approval by the CIM med GmbH construction and engineering department. CIM med GmbH cannot be held responsible for any damage or injury caused by product modifications and/ or amendments without prior permission from CIM med GmbH.

The additional instructions for use and the inspection plan are intended for end users.

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FIXED RAIL CLAMP WITH LEVELLING FOOT (2F, 2K, 2L)

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

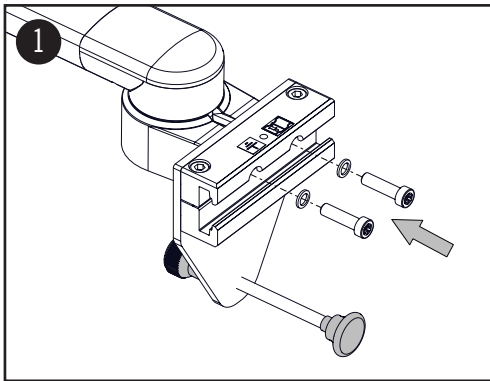
ASSEMBLY STEPS

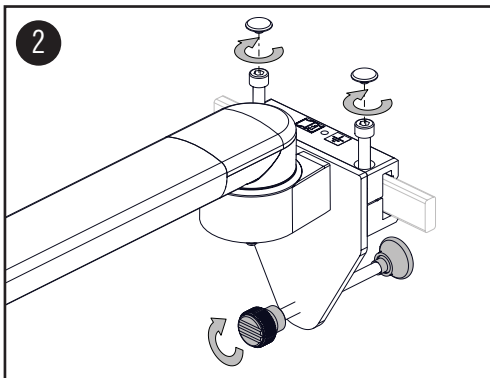Mount the arm to the rail clamp using the (2) M8x30 Torx 45 pan head screws and detent edged washers M8.
Torque moment 25 Nm (221 in-lbs)

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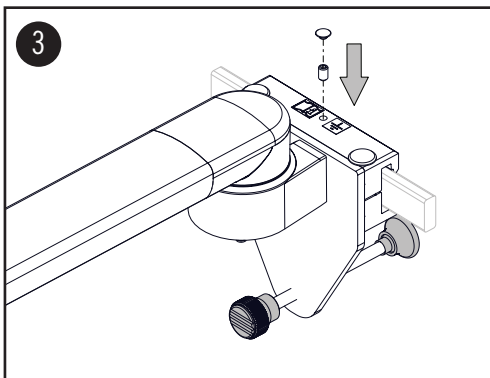Loosen the two pre-assembled hexagon socket head screws M8x45. Attach the rail clamp to the DIN rail and mount it by tightening the hex screws. Cover the screws with the end caps provided.
The levelling foot must be adjusted for support against the wall.

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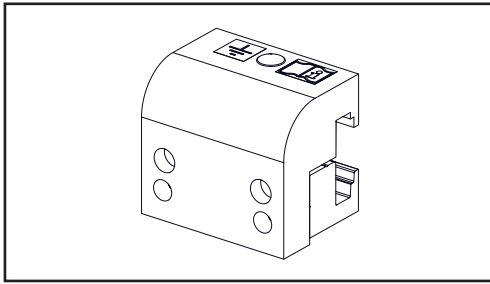
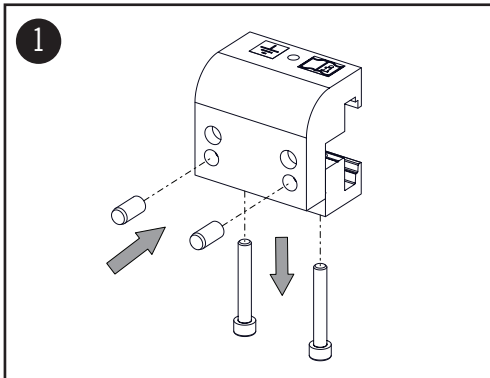
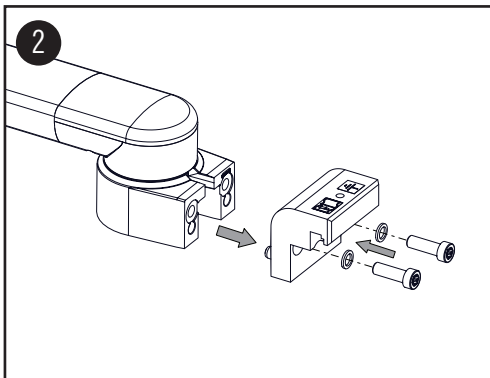
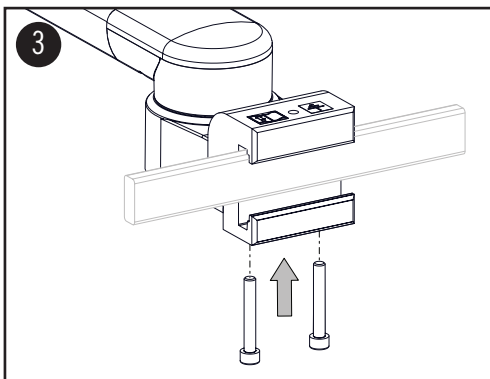
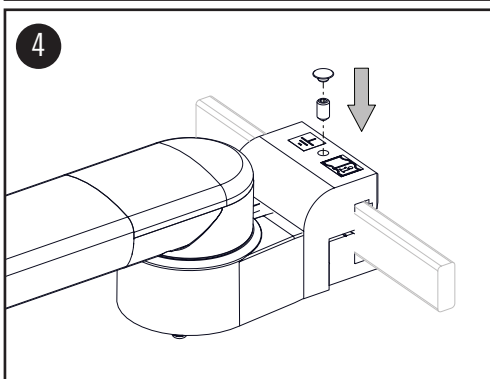
ASSEMBLY STEPS

Push dowel pins Ø8x18 into the bore holes. Loosen the two hexagon socket head screws M6x40 of the lower clamping jaw.

Mount the top clamp to the arm using the (2) M8x25 Torx 45 pan head screws and detent edged washers M8.
Torque moment 25 Nm (221 in-lbs)

Attach the rail clamp to the DIN rail and mount it by tightening the two hex screws M6x40 in the lower clamping jaw.

CAUTION

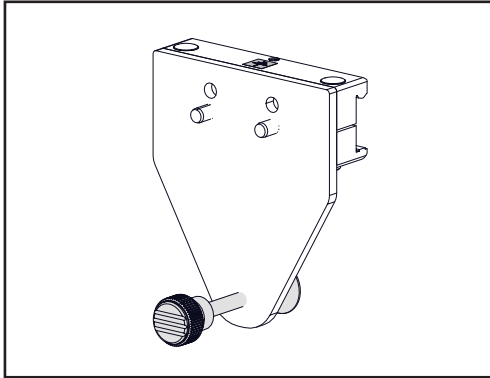
The grounding screw must be tightened for a safe component grounding. Cover the screw with the end cap provided.

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FIXED RAIL CLAMP WITH LEVELLING FOOT (2W, 2WK, 2WL)

FOR SKANDINAVIAN RAILS UNIFLEX, DUOFLEX, MODURA

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

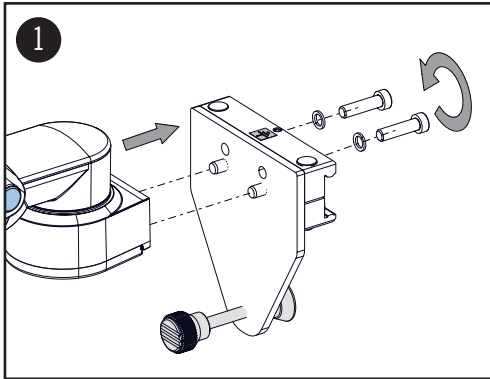
ASSEMBLY STEPS

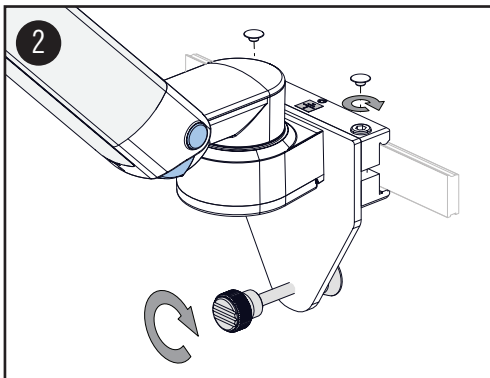Mount the arm to the rail clamp using the (2) M8x30 Torx 45 pan head screws and detent edged washers M8. Torque moment 25 Nm (221 in-lbs)

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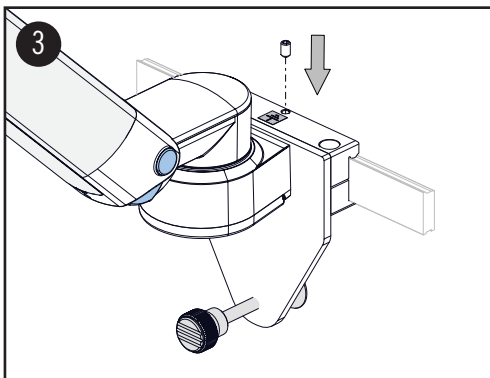Loosen the two pre-assembled hexagon socket head screws M8x45. Attach the rail clamp to the DIN rail and mount it by tightening the hex screws. Cover the screws with the end caps provided. The levelling foot must be adjusted for support against the wall.

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CAUTION

The grounding screw must be tightened for a safe component grounding.