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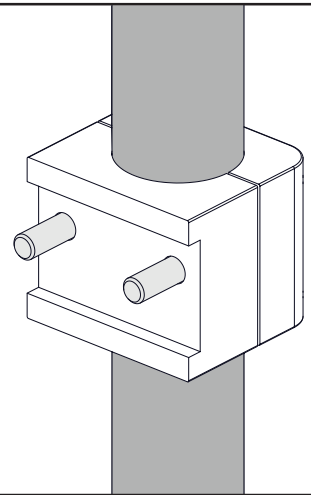
# FIXE RUNDROHRKLEMME UNIVERSELLE RUNDROHRKLEMME

## ASSEMBLY INSTRUCTION

# FIXED POLE CLAMP UNIVERSAL POLE CLAMP

**WARNHINWEISE** 

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**WARNING** 

Please check the weight of the ME equipment to be mounted. The permissible total weight is mentioned on the corresponding data sheet.

Failure to follow these warnings may result in damage to equipment or injury to people.

CIM med GmbH cannot be held responsible for any damage due to poor installation.

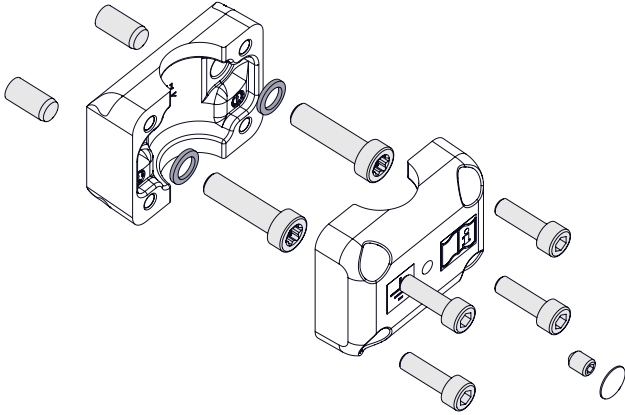
Product modifications, disassembling and reassembling and/or mounting any of the components in any other way than described below are permitted only after written approval by the CIM med GmbH construction and engineering department. CIM med GmbH cannot be held responsible for any damage or injury caused by product modifications and/ or amendments without prior permission from CIM med GmbH.

The additional instructions for use and the inspection plan are intended for end users.

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

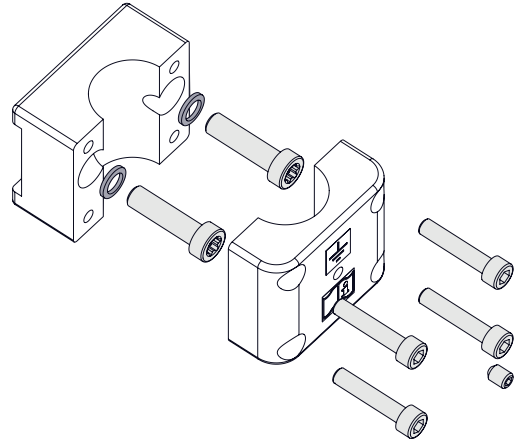
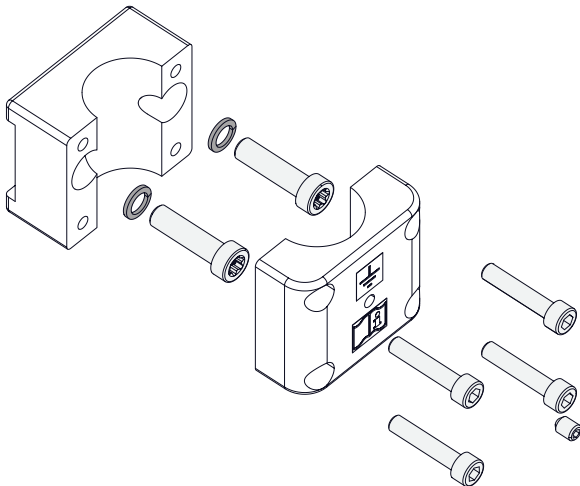
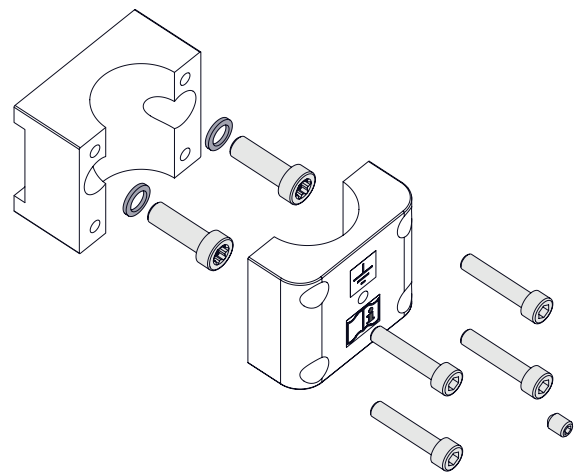
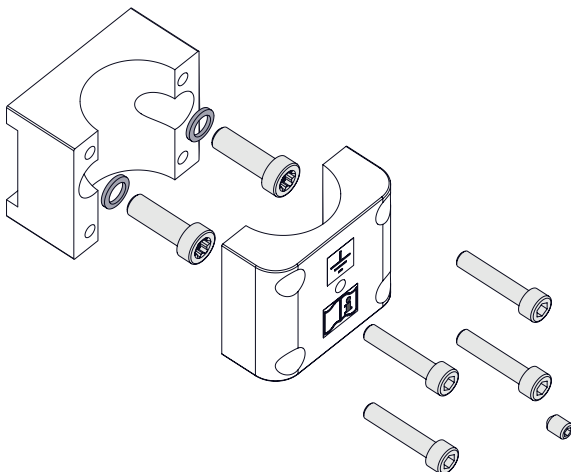
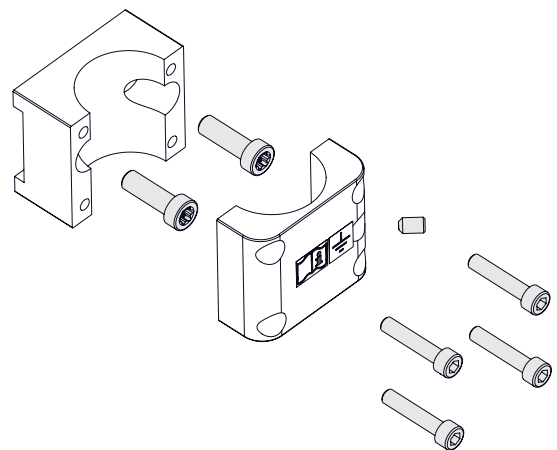
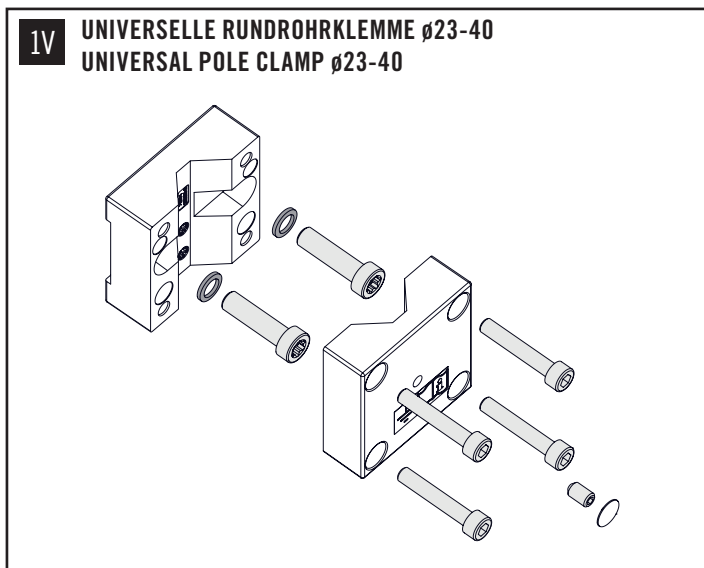
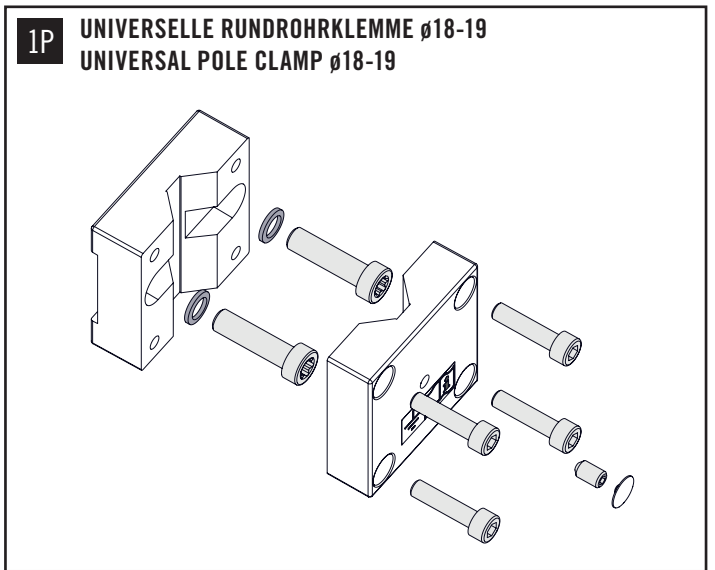
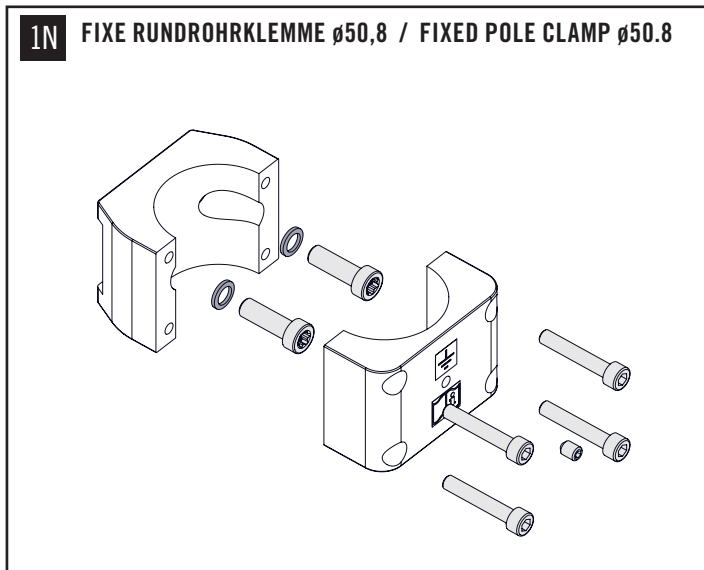
**⚠ CAUTION**

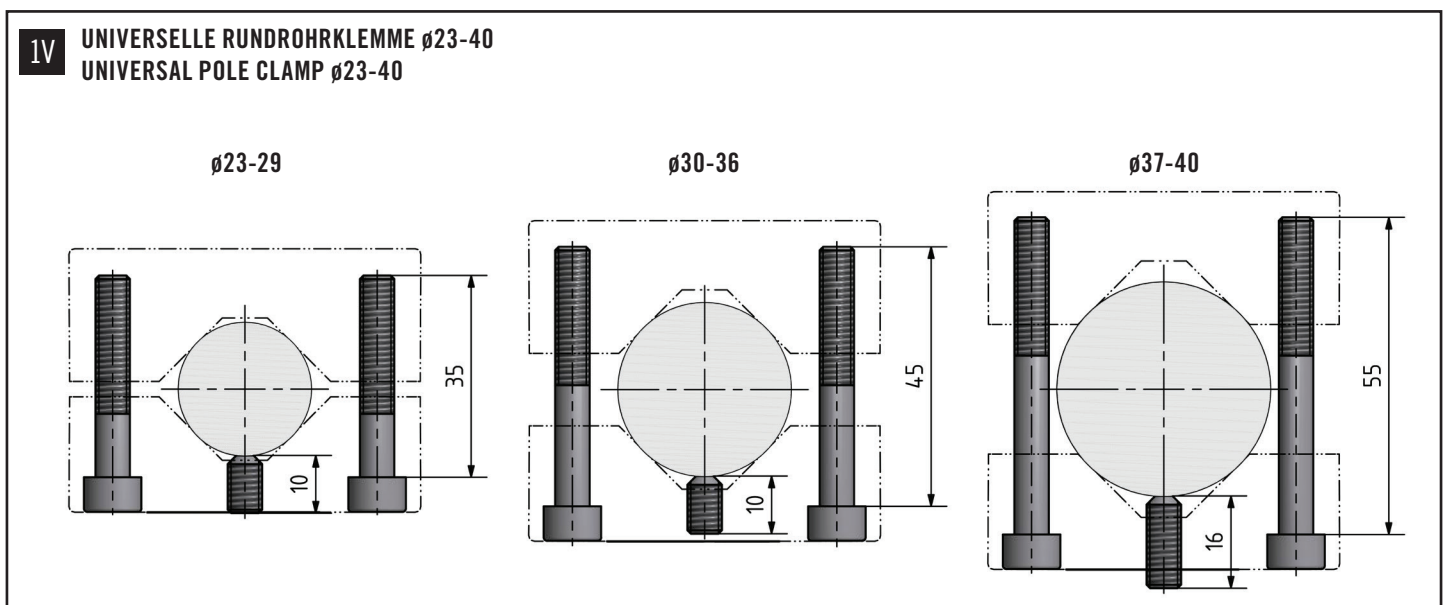The scope of delivery for the universal pole clamp  $\phi 23-40$  consists of 3 different lengths of Allen key screws and 2 different lengths of earthing screws. Please refer to the diagram for information.

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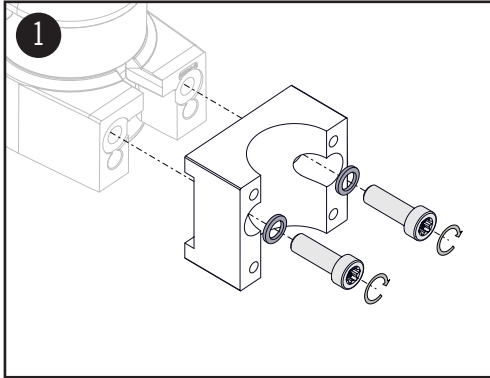
## ASSEMBLY STEPS

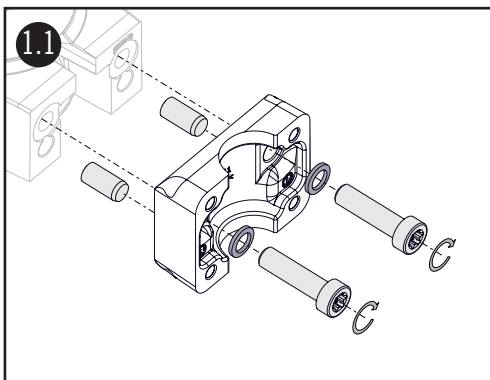Mount the adapter to arm using the 2 x M8 screws and detent washers.

Torque moment: 25 Nm (221 in-lbs)

**!** To guarantee a secure components grounding it is essential to use the both detent washers.

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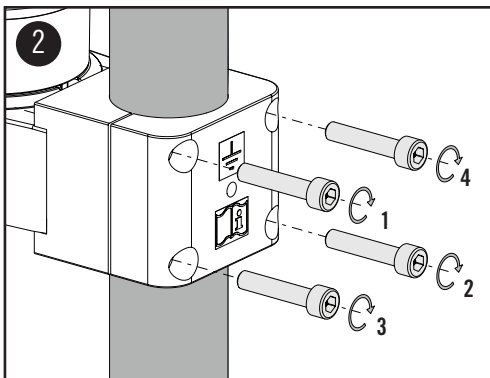
Anziehmoment: 25 Nm

### Fixed pole clamp $\varnothing 25$


For a secure fixation it is necessary to insert the two cylinder pins into the lower two holes.

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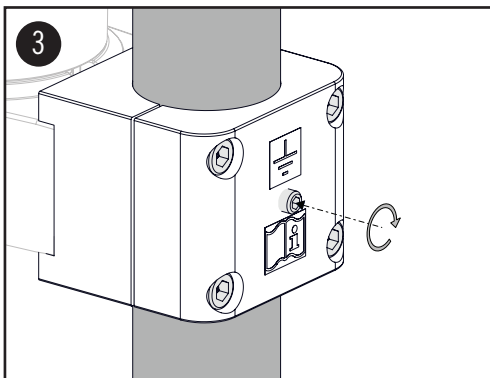Tighten the screws evenly to provide the best contact to the pole.

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### **!** CAUTION

The grounding screw must be tightened for a safe component grounding. If existant, cover the screw with the cover cap provided.