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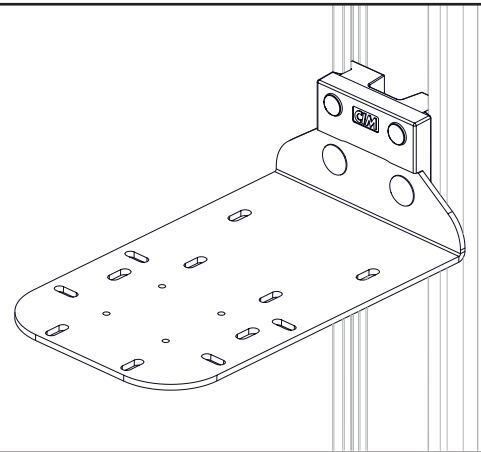
ASSEMBLY INSTRUCTION

# L-BRACKET FOR GE FRAME F2/F5

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**PLEASE READ THIS INSTRUCTION CAREFULLY BEFORE INSTALLING!**

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Clarification of non-standardized Symbols:

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**AREA OF APPLICATION/ INTENDED USE**

CIM medical products are intended to mount and position medical electrical equipment (hereafter referred to as ME equipment) and accessories to those and are used in operating theatres as well as treatment rooms as auxiliary devices for doctors and medical staff.

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**GENERAL NOTES**

Verify that the CIM wall channel has been installed and approved according to the CIM channel installation instructions. If you have a wall channel of another manufacturer, please read the corresponding assembly instruction of the manufacturer carefully.

If you want to attach the L-bracket to another rail, please verify that the rail can hold the weight. Please check the weight of the electric equipment to be mounted. The permissible total weight is mentioned on the corresponding data sheet.

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**WARNING Installation, Operation and Maintenance **

This section contains warnings regarding the installation, operation and maintenance of L-bracket. This section must be read in its entirety before installing, operating and maintaining the L-bracket. Failure to follow these warnings may result in damage to equipment or injury to people.

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Installation, operation and maintenance may only be carried out by qualified personnel. Assembly may only be done by qualified technical personnel. CIM med GmbH cannot be held responsible for any damage due to poor installation.

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Product modifications, disassembling and reassembling and/or mounting any of the components in any other way than described below are permitted only after written approval by the CIM med GmbH construction and engineering department. CIM med GmbH cannot be held responsible for any damage or injury caused by product modifications and/ or amendments without prior permission by CIM med GmbH.

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CIM med GmbH, Margot-Kalinke-Str. 9, 80939 München

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## GENERAL NOTES

### ⚠️ WARNING: MORTAL DANGER BY ELECTRIC SHOCK

The electric cables of electric equipment and electric systems may only be connected to the power source and to the medical device respectively after finishing installation. Prior to maintenance or disassembly all power cables must be disconnected from the power source. Only cables with double insulation may be used.

### CLEANING

The L-bracket may be cleaned with mild, non-abrasive solutions commonly used in hospital environment. Do not use steel wool or other abrasive materials. The product has been designed to the highest specifications to make cleaning as easy as possible. CIM med makes no claims concerning the efficacy of the disinfection processes.

It is recommended to test any cleaning solution on a small area not visible to verify compatibility. Please consult your hospital's infection control manager.

### MAINTENANCE: MANNER AND FREQUENCY

Very little maintenance is required during the life span. Please check all screws on a regular basis (at least once a year) and tighten them if necessary. In case of damage done to the load-bearing parts of the system it has to be strictly refrained from further usage of the mounting system.

### REPORTING OBLIGATION

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

### SAFEKEEPING

For any questions that may arise at a later time please safekeep our manual!

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

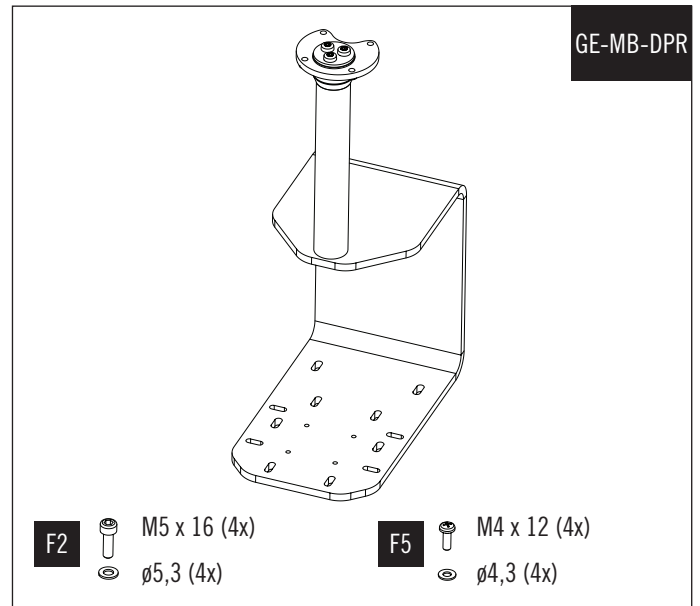
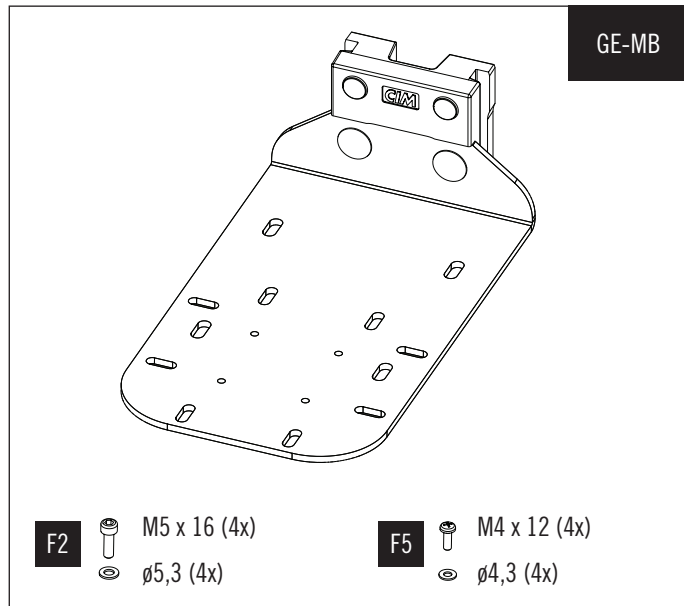
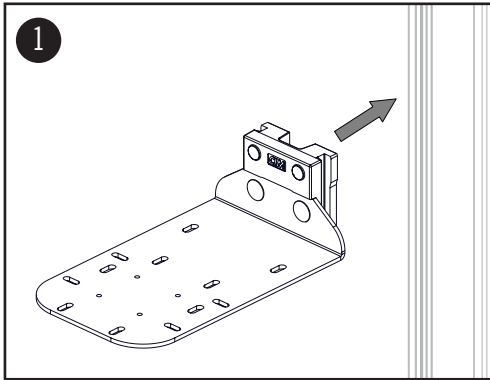
Mount the L-bracket to its designated place (wall channel, tube, DIN rail etc.). The illustration shows mounting to a wall channel.

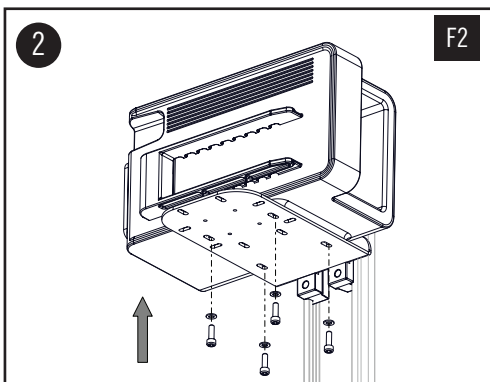**NOTE:**  
Please refer to the corresponding assembly instruction for all other adapters.

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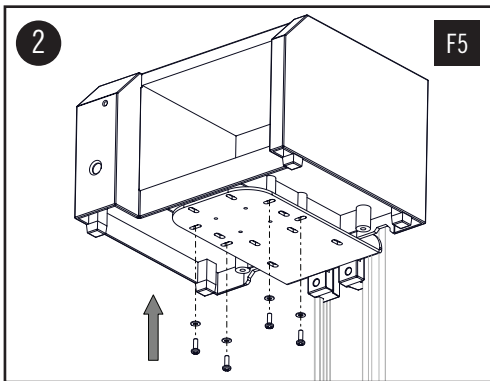Attach the GE Frame F2 using the (4) M5x16 cylinder-head screws and the (4)  $\phi$ 5.3 washers.

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Attach the GE Frame F5 using the (4) M4x12 pan-head screws and the (4)  $\phi$ 4.3 washers.

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CIM med GmbH, Margot-Kalinke-Str. 9, 80939 Munich / Germany