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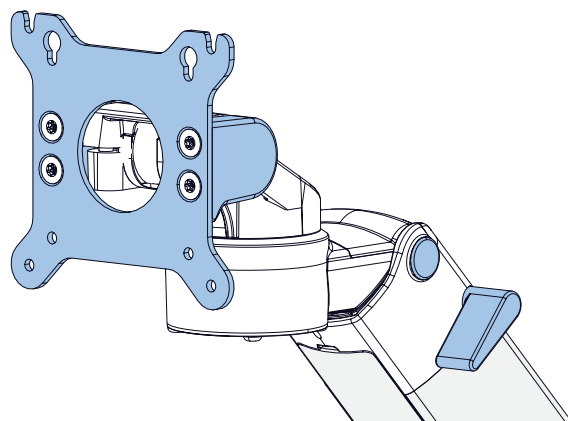
ASSEMBLY INSTRUCTION

MONITOR ADAPTATION VESA

**WARNHINWEISE** ⚠

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WARNING ⚠

Please check the weight of the ME equipment to be mounted. The permissible total weight is mentioned on the corresponding data sheet.

Failure to follow these warnings may result in damage to equipment or injury to people.

CIM med GmbH cannot be held responsible for any damage due to poor installation.

Product modifications, disassembling and reassembling and/or mounting any of the components in any other way than described below are permitted only after written approval by the CIM med GmbH construction and engineering department. CIM med GmbH cannot be held responsible for any damage or injury caused by product modifications and/ or amendments without prior permission from CIM med GmbH.

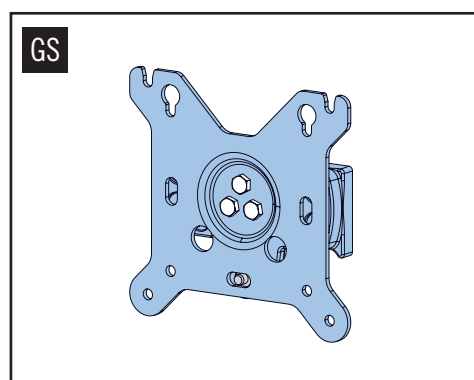
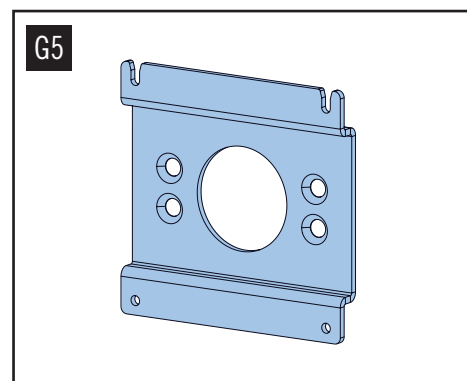
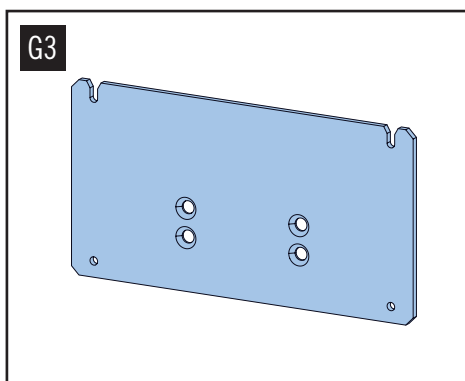
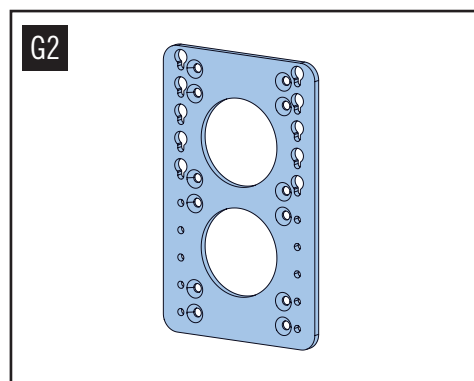
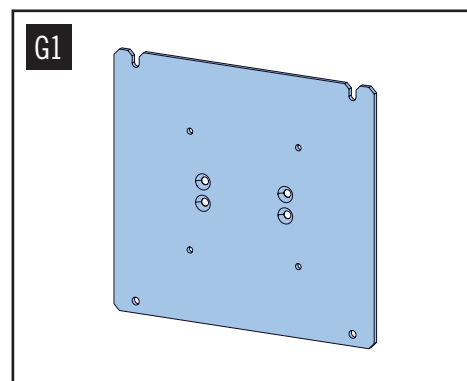
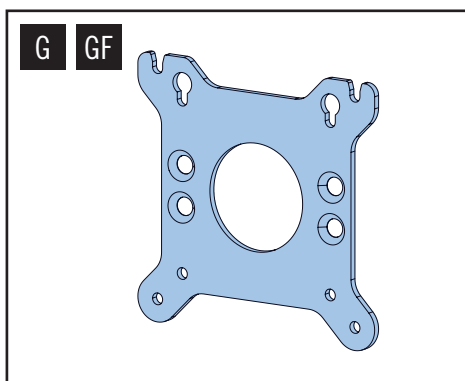
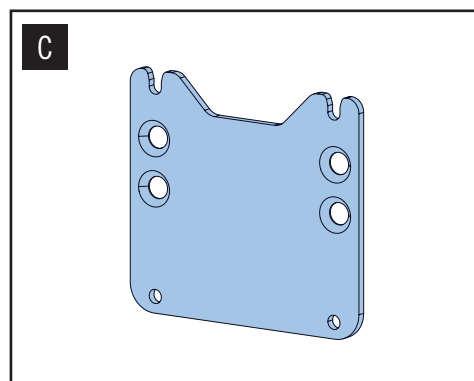
The additional instructions for use and the inspection plan are intended for end users.

SCOPE OF DELIVERY

(4x) Pan head screw M4x16
(4x) Pan head screw M4x12
(4x) Pan head screw M4x10
(4x) Nylon spacer

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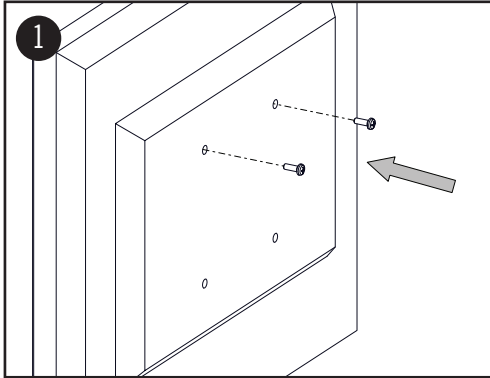
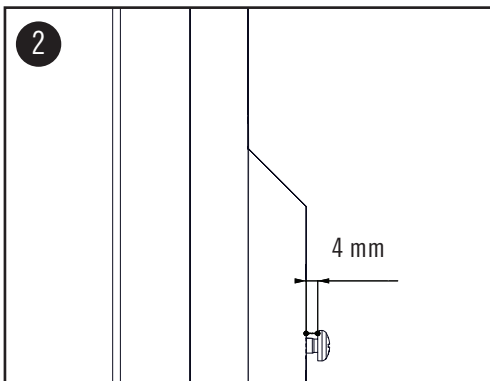
Screw the (2) M4 pan-head screws into the two upper threads on the back of a monitor.

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

NOTE:

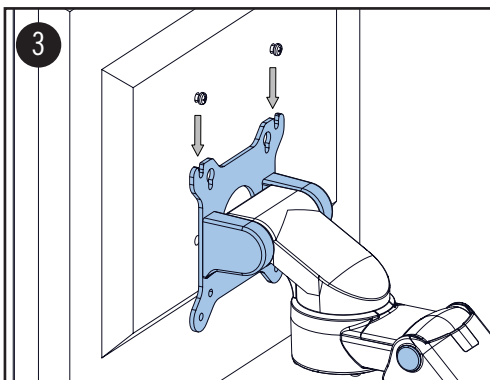For easier installation, do not tighten the screws, but leave a clearance of at least 4 mm.

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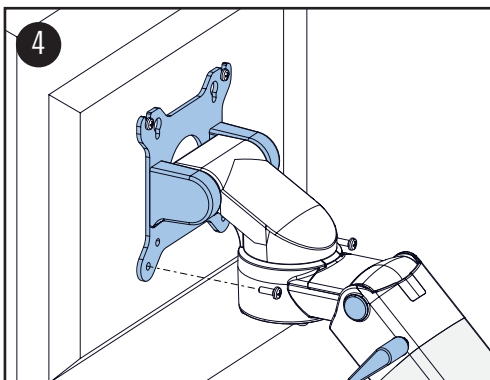Attach the monitor to the VESA plate from above using the installed (2) M4 pan-head screws.

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Mount the two lower M4 pan-head screws and tighten all (4) screws.