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**KOMPONENTENERDUNG AN TRAGARMEN**

## ASSEMBLY INSTRUCTION

**COMPONENTS' GROUNDING AT SUPPORT ARMS**

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**WARNING** 

Please check the weight of the ME equipment to be mounted. The permissible total weight is mentioned on the corresponding data sheet.

Failure to follow these warnings may result in damage to equipment or injury to people.

CIM med GmbH cannot be held responsible for any damage due to poor installation.

Product modifications, disassembling and reassembling and/or mounting any of the components in any other way than described below are permitted only after written approval by the CIM med GmbH construction and engineering department. CIM med GmbH cannot be held responsible for any damage or injury caused by product modifications and/or amendments without prior permission from CIM med GmbH.

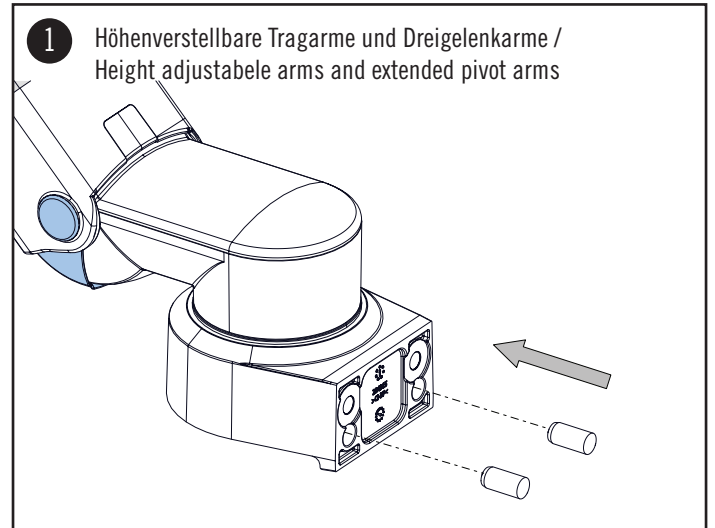
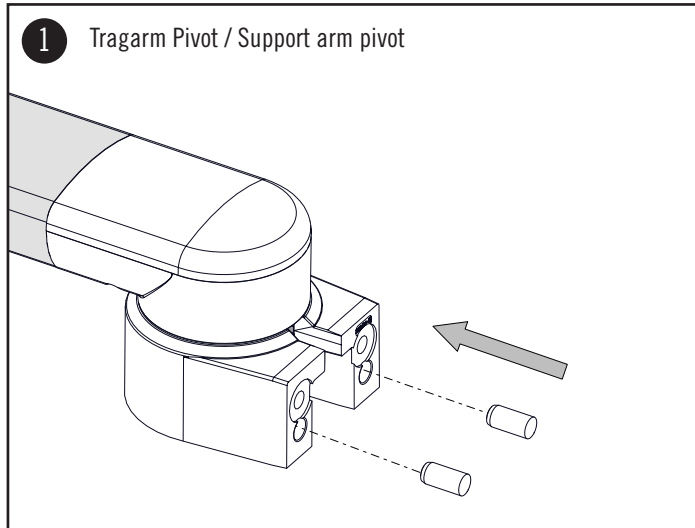
The additional instructions for use and the inspection plan are intended for end users.

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1. Push the Ø8x18 dowel pins into the bore holes.

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There are two ways to connect the support arm to the component grounding of your device:

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**OPTION A:**

Mounting of the support arm on a housing which is conductively connected to the grounding of the device.

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**OPTION B:**

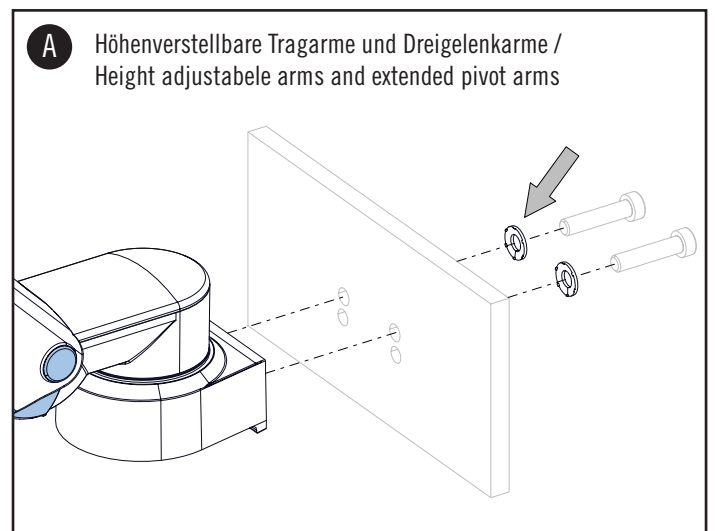
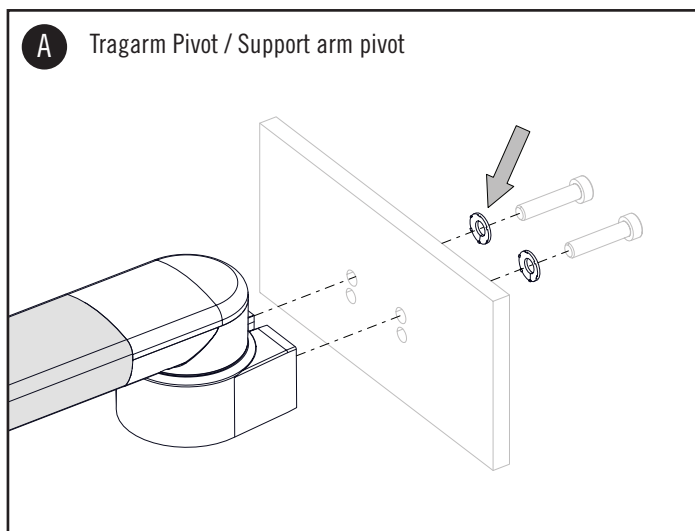
The support arm is mounted on a non-conductive housing.

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When mounting the support arm, use the enclosed contact washers. Torque moment 23 Nm

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
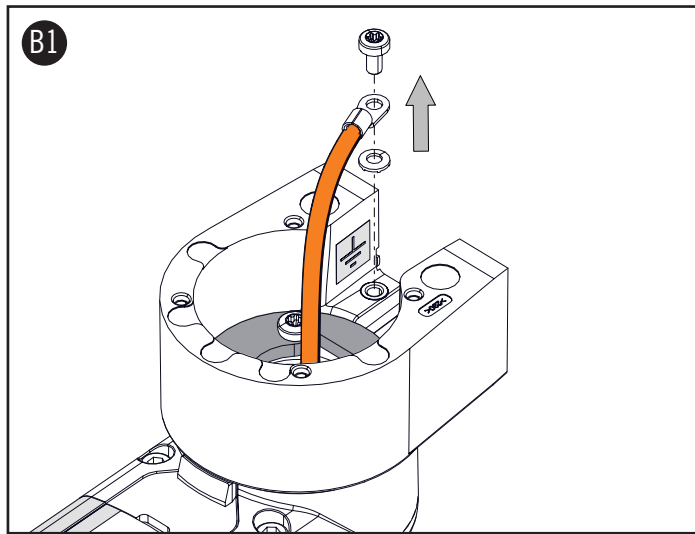
## ASSEMBLY STEPS OPTION B - SUPPORT ARM PIVOT


1. Loosen the M4x8 pan-head screw with which the earthing strap is fastened.
2. Use the enclosed cable lug to ground the component.

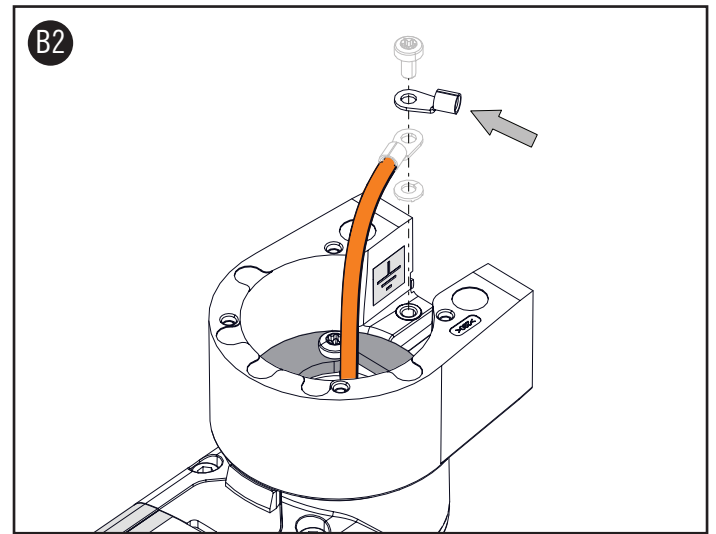**ⓘ NOTE:** Make sure to reconnect the component grounding of the support arm.

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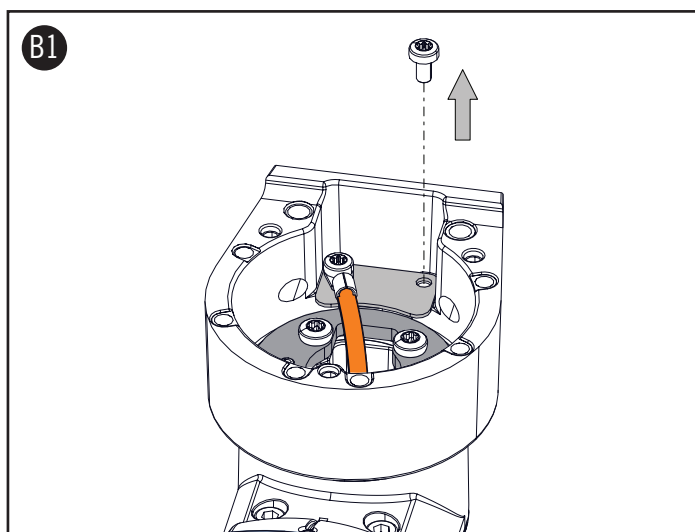## ASSEMBLY STEPS OPTION B - HEIGHT ADJUSTABLE ARMS AND EXTENDED PIVOT ARMS:

1. Loosen the M4x8 pan-head screw to which no grounding strap is attached.
2. Use the enclosed cable lug to ground the component.