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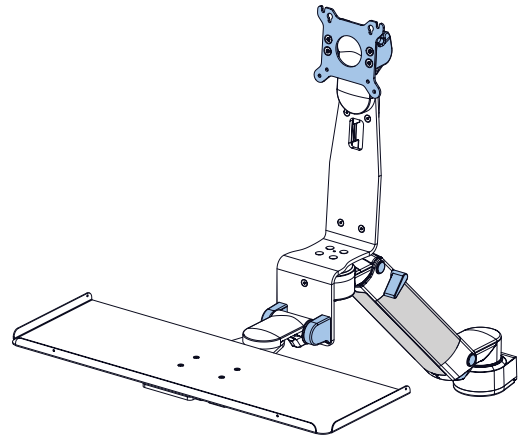
## ASSEMBLY INSTRUCTION

# MONITOR ADAPTATION Z-BRACKET

**WARNHINWEISE** 

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**WARNING** 

Please check the weight of the ME equipment to be mounted. The permissible total weight is mentioned on the corresponding data sheet.

Failure to follow these warnings may result in damage to equipment or injury to people.

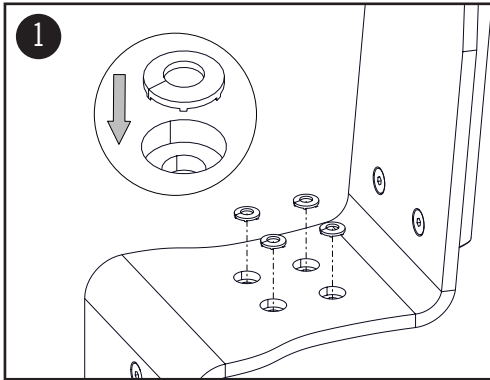
CIM med GmbH cannot be held responsible for any damage due to poor installation.

Product modifications, disassembling and reassembling and/or mounting any of the components in any other way than described below are permitted only after written approval by the CIM med GmbH construction and engineering department. CIM med GmbH cannot be held responsible for any damage or injury caused by product modifications and/ or amendments without prior permission from CIM med GmbH.

The additional instructions for use and the inspection plan are intended for end users.

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
Place the (4) contact wheels in the holes as shown.

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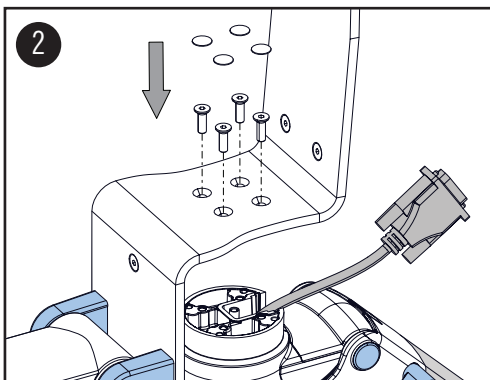**⚠ WARNING**

To guarantee a secure components grounding it is essential to use the (4) contact wheels.

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
Mount the z-bracket to the swivel head by using the (4) M5x16 counter-sunk screws. Attach (4) cover caps to the screws.

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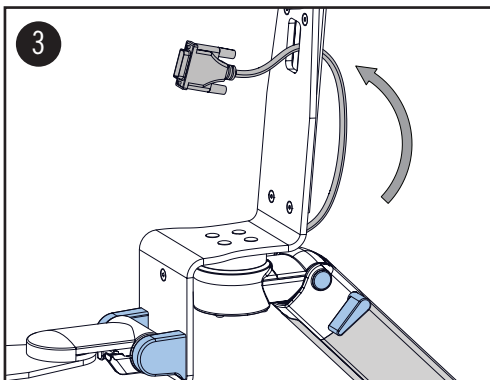
**⚠ WARNING**

Take care the screw head is completely counter-sunk.

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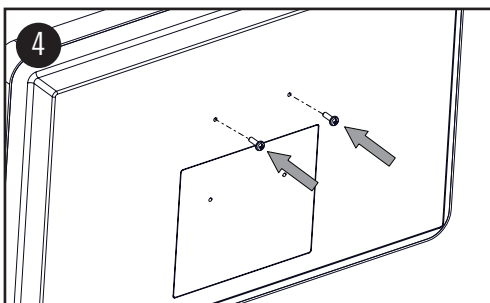
Feed the cables through the bracket from behind.

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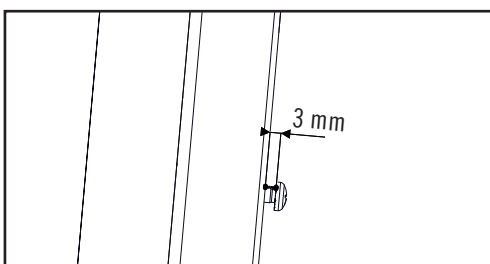
For easy mounting fix the two upper M4 pan head screws to the monitor rear side.

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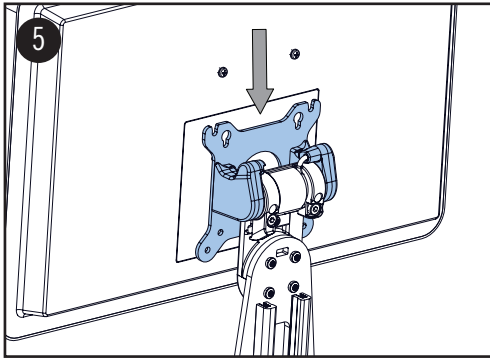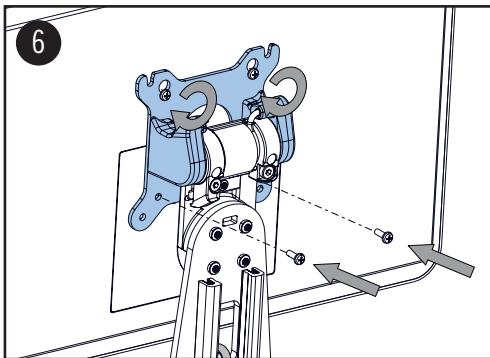
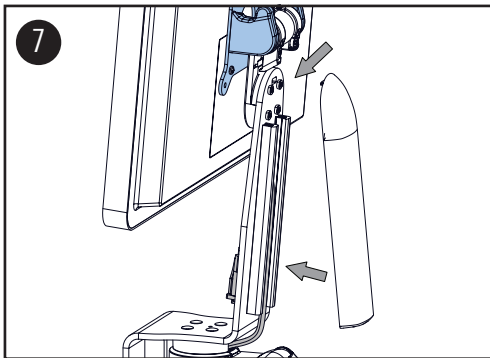
Fix the lower two screws to the monitor. Tighten the (4) screws.

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
Clip the plastic cover into the bracket.

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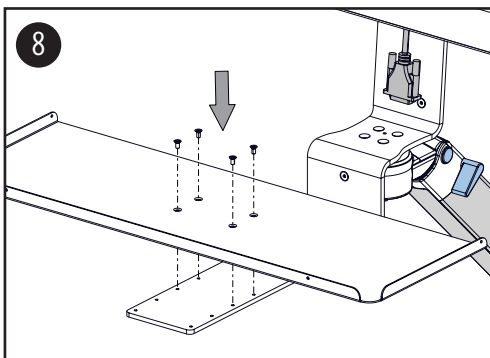
**⚠ WARNING**

Make sure that the cables are placed along the intended cable channel.

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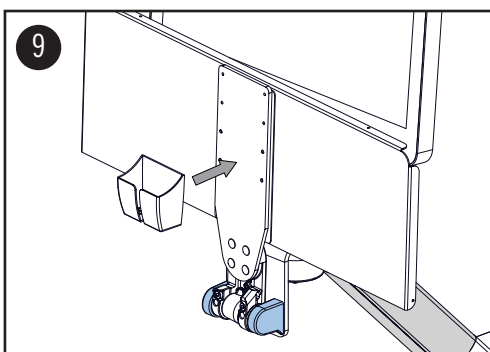Mount keyboard tray by using M4x8 countersunk screws.

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Mouse garage with velcro tape to be fixed at the lower side of the keyboard.

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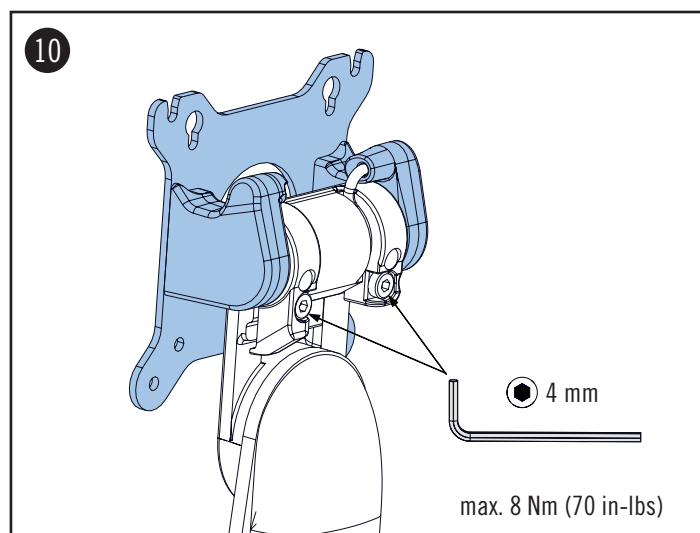
# ASSEMBLY STEPS

## TILT ADJUSTMENT

To adjust the overall tilt tension, tighten or loosen the both hex screws. By tightening both of the screws the tilt function can be locked completely. Maximum permissible tightening torque is 8 Nm (70 in-lbs).

**⚠ WARNING:** In case of overtightening the screw the casting respectively the threads may break which leads to loss of function.

**⚠ WARNING:** When loosening both hex screws the ME equipment will tilt forward.

**⚠ DANGER OF SQUEEZING FINGERS**