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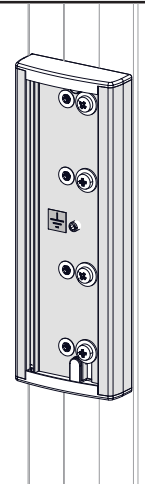
## ASSEMBLY INSTRUCTION

## SIDE CHANNEL ADAPTER FOR GE ANAESTHESIA MACHINES

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**WARNING** ⚠

Please check the weight of the ME equipment to be mounted. The permissible total weight is mentioned on the corresponding data sheet.

Failure to follow these warnings may result in damage to equipment or injury to people.

CIM med GmbH cannot be held responsible for any damage due to poor installation.

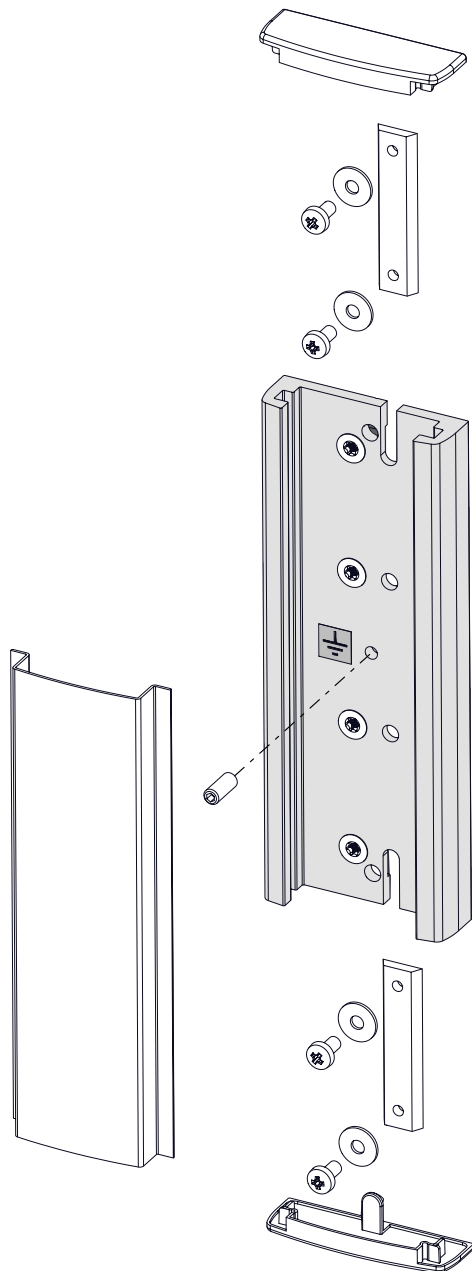
Product modifications, disassembling and reassembling and/or mounting any of the components in any other way than described below are permitted only after written approval by the CIM med GmbH construction and engineering department. CIM med GmbH cannot be held responsible for any damage or injury caused by product modifications and/ or amendments without prior permission from CIM med GmbH.

The additional instructions for use and the inspection plan are intended for end users.

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
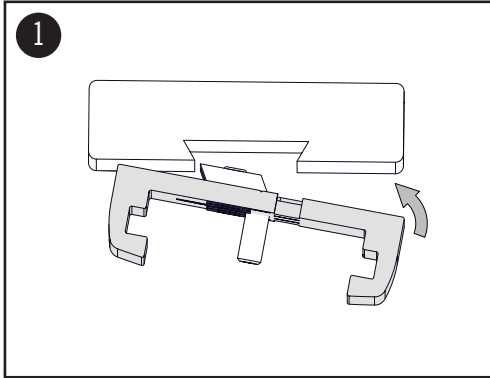
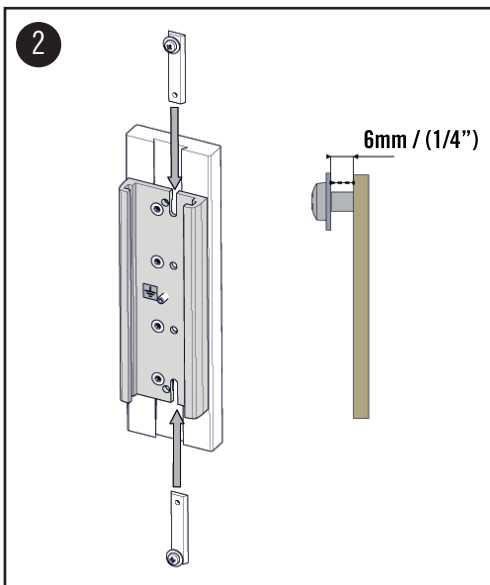
Insert the side adapter into GE dovetail rail as shown on a picture.

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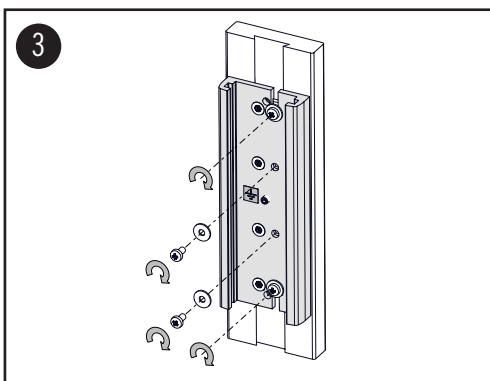
Position the trapezoidal bars with pre-assembled screws in GE dovetail rail, and then slide them into the side adapter. Pay attention to the correct positioning of the washers. Tighten two M5x10 screws slightly.

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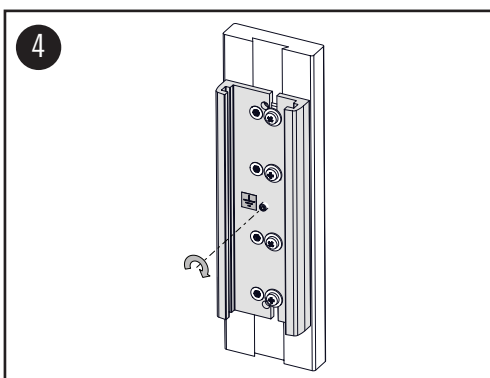Insert the remaining (2) M5x10 screws with the washers. Position the side adapter at the desired height and then tighten all (4) M5x10 screws.

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**⚠ CAUTION**  
The grounding screw must be tightened for a safe component grounding.

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Die CIM med GmbH übernimmt keine Haftung für Schäden, die durch nicht fachgerechte Montage entstanden sind.