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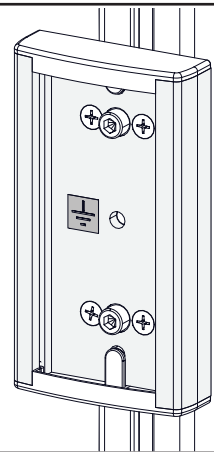
ASSEMBLY INSTRUCTION

ADAPTER FOR DRÄGER ANAESTHESIA MACHINES

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WARNING ⚠

Please check the weight of the ME equipment to be mounted. The permissible total weight is mentioned on the corresponding data sheet.

Failure to follow these warnings may result in damage to equipment or injury to people.

CIM med GmbH cannot be held responsible for any damage due to poor installation.

Product modifications, disassembling and reassembling and/or mounting any of the components in any other way than described below are permitted only after written approval by the CIM med GmbH construction and engineering department. CIM med GmbH cannot be held responsible for any damage or injury caused by product modifications and/ or amendments without prior permission from CIM med GmbH.

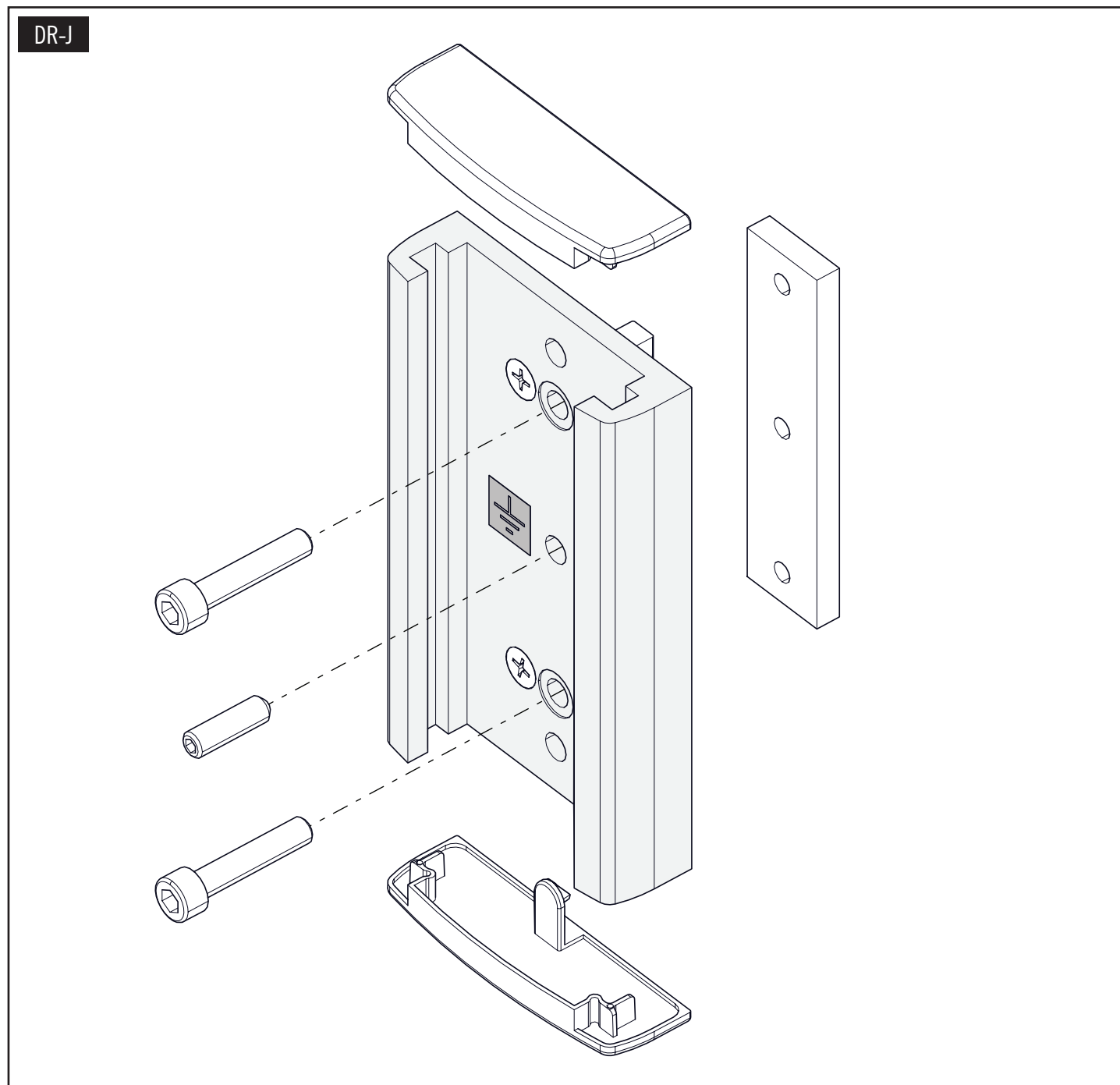
The additional instructions for use and the inspection plan are intended for end users.

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
ADAPTER
FOR DRÄGER ANAESTHESIA MACHINES

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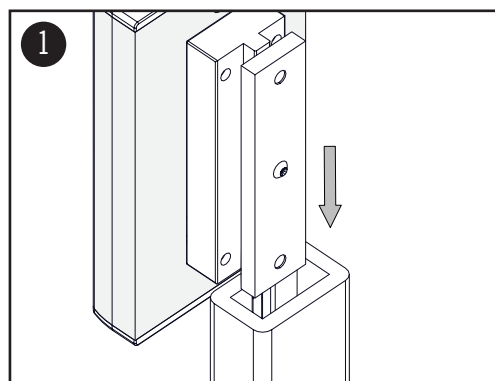
ASSEMBLY STEPS


Insert the adapter into Dräger rail as shown on a picture.

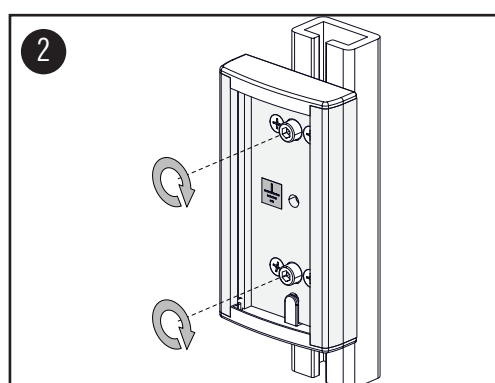NOTE: Loosen the screw connection of the T-nuts to allow threading.

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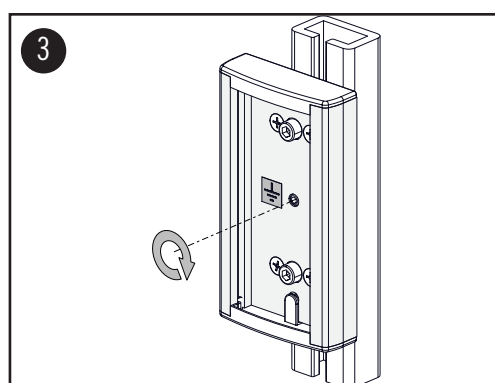Position the adapter at the desired height and then tighten all M6x30 cylinder head screws.

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Now tighten the M6x20 threaded pin.

WARNING: The grounding screw must be tightened for a safe component grounding.

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Die CIM med GmbH übernimmt keine Haftung für Schäden, die durch nicht fachgerechte Montage entstanden sind.