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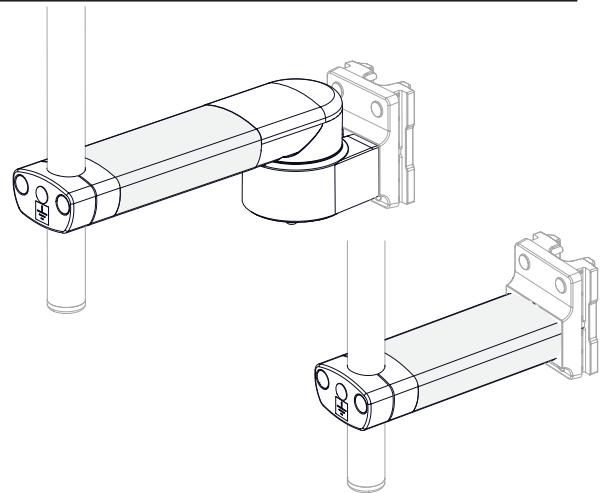
ASSEMBLY INSTRUCTION

SUPPORT ARM FOR INFUSION POLE**WARNHINWEISE** 

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WARNING 

Please check the weight of the ME equipment to be mounted. The permissible total weight is mentioned on the corresponding data sheet.

Failure to follow these warnings may result in damage to equipment or injury to people.

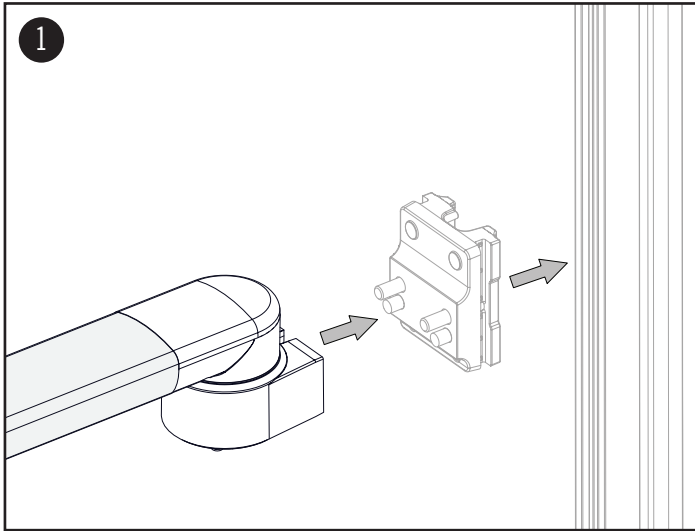
CIM med GmbH cannot be held responsible for any damage due to poor installation.

Product modifications, disassembling and reassembling and/or mounting any of the components in any other way than described below are permitted only after written approval by the CIM med GmbH construction and engineering department. CIM med GmbH cannot be held responsible for any damage or injury caused by product modifications and/or amendments without prior permission from CIM med GmbH.

The additional instructions for use and the inspection plan are intended for end users.

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
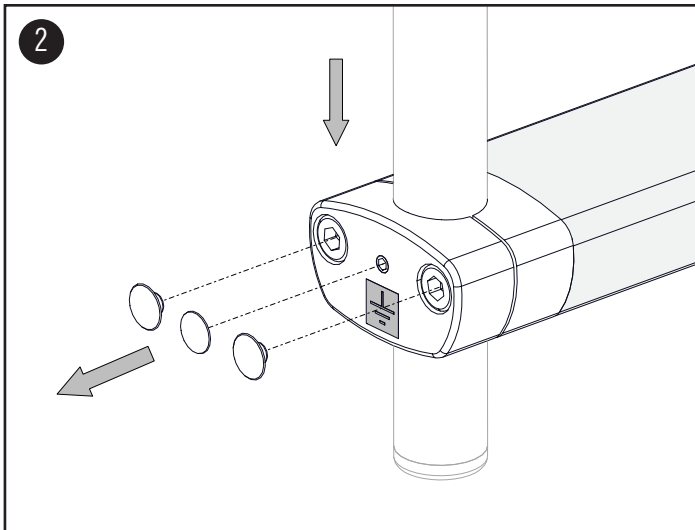
ASSEMBLY STEPS

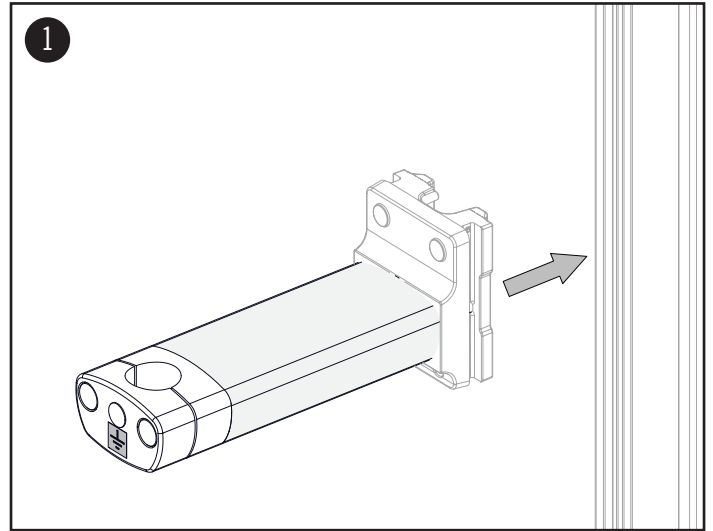1. Mount the support arm to its designated place (wall channel, tube, DIN rail etc.). The illustration shows a wall channel adapter. Please refer to the corresponding assembly instruction for all other adapters.

2. Remove the (3) cover caps and loosen the (2) M8x55 cylinder head screws. Insert the infusion pole.
3. First tighten the (2) M8x55 cylinder head screws evenly. Now tighten the (1) M6x8 threaded pin and cover all screws with the (3) cover caps.

⚠ WARNING: The use of threaded pin is absolutely necessary for a safe grounding of components.

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Die CIM med GmbH übernimmt keine Haftung für Schäden, die durch nicht fachgerechte Montage entstanden sind.