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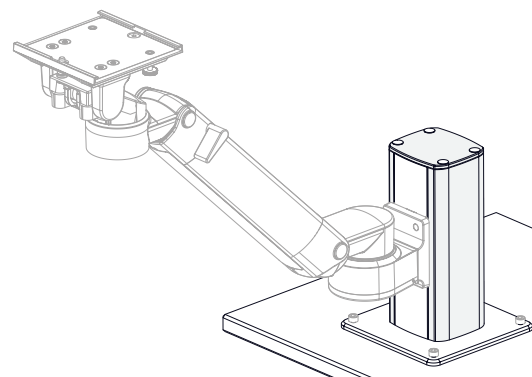
## ASSEMBLY INSTRUCTION

# TABLE COLUMN / FLOOR COLUMN

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**PLEASE READ THIS INSTRUCTION CAREFULLY BEFORE INSTALLING!**

Clarification of non-standardized Symbols:

 Important information

 Warning of a danger point

**AREA OF APPLICATION/ INTENDED USE**

Cable Integrated Mounting systems (CIM) are intended to mount and position monitors, mouse/keyboard, laptops and other medical devices or equipment (hereafter referred to as ME equipment). They are mounted to walls, carts, tables, other surfaces or machines using specific mounting adapters. CIM systems do not fulfil any specific medical purpose on their own.

**GENERAL NOTES**

It shall be the responsibility of the hospital, its consultants or contractors to determine that the table/floor is suitable to safely and securely mount the table and/or floor column. This includes the selection of the suitable fasteners and the proper installation of the same. Please check the weight of the ME equipment to be mounted. The permissible total weight is mentioned on the corresponding data sheet.

**WARNING Installation, Operation and Maintenance** 

This section contains warnings regarding the installation, operation and maintenance. This section must be read in its entirety before installing, operating and maintaining the table and/or floor column. Failure to follow these warnings may result in damage to equipment or injury to people.

Installation, operation and maintenance may only be carried out by qualified personnel. Assembly may only be done by qualified technical personnel. CIM med GmbH cannot be held responsible for any damage due to poor installation.

Product modifications, disassembling and reassembling and/or mounting any of the components in any other way than described below are permitted only after written approval by the CIM med GmbH construction and engineering department. CIM med GmbH cannot be held responsible for any damage or injury caused by product modifications and/ or amendments without prior permission by CIM med GmbH.

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CIM med GmbH  
Margot-Kalinke-Str. 9  
80939 München  
Deutschland

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## CLEANING

The table and/or floor column may be cleaned with mild, non-abrasive solutions commonly used in hospital environment. Do not use steel wool or other abrasive materials. Avoid all liquids entering the column. Your table and/or floor column has been designed to the highest specifications to make cleaning as easy as possible and to avoid liquids from penetrating. CIM med GmbH makes no claims concerning the efficacy of the disinfection processes. It is recommended to test any cleaning solution on a small area of the column not visible to verify compatibility. Please consult your hospital's infection control manager.

Exemplary disinfection tests have been carried out with the following active substance groups: alcoholic quick disinfectants (e.g. Bacillol AF/Hartmann), aldehyde and quaternary ammonium compounds (e.g. B5/Orochemie), alcohol-free quaternary ammonium compounds (e.g. Sani-Cloth AF3/PDI Healthcare). These compounds are VAH / DGHM listed and EPA registered and must be used as per the manufacturers' instructions. The surface finish will be permanently damaged by acetone and trichloroethylene.

## MAINTENANCE: MANNER AND FREQUENCY

Very little maintenance is required during the life span. Please check all screws on a regular basis (at least once a year) and tighten them if necessary.

In case of damage done to the load-bearing parts of the system it has to be strictly refrained from further usage of the mounting system.

## REPORTING OBLIGATION

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

## SAFEKEEPING

For any questions that may arise at a later time please safekeep our manual!

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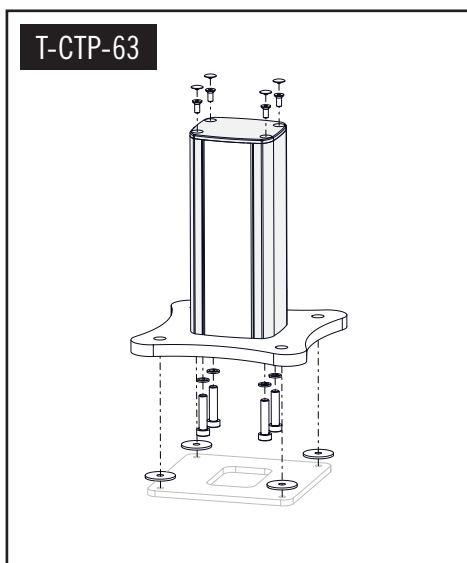
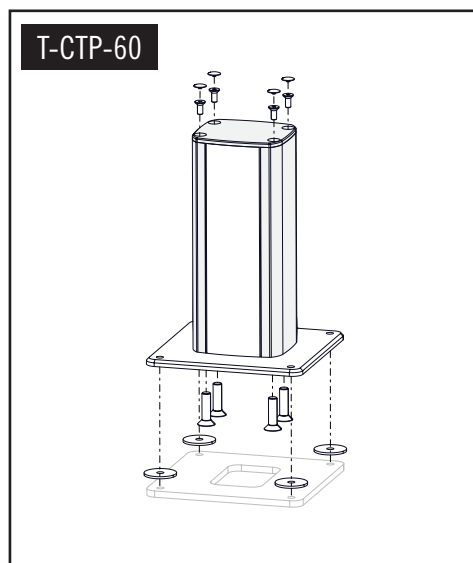
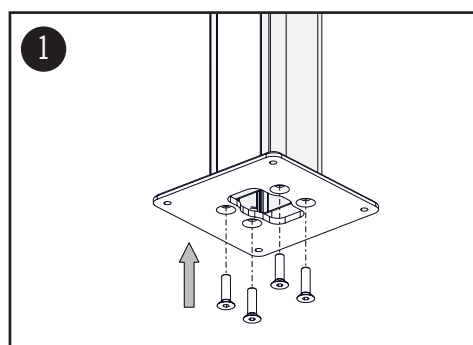
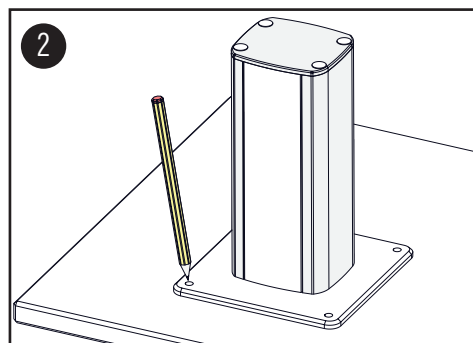
Mount the table/floor column with (4) M8 x 30 countersunk head screws respectively M8 x 35 cylinder head screws and detent edged washers on the base plate.

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
Position the column and mark the required drilling positions.

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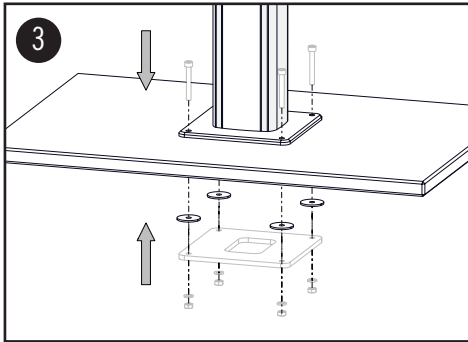
Mount the table/floor column using suitable assembly material and attached washers. Make sure to firmly tighten all screws.

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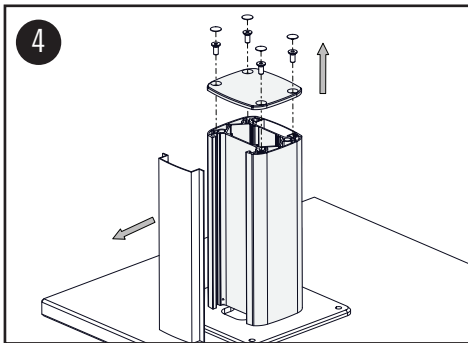
**NOTE:**

Depending on the case, it is recommended to use the optional counter plate.

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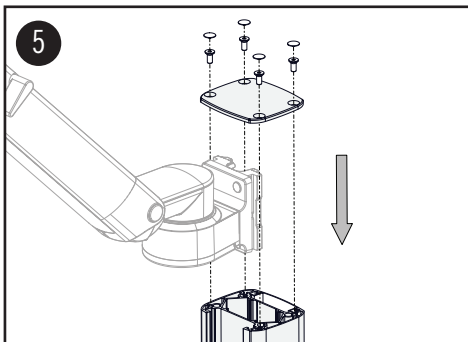
Loosen the (4) M5 x 12 countersunk head screws and remove the top cover and plastic cover.

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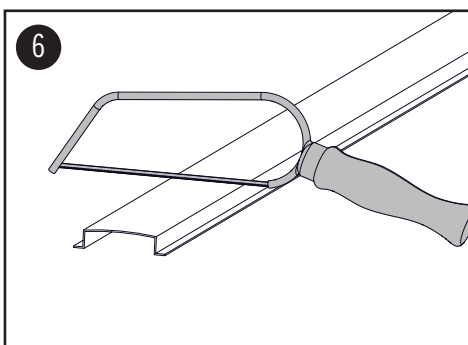
Push the support arm into table/floor column from above and mount it at the favored height. Please refer to the corresponding assembly instruction of the support arm.

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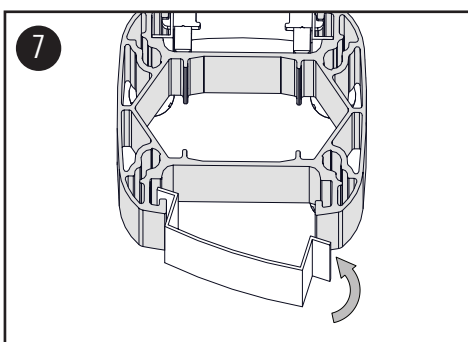Determine the required length of the plastic covers and cut them to size.

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Mount the plastic covers into the column as shown.