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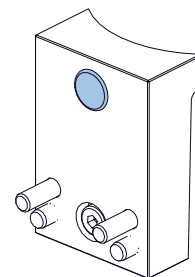
**PLEASE READ THIS INSTRUCTION CAREFULLY BEFORE INSTALLING!**

Clarification of non-standardized Symbols:

Warning of a danger point

**AREA OF APPLICATION/ INTENDED USE**

Cable Integrated Mounting (CIM) systems are intended to mount and position monitors, mouse/keyboard, laptops and other medical devices or equipment (hereafter referred to as ME equipment).

The adapter for Getinge anaesthesia machines is exclusively to be used in conjunction with CIM support arms in the medical sector. CIM systems do not fulfil any specific medical purpose on their own.

**GENERAL NOTES**

Before you want to attach the adapter for Getinge anaesthesia machines, please verify that the column is mounted correctly and can hold the weight. Please check the weight of the ME equipment to be mounted. The permissible total weight is mentioned on the corresponding data sheet.

**WARNING Installation, Operation and Maintenance** 

This section contains warnings regarding the installation, operation and maintenance of the adapter for Getinge anaesthesia machines. This section must be read in its entirety before installing, operating and maintaining the adapter. Failure to follow these warnings may result in damage to equipment or injury to people.

Installation, operation and maintenance may only be carried out by qualified personnel. Assembly may only be done by qualified technical personnel. CIM med GmbH cannot be held responsible for any damage due to poor installation.

Product modifications, disassembling and reassembling and/or mounting any of the components in any other way than described below are permitted only after written approval by the CIM med GmbH construction and engineering department. CIM med GmbH cannot be held responsible for any damage or injury caused by product modifications and/ or amendments without prior permission by CIM med GmbH.

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# GENERAL NOTES

**⚠ WARNING: EARTHING CONNECTION**

The adapter for Getinge anaesthesia machines is connected to the column and the mounting arm via the screws of the adapter.

**TOOLS REQUIRED**

Torx wrench T45  
Hex key 6.0

**CLEANING**

The adapter may be cleaned with mild, non-abrasive solutions commonly used in hospital environment. Do not use steel wool or other abrasive materials. The product has been designed to the highest specifications to make cleaning as easy as possible. CIM med makes no claims concerning the efficacy of the disinfection processes.

It is recommended to test any cleaning solution on a small area not visible to verify compatibility. Please consult your hospital's infection control manager.

**MAINTENANCE: MANNER AND FREQUENCY**

Very little maintenance is required during the life span. Please check all screws on a regular basis (at least once a year) and tighten them if necessary.

In case of damage done to the load-bearing parts of the system it has to be strictly refrained from further usage of the mounting system.

**REPORTING OBLIGATION**

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

**SAFEKEEPING**

For any questions that may arise at a later time please safekeep our manual!

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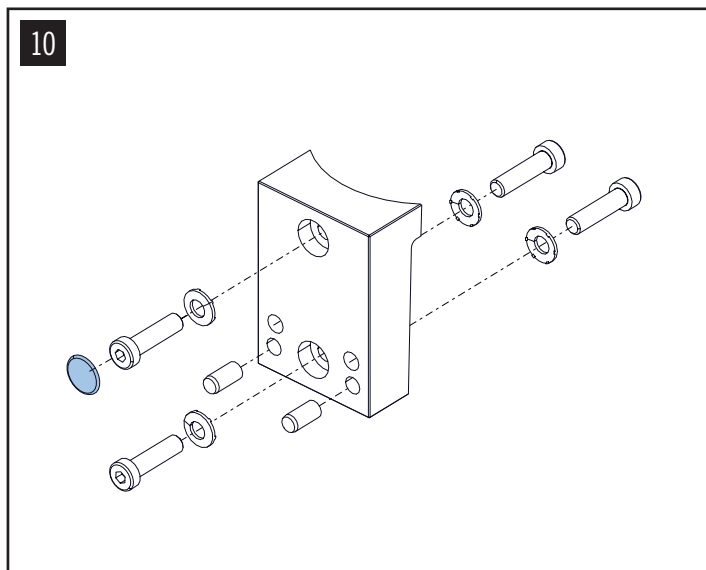
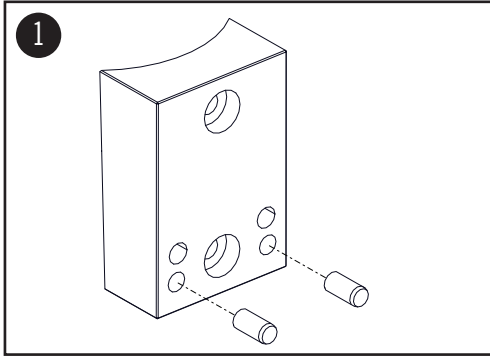
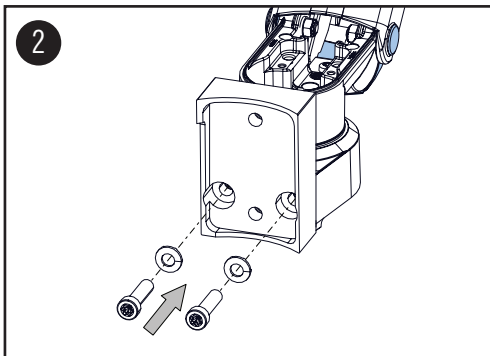
Insert (2) cylinder pins into the lower holes.

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
Mount support arm with (2) M8x30 cylinder head screws (Torx) and (2) contact washers.  
Torque moment: 25 Nm (221 in-lbs)

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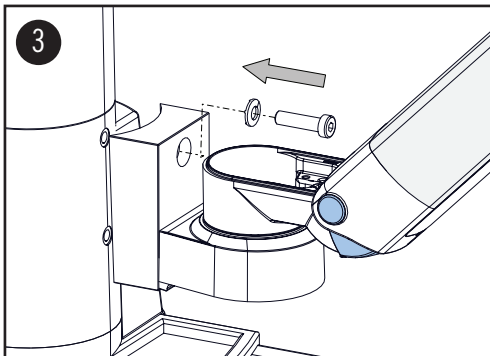**⚠ WARNING:** To guarantee a secure components grounding it is essential to use the both contact washers.

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

Position the adapter on the column and tighten the (2) cylinder head screws and (2) contact washers.

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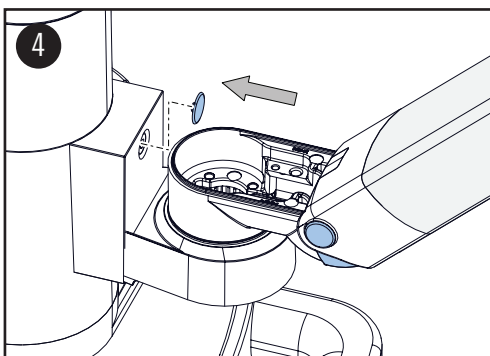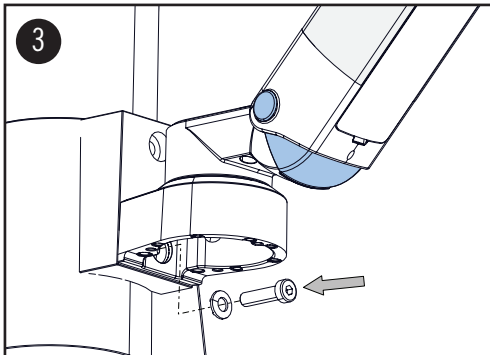Cover the upper screw with the cover cap.