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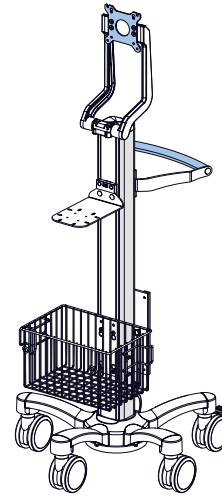
## ASSEMBLY INSTRUCTION

# CART LIGHT DUTY FOR GE - B850

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Name: CIM cart light duty  
Part no. code of the cart: GE-002B850

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**WARNING** 

Please check the weight of the ME equipment to be mounted. The permissible total weight is mentioned on the corresponding data sheet.

Failure to follow these warnings may result in damage to equipment or injury to people.

CIM med GmbH cannot be held responsible for any damage due to poor installation.

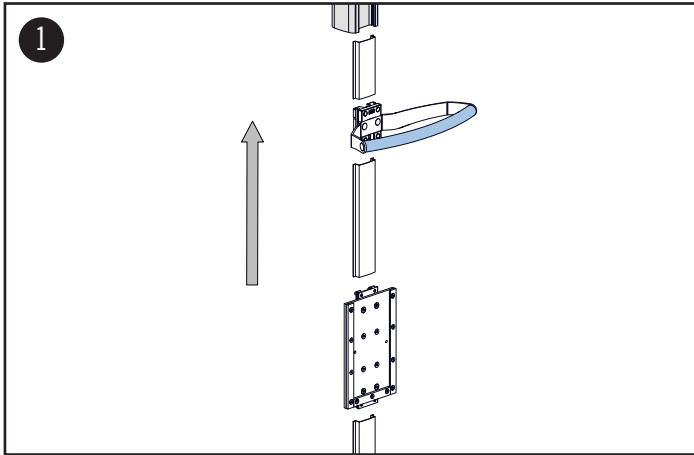
Product modifications, disassembling and reassembling and/or mounting any of the components in any other way than described below are permitted only after written approval by the CIM med GmbH construction and engineering department. CIM med GmbH cannot be held responsible for any damage or injury caused by product modifications and/or amendments without prior permission from CIM med GmbH.

The additional instructions for use and the inspection plan are intended for end users.

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

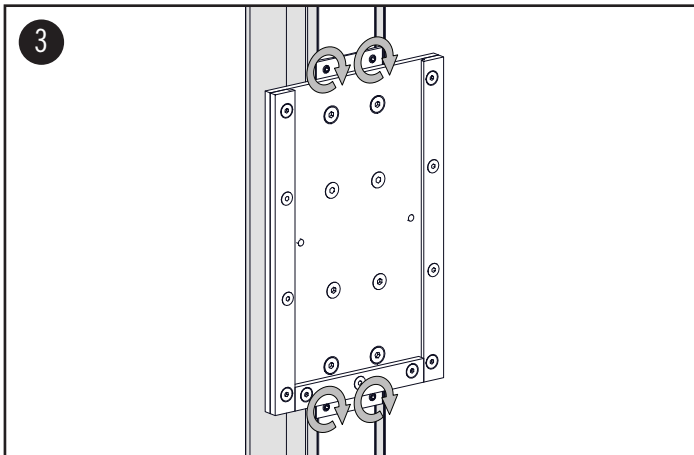
# ASSEMBLY STEPS

## ASSEMBLY STEPS

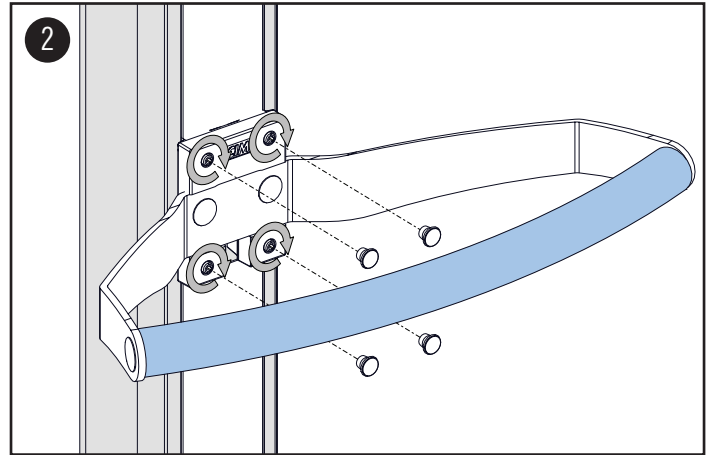1. Push the plastic covers, the handle and the CPU bracket on the designated side into the column.
2. Mount the handle using the (4) M6x20 threaded pins. Attach the (4) cover caps.

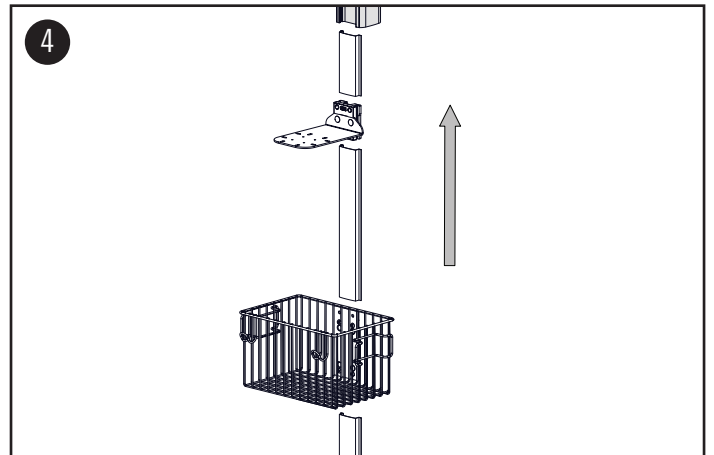3. Mount the CPU bracket using the (4) M6x20 threaded pins.
4. Push the plastic covers, bracket for GE frame F5 and the basket on the designated side into the column.

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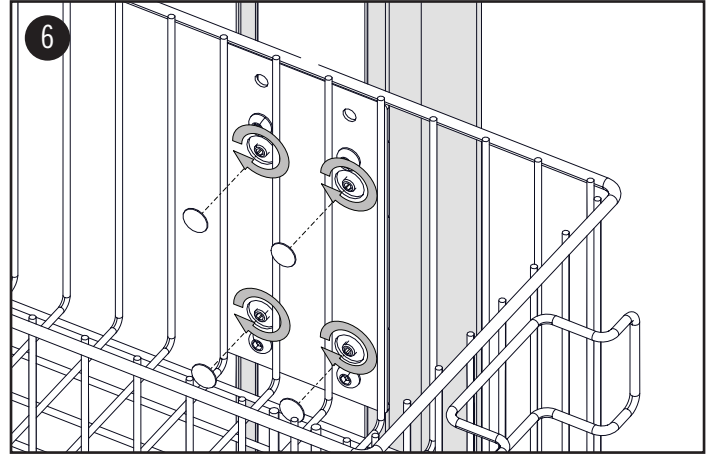
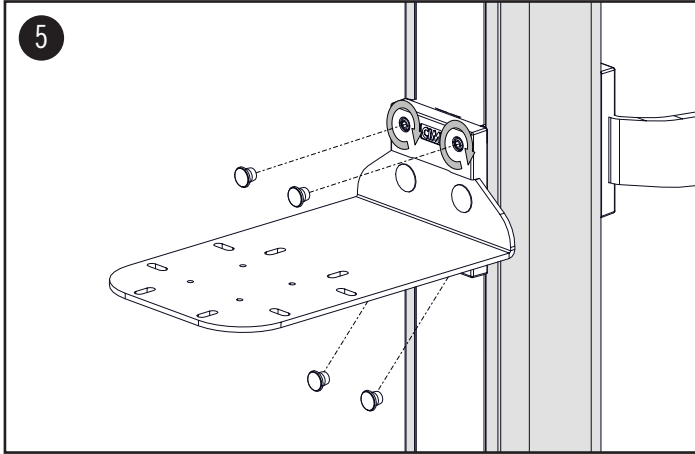
5. Mount the bracket for GE frame F5 using the (4) M6x20 threaded pins. Attach the (4) cover caps.
6. Mount the basket using the (4) M6x20 threaded pins. Attach the (4) cover caps.

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

7. Mount the column to the base with (4) M8x35 cylinder head bolts and (4) contact washers. Tighten the screws securely.
8. Mount the counterweight with the (5) M5x20 pan head screws and the (5) detent edged washers by tightening the screws crosswise.

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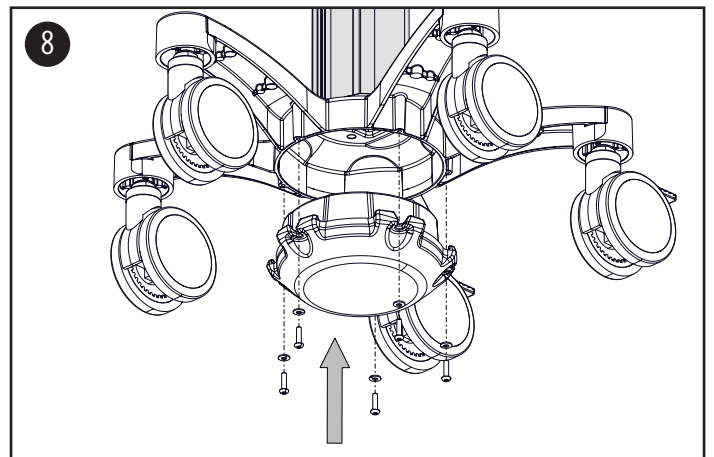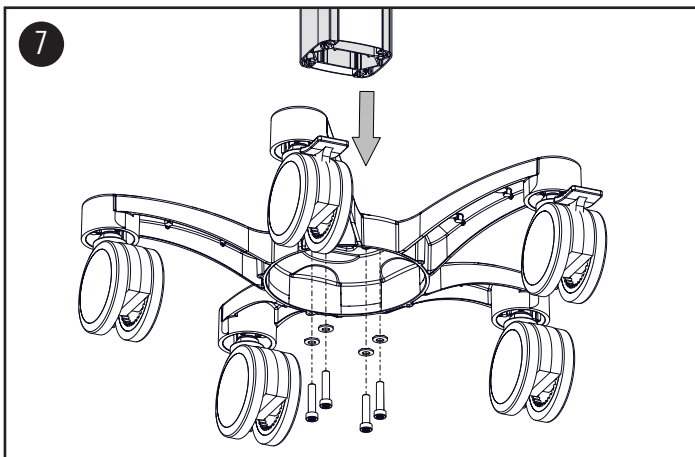
**⚠ WARNING:** To guarantee a secure screw connection, fasten all screws with a tightening torque of 8 Nm (70 in-lbs).

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
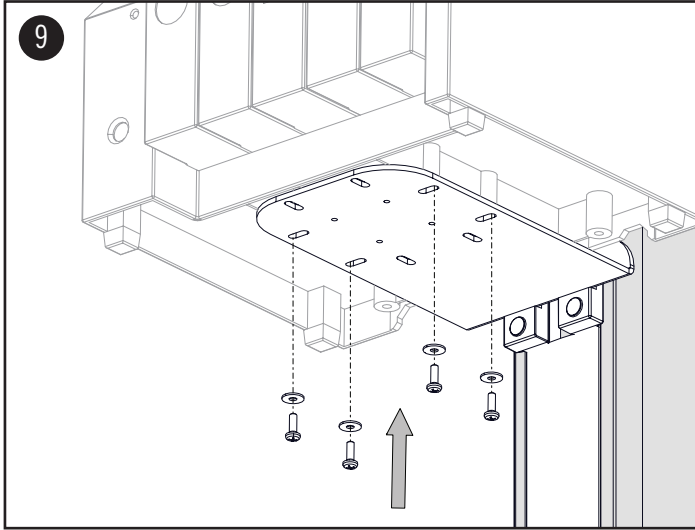
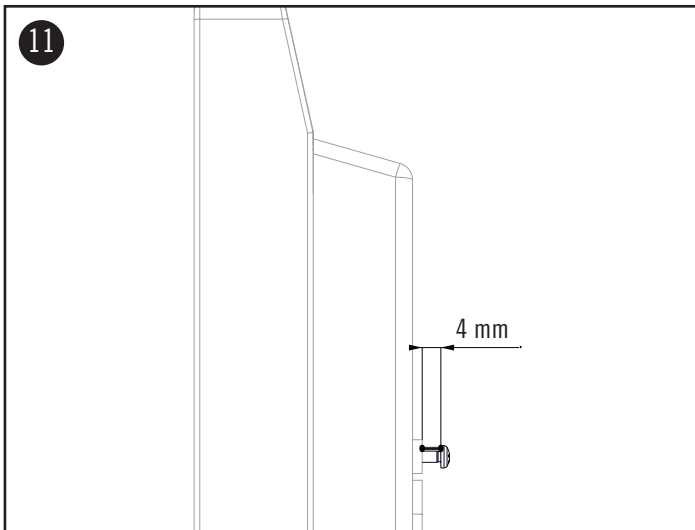
## ASSEMBLY STEPS

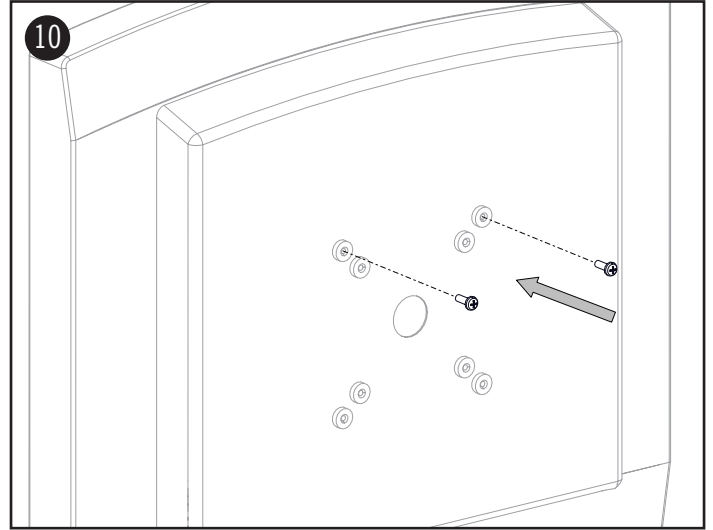9. Mount the ME equipment using the (4) washers and the (4) M4x12 pan head screws.
10. Screw the (2) M4 pan-head screws into the two upper threads on the back of a monitor.

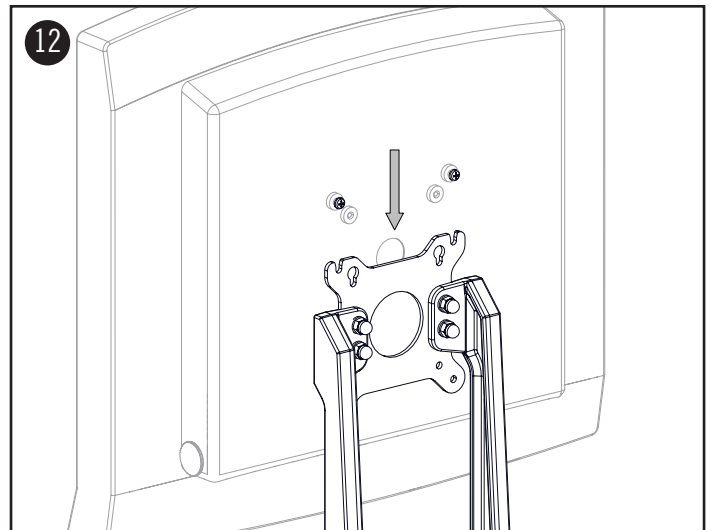
11. **NOTE:** For easier installation, do not tighten the screws, but leave a clearance of at least 4 mm.
12. Attach the monitor to the VESA plate from above using the installed (2) M4 pan-head screws.

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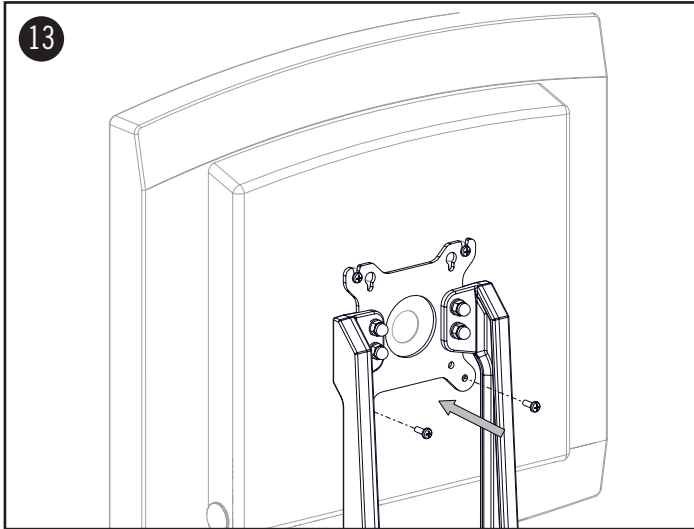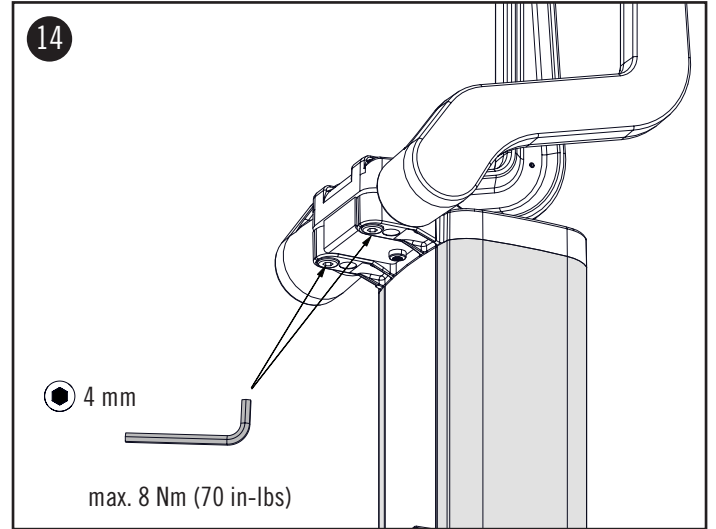
13. Mount the two lower M4 pan-head screws and tighten all (4) screws.

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### TILT ADJUSTMENT

14. To adjust the overall tilt tension, tighten or loosen the both hex screws. By tightening both of the screws the tilt function can be locked completely. Maximum permissible tightening torque is 8 Nm (70 in-lbs).

**⚠ WARNING:** In case of overtightening the screw the casting respectively the threads may break which leads to loss of function.

**⚠ WARNING:** When loosening both hex screws the ME equipment will tilt forward.

**⚠ DANGER OF SQUEEZING FINGERS**