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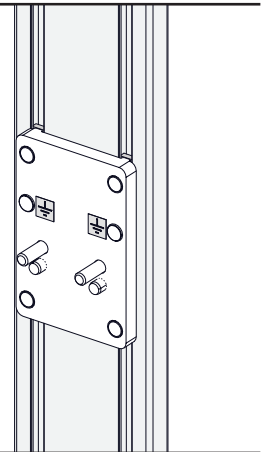
ASSEMBLY INSTRUCTION

ADAPTER FOR LÖWENSTEIN MEDICAL ANAESTHESIA MACHINES

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WARNING ⚠

Please check the weight of the ME equipment to be mounted. The permissible total weight is mentioned on the corresponding data sheet.

Failure to follow these warnings may result in damage to equipment or injury to people.

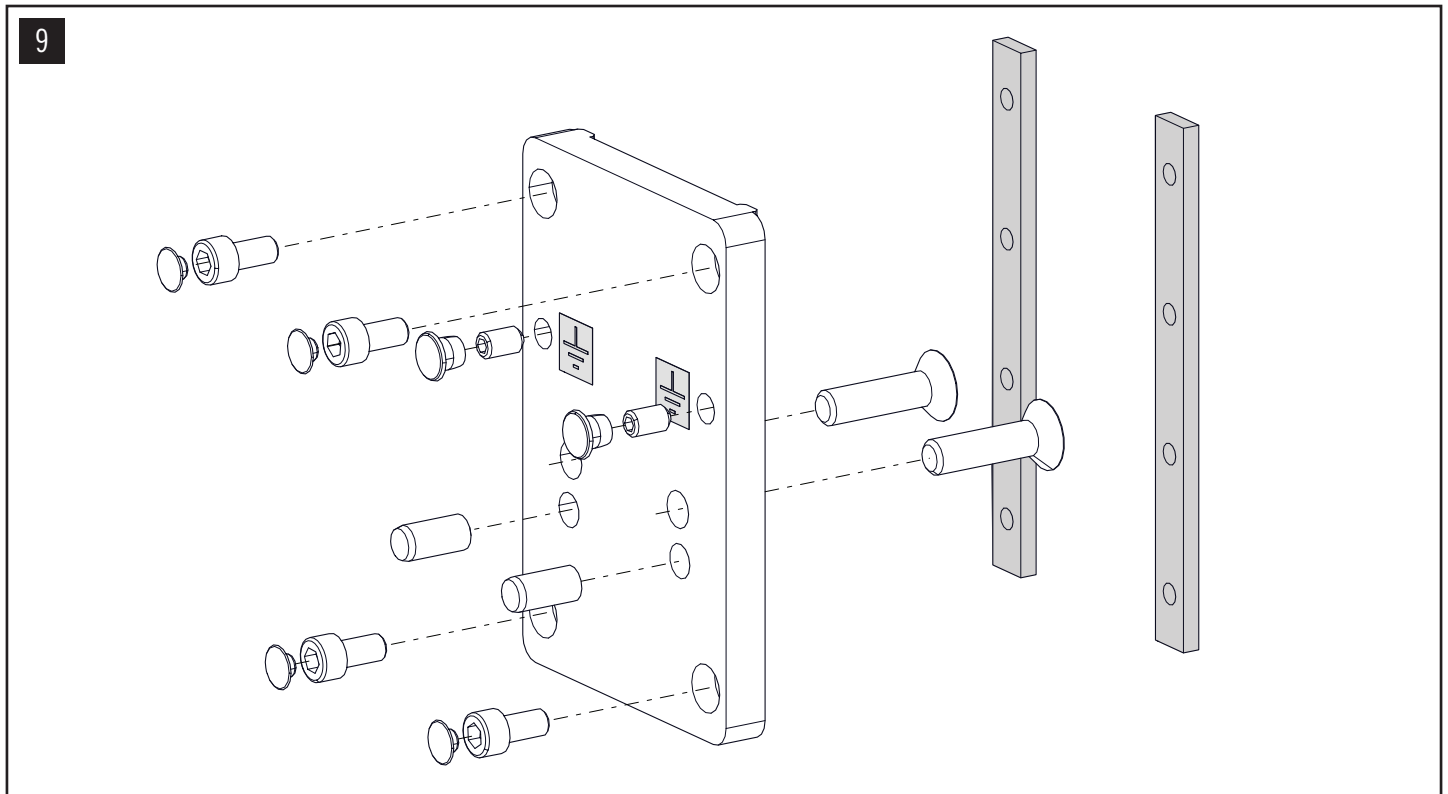
CIM med GmbH cannot be held responsible for any damage due to poor installation.

Product modifications, disassembling and reassembling and/or mounting any of the components in any other way than described below are permitted only after written approval by the CIM med GmbH construction and engineering department. CIM med GmbH cannot be held responsible for any damage or injury caused by product modifications and/ or amendments without prior permission from CIM med GmbH.

The additional instructions for use and the inspection plan are intended for end users.

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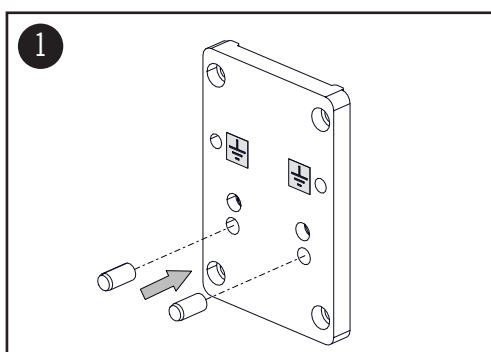
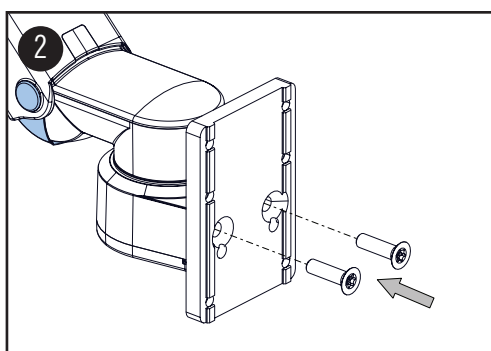
Insert (2) cylinder pins into the lower holes.

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
Mount support arm with (2) M8x30 countersunk head screws (Torx).

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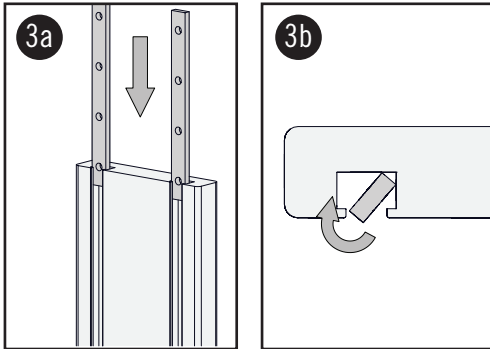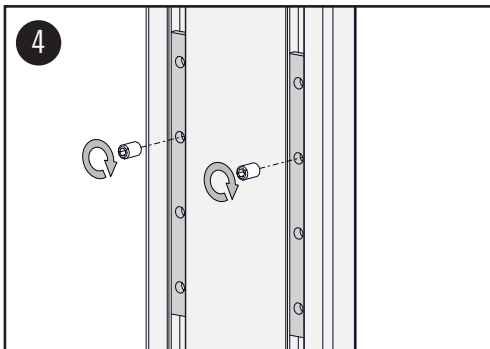
To mount the threaded strips, you can either slide them into the grooves from above or thread them in from the front as shown.

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
Lightly tighten (2) grounding screws for pre-assembly.

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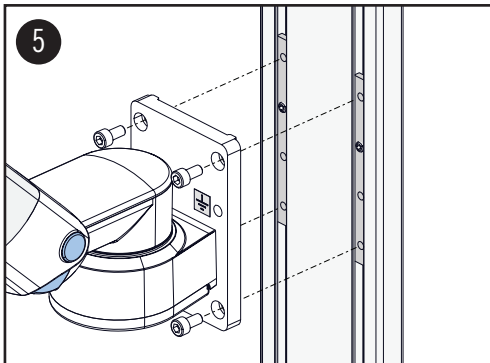⚠ WARNING:
The two threaded strips must be aligned vertically at the same height!

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
Place adapter on the threaded strip and mount it with (4) M6x12 cylinder head screws.

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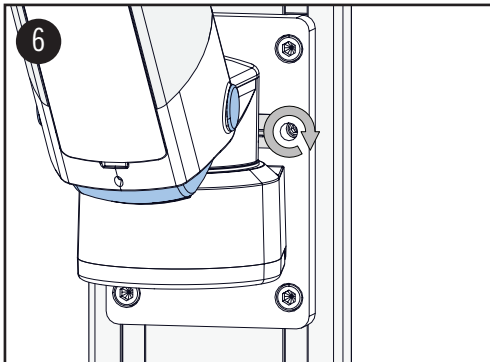⚠ WARNING:
The pre-assembled support arm may only be mounted without payload.

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
Tighten the (2) M6x20 grounding screws for final assembly.

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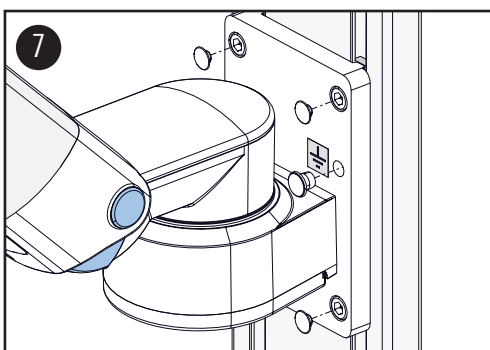⚠ WARNING:
The grounding screw must be tightened for a safe component grounding.

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