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**WARNING** 

Please check the weight of the ME equipment to be mounted. The permissible total weight is mentioned on the corresponding data sheet.

Failure to follow these warnings may result in damage to equipment or injury to people.

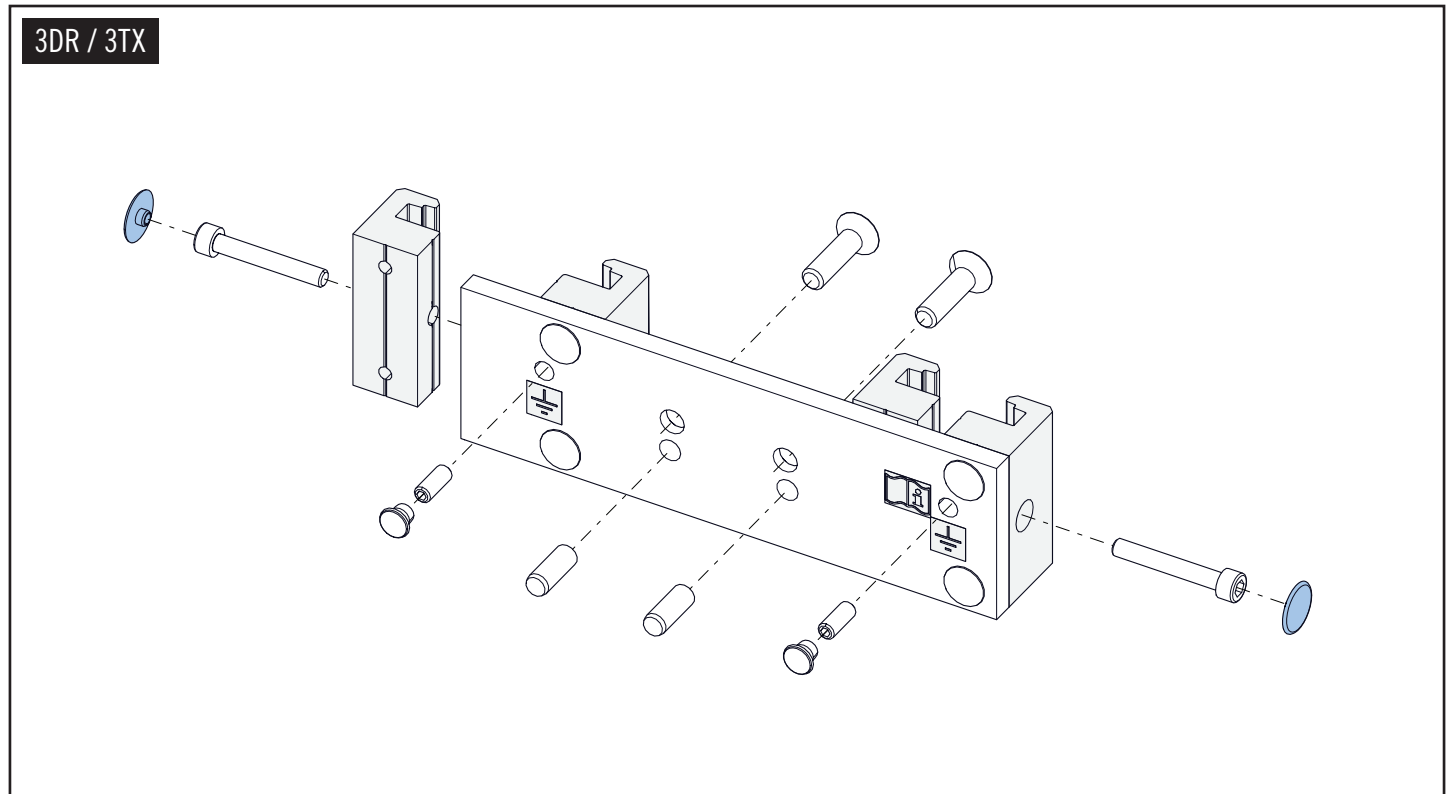
CIM med GmbH cannot be held responsible for any damage due to poor installation.

Product modifications, disassembling and reassembling and/or mounting any of the components in any other way than described below are permitted only after written approval by the CIM med GmbH construction and engineering department. CIM med GmbH cannot be held responsible for any damage or injury caused by product modifications and/or amendments without prior permission from CIM med GmbH.

The additional instructions for use and the inspection plan are intended for end users.

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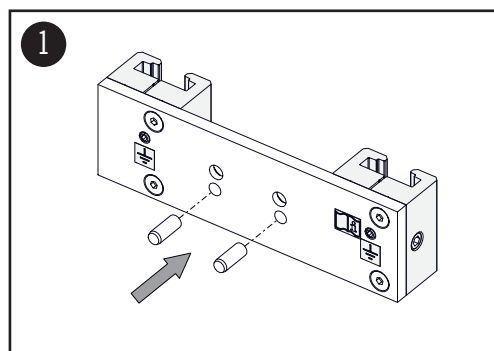
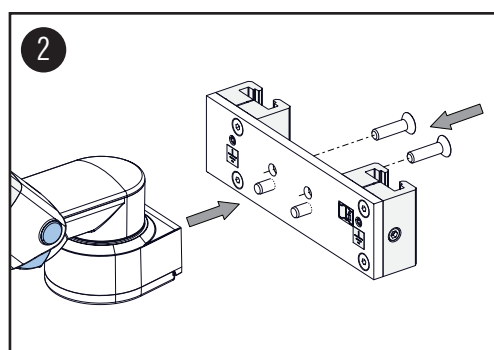
Insert (2) cylinder pins into the lower holes.

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
Mount support arm with (2) M8x30 countersunk head screws.

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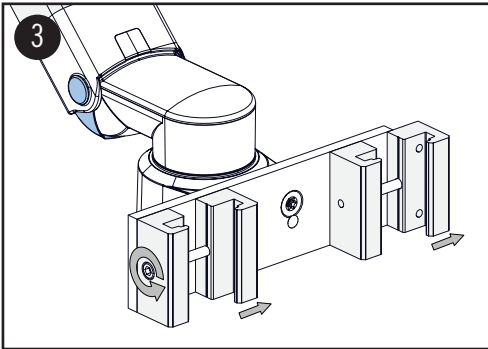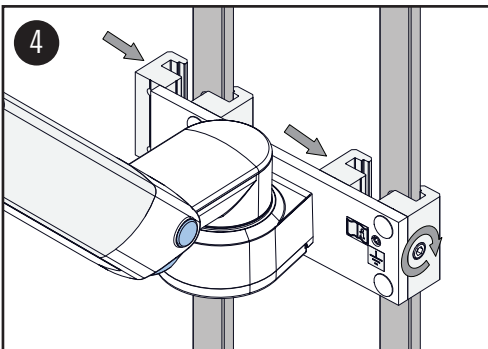
Loosen the (2) M6x40 cylinder head screws and place the adapter onto the two vertical rails.

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
Mount the rail clamp using (2) M6x40 screws.

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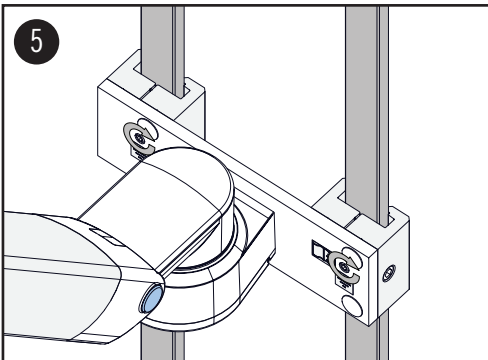**⚠ WARNING:**  
The pre-assembled support arm may only be mounted without payload.

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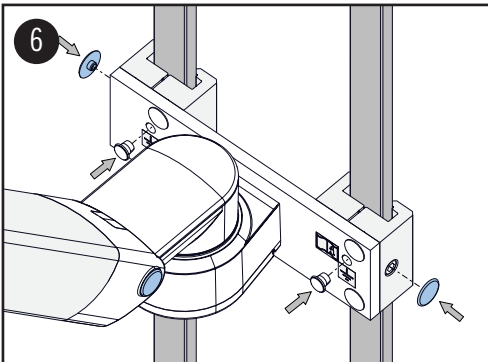
**⚠ WARNING:**  
The grounding screw must be tightened for a safe component grounding.

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