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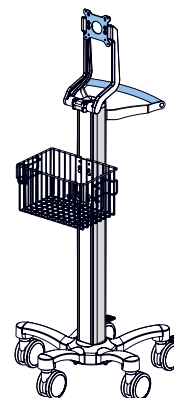
ROLLSTÄNDER LIGHT DUTY

ASSEMBLY INSTRUCTION

CART LIGHT DUTY

Name: CIM Rollständer Light Duty

Artikelnummern-Code des Rollständers: R-002, ACU-002, AN-002, CHM-002, CO-002, DIC-002, DR-002, FK-002, GE-002, HE-002, ID-002, MD-002, PH-002, P-R-002, SU-002, TRI-002, XI-002

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Part no. code of the monitor adaptation: A, C, FK, G, G1, GE, K, KQ, MD, NS, PH, Q, SLX, TN, X2X3

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WARNING 

Please check the weight of the ME equipment to be mounted. The permissible total weight is mentioned on the corresponding data sheet.

Failure to follow these warnings may result in damage to equipment or injury to people.

CIM med GmbH cannot be held responsible for any damage due to poor installation.

Product modifications, disassembling and reassembling and/or mounting any of the components in any other way than described below are permitted only after written approval by the CIM med GmbH construction and engineering department. CIM med GmbH cannot be held responsible for any damage or injury caused by product modifications and/ or amendments without prior permission from CIM med GmbH.

The additional instructions for use and the inspection plan are intended for end users.

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
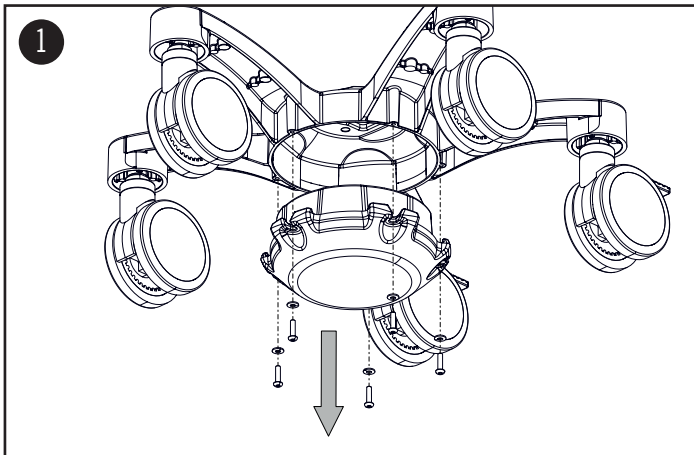
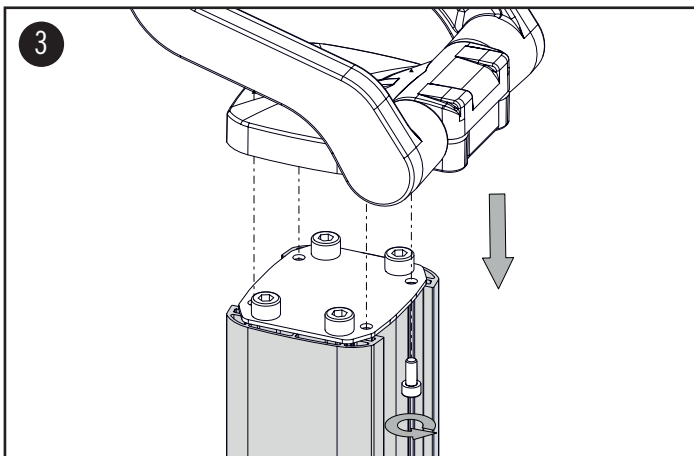
ASSEMBLY STEPS

ASSEMBLY STEPS

1. Loosen the (5) M5x20 pan head screws and remove the counterweight if existing.

NOTE: Hold the counterweight while loosening the screws.

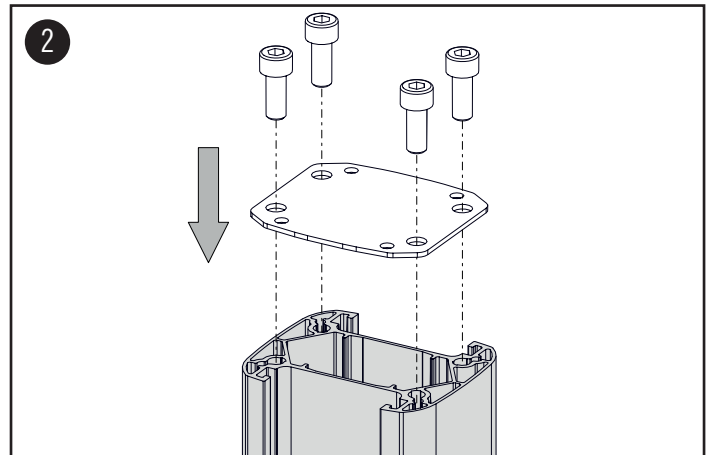
2. Mount the plate with the (4) M8x20 socket head screws to the column profile.

3. Mount the monitor adaption with the (4) M5x12 socket head screws.

4. Push the accessory on the designated side into the column and mount it to the given height.

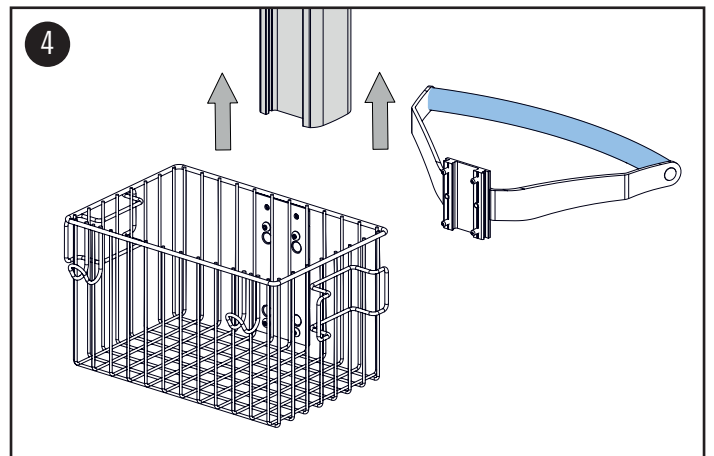
NOTE: Please refer to the corresponding assembly instruction of the accessory.

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
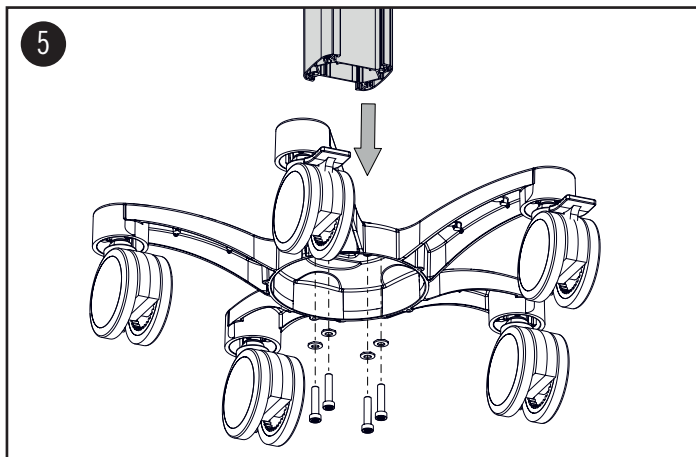
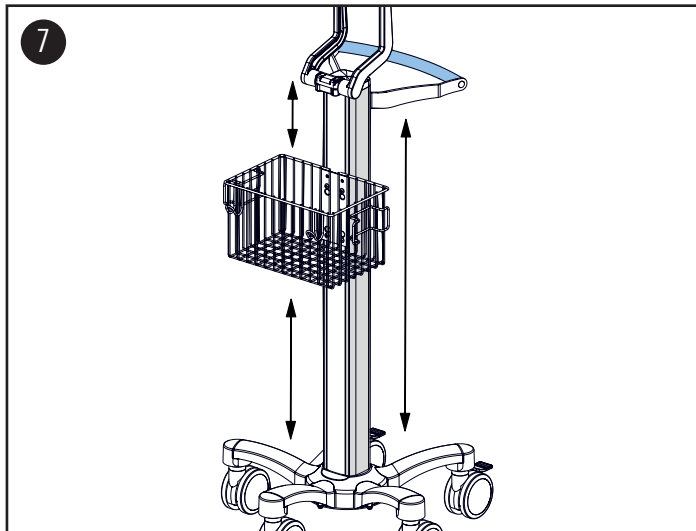
ASSEMBLY STEPS

5. Mount the column to the base with (4) M8x35 cylinder head bolts and (4) contact washers. Tighten the screws securely.
6. Mount the counterweight with the (5) M5x20 pan head screws and the (5) detent edged washers by tightening the screws crosswise.

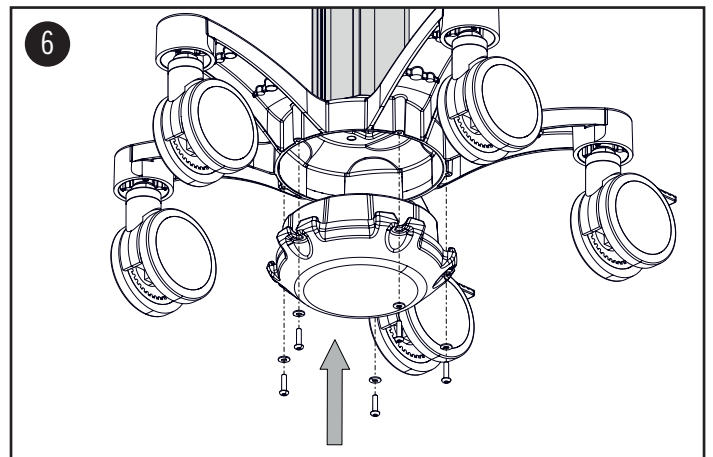⚠ WARNING: To guarantee a secure screw connection, fasten all screws with a tightening torque of 8 Nm (70 in-lbs).

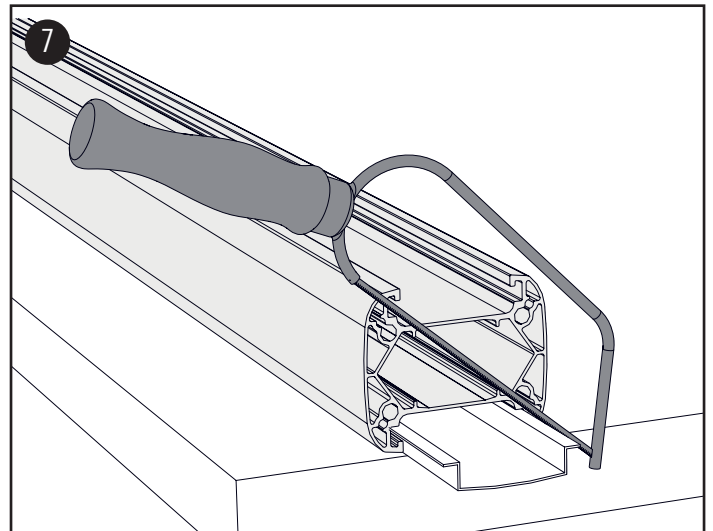
7. Choose the required length of the plastic covers and then cut them as shown.

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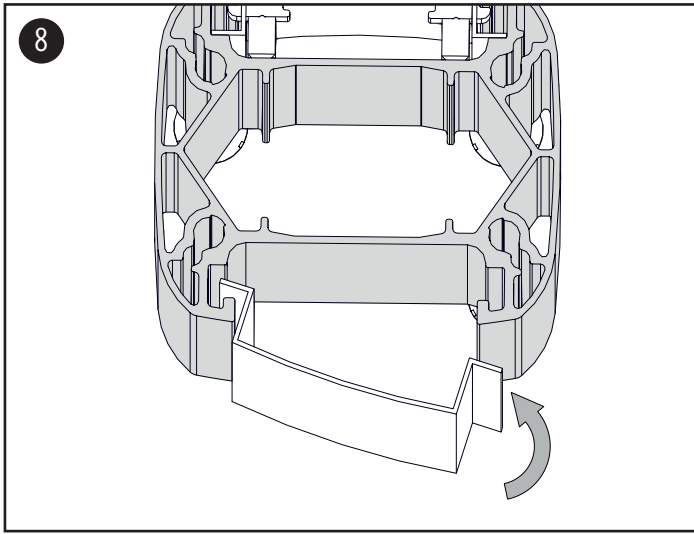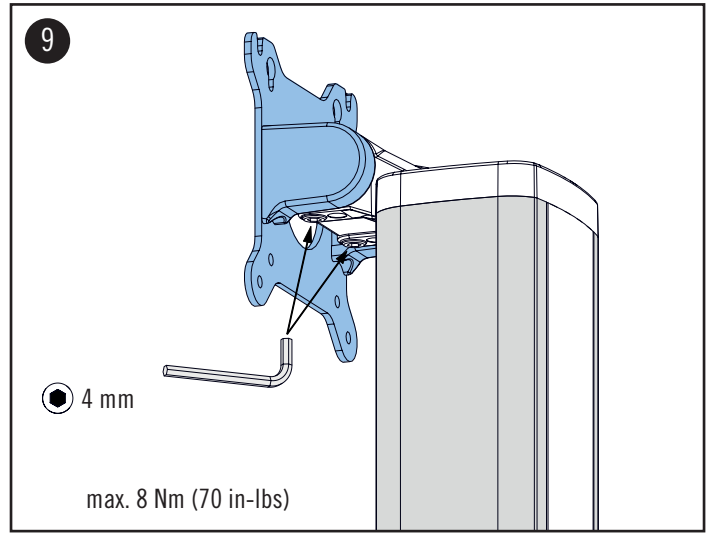
ASSEMBLY STEPS

8. Clip the plastic covers into the column.

TILT ADJUSTMENT

9. To adjust the overall tilt tension, tighten or loosen the both hex screws. By tightening both of the screws the tilt function can be locked completely. Maximum permissible tightening torque is 8 Nm (70 in-lbs).

⚠️ WARNING: In case of overtightening the screw the casting respectively the threads may break which leads to loss of function.

⚠️ WARNING: When loosening both hex screws the ME equipment will tilt forward.

⚠️ DANGER OF SQUEEZING FINGERS