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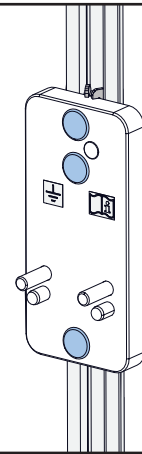
ADAPTER FÜR AMICO DECKENSTATIVE

ASSEMBLY INSTRUCTION

ADAPTER FOR AMICO PENDANTS & HEADWALLS

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WARNING 

Please check the weight of the ME equipment to be mounted. The permissible total weight is mentioned on the corresponding data sheet.

Failure to follow these warnings may result in damage to equipment or injury to people.

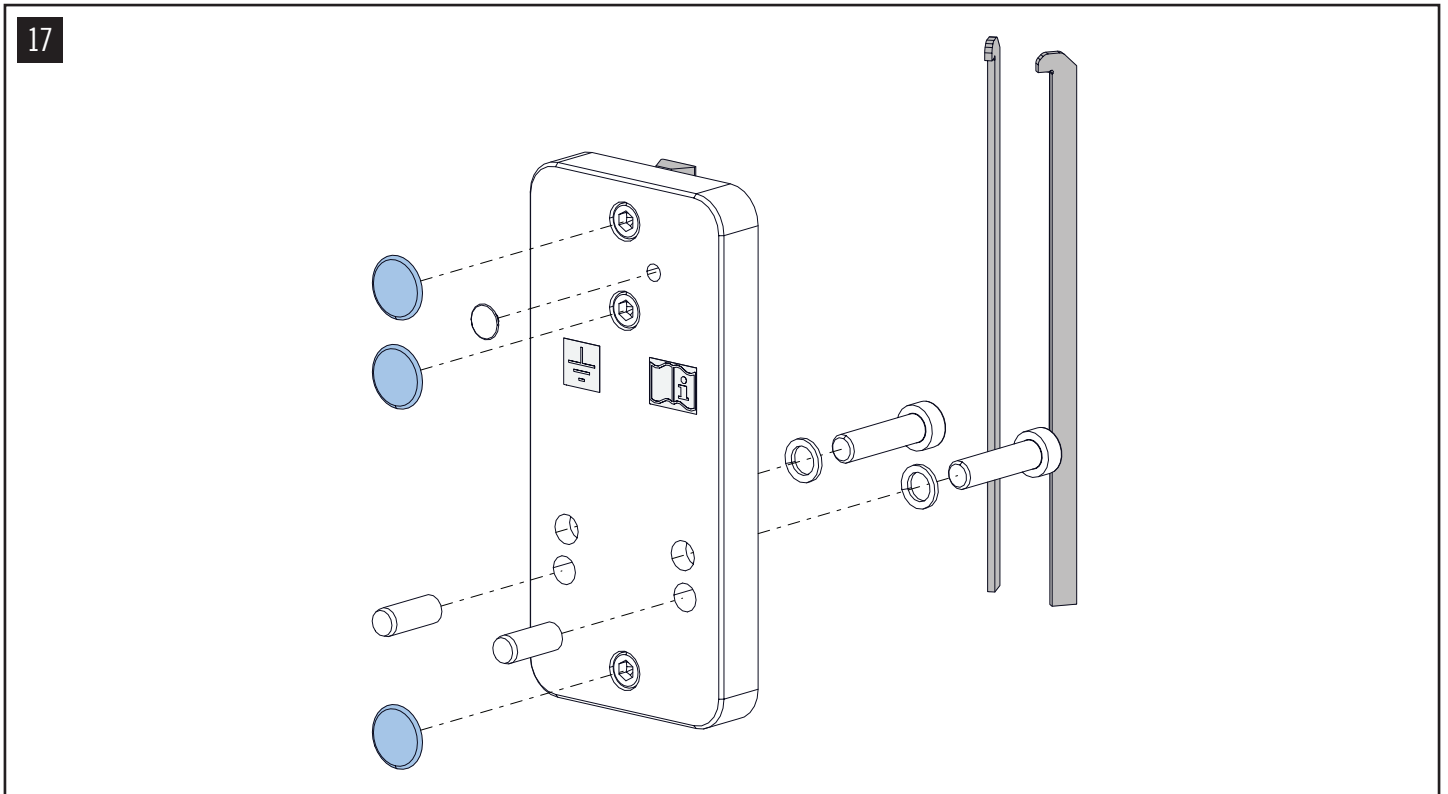
CIM med GmbH cannot be held responsible for any damage due to poor installation.

Product modifications, disassembling and reassembling and/or mounting any of the components in any other way than described below are permitted only after written approval by the CIM med GmbH construction and engineering department. CIM med GmbH cannot be held responsible for any damage or injury caused by product modifications and/ or amendments without prior permission from CIM med GmbH.

The additional instructions for use and the inspection plan are intended for end users.

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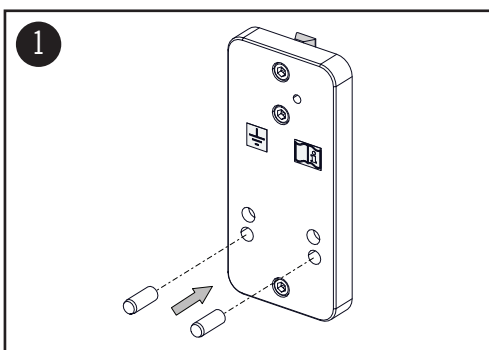
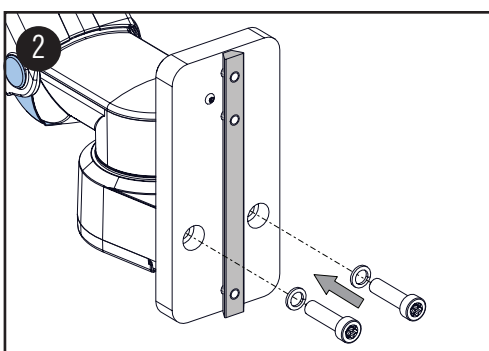
Insert (2) cylinder pins into the lower holes.

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Mount support arm with (2) M8x30 cylinder head screws (Torx) and (2) detent edged washers.

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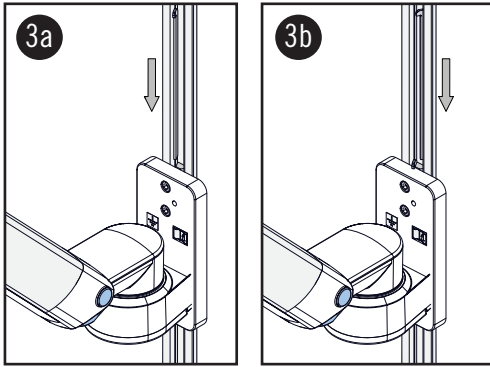
⚠ CAUTION

To guarantee a secure components grounding it is essential to use the both detent edged washers.

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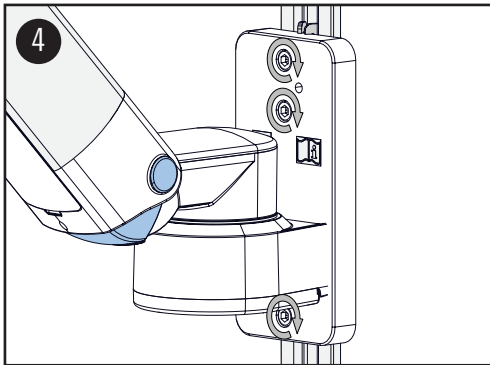
Place the adapter in the channel of medical device. Then slide the two slot nuts into the slot from above as shown.

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NOTE:
Loosen the screw connection of the T-nut to allow threading.

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
WARNING:
The pre-assembled support arm may only be mounted without payload.

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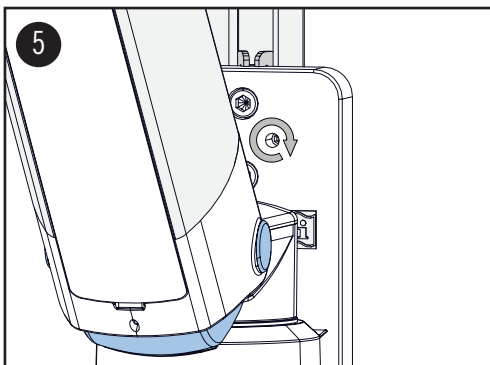Tighten the (3) M6x16 cylinder head screws.

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
Tighten the M6 x 8 grounding screw for final assembly.

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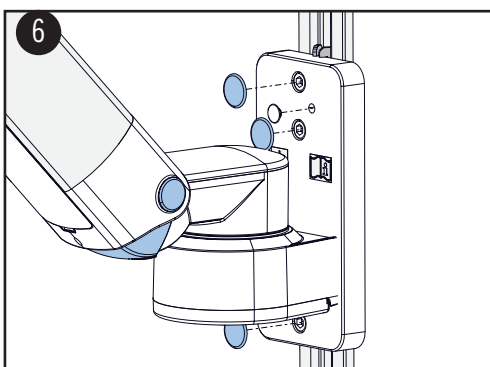WARNING:
The grounding screw must be tightened for a safe component grounding.

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