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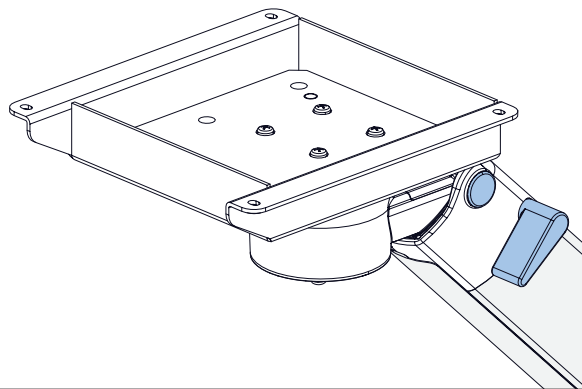
## ASSEMBLY INSTRUCTION

## MOUNTING ADAPTER FOR MINDRAY SV300 / SV70

**WARNHINWEISE** ⚠

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**WARNING** ⚠

Please check the weight of the ME equipment to be mounted. The permissible total weight is mentioned on the corresponding data sheet.

Failure to follow these warnings may result in damage to equipment or injury to people.

CIM med GmbH cannot be held responsible for any damage due to poor installation.

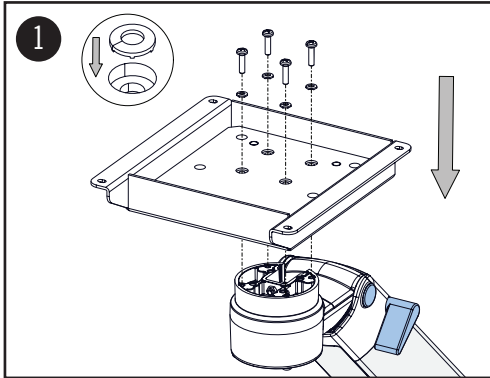
Product modifications, disassembling and reassembling and/or mounting any of the components in any other way than described below are permitted only after written approval by the CIM med GmbH construction and engineering department. CIM med GmbH cannot be held responsible for any damage or injury caused by product modifications and/ or amendments without prior permission from CIM med GmbH.

The additional instructions for use and the inspection plan are intended for end users.

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
Mount the adapter plate to the support arm using the (4) M4x16 pan-head screws and the (4) contact wheels.

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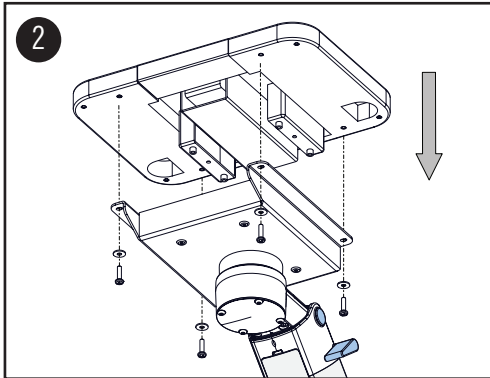**⚠ WARNING:**  
To guarantee a secure components grounding it is essential to use the (4) contact wheels.

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
Mount the Mindray SV300 base plate to the adaption plate using the (4) M4x16 pan-head screws and the (4) washers.

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**ℹ NOTE:**  
The Mindray SV300 base plate is provided by Mindray.

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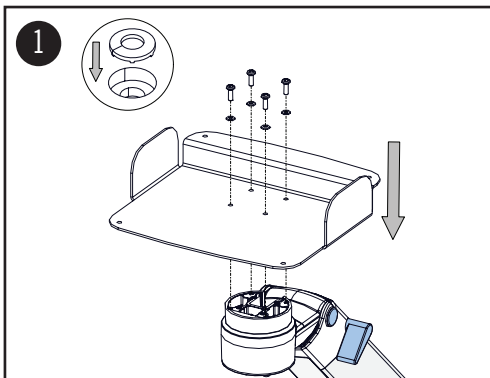
Mount the adapter plate to the support arm using the (4) M4x12 pan-head screws and the (4) contact wheels.

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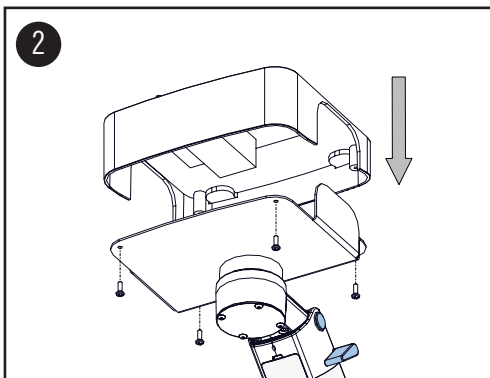Mount the Mindray SV70 base plate to the adaption plate using the (4) M4x12 pan-head screws.

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**ℹ NOTE:**  
The Mindray SV70 base plate is provided by Mindray.