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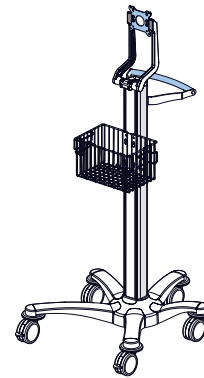
ROLLSTÄNDER HEAVY DUTY

ASSEMBLY INSTRUCTION

CART HEAVY DUTY

Name: CIM Rollständer Heavy Duty

Artikelnummern-Code des Rollständers: AKT-001, EKI-001, EKU-001, IM-001, K2-001, MD-001, OL-001, R-001, SUS-001, XI-001

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Part no. code of the monitor adaptation: A, G, K, PH, TN

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WARNING 

Please check the weight of the ME equipment to be mounted. The permissible total weight is mentioned on the corresponding data sheet.

Failure to follow these warnings may result in damage to equipment or injury to people.

CIM med GmbH cannot be held responsible for any damage due to poor installation.

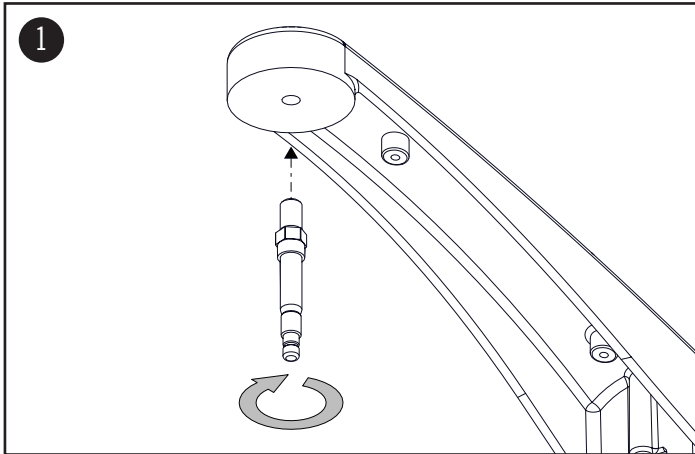
Product modifications, disassembling and reassembling and/or mounting any of the components in any other way than described below are permitted only after written approval by the CIM med GmbH construction and engineering department. CIM med GmbH cannot be held responsible for any damage or injury caused by product modifications and/ or amendments without prior permission from CIM med GmbH.

The additional instructions for use and the inspection plan are intended for end users.

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
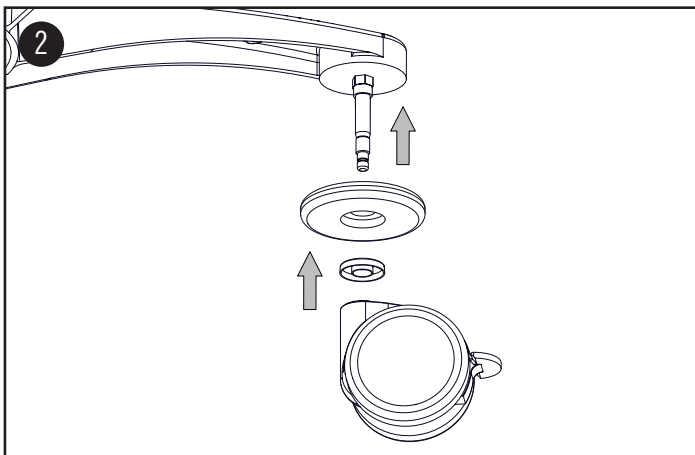
ASSEMBLY STEPS

ASSEMBLY STEPS

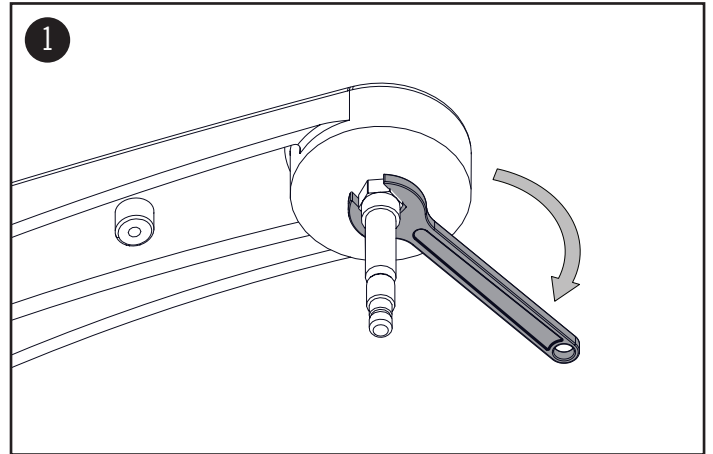
1. Screw the assembly pins of the castors into the base and tighten them with a open-end wrench.

2. Insert the deflector and the washer on the assembly pin.


NOTE: Pay attention to the positioning of the two components shown.

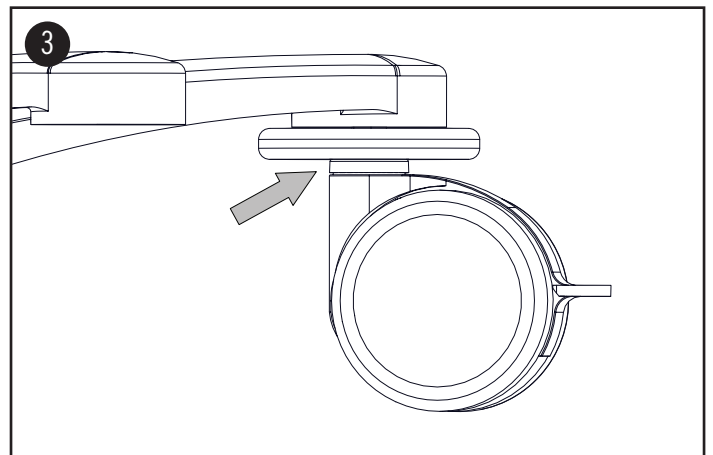3. Press the castor firmly onto the assembly pin. The roller snaps in and rests flush with the washer.

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
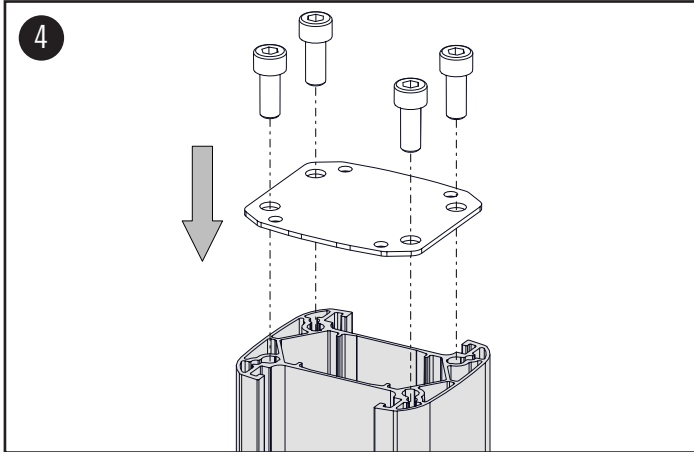
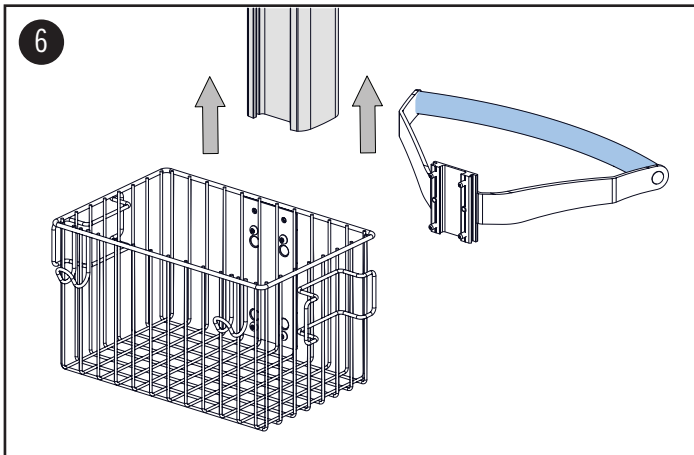
ASSEMBLY STEPS

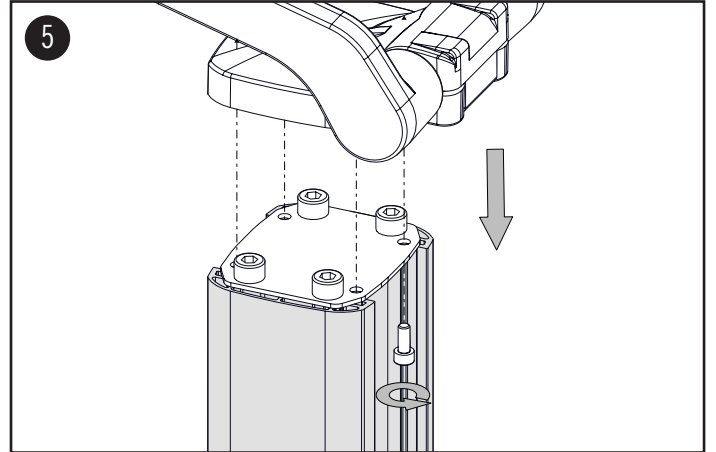4. Mount the plate with the (4) M8 x 20 socket head screws to the columns profile.
5. Mount the monitor adaption with the (4) M5 x 12 socket head screws.

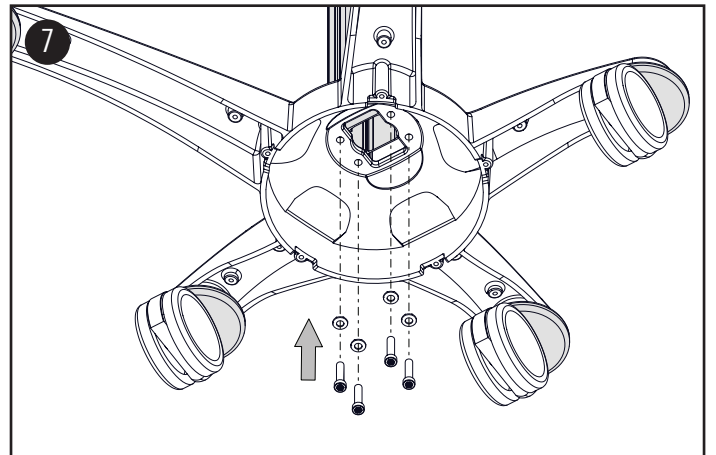
6. Push the accessory on the designated side into the column and mount it to the given height. Please refer to the corresponding assembly instruction of the accessory.
7. Mount the column to the base from underneath with (4) M8x35 cylinder head bolts and (4) contact washers. Tighten the screws securely.

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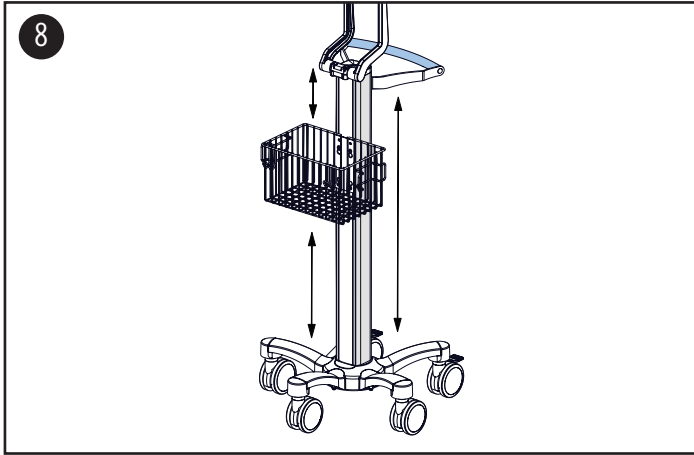
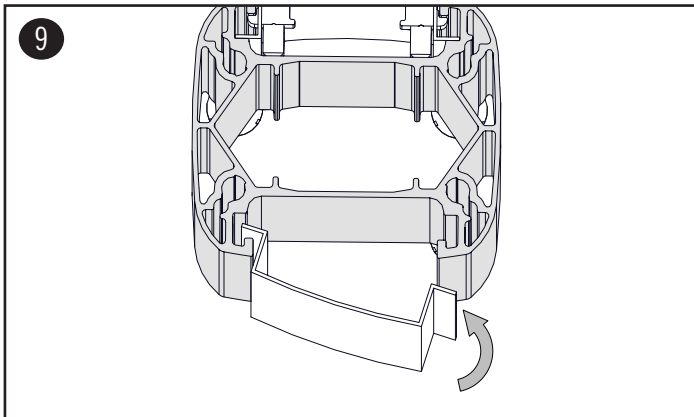
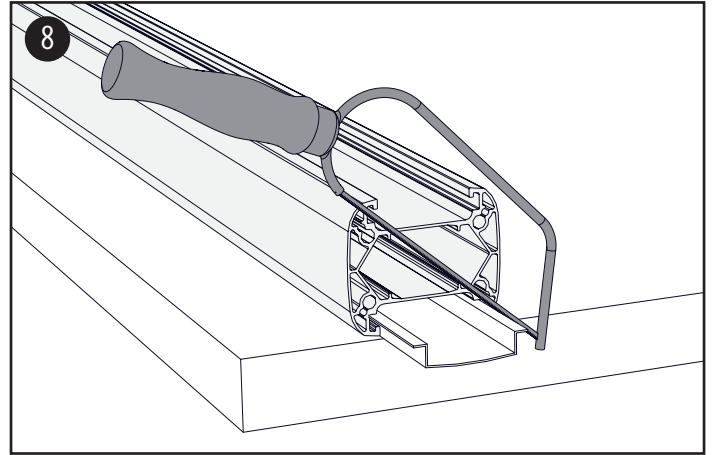
ASSEMBLY STEPS

8. Choose the required length of the plastic covers and then cut them as shown.

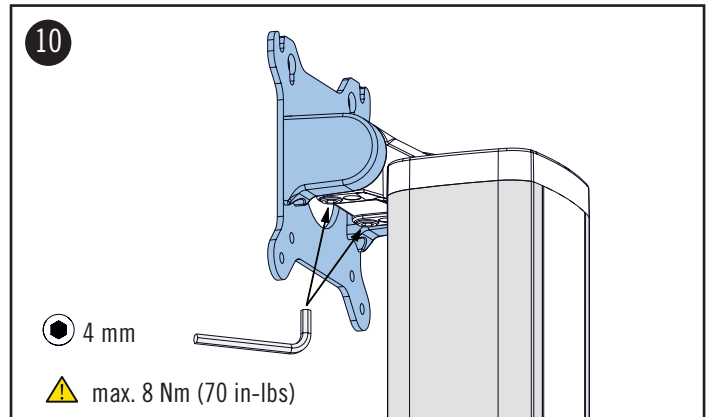9. Clip the plastic covers into the column.

TILT ADJUSTMENT

10. To adjust the overall tilt tension, tighten or loosen the both hex screws with the hex key 4 mm provided. By tightening both of the screws the tilt function can be locked completely. Maximum permissible tightening torque is 8 Nm (70 in-lbs).

⚠️ WARNING: In case of overtightening the screw (> 10 Nm/ 88 in-lbs) the casting respectively the threads may break which leads to loss of function.

⚠️ WARNING: When loosening both hex screws the ME equipment will tilt forward.

⚠️ DANGER OF SQUEEZING FINGERS