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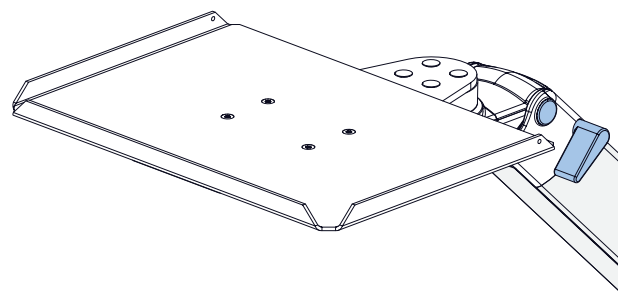
TASTATURABLAGE

ASSEMBLY INSTRUCTION

KEYBOARD TRAY**WARNHINWEISE** 

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WARNING 

Please check the weight of the ME equipment to be mounted. The permissible total weight is mentioned on the corresponding data sheet.

Failure to follow these warnings may result in damage to equipment or injury to people.

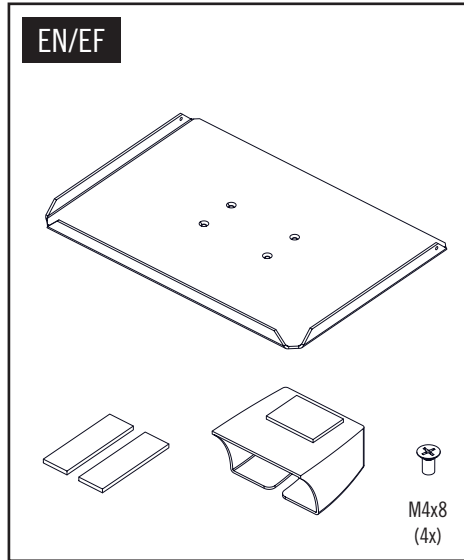
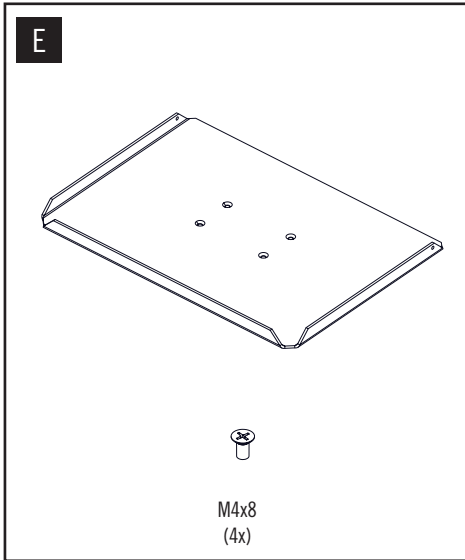
CIM med GmbH cannot be held responsible for any damage due to poor installation.

Product modifications, disassembling and reassembling and/or mounting any of the components in any other way than described below are permitted only after written approval by the CIM med GmbH construction and engineering department. CIM med GmbH cannot be held responsible for any damage or injury caused by product modifications and/ or amendments without prior permission from CIM med GmbH.

The additional instructions for use and the inspection plan are intended for end users.

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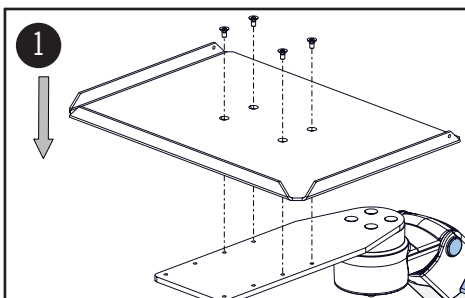
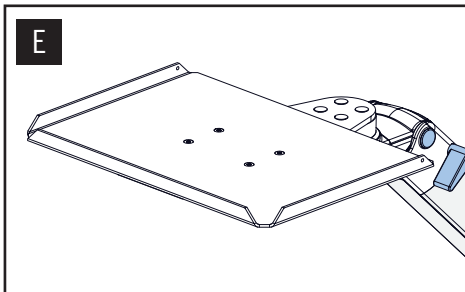
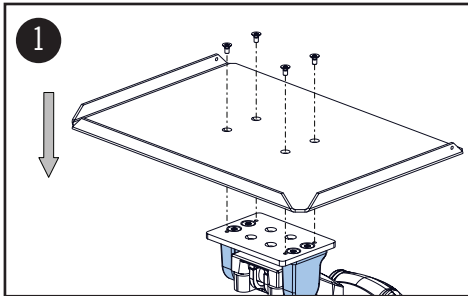
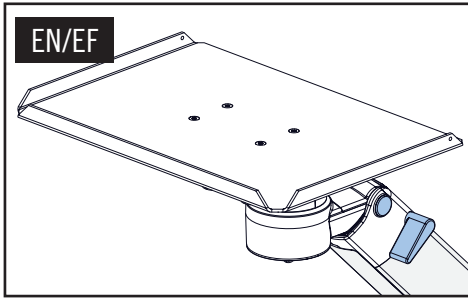
Mount the keyboard tray with (4) M4 x 8 countersunk-head screws.

NOTE:
Keyboard tray can be mounted to the bracket in various positions.

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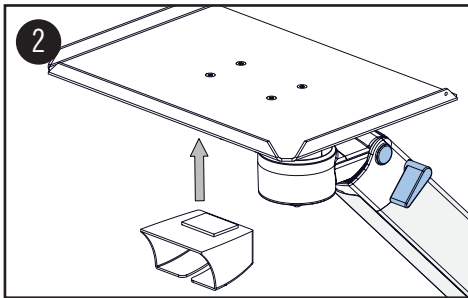
Attach the mouse-garage on the keyboard tray using the hook-and-loop fastener.

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Die CIM med GmbH übernimmt keine Haftung für Schäden, die durch nicht fachgerechte Montage entstanden sind.