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CIM WANDKANAL

ASSEMBLY INSTRUCTION

CIM WALL CHANNEL

**WARNHINWEISE** ⚠

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WARNING ⚠

Please check the weight of the ME equipment to be mounted. The permissible total weight is mentioned on the corresponding data sheet.

Failure to follow these warnings may result in damage to equipment or injury to people.

CIM med GmbH cannot be held responsible for any damage due to poor installation.

Product modifications, disassembling and reassembling and/or mounting any of the components in any other way than described below are permitted only after written approval by the CIM med GmbH construction and engineering department. CIM med GmbH cannot be held responsible for any damage or injury caused by product modifications and/ or amendments without prior permission from CIM med GmbH.

The additional instructions for use and the inspection plan are intended for end users.

MOUNTING MATERIAL

Mounting material is not included in the scope of delivery. It is important to check the load capacity of the wall and to make sure that the adequate and recommended mounting material is used. Please consult your technical department.

DRYWALL

The wall channel must be anchored into the studs. Make sure that the appropriate fasteners and mounting material is used and the manufacturers' instructions for proper installation are adhered to. Make sure that the studs are safe to mount the wall channel.

CONCRETE WALL

Make sure that the appropriate fasteners and mounting material is used and the manufacturers' instructions for proper installation are adhered to. Refer to qualified personnel.

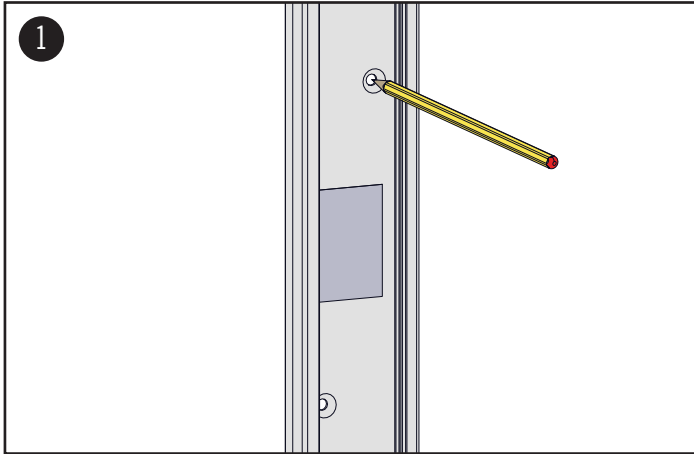
METAL SHEET OR WOODEN WALL

Make sure that the appropriate fasteners and mounting material is used and the manufacturers' instructions for proper installation are adhered to. Make sure that the metal sheet or wooden wall is safe to mount the wall channel.

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
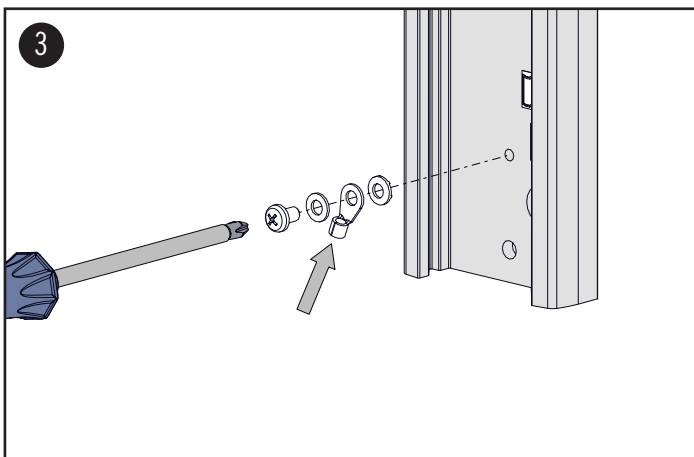
ASSEMBLY STEPS

1. Position the wall channel. Mark the requested bore position by using wall channel's hole pattern.

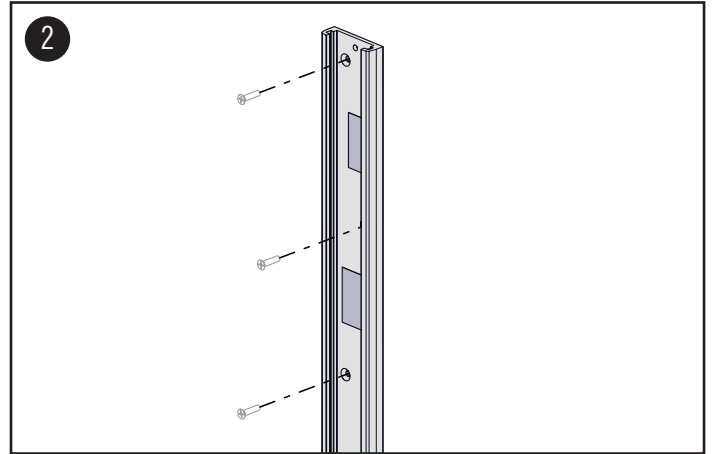NOTE: To level the wall channel use an air level.

2. Mount the wall channel using suitable mounting hardware. Make sure to firmly tighten all screws.
3. Connect the wall channel to the building's grounding by using the pre-assembled eyelet connector. Make sure that the contact disk are positioned as shown.

CAUTION: The support arm's component grounding requires connection to the building's grounding.