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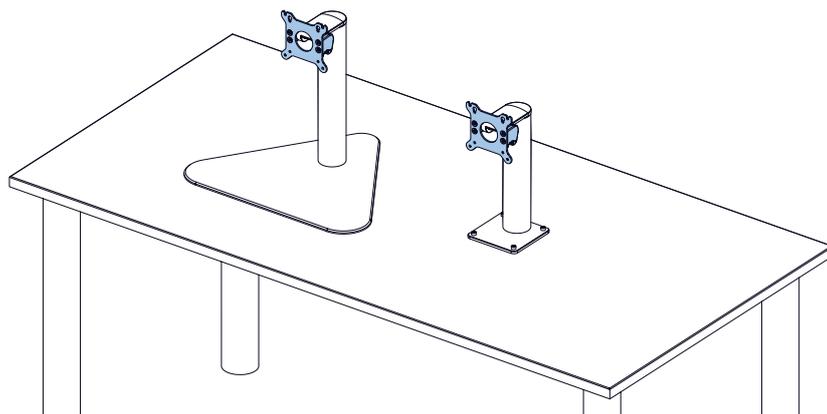
TISCHTRÄGER

ASSEMBLY INSTRUCTION

COUNTER TOP

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Part no. code of the monitor adaptation: A, FK, G, K, MD, PH, Q, SLX, X2X3

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WARNING

Please check the weight of the ME equipment to be mounted. The permissible total weight is mentioned on the corresponding data sheet.

Failure to follow these warnings may result in damage to equipment or injury to people.

CIM med GmbH cannot be held responsible for any damage due to poor installation.

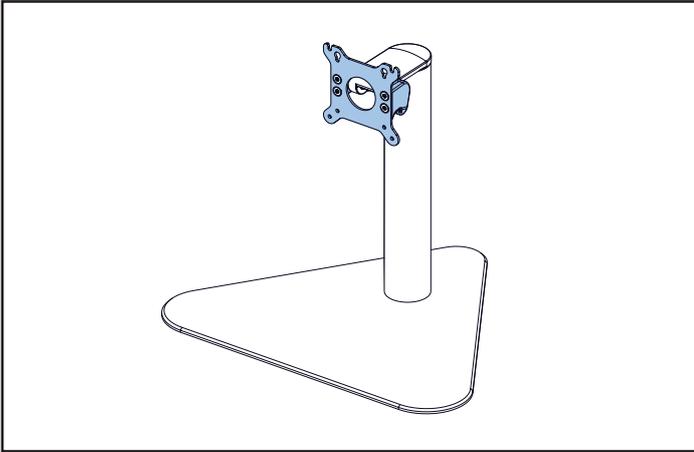
Product modifications, disassembling and reassembling and/or mounting any of the components in any other way than described below are permitted only after written approval by the CIM med GmbH construction and engineering department. CIM med GmbH cannot be held responsible for any damage or injury caused by product modifications and/ or amendments without prior permission from CIM med GmbH.

The additional instructions for use and the inspection plan are intended for end users.

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
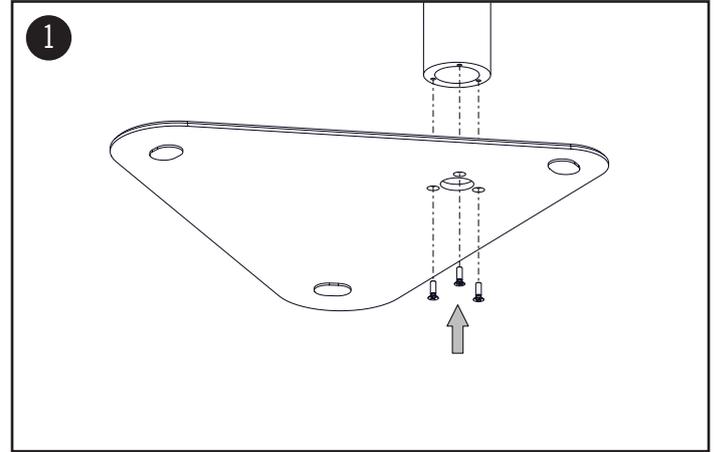
COUNTER TOP FREE-STANDING / WITHOUT BASE PLATE

1. Mount the counter top with (3) M5x16 counter sunk screws to the base plate or on the existing table area.

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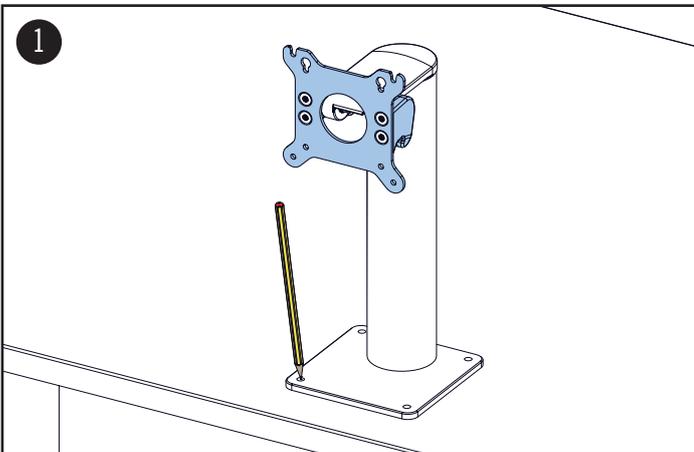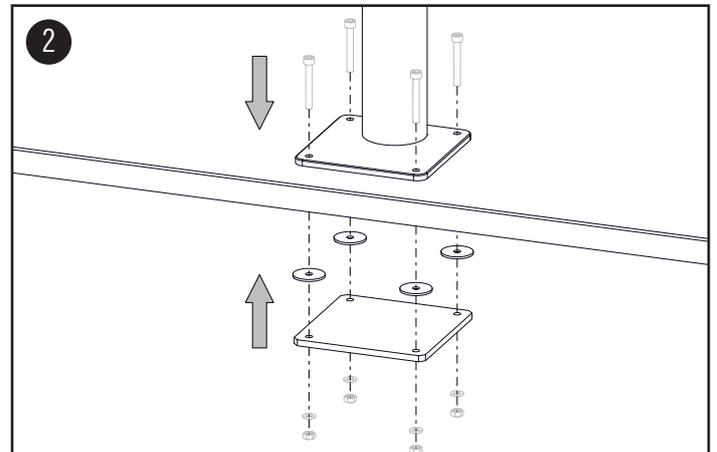
COUNTER TOP WITH BASE PLATE TO MOUNT

1. Position the counter top. Mark the requested bore position.
2. Mount the counter top using suitable mounting hardware and attached washers. Screw all securely.

ⓘ NOTE: Depending on the type of table, it is recommended to use the optional counter plate.

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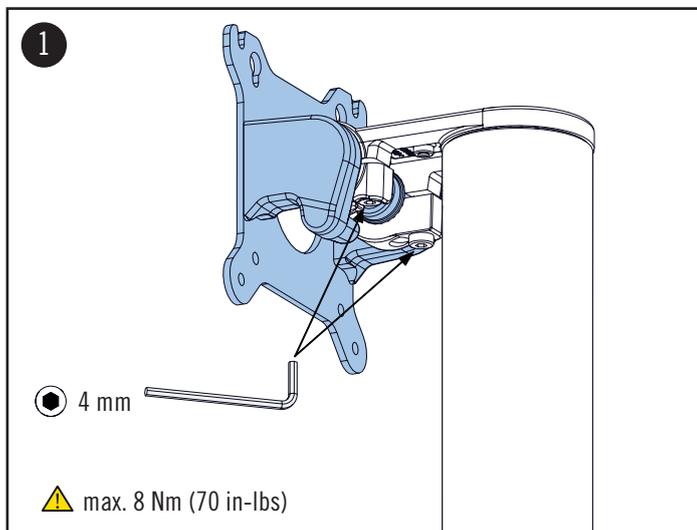
TILT ADJUSTMENT

1. To adjust the overall tilt tension, tighten or loosen the both hex screws. By tightening both of the screws the tilt function can be locked completely. Maximum permissible tightening torque is 8 Nm (70 in-lbs).

⚠ WARNING: In case of overtightening the screw the casting respectively the threads may break which leads to loss of function.

⚠ WARNING: When loosening both hex screws the ME equipment will tilt forward.

⚠ DANGER OF SQUEEZING FINGERS