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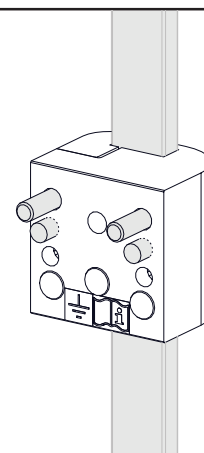
VERTIKALE NORMSCHIENENKLEMME 10 x 25 MM

ASSEMBLY INSTRUCTION

VERTICAL RAIL CLAMP 10 x 25 MM**WARNHINWEISE** 

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WARNING 

Please check the weight of the ME equipment to be mounted. The permissible total weight is mentioned on the corresponding data sheet.

Failure to follow these warnings may result in damage to equipment or injury to people.

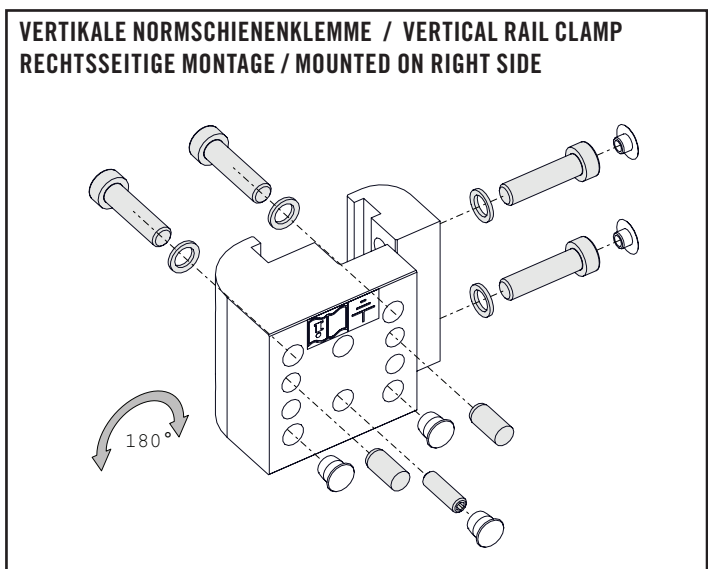
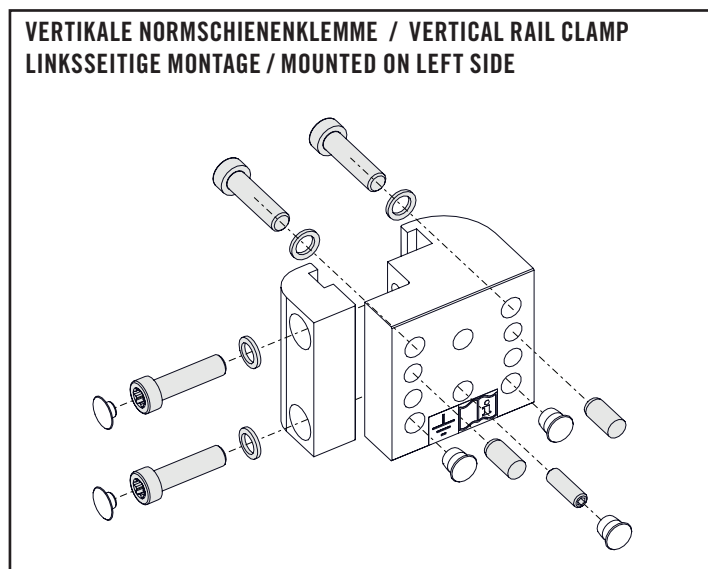
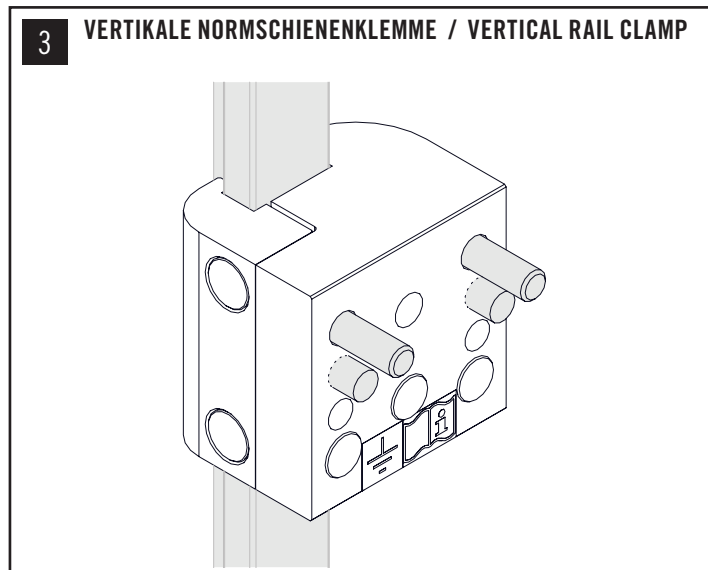
CIM med GmbH cannot be held responsible for any damage due to poor installation.

Product modifications, disassembling and reassembling and/or mounting any of the components in any other way than described below are permitted only after written approval by the CIM med GmbH construction and engineering department. CIM med GmbH cannot be held responsible for any damage or injury caused by product modifications and/ or amendments without prior permission from CIM med GmbH.

The additional instructions for use and the inspection plan are intended for end users.

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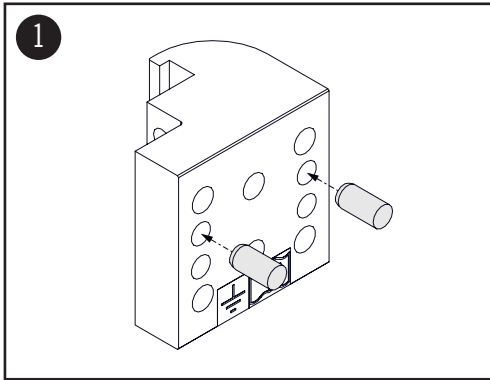
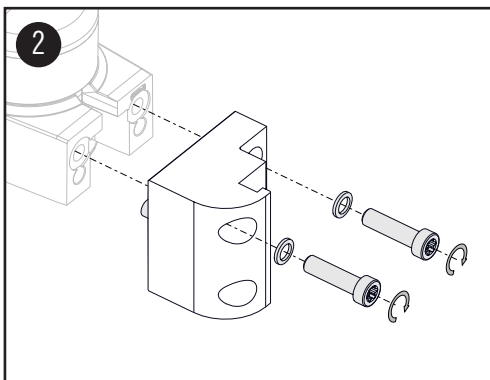
Place locating pins in the bore holes below the holes for the screws. It is the second row from top.

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
Mount the adapter to arm using the 2 x M8 screws and detent edged washers.
Torque moment: 25 Nm (221 in-lbs)

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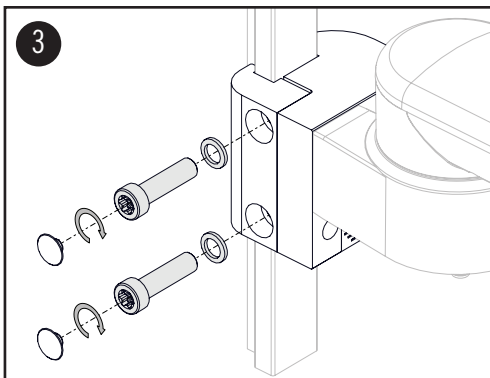
! To guarantee a secure components grounding it is essential to use the both detent edged washers.

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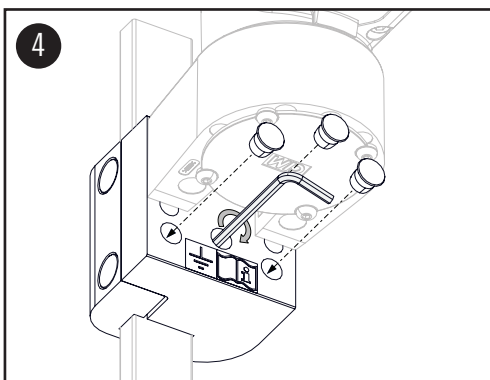Place the rail clamp to the vertical rail and fasten it with two TX45 screws.
Apply the cover caps on top of the screws.

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! **CAUTION**
The grounding screw must be tightened for a safe component grounding.

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Die CIM med GmbH übernimmt keine Haftung für Schäden, die durch nicht fachgerechte Montage entstanden sind.