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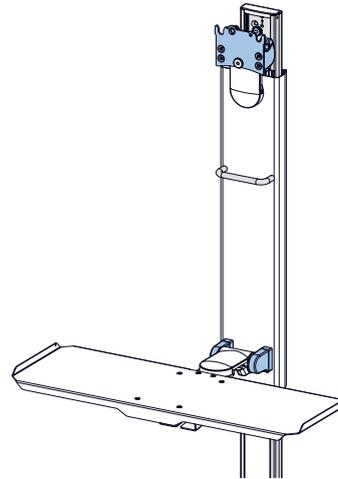
IT LIFT SYSTEM

ASSEMBLY INSTRUCTION

IT LIFT SYSTEM

WARNHINWEISE

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WARNING

Please check the weight of the ME equipment to be mounted. The permissible total weight is mentioned on the corresponding data sheet.

Failure to follow these warnings may result in damage to equipment or injury to people.

CIM med GmbH cannot be held responsible for any damage due to poor installation.

Product modifications, disassembling and reassembling and/or mounting any of the components in any other way than described below are permitted only after written approval by the CIM med GmbH construction and engineering department. CIM med GmbH cannot be held responsible for any damage or injury caused by product modifications and/ or amendments without prior permission from CIM med GmbH.

The additional instructions for use and the inspection plan are intended for end users.

MOUNTING MATERIAL

Mounting material is not included in the scope of delivery. It is important to check the load capacity of the wall and to make sure that the adequate and recommended mounting material is used. Please consult your technical department.

DRYWALL

The wall channel must be anchored into the studs. Make sure that the appropriate fasteners and mounting material is used and the manufacturers' instructions for proper installation are adhered to. Make sure that the studs are safe to mount the wall channel.

CONCRETE WALL

Make sure that the appropriate fasteners and mounting material is used and the manufacturers' instructions for proper installation are adhered to. Refer to qualified personnel.

METAL SHEET OR WOODEN WALL


Make sure that the appropriate fasteners and mounting material is used and the manufacturers' instructions for proper installation are adhered to. Make sure that the metal sheet or wooden wall is safe to mount the wall channel.

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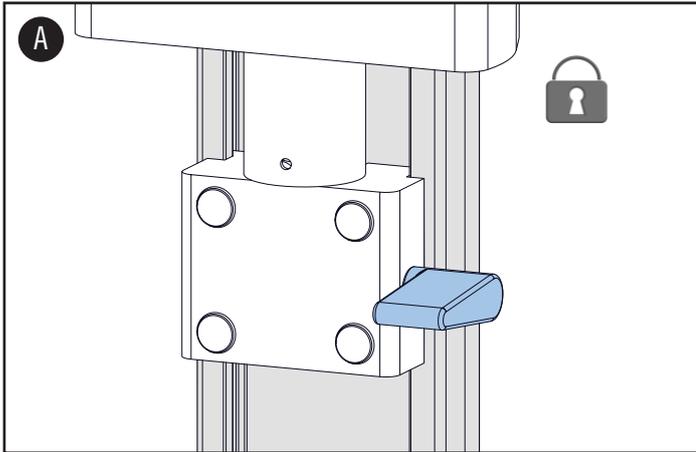
ASSEMBLY STEPS

RELEASING THE PRETENTION

Support arm is supplied pretensioned. Safe release of the support arm is only possible after assembly has been completed.

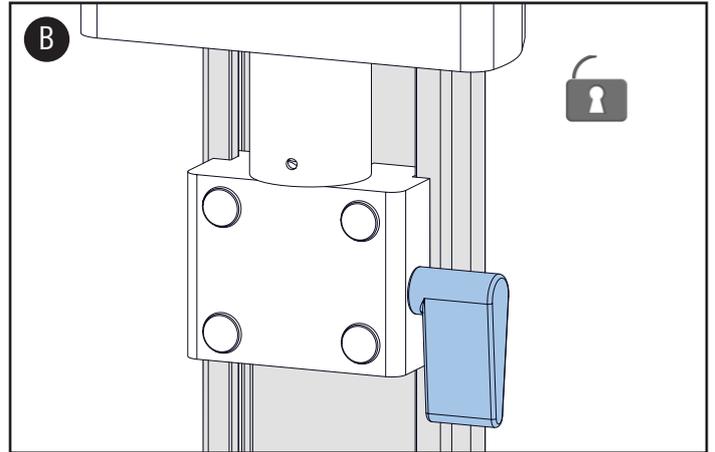Illustration A shows the support arm with the locking lever engaged.
Illustration B shows the support arm with the locking lever not engaged.

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
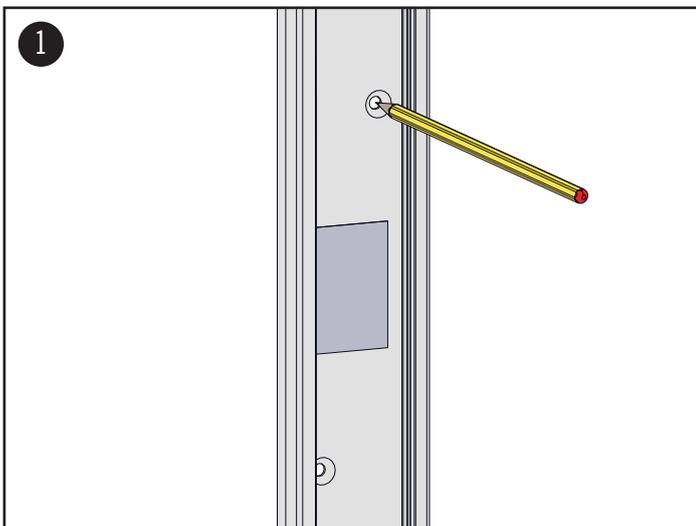
ASSEMBLY STEPS

1. Position the wall channel. Mark the requested bore position by using wall channel's hole pattern.

ⓘ NOTE: To level the wall channel use an air level.

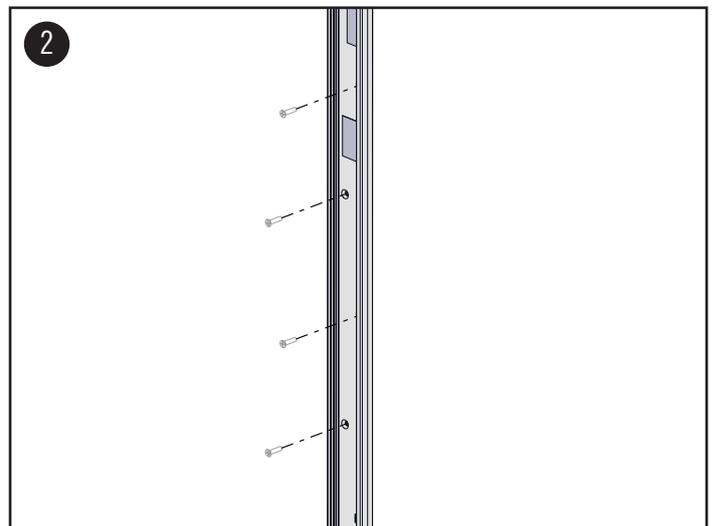
2. Mount the wall channel using suitable mounting hardware. Make sure to firmly tighten all screws.

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
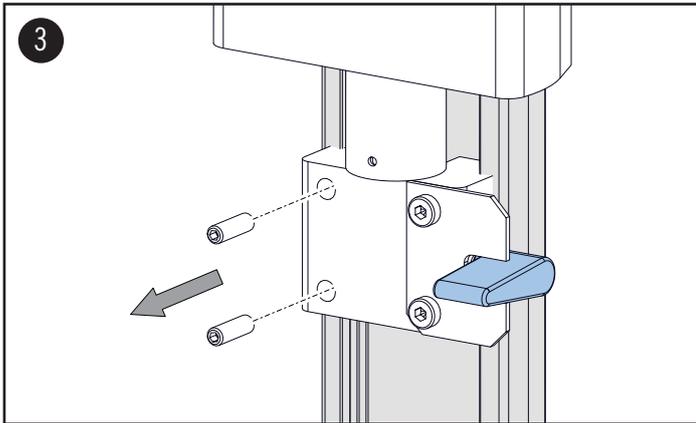
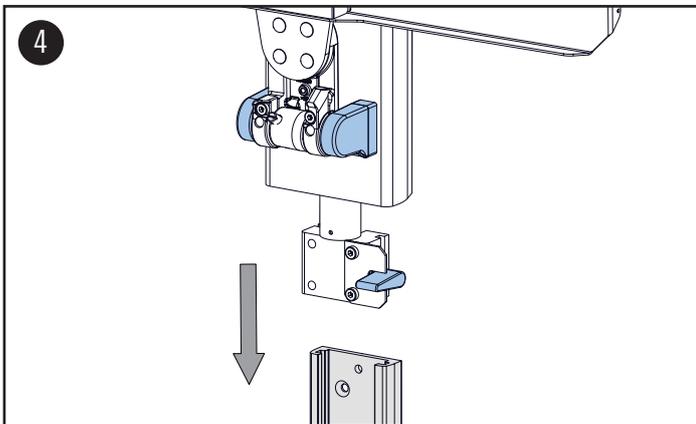
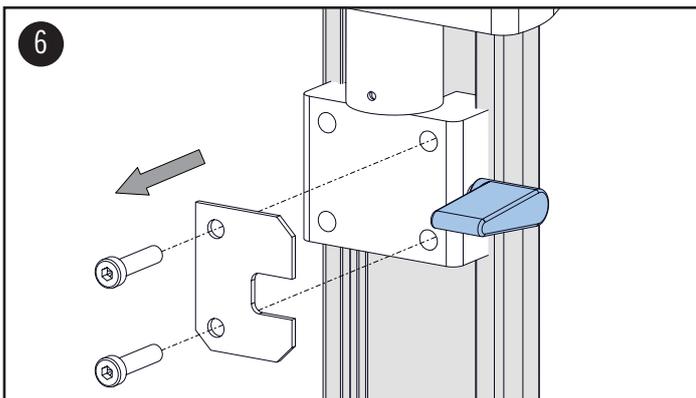
ASSEMBLY STEPS

3. Unscrew the (2) M6x20 threaded pins and remove the wall channel, mounted there for transportation protection.

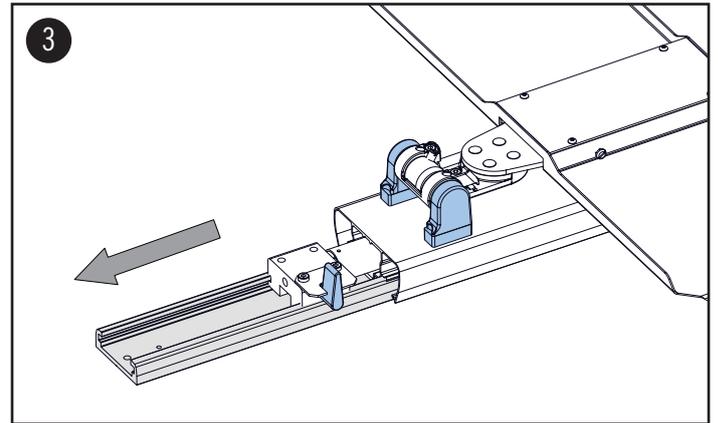⚠ WARNING: Do not loosen the transport lock. This prevents unintentional extension, which can lead to considerable injuries.

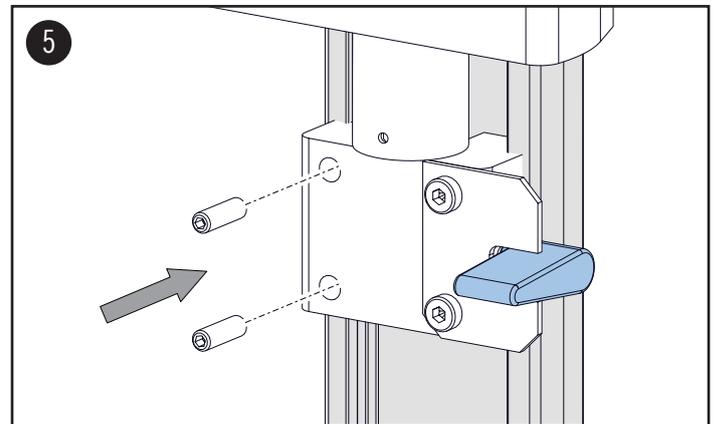
4. Insert support arm into the wall channel. Make sure that the slide bars are positioned correctly.
5. Mount support arm at the desired height using (2) M6x20 threaded pins.

6. Remove the transport lock by unscrewing (2) M6x25 cylinder head screws.

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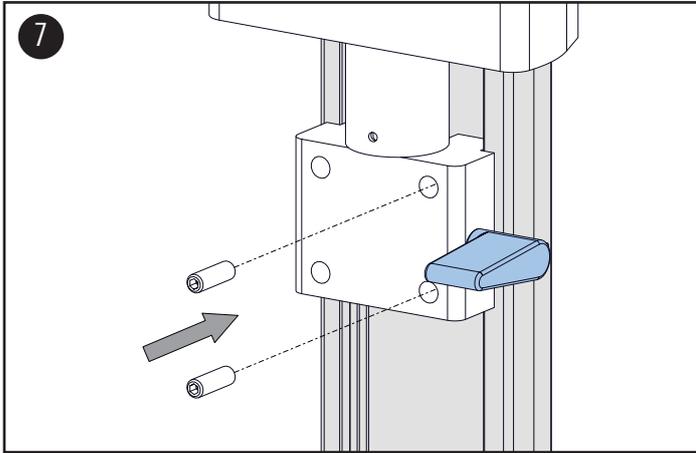
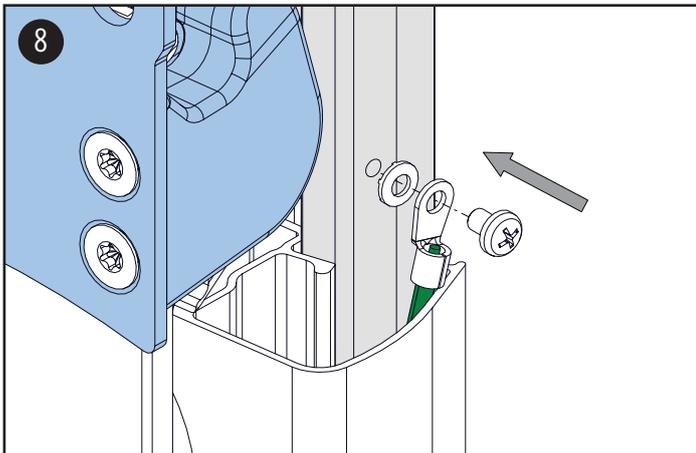
ASSEMBLY STEPS

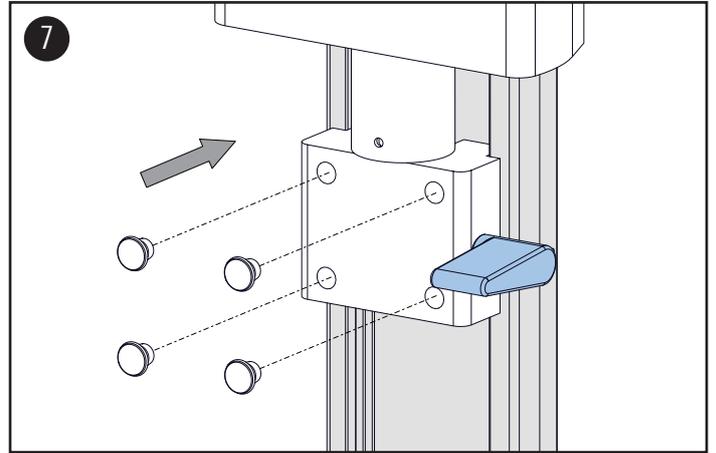
7. Insert (2) M6x20 threaded pins and tighten them. Cover the holes with (4) cover caps.

8. Connect the component grounding of the support arm to the wall channel using the pan head screw.

NOTE: There are several drilling positions for the earthing connection on the side of the wall channel. Use the closest earthing connection.

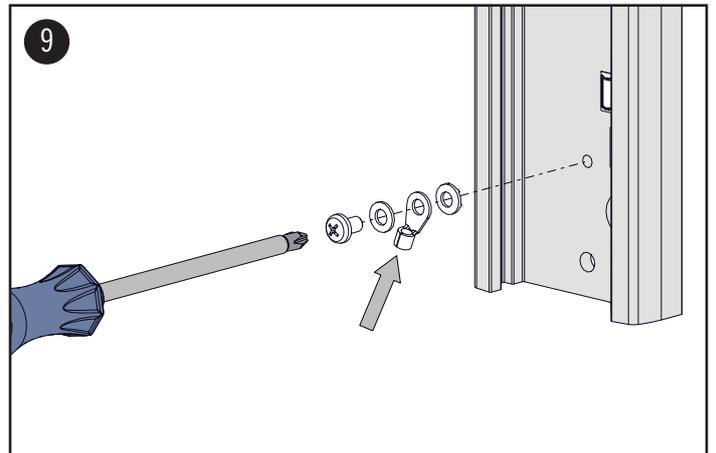9. Connect the wall channel to the building grounding using pre-assembled cable lug. When installing the cable lug, pay attention to the positioning of the contact washers as shown.

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
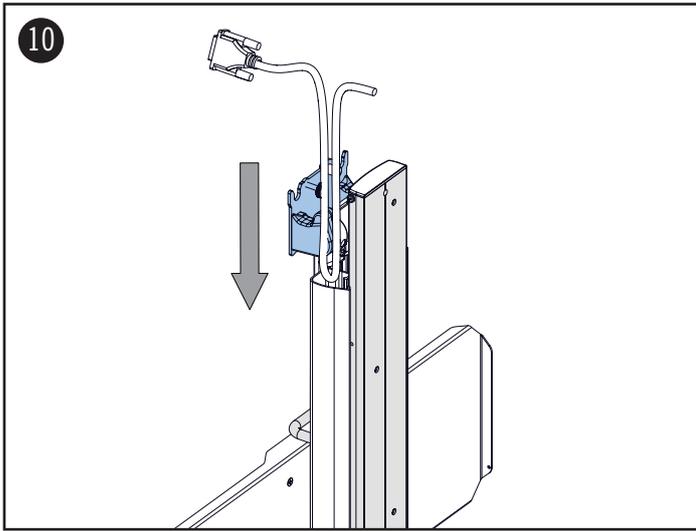
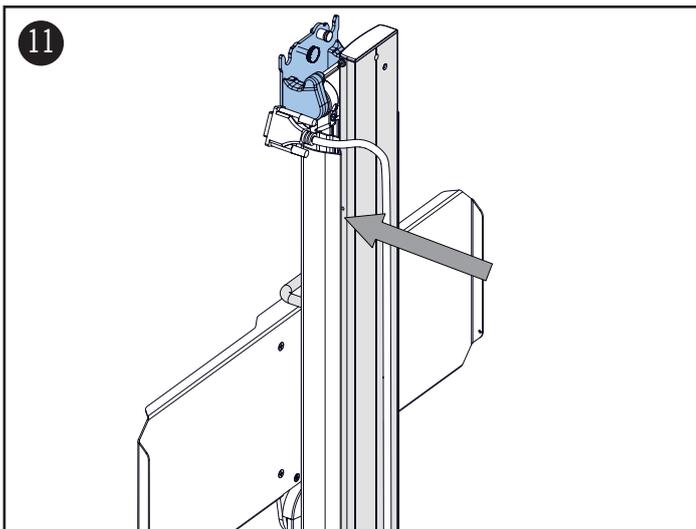
ASSEMBLY STEPS

10. OPTION A

Guide the cable into the guide rail from above as shown.

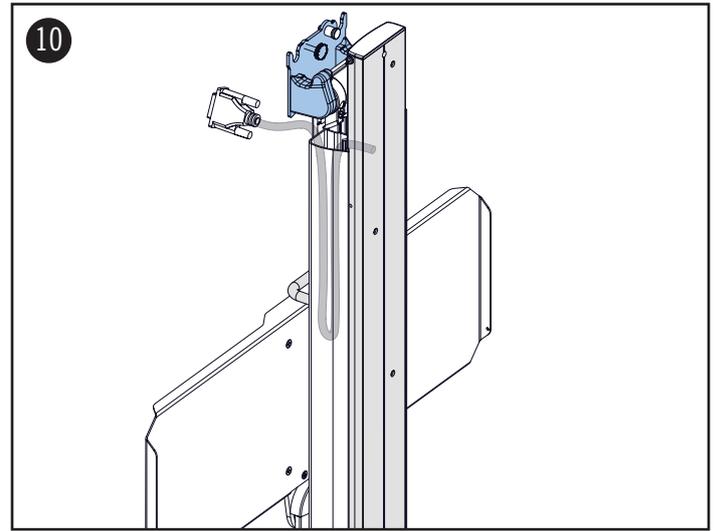
NOTE: Ensure that there is sufficient cable length for the entire range of motion of the support arm.

11. OPTION B:

Guide the cable into the guide rail from behind as shown.

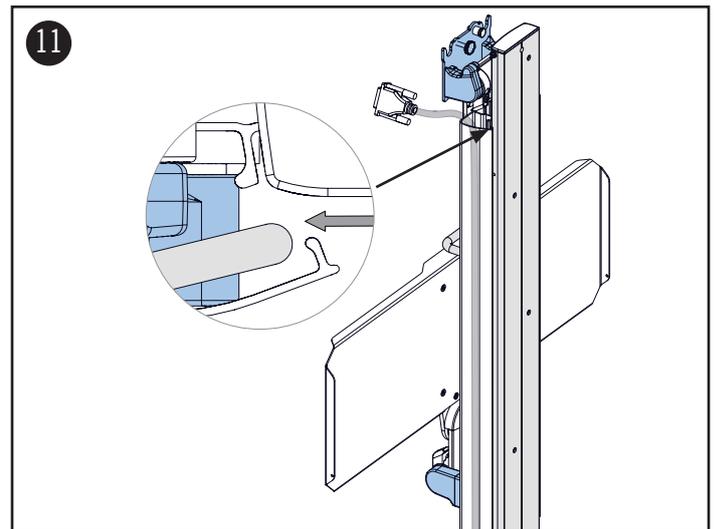
NOTE: Ensure that there is sufficient cable length for the entire range of motion of the support arm.

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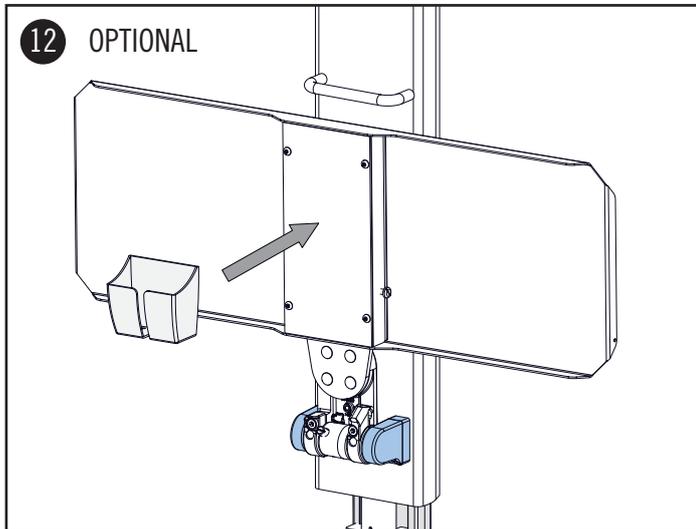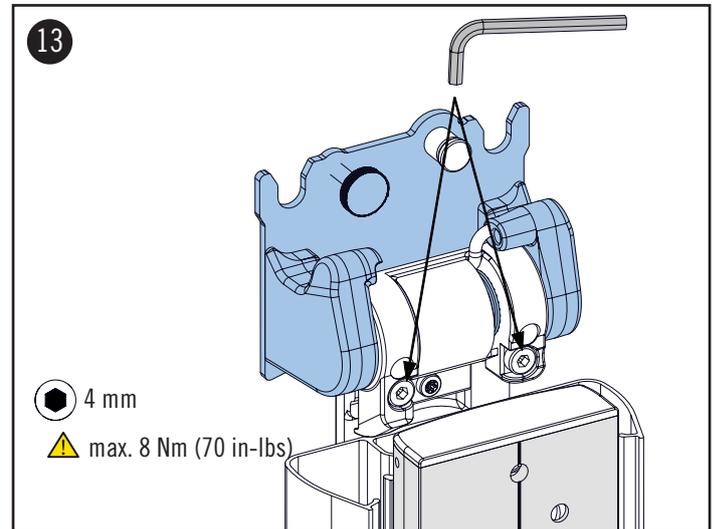
ASSEMBLY STEPS

12. OPTIONAL:

Mount the mouse holder with the enclosed double-sided adhesive tape.

TILT ADJUSTMENT

13. To adjust the overall tilt tension, tighten or loosen the both hex screws with the hex key 4 mm provided. By tightening both of the screws the tilt function can be locked completely. Maximum permissible tightening torque is 8 Nm (70 in-lbs).

⚠ WARNING: In case of overtightening the screw (> 10 Nm/ 88 in-lbs) the casting respectively the threads may break which leads to loss of function.

⚠ WARNING: When loosening both hex screws the ME equipment will tilt forward.

⚠ DANGER OF SQUEEZING FINGERS