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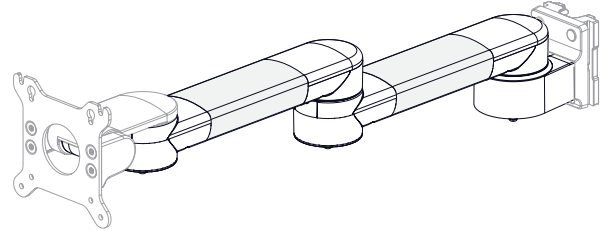
DREIGELENKARM LIGHT DUTY

ASSEMBLY INSTRUCTION

EXTENDED PIVOT ARM LIGHT DUTY

Name: CIM Dreigelenkarm Light Duty

Artikelnummern-Code des Tragarmes: 32424, 33030

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Name: CIM Extended pivot arm light duty

Part no. code of the arm: 32424, 33030

Part no. code of the monitor adaptation: A, C, E, EF, FMS, G, GE, GS, MD, MP2, MP2PS, NS, PH, Q, SMR, TN, X2X3

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WARNING 

Please check the weight of the ME equipment to be mounted. The permissible total weight is mentioned on the corresponding data sheet.

Failure to follow these warnings may result in damage to equipment or injury to people.

CIM med GmbH cannot be held responsible for any damage due to poor installation.

Product modifications, disassembling and reassembling and/or mounting any of the components in any other way than described below are permitted only after written approval by the CIM med GmbH construction and engineering department. CIM med GmbH cannot be held responsible for any damage or injury caused by product modifications and/or amendments without prior permission from CIM med GmbH.

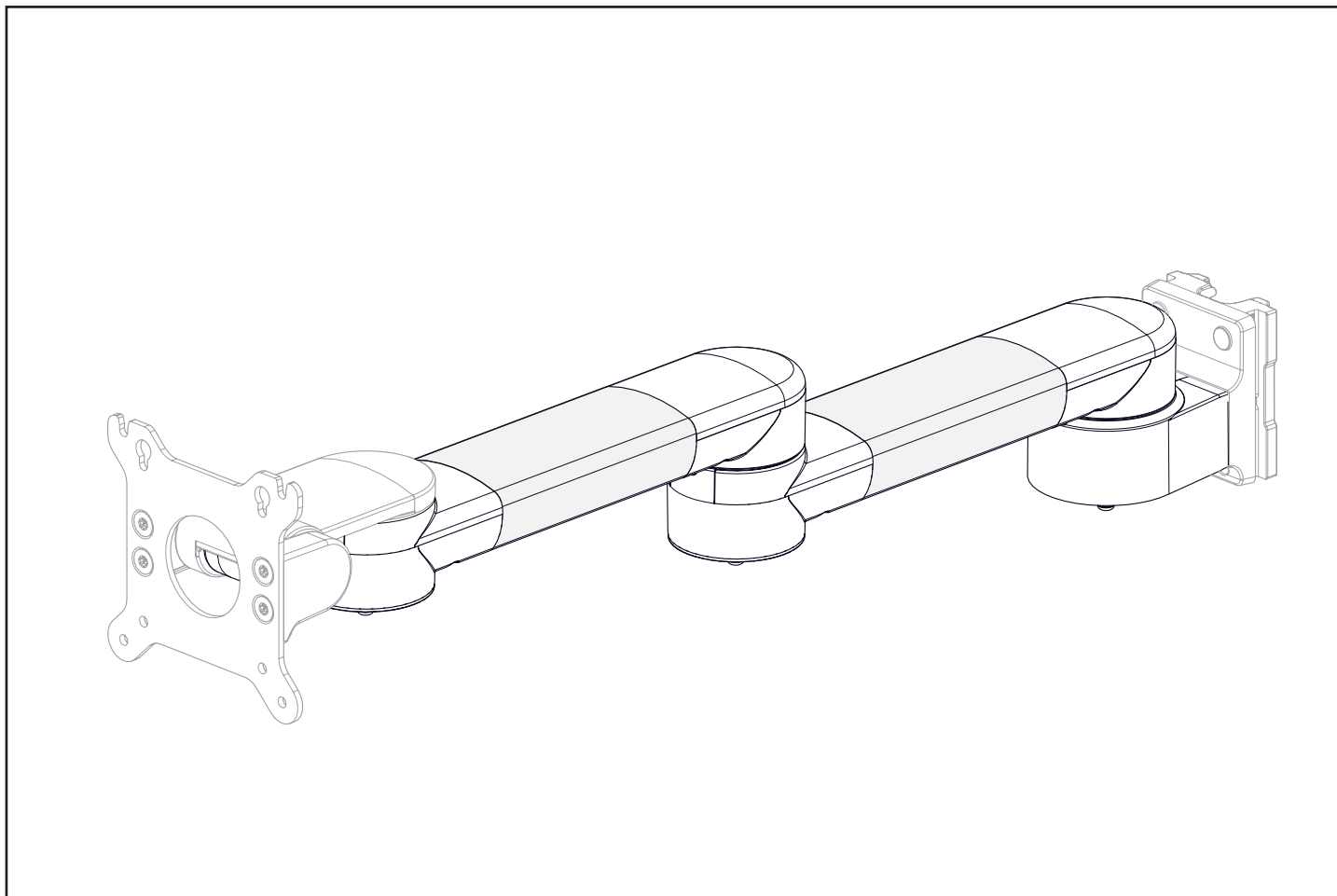
The additional instructions for use and the inspection plan are intended for end users.

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
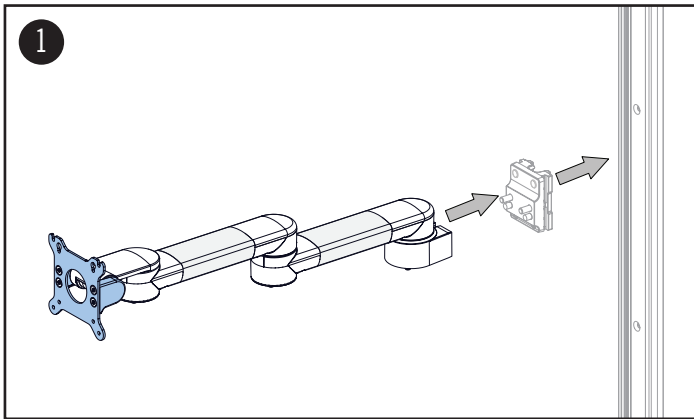
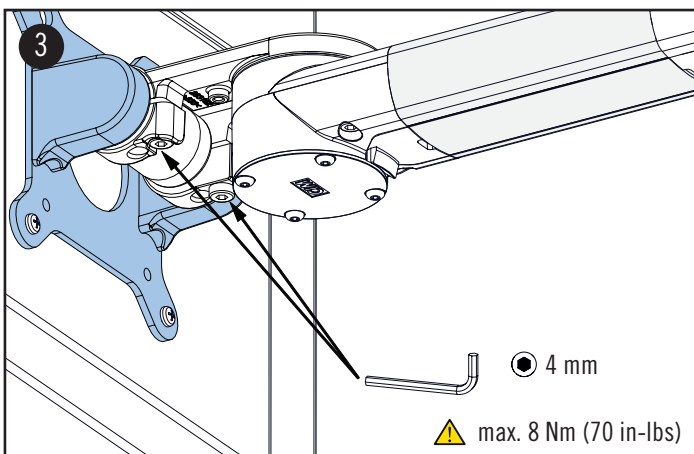
ASSEMBLY STEPS

ASSEMBLY STEPS

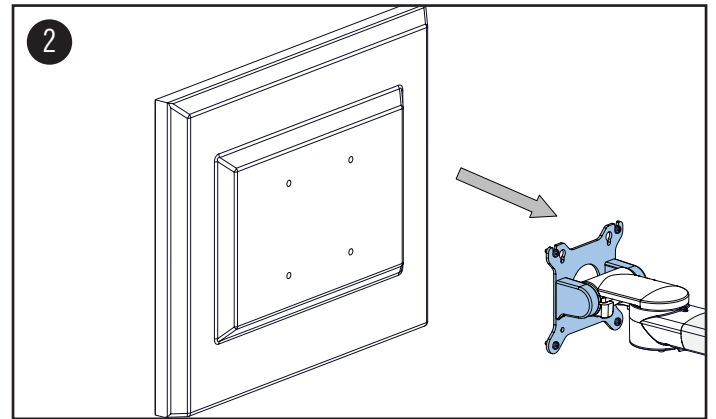1. Mount the support arm to its designated place (wall channel, tube, DIN rail etc.). The illustration shows a wall channel adapter. Please refer to the corresponding assembly instruction for all other adapters.
2. Mount the ME equipment to its designated monitor adaptation. The illustration shows a VESA plate. Please refer to the corresponding assembly instruction for all other monitor adaptations.

TILT ADJUSTMENT

3. To adjust the overall tilt tension, tighten or loosen the both hex screws. By tightening both of the screws the tilt function can be locked completely. Maximum permissible tightening torque is 8 Nm (70 in-lbs).

⚠ WARNING: In case of overtightening the screw the casting respectively the threads may break which leads to loss of function.

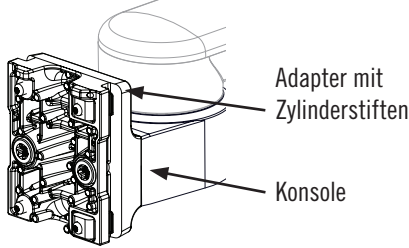
⚠ WARNING: When loosening both hex screws the ME equipment will tilt forward.

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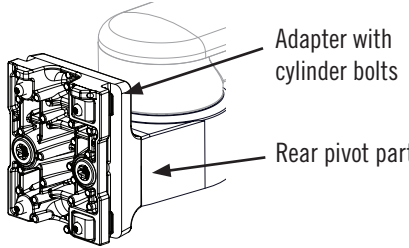
TROUBLE SHOOTING

Problem	Problem caused by	Remedy
ME equipment does not appear level or parallel to floor	Channel not even/ straight	Adjust wall channel or reinstall channel.
	Adapter and rear pivot part are not tightened completely. 	Please check that there is no crack or movement between these parts. Tighten the screws and make sure that adapter and console are pulled tight with a torque moment of 25 Nm.
	Adapter not parallel to wall channel: (4) cylinder bolts are pulled tight without having adjusted the adapter within the channel.	Remove cylinder bolts on both sides. Tighten first the cylinder pins on the left side. See assembly instruction adapter vertical/ horizontal wall channel.
	Monitor adaption VESA / flat panel are not mounted parallel to floor.	Loosen the (4) M4 screws, adjust the monitor and pull the (4) screws tight again. Make sure that the monitor appears level or parallel to the floor.