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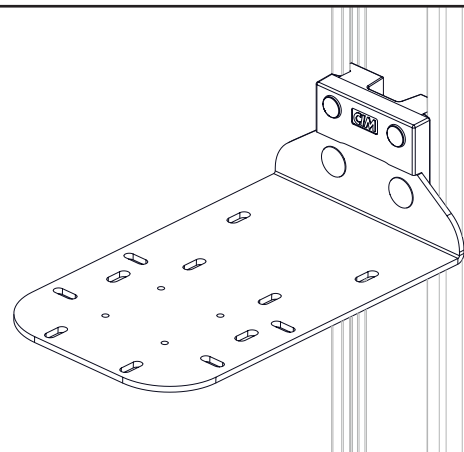
## ASSEMBLY INSTRUCTION

**L-BRACKET FOR GE FRAME F2/F5****WARNHINWEISE** 

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**WARNING** 

Please check the weight of the ME equipment to be mounted. The permissible total weight is mentioned on the corresponding data sheet.

Failure to follow these warnings may result in damage to equipment or injury to people.

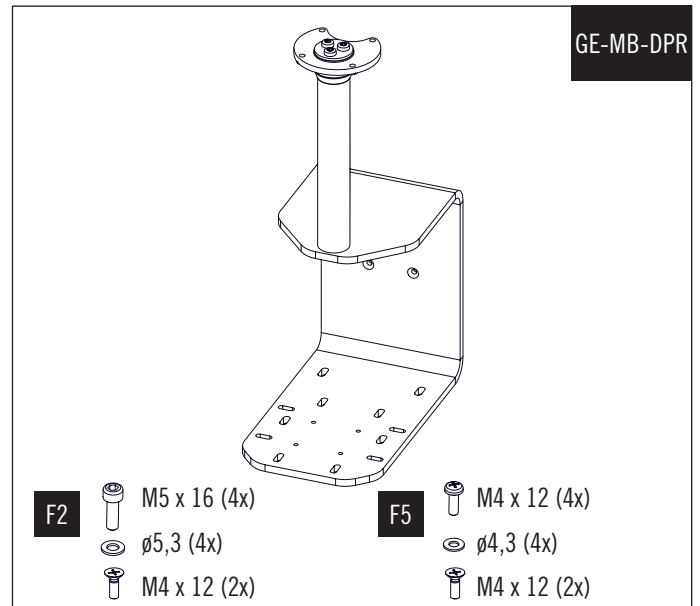
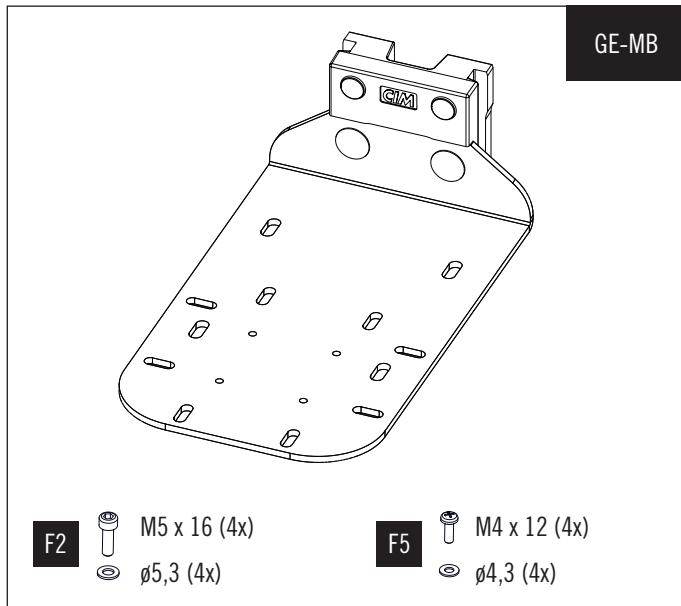
CIM med GmbH cannot be held responsible for any damage due to poor installation.

Product modifications, disassembling and reassembling and/or mounting any of the components in any other way than described below are permitted only after written approval by the CIM med GmbH construction and engineering department. CIM med GmbH cannot be held responsible for any damage or injury caused by product modifications and/ or amendments without prior permission from CIM med GmbH.

The additional instructions for use and the inspection plan are intended for end users.

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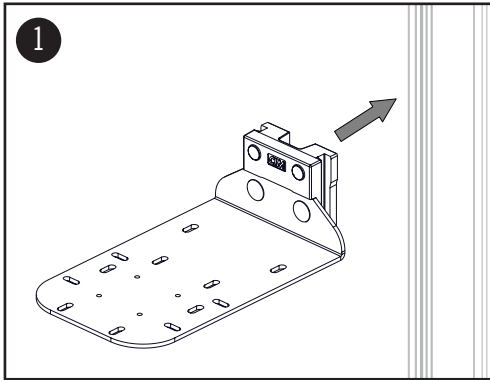
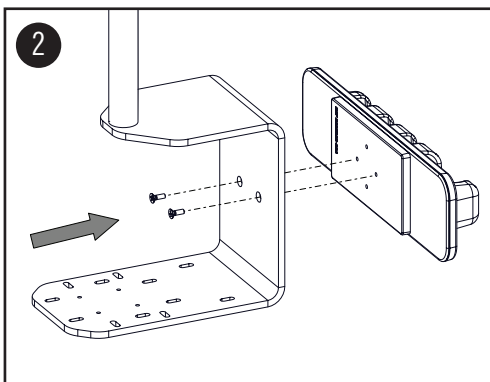
Mount the L-bracket to its designated place (wall channel, tube, DIN rail etc.). The illustration shows mounting to a wall channel.

**NOTE:**

Please refer to the corresponding assembly instruction for all other adapters.

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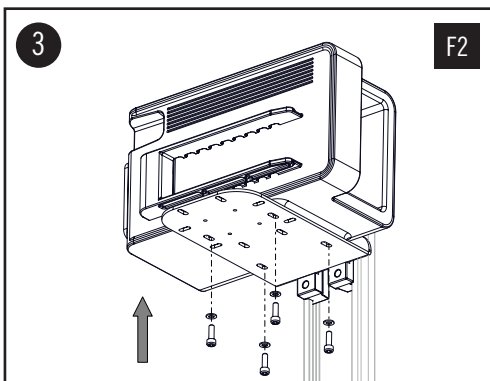
Mount the bracket using the (2) M4x12 counter-sunk screws.


**NOTE:**

This step is only possible with the GE-MB-DRP mounting bracket.

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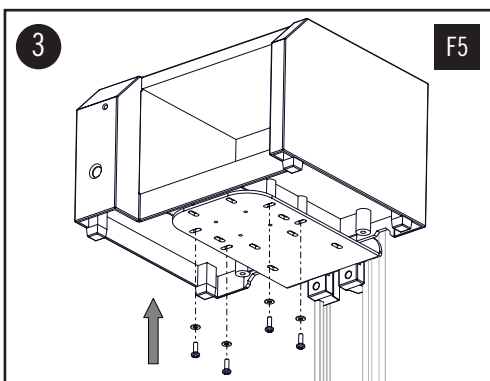Attach the GE Frame F2 using the (4) M5x16 cylinder-head screws and the (4)  $\phi$ 5.3 washers.

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Attach the GE Frame F5 using the (4) M4x12 pan-head screws and the (4)  $\phi$ 4.3 washers.

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Die CIM med GmbH übernimmt keine Haftung für Schäden, die durch nicht fachgerechte Montage entstanden sind.