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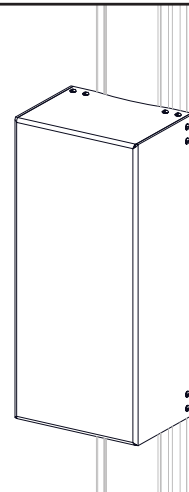
ASSEMBLY INSTRUCTION

POWER SUPPLY ENCLOSURE**WARNHINWEISE** 

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WARNING 

Please check the weight of the ME equipment to be mounted. The permissible total weight is mentioned on the corresponding data sheet.

Failure to follow these warnings may result in damage to equipment or injury to people.

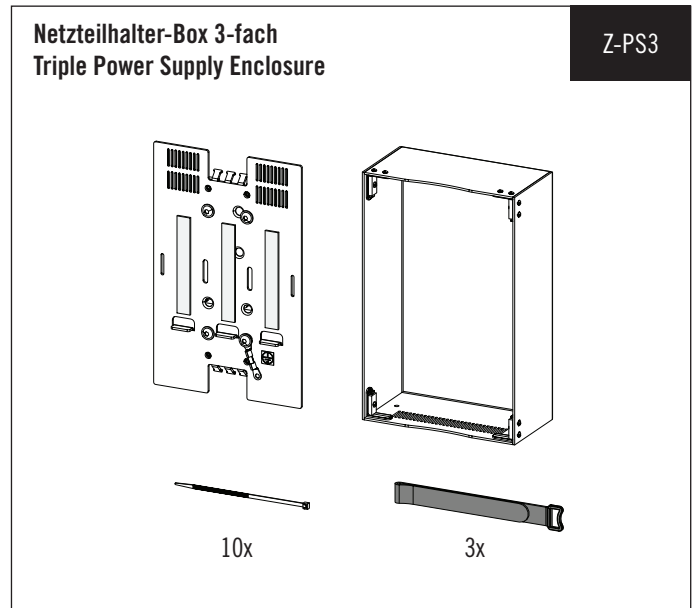
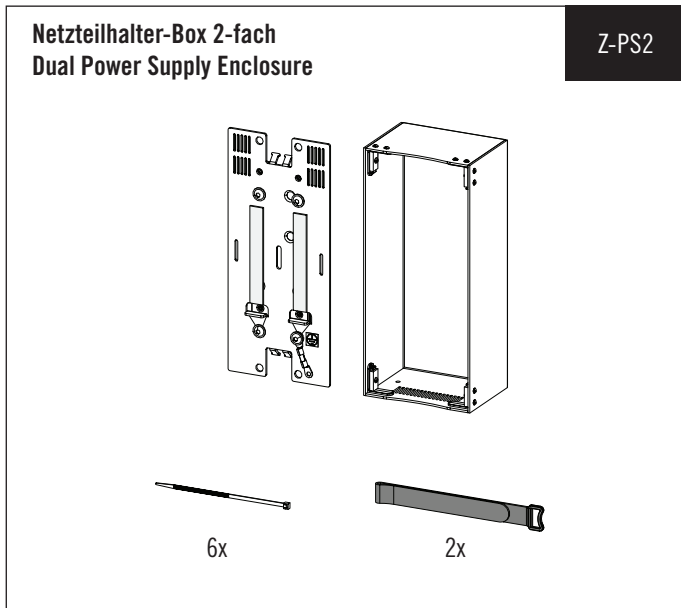
CIM med GmbH cannot be held responsible for any damage due to poor installation.

Product modifications, disassembling and reassembling and/or mounting any of the components in any other way than described below are permitted only after written approval by the CIM med GmbH construction and engineering department. CIM med GmbH cannot be held responsible for any damage or injury caused by product modifications and/ or amendments without prior permission from CIM med GmbH.

The additional instructions for use and the inspection plan are intended for end users.

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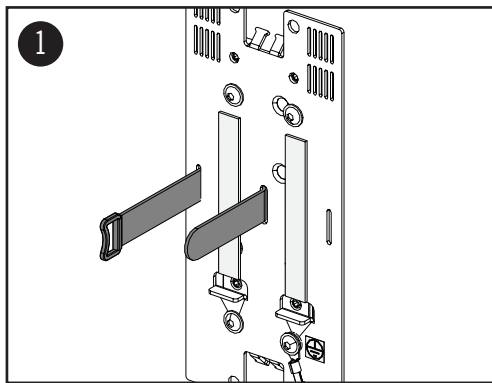
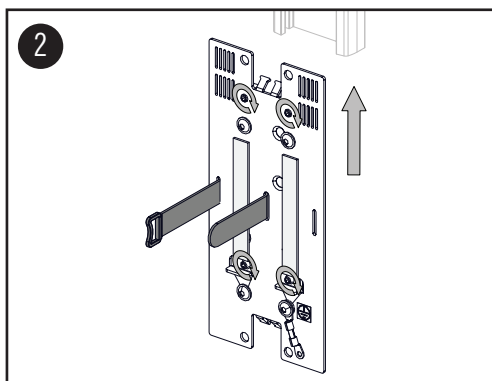
Thread the hook-and-loop fastener through the recess in the base plate.

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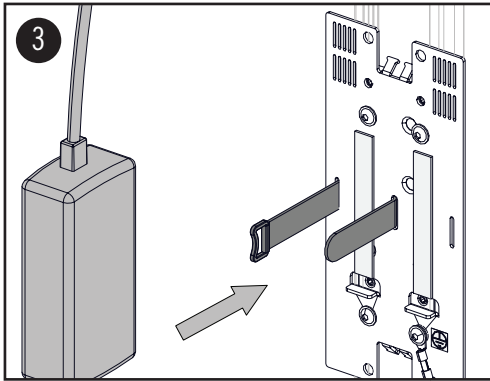
Mount the power supply enclosure to its designated place and tighten the (4) M6x20 threaded pins. The illustration shows mounting to a CIM wall channel.

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NOTE:
Please refer to the corresponding assembly instruction for all other adapters.

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
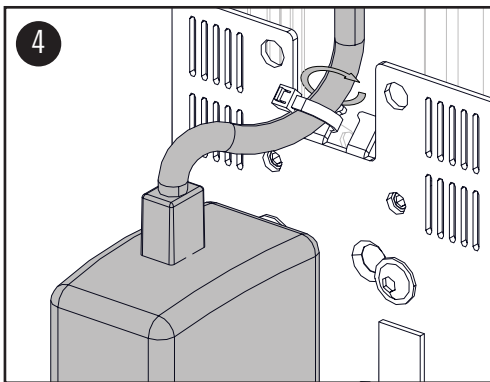
Attach the power supplies by using the pre-assembled hook-and-loop fastener.

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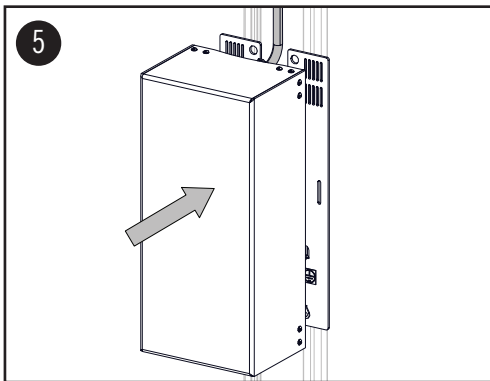Use the cable ties to fix the cables to the designated lug of the base plate.

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Die CIM med GmbH übernimmt keine Haftung für Schäden, die durch nicht fachgerechte Montage entstanden sind.