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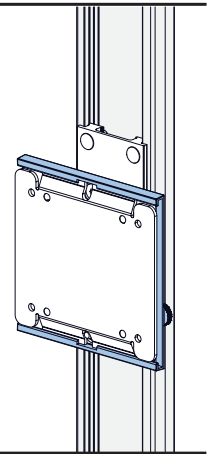
## ASSEMBLY INSTRUCTION

# BRACKET FOR DRÄGER P2500 POWER SUPPLY

**WARNHINWEISE** 

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**WARNING** 

Please check the weight of the ME equipment to be mounted. The permissible total weight is mentioned on the corresponding data sheet.

Failure to follow these warnings may result in damage to equipment or injury to people.

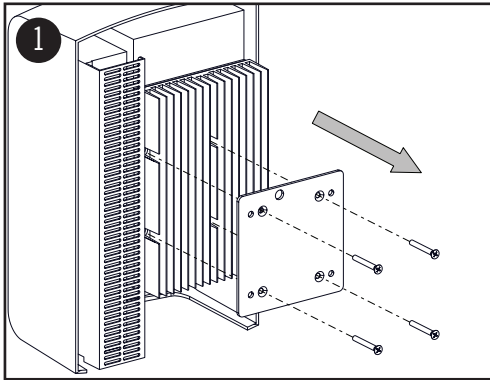
CIM med GmbH cannot be held responsible for any damage due to poor installation.

Product modifications, disassembling and reassembling and/or mounting any of the components in any other way than described below are permitted only after written approval by the CIM med GmbH construction and engineering department. CIM med GmbH cannot be held responsible for any damage or injury caused by product modifications and/ or amendments without prior permission from CIM med GmbH.

The additional instructions for use and the inspection plan are intended for end users.

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
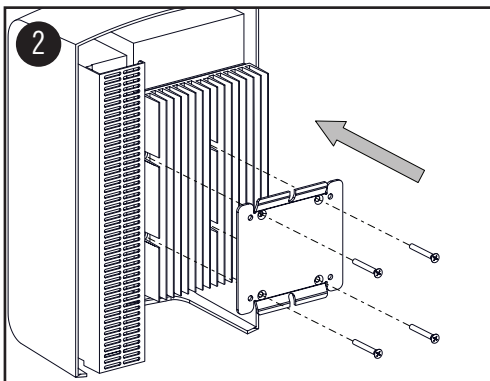
Loosen the (4) countersunk screws and remove the preassembled adapter plate.

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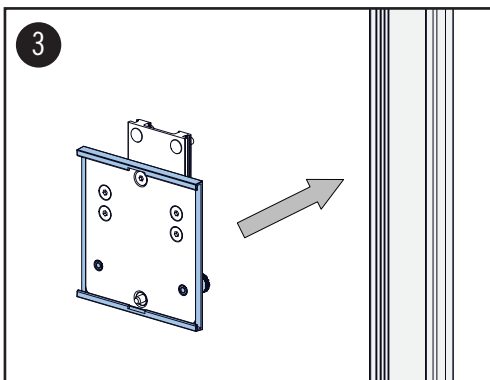
Mount the supplied adapter plate using the enclosed (4) M5x40 countersunk screws.

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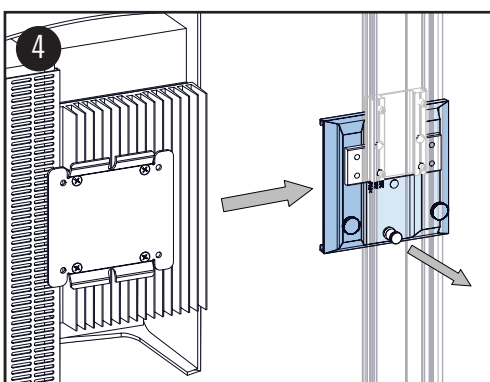Mount the bracket to its designated place (wall channel, tube, DIN rail etc.). The illustration shows a wall channel adapter. Please refer to the corresponding assembly instruction for all other adapters.

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
Slide the mounted adapter plate into the bracket. Pull out the spring bolt slightly.

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
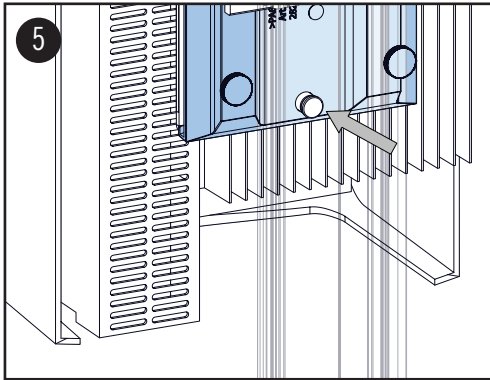
**NOTE:**

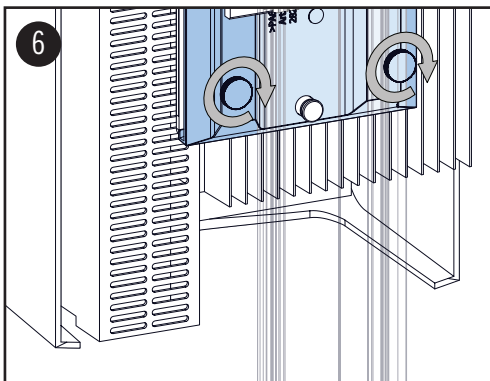Ensure that the spring bolt is engaged.

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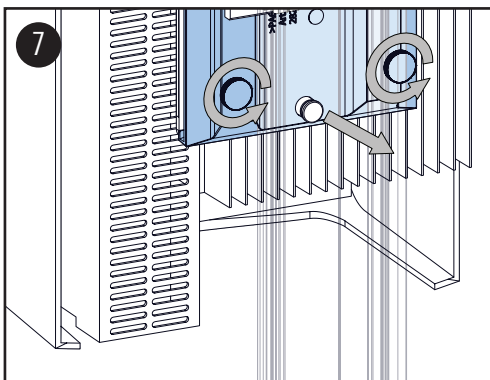Tighten the plastic screws to lock the adapter plate.

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To dismantle the ME equipment, loosen the plastic screws and pull out the spring bolt slightly.

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Die CIM med GmbH übernimmt keine Haftung für Schäden, die durch nicht fachgerechte Montage entstanden sind.