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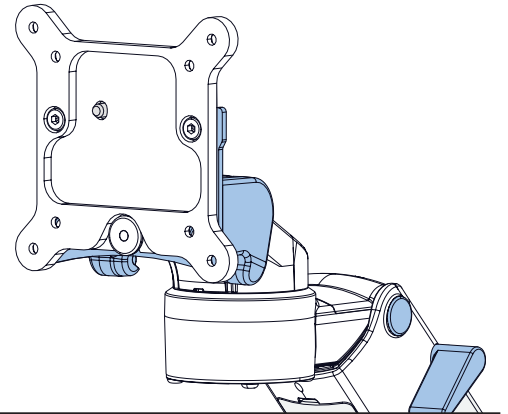
ASSEMBLY INSTRUCTION

MONITOR ADAPTATION QUICK RELEASE

**WARNHINWEISE** ⚠

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WARNING ⚠

Please check the weight of the ME equipment to be mounted. The permissible total weight is mentioned on the corresponding data sheet.

Failure to follow these warnings may result in damage to equipment or injury to people.

CIM med GmbH cannot be held responsible for any damage due to poor installation.

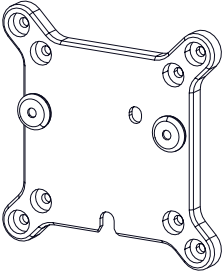
Product modifications, disassembling and reassembling and/or mounting any of the components in any other way than described below are permitted only after written approval by the CIM med GmbH construction and engineering department. CIM med GmbH cannot be held responsible for any damage or injury caused by product modifications and/ or amendments without prior permission from CIM med GmbH.

The additional instructions for use and the inspection plan are intended for end users.

VARIANTEN

VERSIONS


Z-QR-G

M4x16 (4x) M4x12 (4x) M4x10 (4x) Spacer (4x)

The Z-QR-G variant is a square-shaped metal bracket with rounded corners. It features four large circular mounting holes at the corners and four smaller circular holes near the center. A notch is present at the bottom edge. The diagram shows four screws of different lengths and four spacers used for assembly.

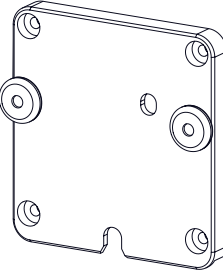
Z-QR-C

M4x16 (4x) M4x12 (4x) M4x10 (4x) Spacer (4x)

The Z-QR-C variant is a square-shaped metal bracket with rounded corners, similar to Z-QR-G but with a different hole pattern. It has four large circular mounting holes at the corners and four smaller circular holes near the center. A notch is present at the bottom edge. The diagram shows four screws of different lengths and four spacers used for assembly.

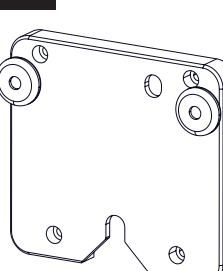
MD-QR-SMR

M3x12 (4x)

The MD-QR-SMR variant is a square-shaped metal bracket with rounded corners. It features four large circular mounting holes at the corners and four smaller circular holes near the center. A notch is present at the bottom edge. The diagram shows four screws used for assembly.

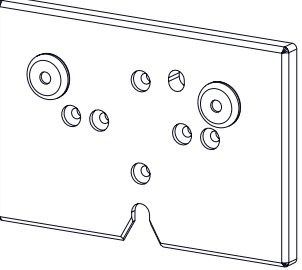PH-QR-FMS


M4x12 (4x)

The PH-QR-FMS variant is a rectangular metal plate with rounded corners. It features four large circular mounting holes at the corners and four smaller circular holes near the center. A notch is present at the bottom edge. The diagram shows four screws used for assembly.

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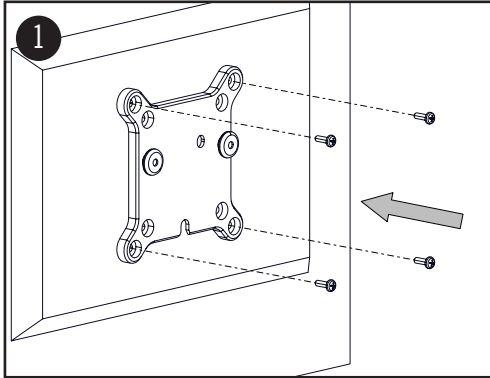
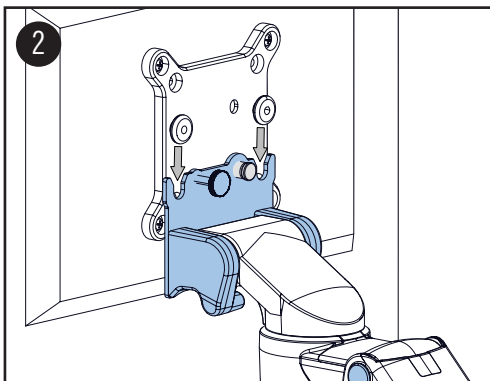
Screw adapter plate to the ME equipment. Use the screws provided.

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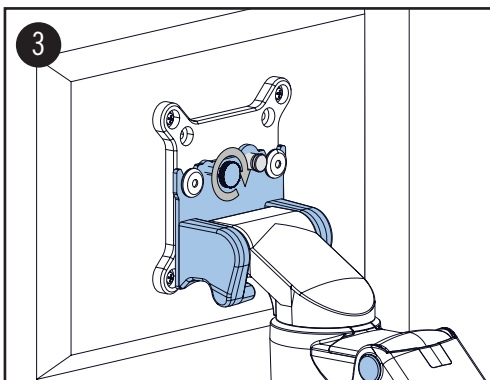
Slide the mounted adapter plate from above into the monitor mount on the support arm.

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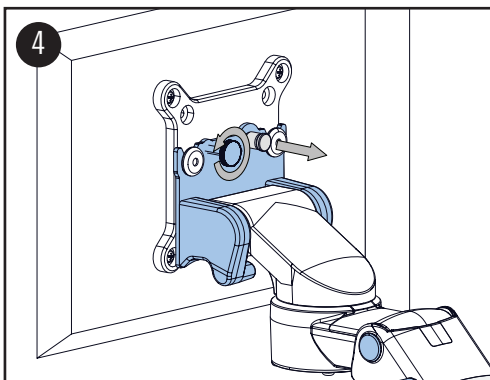Tighten the plastic screw to lock the adapter plate.

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To dismantle the ME equipment, loosen the plastic screw and pull out the spring bolt slightly.