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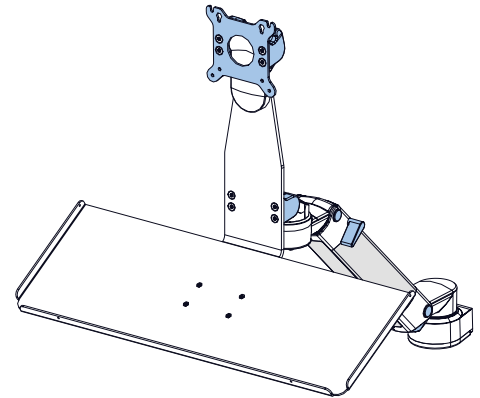
ASSEMBLY INSTRUCTION

MONITOR ADAPTATION L-BRACKET**WARNHINWEISE** 

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WARNING 

Please check the weight of the ME equipment to be mounted. The permissible total weight is mentioned on the corresponding data sheet.

Failure to follow these warnings may result in damage to equipment or injury to people.

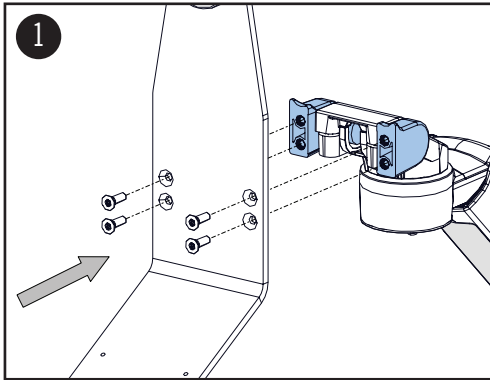
CIM med GmbH cannot be held responsible for any damage due to poor installation.

Product modifications, disassembling and reassembling and/or mounting any of the components in any other way than described below are permitted only after written approval by the CIM med GmbH construction and engineering department. CIM med GmbH cannot be held responsible for any damage or injury caused by product modifications and/or amendments without prior permission from CIM med GmbH.

The additional instructions for use and the inspection plan are intended for end users.

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

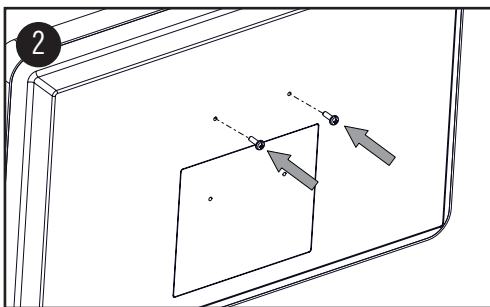
Mount the l-bracket to the plate holders using the (4) 6x19 counter sunk screws and tighten all screws firmly.

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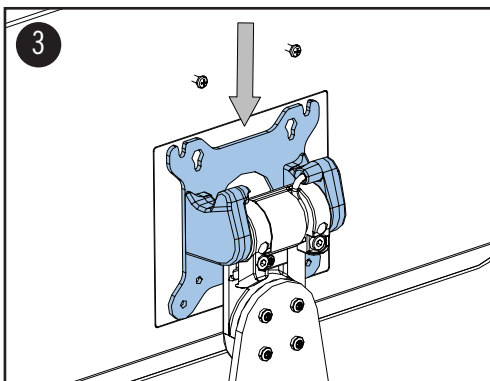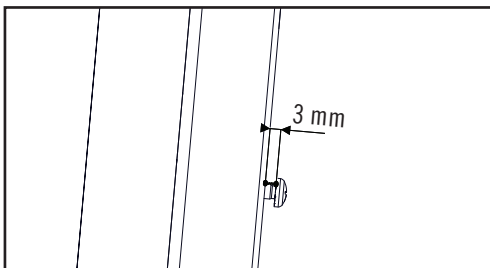
For easy mounting fix the two upper M4 pan head screws to the monitor rear side.

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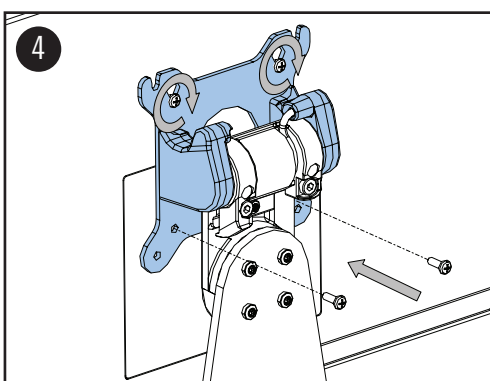
Fix the lower two screws to the monitor. Tighten the 4 screws.

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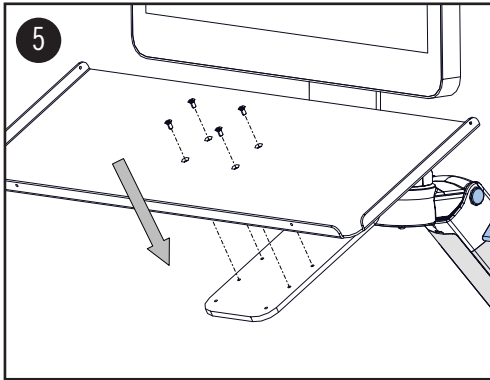Mount keyboard tray by using (4) M4 x 8 counter-sunk screws.

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TILT ADJUSTMENT

To adjust the overall tilt tension, tighten or loosen the both hex screws with the hex key 4 mm provided. By tightening both of the screws the tilt function can be locked completely. Maximum permissible tightening torque is 8 Nm (70 in-lbs).

⚠ WARNING: In case of overtightening the screw the casting respectively the threads may break which leads to loss of function.

⚠ WARNING: When loosening both hex screws the ME equipment will tilt forward.

⚠ DANGER OF SQUEEZING FINGERS