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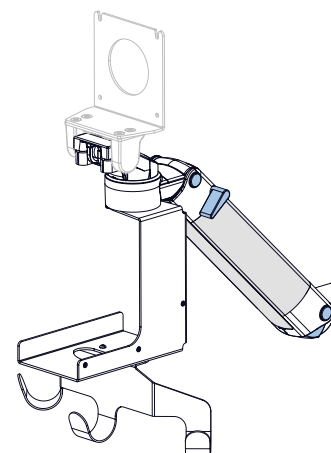
## ASSEMBLY INSTRUCTION

## BRACKET FOR FUKUDA SUPER UNIT AND FUKUDA SUPER UNIT WITH GAS MODULE

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**WARNING** ⚠

Please check the weight of the ME equipment to be mounted. The permissible total weight is mentioned on the corresponding data sheet.

Failure to follow these warnings may result in damage to equipment or injury to people.

CIM med GmbH cannot be held responsible for any damage due to poor installation.

Product modifications, disassembling and reassembling and/or mounting any of the components in any other way than described below are permitted only after written approval by the CIM med GmbH construction and engineering department. CIM med GmbH cannot be held responsible for any damage or injury caused by product modifications and/ or amendments without prior permission from CIM med GmbH.

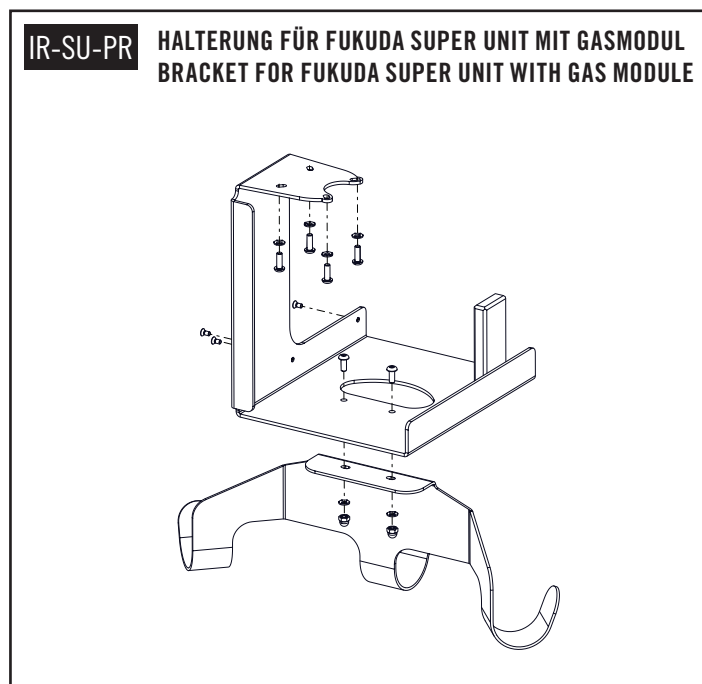
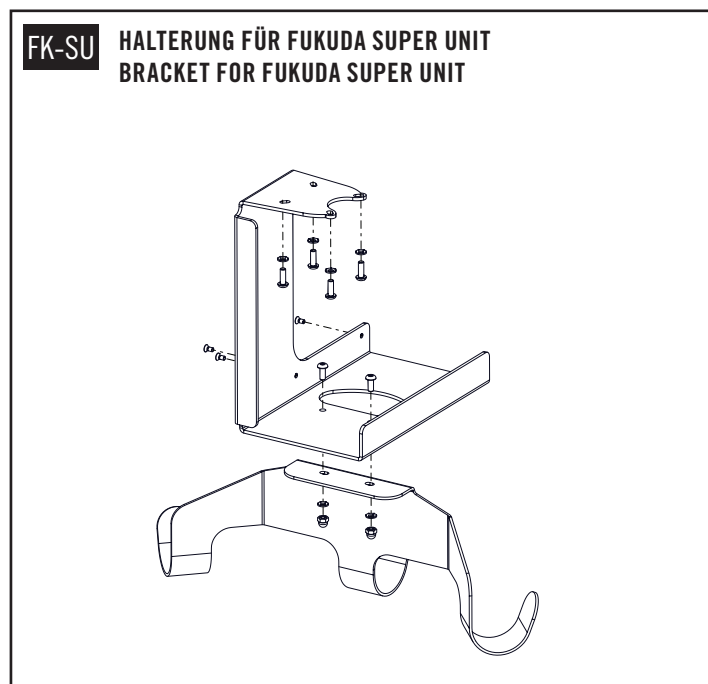
The additional instructions for use and the inspection plan are intended for end users.

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
BRACKET FOR FUKUDA SUPER UNIT  
BRACKET FOR FUKUDA SUPER UNIT WITH GAS MODULE

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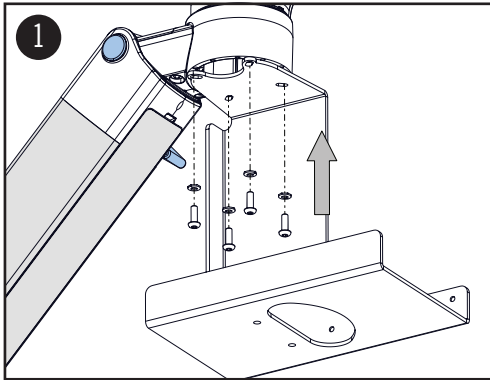
Mount the bracket for Fukuda super unit to the support arm as shown using the (4) M4x12 pan head screws and the (4) contact wheels.

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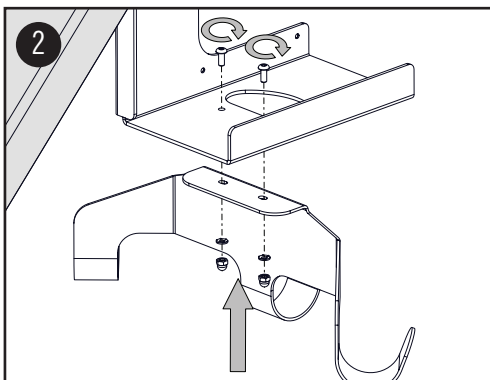
**⚠ WARNING**

To guarantee a secure components grounding it is essential to use the (4) contact wheels.

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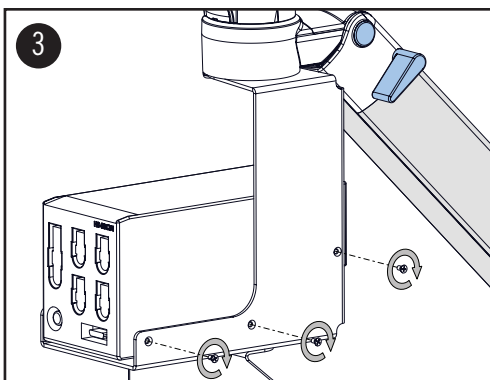Attach the cable hook using the (2) M4x10 pan head screws, the (2) contact wheels and the (2) M4 cap nuts.

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Use the (3) M3x6 counter-sunk screws to fix the ME equipment.

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Die CIM med GmbH übernimmt keine Haftung für Schäden, die durch nicht fachgerechte Montage entstanden sind.