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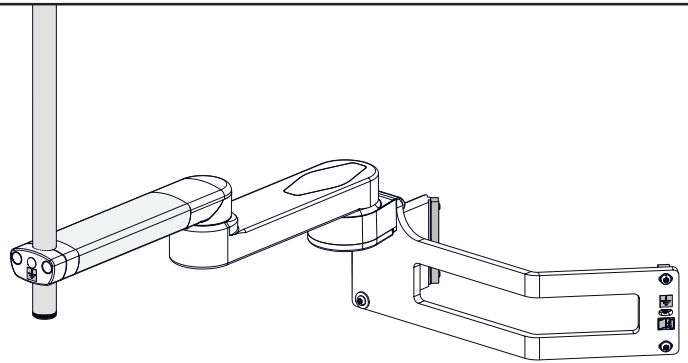
ASSEMBLY INSTRUCTION

**SINGLE MOUNT
FOR GETINGE FLOW-I****WARNHINWEISE** 

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WARNING 

Please check the weight of the ME equipment to be mounted. The permissible total weight is mentioned on the corresponding data sheet.

Failure to follow these warnings may result in damage to equipment or injury to people.

CIM med GmbH cannot be held responsible for any damage due to poor installation.

Product modifications, disassembling and reassembling and/or mounting any of the components in any other way than described below are permitted only after written approval by the CIM med GmbH construction and engineering department. CIM med GmbH cannot be held responsible for any damage or injury caused by product modifications and/ or amendments without prior permission from CIM med GmbH.

The additional instructions for use and the inspection plan are intended for end users.

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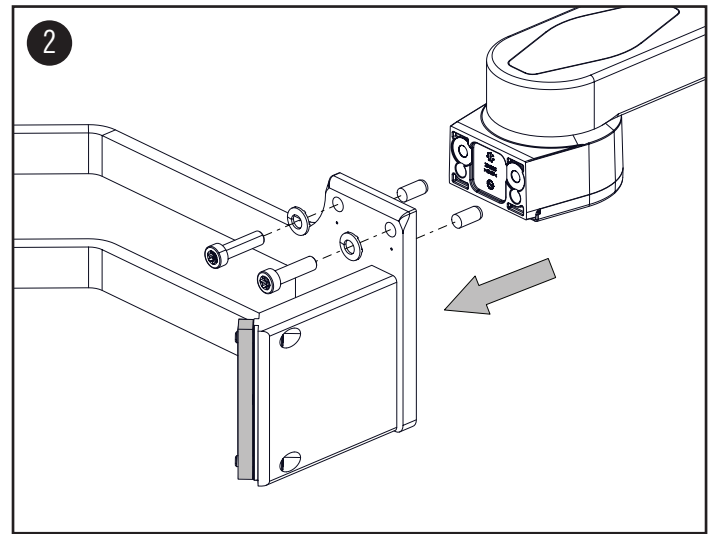
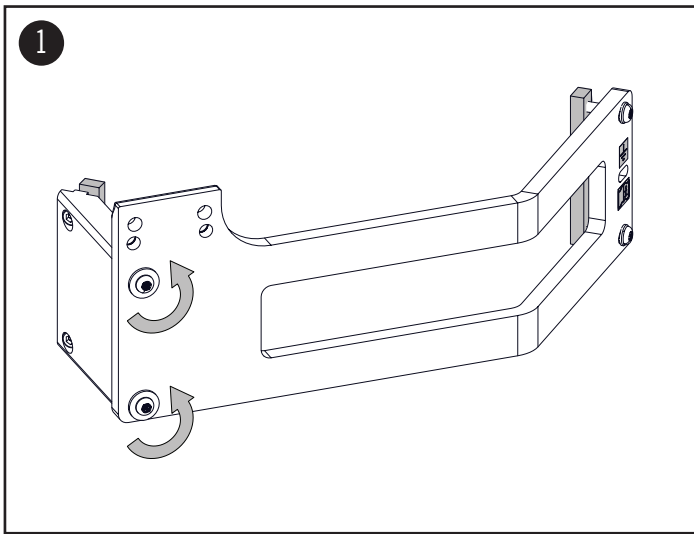
1. Loosen the (2) M6x25 pan head screws for one turn.
2. Insert (2) cylinder pins into the lower holes. Mount support arm with (2) M8x30 cylinder head screws and (2) contact washers.
Torque: 25 Nm

⚠ WARNING: The use of contact washers is absolutely necessary for a safe grounding of components.

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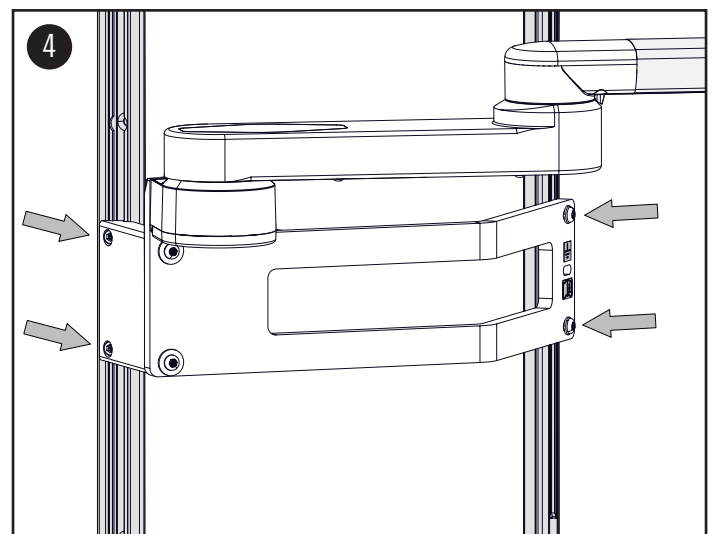
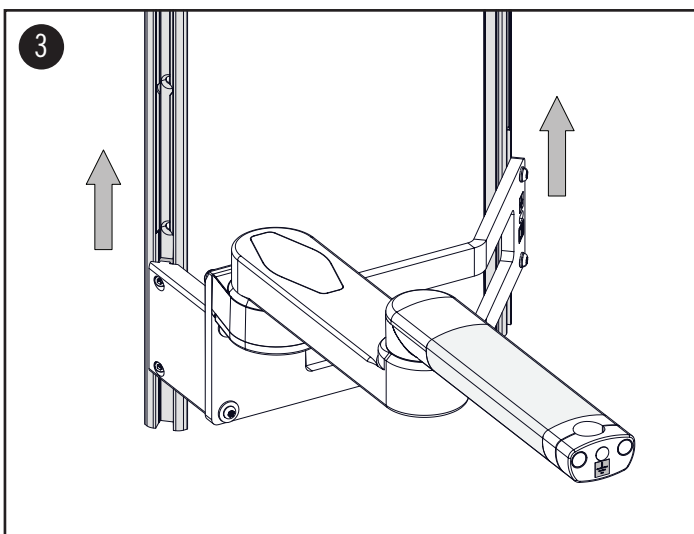
3. Thread single mount into the channel of anaesthesia machine.

ⓘ NOTE: Loosen the screw connection of the T-nuts to allow threading.


4. Tighten the (2) M6x20 cylinder head screws and (2) M6x25 pan head screws.

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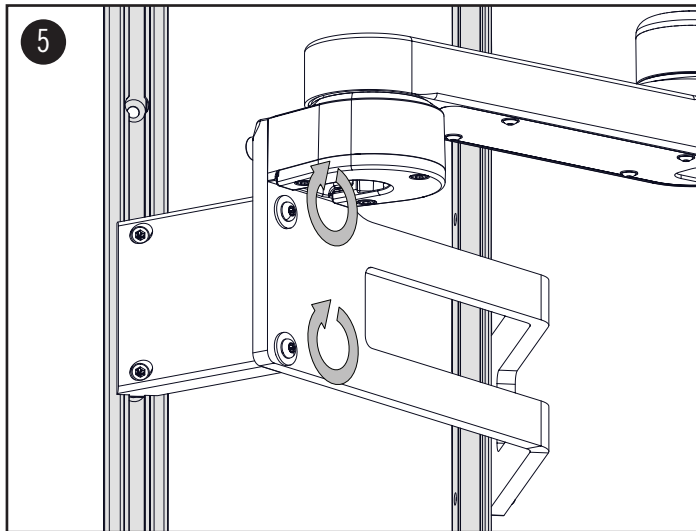
ASSEMBLY STEPS


5. Fasten the (2) M6x25 pan head screws to fix the bridge and support plate.
6. Mount the (1) M6x20 threaded pin.

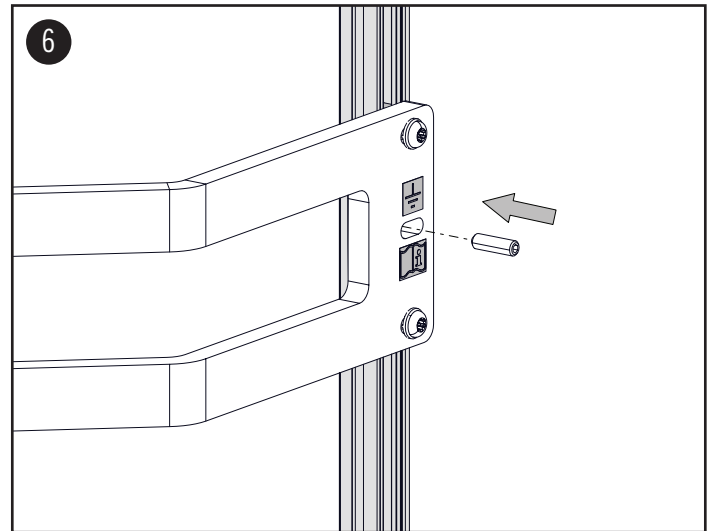⚠ WARNING: The use of threaded pin is absolutely necessary for a safe grounding of components.

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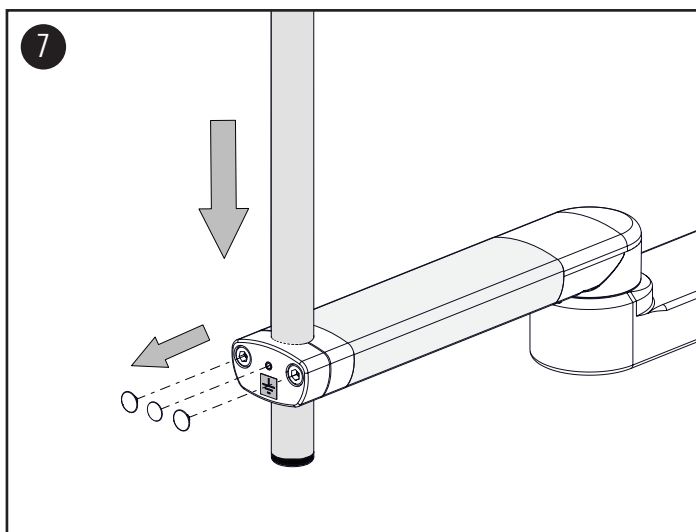7. Remove the (3) cover caps and loosen the (2) M8x55 cylinder head screws. Insert the infusion pole.

8. First tighten the (2) M8x55 cylinder head screws evenly. Now tighten the (1) M6x8 threaded pin and cover all screws with the (3) cover caps.

⚠ WARNING: The use of threaded pin is absolutely necessary for a safe grounding of components.