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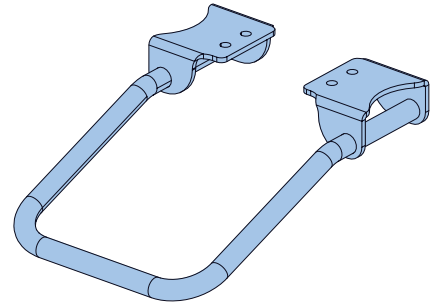
ASSEMBLY INSTRUCTION

ACCESSORIES HANDLE

FOR MONITOR MOUNT 5“, MINDRAY AND PHILIPS

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WARNING 

Please check the weight of the ME equipment to be mounted. The permissible total weight is mentioned on the corresponding data sheet.

Failure to follow these warnings may result in damage to equipment or injury to people.

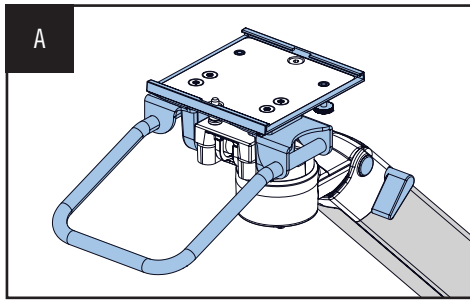
CIM med GmbH cannot be held responsible for any damage due to poor installation.

Product modifications, disassembling and reassembling and/or mounting any of the components in any other way than described below are permitted only after written approval by the CIM med GmbH construction and engineering department. CIM med GmbH cannot be held responsible for any damage or injury caused by product modifications and/ or amendments without prior permission from CIM med GmbH.

The additional instructions for use and the inspection plan are intended for end users.

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
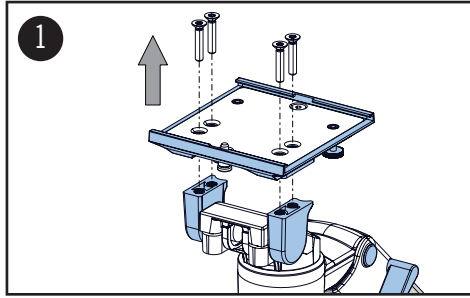
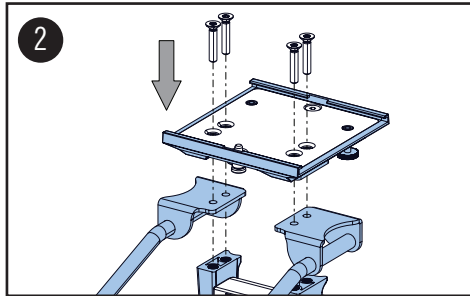
Unscrew the (4) 6x32 counter-sunk screws and remove the 5" plunge plate.

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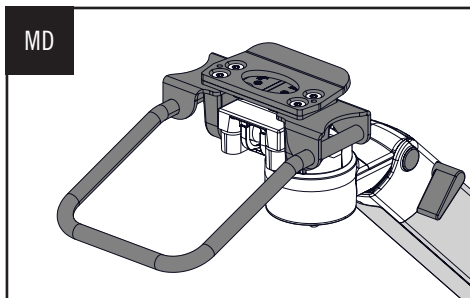
Mount handle and 5" plunge plate using the (4) 6x32 counter-sunk screws.

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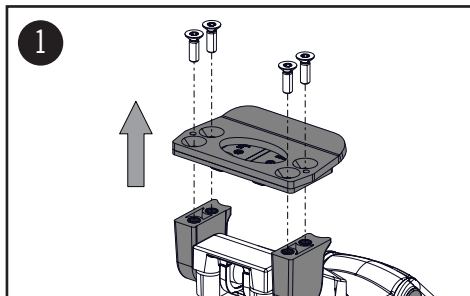
HANDLE FOR
MINDRAY QUICK RELEASE MOUNTING ADAPTER

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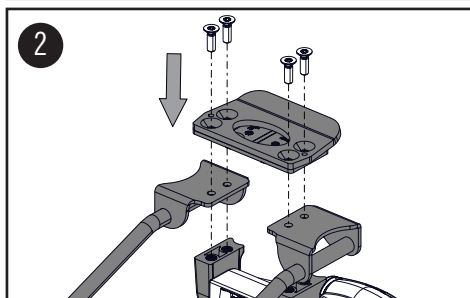Unscrew the (4) 6x19 counter-sunk screws and remove the Mindray quick release mounting adapter.

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
Mount handle and Mindray quick release mounting adapter using the (4) 6x19 counter-sunk screws.

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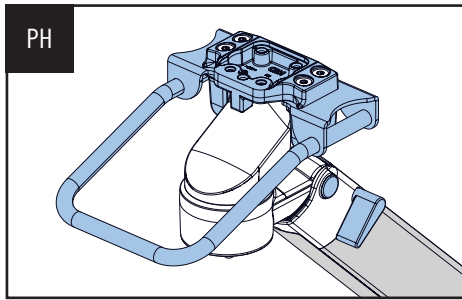
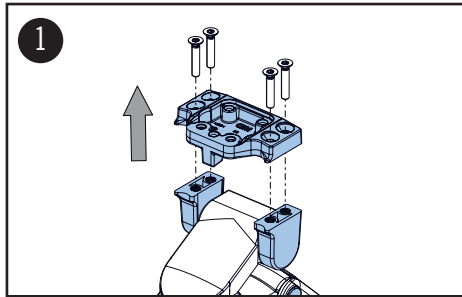
**HANDLE FOR
PHILIPS MONITORING INTELLIVUE ADAPTER**

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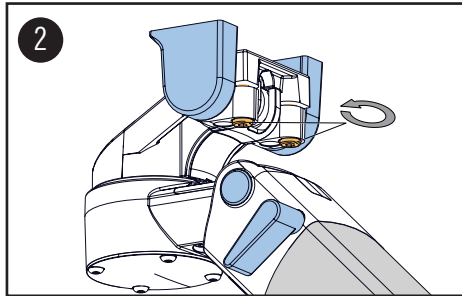
Unscrew the (4) 6x32 counter-sunk screws and remove the Philips Monitoring IntelliVue adapter.

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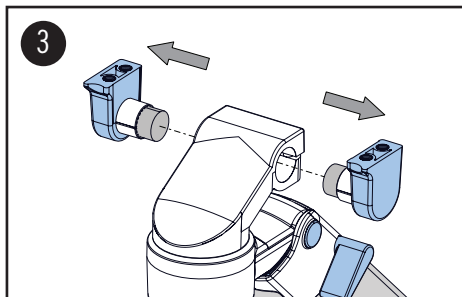
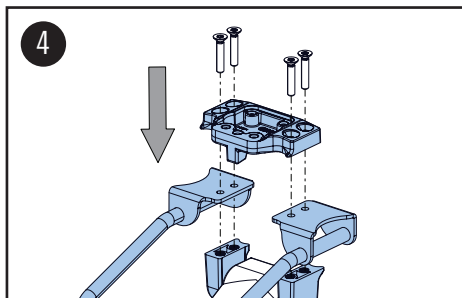
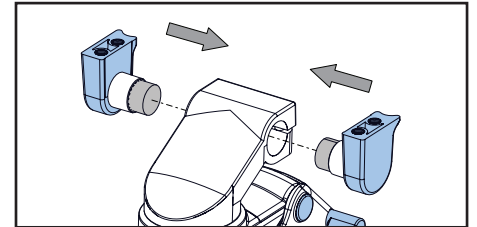
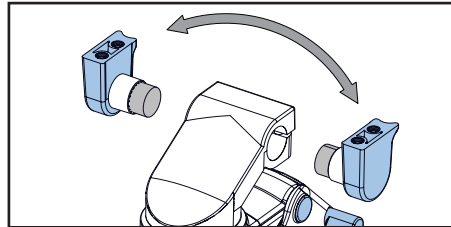
Change the right plate holder with the left one.

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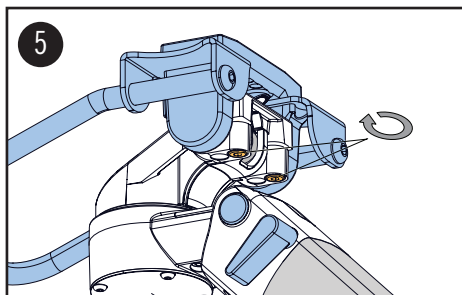Mount handle and Philips Monitoring IntelliVue adapter using the (4) 6x32 counter-sunk screws.

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Tighten the (2) M6x25 cylinder head screws until tilt function is adjusted as necessary.

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⚠ CAUTION

In case of overtightening the screw (> 10 Nm/ 88 in-lbs) the casting respectively the threads may break which leads to loss of function.