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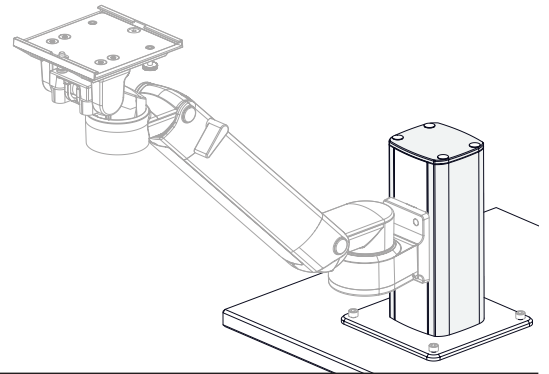
## ASSEMBLY INSTRUCTION

**TABLE COLUMN / FLOOR COLUMN****WARNHINWEISE** ⚠

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**WARNING** ⚠

Please check the weight of the ME equipment to be mounted. The permissible total weight is mentioned on the corresponding data sheet.

Failure to follow these warnings may result in damage to equipment or injury to people.

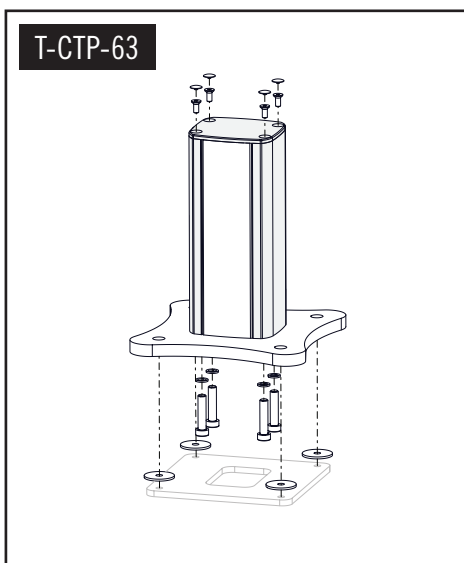
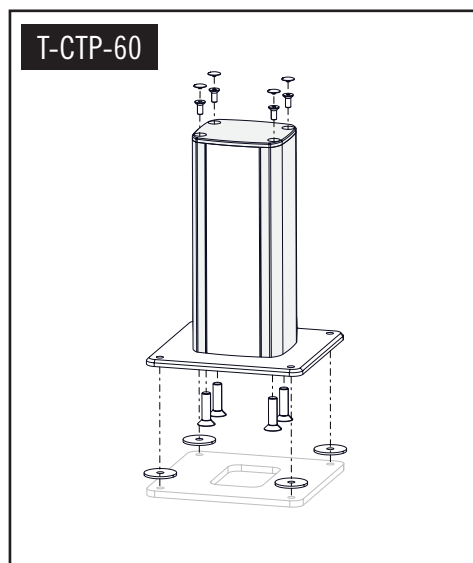
CIM med GmbH cannot be held responsible for any damage due to poor installation.

Product modifications, disassembling and reassembling and/or mounting any of the components in any other way than described below are permitted only after written approval by the CIM med GmbH construction and engineering department. CIM med GmbH cannot be held responsible for any damage or injury caused by product modifications and/ or amendments without prior permission from CIM med GmbH.

The additional instructions for use and the inspection plan are intended for end users.

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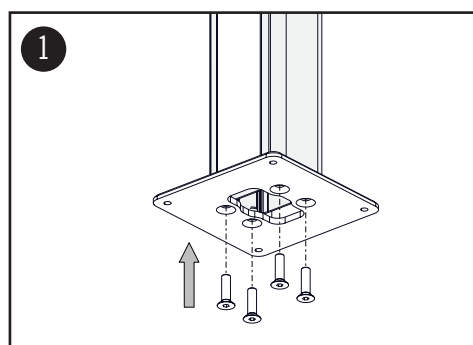
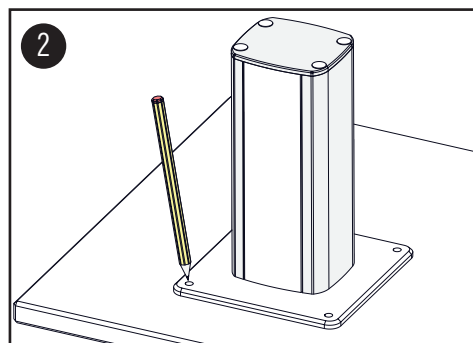
Mount the table/floor column with (4) M8 x 30 countersunk head screws respectively M8 x 35 cylinder head screws and detent edged washers on the base plate.

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
Position the column and mark the required drilling positions.

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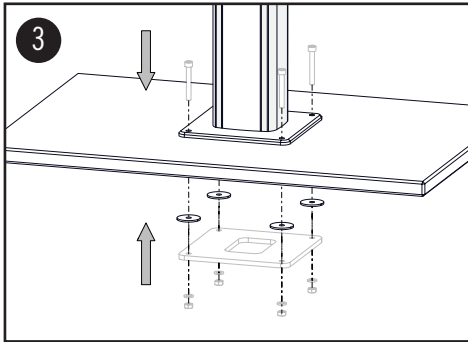
Mount the table/floor column using suitable assembly material and attached washers. Make sure to firmly tighten all screws.

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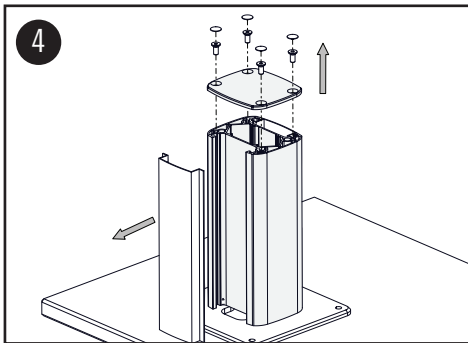
**NOTE:**

Depending on the case, it is recommended to use the optional counter plate.

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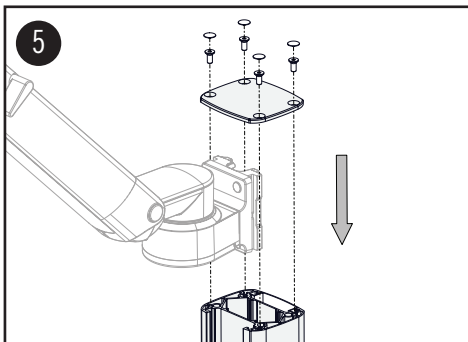
Loosen the (4) M5 x 12 countersunk head screws and remove the top cover and plastic cover.

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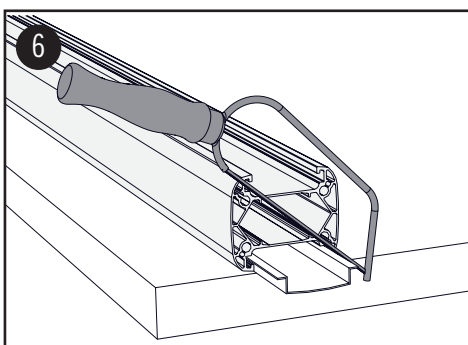
Push the support arm into table/floor column from above and mount it at the favored height. Please refer to the corresponding assembly instruction of the support arm.

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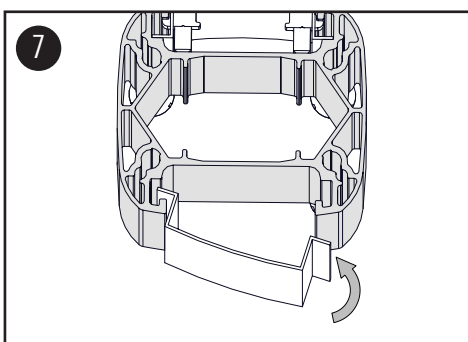Define the necessary length of the channel cover and cut it as shown.

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Mount the plastic covers into the column as shown.