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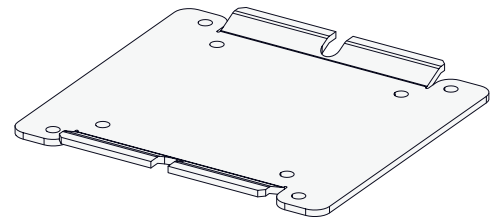
# ADAPTERPLATTEN

## ASSEMBLY INSTRUCTION

# ADAPTER PLATES

### WARNHINWEISE ⚠

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### WARNING ⚠

Please check the weight of the ME equipment to be mounted. The permissible total weight is mentioned on the corresponding data sheet.

Failure to follow these warnings may result in damage to equipment or injury to people.

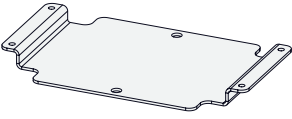
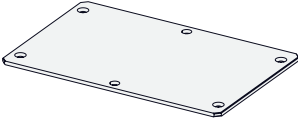
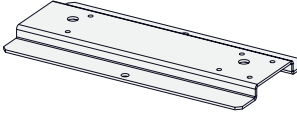
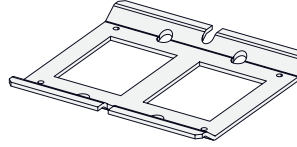
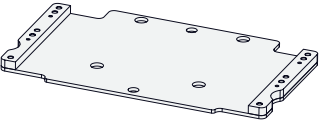
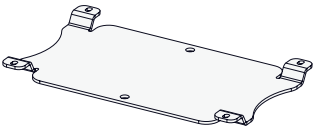
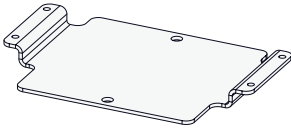
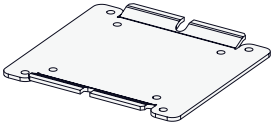
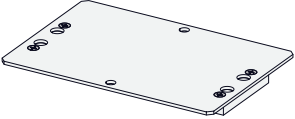
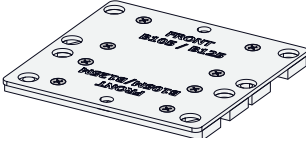
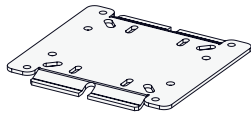
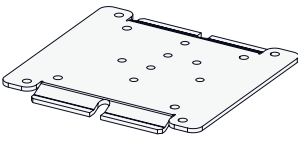
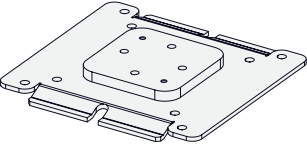
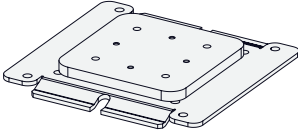
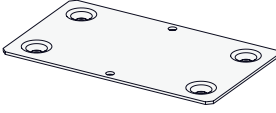
CIM med GmbH cannot be held responsible for any damage due to poor installation.

Product modifications, disassembling and reassembling and/or mounting any of the components in any other way than described below are permitted only after written approval by the CIM med GmbH construction and engineering department. CIM med GmbH cannot be held responsible for any damage or injury caused by product modifications and/ or amendments without prior permission from CIM med GmbH.

The additional instructions for use and the inspection plan are intended for end users.

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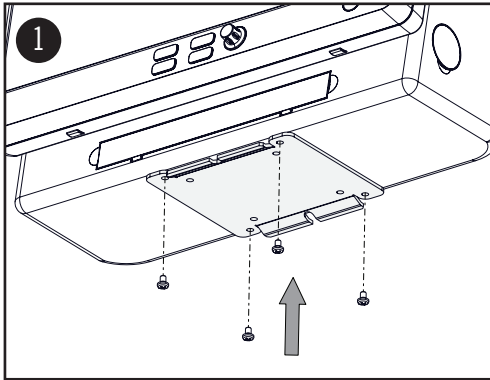
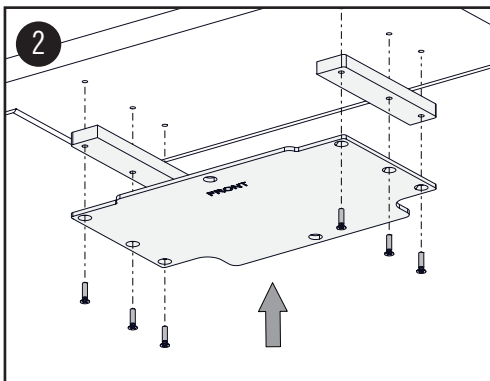
## VERSION 1: ADAPTER PLATE

Screw adapter plate to the ME equipment. Use the screws provided.

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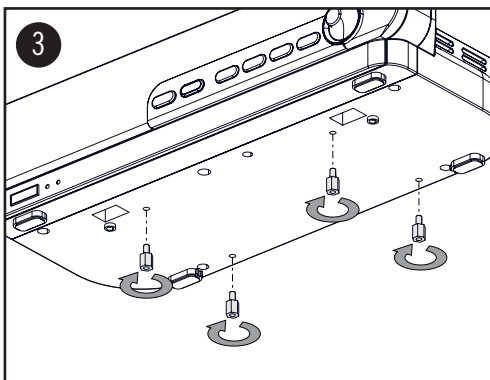
## VERSION 2: ADAPTER PLATE WITH FLAT RODS

Screw adapter plate and flat rods to the ME equipment. Use the screws provided.

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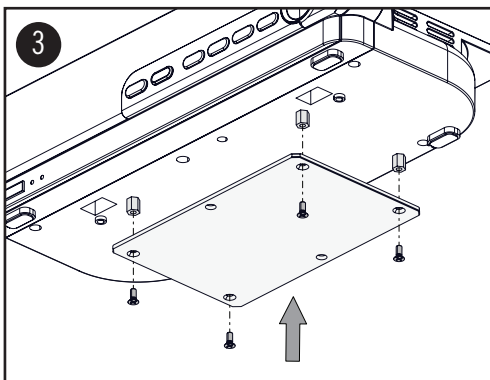
## VERSION 3: ADAPTER PLATE WITH SPACER BOLTS

Screw spacer bolts to the ME equipment.

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Screw the adapter plate to the spacer bolts. Use the screws provided.

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Die CIM med GmbH übernimmt keine Haftung für Schäden, die durch nicht fachgerechte Montage entstanden sind.